



# Propaq<sup>®</sup> MD Operator's Guide

---



The issue date for the Propaq MD Operator's Guide (**REF 9650-0802-01 Rev. D**) is **September, 2012**.

If more than 3 years have elapsed since the issue date, contact ZOLL Medical Corporation to determine if additional product information updates are available.

Copyright © 2012 ZOLL Medical Corporation. All rights reserved. pedi-padz, Rectilinear Biphasic, stat-padz, SurePower, and ZOLL are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are property of their respective owners.

IC Model: XSCP-1



**ZOLL Medical Corporation**  
269 Mill Road  
Chelmsford, MA USA  
01824-4105



**ZOLL International Holding B.V.**  
Newtonweg 18  
6662 PV ELST  
The Netherlands



**0123**

---

# Table of Contents

## Chapter 1 General Information

|  |      |
|--|------|
| Product Description .....                    | 1-1  |
| Propaq MD Optional Features .....            | 1-2  |
| How to Use This Manual.....                  | 1-2  |
| Operator's Guide Updates .....               | 1-2  |
| Unpacking.....                               | 1-2  |
| Symbols Used on the Equipment .....          | 1-3  |
| Conventions.....                             | 1-6  |
| Propaq MD Indications for Use .....          | 1-6  |
| Manual Defibrillation .....                  | 1-7  |
| Semiautomatic Operation (AED) .....          | 1-7  |
| ECG Monitoring .....                         | 1-7  |
| External Transcutaneous Pacing .....         | 1-8  |
| Non-Invasive Blood Pressure Monitoring ..... | 1-8  |
| Temperature Monitoring .....                 | 1-8  |
| SpO2 Monitoring .....                        | 1-8  |
| Respiration Monitoring .....                 | 1-8  |
| CO2 Monitoring .....                         | 1-8  |
| Invasive Pressure Monitoring .....           | 1-9  |
| 12-Lead Analysis .....                       | 1-9  |
| Propaq MD Product Functions.....             | 1-9  |
| Defibrillator Function .....                 | 1-9  |
| Defibrillator Output Energy .....            | 1-9  |
| External Pacemaker .....                     | 1-9  |
| ECG Monitoring .....                         | 1-10 |
| Electrodes .....                             | 1-10 |
| Batteries .....                              | 1-11 |
| Ready For Use (RFU) Indicator .....          | 1-12 |
| Safety Considerations.....                   | 1-13 |

|                                      |      |
|--------------------------------------|------|
| Warnings.....                        | 1-13 |
| General .....                        | 1-13 |
| ECG Monitoring .....                 | 1-14 |
| Defibrillation .....                 | 1-15 |
| Pacing .....                         | 1-16 |
| Pulse Oximeter .....                 | 1-17 |
| Noninvasive Blood Pressure .....     | 1-18 |
| IBP .....                            | 1-18 |
| CO2 .....                            | 1-18 |
| Respiration .....                    | 1-19 |
| Ferromagnetic Equipment .....        | 1-19 |
| Battery .....                        | 1-19 |
| Operator Safety .....                | 1-20 |
| Patient Safety .....                 | 1-21 |
| Cautions.....                        | 1-22 |
| Restarting the Defibrillator .....   | 1-22 |
| FDA Tracking Requirements.....       | 1-23 |
| Notification of Adverse Events ..... | 1-23 |
| Software License .....               | 1-24 |
| Service.....                         | 1-24 |
| The ZOLL Serial Number.....          | 1-26 |

## Chapter 2 Product Overview

|   |      |
|---|------|
| Defibrillator Controls and Indicators.....          | 2-1  |
| The Front Panel .....                               | 2-2  |
| Display Screen .....                                | 2-4  |
| Battery Status and Auxiliary Power Indicators ..... | 2-6  |
| Patient Cables and Connectors .....                 | 2-7  |
| External Paddles .....                              | 2-9  |
| Auxiliary Power Adapter .....                       | 2-11 |
| Navigating the Display Screen.....                  | 2-12 |
| Quick Access Keys .....                             | 2-12 |
| Navigation Keys .....                               | 2-15 |
| Display Brightness .....                            | 2-15 |
| Common Tasks .....                                  | 2-16 |
| Changing the Display Brightness .....               | 2-16 |
| Replacing a Battery Pack on the Propaq MD .....     | 2-16 |
| Using Treatment Buttons .....                       | 2-17 |

---

## Chapter 3 Monitoring Overview

|  |     |
|--|-----|
| Propaq MD Monitoring Functions .....     | 3-1 |
| ECG .....                                | 3-2 |
| Heart Rate .....                         | 3-2 |
| Respiration Rate .....                   | 3-2 |
| Temperature .....                        | 3-2 |
| Invasive Pressures (IBP) .....           | 3-2 |
| Non-Invasive Blood Pressure (NIBP) ..... | 3-2 |
| Capnography (CO <sub>2</sub> ) .....     | 3-3 |
| Pulse Oximetry (SpO <sub>2</sub> ) ..... | 3-3 |
| Monitoring Display Options.....          | 3-3 |
| Configuring the Waveform Display .....   | 3-7 |

## Chapter 4 Trends

|  |     |
|--|-----|
| Displaying the Trends Status Window .....      | 4-1 |
| Displaying and Printing Trend Information..... | 4-2 |
| Changing the Trends Status Window Display..... | 4-3 |

## Chapter 5 Alarms

|   |     |
|---|-----|
| Visual Alarm Indicators .....   | 5-1 |
| Audible Alarm Indicators.....   | 5-2 |
| Alarm Indicator Self-Test.....  | 5-2 |
| Patient Alarm Display .....   | 5-3 |
| Life Threatening Rhythm Alarms .....                                  | 5-3 |
| Equipment Alert Display .....   | 5-3 |
| Responding to Active Alarms -- Silencing the Alarm .....              | 5-4 |
| Re-enabling an Alarm .....  | 5-4 |
| Suspending Alarms.....  | 5-5 |
| The Alarm Suspension Timer .....                                      | 5-6 |
| Alarm Options .....   | 5-7 |
| Selecting Default Alarm Limits .....                                  | 5-8 |
| Setting Alarm Limits Relative to the Patient -- Stat Set Option ..... | 5-8 |

## Chapter 6 Monitoring ECG

|   |     |
|---|-----|
| ECG Monitoring Setup.....                             | 6-2 |
| Preparing the Patient for Electrode Application ..... | 6-2 |
| Applying Electrodes to the Patient .....              | 6-3 |
| Connecting the ECG Cable To the Propaq MD Unit .....  | 6-5 |
| Selecting ECG Waveforms for Display .....             | 6-6 |
| Selecting the Waveform Trace Size .....               | 6-8 |
| ECG Monitoring and Pacemakers .....                   | 6-9 |
| ECG System Messages.....                              | 6-9 |

## Chapter 7 Monitoring Respiration (Resp) and Heart Rate (HR)

|  |     |
|--|-----|
| Respiration/Breath Rate Meter .....                            | 7-2 |
| Using Impedance Pneumography to Measure Respiration .....      | 7-2 |
| Configuring Respiration (RR/BR) Alarms and Settings .....      | 7-3 |
| Enabling/Disabling RR/BR Alarms and Setting Alarm Limits ..... | 7-3 |
| Using the Resp Parameter Control Panel .....                   | 7-4 |
| Heart Rate Meter .....   | 7-5 |
| Configuring Heart Rate (HR) Meter Alarms .....                 | 7-5 |
| Enabling/Disabling HR Alarms and Setting Alarm Limits .....    | 7-6 |
| Life Threatening Rhythm Alarms .....                           | 7-7 |
| Using the Heart Rate Parameter Control Panel .....             | 7-9 |

## Chapter 8 Monitoring Non-Invasive Blood Pressure (NIBP)

|   |      |
|---|------|
| How does NIBP Work? .....                                     | 8-2  |
| The NIBP Numeric Display .....                                | 8-3  |
| NIBP Setup and Use.....                                       | 8-3  |
| Selecting the NIBP Cuff .....                                 | 8-4  |
| Connecting the NIBP Cuff.....                                 | 8-6  |
| Applying the Cuff to the Patient .....                        | 8-8  |
| Ensuring Correct Cuff Inflation Settings.....                 | 8-9  |
| Configuring NIBP Alarms and Settings.....                     | 8-10 |
| Enabling/Disabling NIBP Alarms and Setting Alarm Limits ..... | 8-10 |
| Using the NIBP Parameter Control Panel .....                  | 8-12 |
| NIBP System Messages .....                                    | 8-15 |

## Chapter 9 Monitoring Pulse Oximetry (SpO<sub>2</sub>)

|   |      |
|---|------|
| Warnings -- SpO <sub>2</sub> , General.....                               | 9-2  |
| Warnings -- SpO <sub>2</sub> , Oximeter Sensor.....                       | 9-2  |
| SpO <sub>2</sub> Setup and Use .....                                      | 9-3  |
| Selecting the SpO <sub>2</sub> Sensor .....                               | 9-3  |
| Applying the SpO <sub>2</sub> Sensor .....                                | 9-3  |
| Applying a Reusable SpO <sub>2</sub> Sensor .....                         | 9-4  |
| Applying a Single-Use Sensor .....  | 9-5  |
| Connecting the SpO <sub>2</sub> Sensor.....                               | 9-6  |
| Displaying SpO <sub>2</sub> Measurements.....                             | 9-7  |
| Enabling/Disabling SpO <sub>2</sub> Alarms and Setting Alarm Limits ..... | 9-7  |
| Setting Upper and Lower Systolic Alarm Limits .....                       | 9-8  |
| Using the SpO <sub>2</sub> Parameter Control Panel .....                  | 9-8  |
| Selecting the Heart Rate/ Pulse Rate (HR/PR) Tone .....                   | 9-9  |
| SpO <sub>2</sub> System Messages.....                                     | 9-9  |
| Functional Testers and Patient Simulators .....                           | 9-10 |

---

## Chapter 10 Monitoring CO2

|  |       |
|--|-------|
| Overview .....   | 10-1  |
| CO2 Monitoring Setup and Use .....                           | 10-2  |
| Selecting the CO2 Sampling Line .....                        | 10-3  |
| Connecting the CO2 Sampling Lines .....                      | 10-4  |
| Applying a FilterLine Set .....                              | 10-5  |
| Applying a Smart CapnoLine Nasal or Nasal/Oral Cannula ..... | 10-6  |
| Measuring CO2.....   | 10-7  |
| Setting CO2 and Respiration Rate Alarms .....                | 10-8  |
| Enabling/Disabling Alarms and Setting CO2 Alarm Limits ..... | 10-8  |
| Using the CO2 Parameter Control Panel .....                  | 10-10 |
| System Messages .....  | 10-11 |
| Patents.....   | 10-12 |

## Chapter 11 Monitoring Invasive Pressures (IBP)

|  |      |
|--|------|
| Invasive Pressure Transducers .....                                      | 11-1 |
| IBP Setup.....   | 11-2 |
| Attaching the Invasive Pressure Transducer .....                         | 11-2 |
| Zeroing the Transducer.....  | 11-3 |
| Rezeroing a Transducer .....   | 11-4 |
| Displaying IBP Measurements.....   | 11-5 |
| Conditions Affecting IBP Measurements .....                              | 11-5 |
| Enabling/Disabling IBP Alarms and Setting Alarm Limits .....             | 11-6 |
| Setting Upper and Lower Systolic (SYS) Alarm Limits .....                | 11-6 |
| Setting Upper and Lower Diastolic (DIA) Alarm Limits .....               | 11-7 |
| Setting Upper and Lower Mean Arterial Pressure (MEAN) Alarm Limits ..... | 11-7 |
| Setting IBP Source Label .....   | 11-8 |
| IBP System Messages.....   | 11-9 |

## Chapter 12 Monitoring Temperature

|   |      |
|---|------|
| Temperature Monitoring Setup .....                                  | 12-1 |
| Selecting and Applying Temperature Probes.....                      | 12-1 |
| Connecting the Temperature Probe .....                              | 12-2 |
| Displaying Temperature .....  | 12-2 |
| Enabling/Disabling Temperature Alarms and Setting Alarm Limits..... | 12-3 |
| Setting Upper and Lower Temperature Alarm Limits .....              | 12-3 |
| Setting Upper and Lower Temperature Alarm Limits .....              | 12-4 |
| Selecting the Temperature Label .....                               | 12-4 |
| Temperature System Messages .....                                   | 12-5 |

## Chapter 13 Automated External Defibrillator (AED) Operation

|   |      |
|---|------|
| AED Operation.....  | 13-2 |
| Determine Patient Condition Following Medical Protocols ..... | 13-2 |
| Begin CPR Following Medical Protocols .....                   | 13-2 |
| Prepare Patient .....   | 13-2 |
| 1 Turn on unit .....  | 13-3 |
| 2 Analyze .....   | 13-4 |
| 3 Press SHOCK .....   | 13-7 |
| Operating Messages .....                                      | 13-8 |
| Audio and Display Messages .....                              | 13-8 |
| Switching to Manual Mode Operation.....                       | 13-9 |

## Chapter 14 12-Lead ECG Interpretive Analysis

|  |       |
|--|-------|
| Entering Patient Information .....                             | 14-2  |
| Entering the Patient Name and ID .....                         | 14-3  |
| Entering Patient Age and Gender .....                          | 14-4  |
| 12-Lead ECG Monitoring Setup.....                              | 14-4  |
| Preparing the Patient for Electrode Application .....          | 14-4  |
| Applying Electrodes to the Patient .....                       | 14-5  |
| Connecting the 12-Lead Cable .....                             | 14-7  |
| Observing the 12-Lead Waveform Traces .....                    | 14-8  |
| 12-Lead Interpretive Analysis .....                            | 14-8  |
| Fault Conditions Affecting 12-Lead Interpretive Analysis ..... | 14-11 |
| Printing 12-Lead Waveform Traces .....                         | 14-12 |
| 12-Lead Print and Display Options .....                        | 14-13 |
| Selecting 12-Lead Acquire .....                                | 14-13 |
| Specifying the Number of 12-Lead Print Copies .....            | 14-13 |
| Specifying the 12-Lead Print Format .....                      | 14-13 |
| Printing 10 Seconds of the Lead II Waveform Trace .....        | 14-13 |
| Specifying the 12-Lead Frequency Response .....                | 14-14 |

## Chapter 15 Manual Defibrillation

|   |      |
|---|------|
| Emergency Defibrillation Procedure with Paddles.....                      | 15-1 |
| Determine the Patient's Condition Following Local Medical Protocols ..... | 15-2 |
| Begin CPR Following Local Medical Protocols .....                         | 15-2 |
| Turn Unit On .....  | 15-2 |
| 1 Select Energy Level .....   | 15-2 |
| 2 Charge Defibrillator .....  | 15-3 |
| 3 Deliver Shock .....   | 15-4 |

---

|   |       |
|---|-------|
| Emergency Defibrillation Procedure with Hands-Free Therapy Electrodes.....                    | 15-5  |
| Determine the Patient's Condition Following Local Medical Protocols .....                     | 15-5  |
| Begin CPR Following Medical Protocols .....   | 15-6  |
| Prepare Patient .....   | 15-6  |
| Turn Unit On .....  | 15-6  |
| 1 Select Energy Level .....   | 15-7  |
| 2 Charge Defibrillator .....  | 15-7  |
| 3 Deliver Shock .....   | 15-8  |
| Internal Paddles.....   | 15-9  |
| Verification Prior to Use .....   | 15-9  |
| Synchronized Cardioversion.....   | 15-10 |
| Synchronized Cardioversion Procedure .....  | 15-11 |
| Determine the Patient's Condition and Provide Care Following Local Medical<br>Protocols ..... | 15-11 |
| Prepare Patient .....   | 15-11 |
| Turn the Unit On .....  | 15-11 |
| Press the Sync Key .....  | 15-11 |
| 1 Select Energy Level .....   | 15-12 |
| 2 Charge Defibrillator .....  | 15-12 |
| 3 Deliver Shock .....   | 15-13 |

## Chapter 16 Advisory Defibrillation

|   |      |
|---|------|
| Advisory Defibrillation Procedure.....                                    | 16-2 |
| Determine the Patient's Condition Following Local Medical Protocols ..... | 16-2 |
| Begin CPR Following Local Medical Protocols .....                         | 16-2 |
| Prepare Patient .....   | 16-2 |
| 1 Turn on unit .....  | 16-2 |
| 2 Press ANALYZE Key .....   | 16-3 |
| 3 Press SHOCK .....   | 16-5 |

## Chapter 17 Advisory/CPR Protocol Defibrillation

|   |      |
|---|------|
| Advisory/CPR Protocol Defibrillation Procedure.....                       | 17-2 |
| Determine the Patient's Condition Following Local Medical Protocols ..... | 17-2 |
| Begin CPR Following Local Medical Protocols .....                         | 17-2 |
| Prepare Patient .....   | 17-2 |
| 1 Turn on unit .....  | 17-3 |
| 2 Press ANALYZE Button .....  | 17-4 |
| 3 Press SHOCK button .....  | 17-5 |

## Chapter 18 External Pacing

|   |      |
|---|------|
| External Pacing.....  | 18-2 |
| Pacer Modes .....   | 18-2 |
| Pacing in Demand Mode .....   | 18-2 |
| Determine Patient Condition and Provide Care Following Local Medical Protocols. | 18-2 |
| Prepare the Patient .....   | 18-2 |
| 1 Turn Unit On .....  | 18-2 |
| 2 Apply ECG Electrodes/Hands-Free Therapy Electrodes .....                      | 18-2 |
| 3 Press Pacer button .....  | 18-3 |
| 4 Set Mode .....  | 18-3 |
| 5 Set Pacer Rate .....  | 18-4 |
| 6 Turn Pacer On .....   | 18-4 |
| 7 Set Pacer Output .....  | 18-4 |
| 8 Determine Capture .....   | 18-4 |
| 9 Determine Optimum Threshold .....   | 18-5 |
| Pacing in Fixed Mode .....  | 18-5 |
| 1 Turn Unit On .....  | 18-5 |
| 2 Apply ECG Electrodes/Hands-Free Therapy Electrodes .....                      | 18-5 |
| 3 Press Pacer button .....  | 18-6 |
| 4 Set Mode .....  | 18-6 |
| 5 Set Pacer Rate .....  | 18-7 |
| 6 Turn Pacer On .....   | 18-7 |
| 7 Set Pacer Output .....  | 18-7 |
| 8 Determine Capture .....   | 18-7 |
| 9 Determine Optimum Threshold .....   | 18-8 |
| Pediatric Pacing .....  | 18-9 |

## Chapter 19 Patient Data

|   |      |
|---|------|
| Storing Data .....                      | 19-1 |
| Capturing a Data Snapshot.....          | 19-2 |
| Reviewing and printing snapshots .....  | 19-2 |
| Treatment Summary Report .....          | 19-2 |
| Printing Treatment Summary Report ..... | 19-3 |
| Transferring Data to a USB Device.....  | 19-3 |
| Clearing the Log .....                  | 19-4 |

## Chapter 20 Printing

|                             |      |
|-----------------------------|------|
| Printing Patient Data ..... | 20-1 |
| Printer Setup .....         | 20-2 |
| Automatic Prints .....      | 20-2 |
| Printing Waveforms .....    | 20-2 |
| Printing Reports .....      | 20-3 |
| Printing Trends .....       | 20-4 |

---

## Chapter 21 Maintenance

|   |       |
|---|-------|
| Daily/Shift Check Procedure .....                                 | 21-2  |
| Inspection .....  | 21-2  |
| Defibrillator/Pacing Test with Hands-Free Therapy Electrodes..... | 21-3  |
| Defibrillator Testing with External Paddles.....                  | 21-5  |
| Recommended Minimum Preventive Maintenance Schedule .....         | 21-7  |
| Annually .....  | 21-7  |
| Guidelines for Maintaining Peak Battery Performance .....         | 21-7  |
| Cleaning instructions .....                                       | 21-8  |
| Cleaning the Propaq MD unit .....                                 | 21-8  |
| Cleaning the NIBP Blood Pressure Cuff .....                       | 21-8  |
| Cleaning SpO2 Sensors .....                                       | 21-9  |
| Cleaning Cables and Accessories .....                             | 21-9  |
| Loading Recorder Paper .....                                      | 21-9  |
| Cleaning the Print Head .....                                     | 21-10 |

## Appendix A Specifications

|   |      |
|---|------|
| Defibrillator.....  | A-2  |
| Monitor/Display .....   | A-14 |
| Impedance Pneumography.....   | A-15 |
| Alarms.....   | A-15 |
| Recorder .....  | A-16 |
| Battery .....   | A-17 |
| General .....   | A-17 |
| Pacer .....   | A-18 |
| CO2 .....   | A-19 |
| Pulse Oximeter .....  | A-20 |
| Non-Invasive Blood Pressure .....   | A-20 |
| Invasive Pressures .....  | A-22 |
| Temperature.....  | A-22 |
| Clinical Trial Results for the Biphasic Waveform .....  | A-23 |
| Randomized Multicenter Clinical Trial for Defibrillation of Ventricular Fibrillation<br>(VF) and Ventricular Tachycardia (VT) ..... | A-23 |
| Randomized Multi-Center Clinical trial for Cardioversion of Atrial Fibrillation (AF) ...  | A-24 |
| Synchronized Cardioversion of Atrial Fibrillation .....   | A-25 |
| Electromagnetic Compatibility Guidance and Manufacturer's Declaration .....   | A-27 |
| ECG Analysis Algorithm Accuracy.....  | A-31 |
| Clinical Performance Results .....  | A-31 |

## Appendix B Accessories



# Chapter 1

## General Information

---

### Product Description

The ZOLL® Propaq® MD unit is an easy-to-use portable defibrillator that combines defibrillation and external pacing with the following monitoring capabilities: ECG, Pulse Oximeter, Non-invasive Blood Pressure, IBP, CO<sub>2</sub>, Temperature, and Respiration. It has been designed for all resuscitation situations and its rugged, compact, lightweight design makes it ideal for transport situations. It is powered by auxiliary power and an easily replaced battery pack that is quickly recharged in the device when it is connected to auxiliary power. In addition, the unit's battery may be recharged and tested using a ZOLL *SurePower™ Battery Charger Station*.

**Note:** The Propaq MD has defibrillation and pacing functionality, but some of the monitoring functions are optional features. See the complete list of options in Fig. 1-1. Optional features are specified as “optional” within this guide.

The unit has a large colorful LCD display of numerics and waveform data that provides easy visibility from across the room and at any angle. ECG, plethysmograph, and respiration waveform traces can be displayed simultaneously, giving easy access to all patient monitoring data at once. The display screen is configurable, so you can choose the best visual layout to fit your monitoring needs. The Propaq MD includes a transcutaneous pacemaker consisting of a pulse generator and ECG sensing circuitry. Pacing supports both demand and fixed noninvasive pacing for adult patients and adolescent, child, and infant pediatric patients.

The Propaq MD has a patient data review and collection system that allows you to view, store, and transfer patient data. The Propaq MD unit contains a printer and USB port, which you can use to print the data and transfer it to a PC.

## Propaq MD Optional Features

The following features are optional in the Propaq MD unit.

**Figure 1-1 Propaq MD Optional Features**

| Optional Feature                          |
|---|
| 12 Lead ECG                               |
| SpO <sub>2</sub> (Nellcor™)               |
| NIBP (with Smartcuf® and SureBP™)         |
| EtCO <sub>2</sub> (Oridion® Microstream®) |
| Temperature                               |
| Invasive Pressures (3 Channels)           |
| Printer                                   |

## How to Use This Manual

The Propaq MD Operator's Guide provides information operators need for the safe and effective use and care of the Propaq MD product. It is important that all persons using this device read and understand all the information contained within.

Please thoroughly read the safety considerations and warnings section.

Procedures for daily checkout and unit care are located in the Chapter 21, "Maintenance".

## Operator's Guide Updates

An issue or revision date for this manual is shown on the front cover. If more than three years have elapsed since this date, contact ZOLL Medical Corporation to determine if additional product information updates are available.

All users should carefully review each manual update to understand its significance and then file it in its appropriate section within this manual for subsequent reference.

Product documentation is available through the ZOLL website at [www.zoll.com](http://www.zoll.com). From the Products menu, choose Product Manuals.

## Unpacking

Carefully inspect each container for damage. If the shipping container or cushion material is damaged, keep it until the contents have been checked for completeness and the instrument has been checked for mechanical and electrical integrity. If the contents are incomplete, if there is mechanical damage, or if the defibrillator does not pass its electrical self-test, U.S.A. customers should call ZOLL Medical Corporation (1-800-348-9011). Customers outside of the U.S.A. should contact the nearest ZOLL authorized representative. If the shipping container is damaged, also notify the carrier.

## Symbols Used on the Equipment

Any or all of the following symbols may be used in this manual or on this equipment:

| Symbol  | Description  |
|---|--|
|    | Dangerous voltage.   |
|    | Attention, consult accompanying documents.                                     |
|    | Fragile, handle with care.   |
|    | Keep dry.  |
|    | This end up.   |
|    | Temperature limitation.  |
|  | <b>Conformité Européenne</b> Complies with medical device directive 93/42/EEC. |
|  | Type B patient connection.   |
|  | Type BF patient connection.  |
|  | Type CF patient connection.  |
|  | Defibrillator-proof type BF patient connection.                                |
|  | Defibrillator-proof type CF patient connection.                                |
|  | Fusible link.  |

| Symbol  | Description  |
|---|--|
|    | Equipotentiality.                                  |
|    | Alternating current (ac).                          |
|    | Direct current (dc).                               |
|    | Auxiliary power adapter operation.                 |
|    | Caution, high voltage.                             |
|    | Earth (ground).                                    |
|   | Negative input terminal.                           |
|  | Positive input terminal.                           |
|  | Power On/Off.                                      |
|  | Protective earth (ground).                         |
|  | Contains lithium. Recycle or dispose of properly.  |
|  | Keep away from open flame and high heat.           |
|  | Do not open, disassemble, or intentionally damage. |
|  | Do not crush.                                      |

| Symbol  | Description   |
|---|---|
|    | Do not discard in trash. Recycle or dispose of properly.  |
|    | Return to a collection site intended for waste electrical and electronic equipment (WEEE). Do not dispose of in unsorted trash. |
|    | Date of manufacture.  |
|    | Use by.   |
|    | Latex-free.   |
|    | Do not reuse.   |
|   | Do not fold.  |
|  | Not sterile.  |
|  | Manufacturer.   |
|  | Authorized representative in the European Community.  |
|  | Serial Number.  |
|  | Catalogue number.   |
|  | Consult instructions for use.   |
|  | Prescription only.  |

| Symbol  | Description   |
|---|---|
|  | Battery charging status.                                    |
|  | Do not use device, cables, or probes in an MRI environment. |

## Conventions

This guide uses the following conventions:

Within text, the names and labels for physical buttons and softkeys appear in **boldface** type (for example, “Press the **Charge** button or press the **Pacer** button”).

This guide uses uppercase italics for audible prompts and for text messages displayed on the screen (for example, *LEAD FAULT*).

---

**Warning!**     **Warning statements alert you to conditions or actions that can result in personal injury or death.**

---



---

**Caution**     Caution statements alert you to conditions or actions that can result in damage to the unit.

---

## Propaq MD Indications for Use

The Propaq MD is intended for use by trained medical personnel who are familiar with basic monitoring, vital sign assessment, emergency cardiac care and the use of the Propaq MD. The Propaq MD is also intended for use by (or on the order of) physicians at the scene of an emergency or in a hospital emergency room, intensive care unit, cardiac care unit, or other similar areas of a hospital. The usage may be in an ambulance or at the scene of an emergency. It is also intended to be used during the transport of patients. The Propaq MD will be used primarily on patients experiencing symptoms of cardiac arrest or in post trauma situation. It may also be used whenever it is required to monitor any of those functions that are included (as options) in the device. The Propaq MD can be used on pediatric patients (as described in the following table) and on adult patients (21 years of age or older) with and without heart dysfunction.

| Pediatric Patient Subpopulation | Approximate Age Range      |
|---------------------------------|----------------------------|
| Newborn (neonate)               | Birth to 1 month of age.   |
| Infant                          | 1 month to 2 years of age. |
| Child                           | 2 to 12 years of age.      |
| Adolescent                      | 12 to 21 years of age.     |

When the pediatric patient is less than 8 years of age or weighs less than 55 lbs. (25 kg.), use ZOLL pedi-padz<sup>®</sup> pediatric defibrillation electrodes. Do not delay therapy to determine the patient's exact age or weight. AED mode and ALS rescue protocol are not intended for use on neonatal patients.

## Manual Defibrillation

Use of the Propaq MD in the manual mode for external and internal defibrillation is indicated on victims of cardiac arrest where there is apparent lack of circulation as indicated by:

- Unconsciousness.
- Absence of breathing.
- Absence of pulse.

This product should be used only by qualified medical personnel for converting ventricular fibrillation and rapid ventricular tachycardia to sinus rhythm or other cardiac rhythms capable of producing hemodynamically significant heart beats.

The unit can also be used for synchronized cardioversion of certain atrial or ventricular arrhythmias. Qualified medical personnel must decide when synchronized cardioversion is appropriate.

The patient population will range from newborn (neonate) to adult.

## Semiautomatic Operation (AED)

The Propaq MD products are designed for use by emergency care personnel who have completed training and certification requirements applicable to the use of a defibrillator where the device operator controls delivery of shocks to the patient.

They are specifically designed for use in early defibrillation programs where the delivery of a defibrillator shock during resuscitation involving CPR, transportation, and definitive care are incorporated into a medically-approved patient care protocol.

Use of the Propaq MD in the Semiautomatic mode for defibrillation is indicated on victims of cardiac arrest where there is apparent lack of circulation as indicated by:

- Unconsciousness.
- Absence of breathing.
- Absence of pulse.

When the patient is less than 8 years of age or weighs less than 55 lbs. (25 Kg), use ZOLL pediatric defibrillation electrodes. Do not delay therapy to determine patient's exact age or weight.

## ECG Monitoring

The Propaq MD is intended for use to monitor and/or record 3-, 5-, or 12-lead ECG waveform and heart rate, and to alarm when heart rate is above or below limits set by the operator. The patient population will range from newborn (neonate) to adult, with and without heart dysfunction.

## External Transcutaneous Pacing

This product can be used for temporary external cardiac pacing in conscious or unconscious patients as an alternative to endocardial stimulation.

The purposes of pacing include:

- Resuscitation from standstill or bradycardia of any etiology:
- As a standby when standstill or bradycardia might be expected:
- Suppression of tachycardia.
- Pediatric pacing.

## Non-Invasive Blood Pressure Monitoring

The Propaq MD is intended for use to make non-invasive measurements of arterial pressure and heart rate, and to alarm if either parameter is outside of the limits set by the user.

Measurements are made using an inflatable cuff on the patient's arm or leg. The patient population will range from newborn (neonate) to adult.

## Temperature Monitoring

The Propaq MD is intended for use to make continuous temperature measurements of rectal, esophageal, or surface temperatures, and to alarm if the temperature is outside of the limits set by the user. The patient population will range from newborn (neonate) to adult.

## SpO<sub>2</sub> Monitoring

The Propaq MD is intended for use to monitor pulse rate and oxygen saturation of arteriolar hemoglobin, and to alarm if either parameter is outside of the limits set by the user.

Measurements are made non-invasively at remote sites such as a finger, toe, ear lobe, bridge of nose, etc. The patient population will range from newborn (neonate) to adult.

## Respiration Monitoring

The Propaq MD is intended for use to continuously monitor respiration rate and to alarm if the rate falls outside of the range set by the operator. Because the measurement method actually measures respiratory effort, apnea episodes with continued respiratory effort (such as obstructive apnea) may not be detected. It is not intended to be used as an apnea monitor. The patient population will range from newborn (neonate) to adult.

## CO<sub>2</sub> Monitoring

The Propaq MD is intended for use to make continuous noninvasive measurement and monitoring of carbon dioxide concentration of the expired and inspired breath and breath rate. The patient population will range from newborn (neonate) to adult.

## Invasive Pressure Monitoring

The Propaq MD is intended for use to display and make continuous invasive pressure measurements from any compatible pressure transducer. The primary intended uses are arterial blood pressure, central venous pressure and intracranial pressure monitoring. Any contraindications of the particular transducer selected by the user shall apply. The patient population will range from newborn (neonate) to adult.

## 12-Lead Analysis

The 12-lead ECG Analysis is intended for use in acquiring, analyzing and reporting ECG data, and to provide interpretation of the data for consideration by caregivers. The interpretations of ECG data offered by the device are only significant when used in conjunction with caregiver overread as well as consideration of all other relevant patient data. The 12-lead ECG Analysis is intended for use on adults (> 18 years of age).

# Propaq MD Product Functions

## Defibrillator Function

The Propaq MD contains a direct current (dc) defibrillator capable of delivering up to 200 joules. It may be used in synchronized mode to perform synchronized cardioversion using the patient's R-wave as a timing reference. The unit uses paddles or disposable, pregelled electrodes for defibrillation.

## Defibrillator Output Energy

Propaq MD defibrillators can deliver biphasic energy from 1 joule to 200 joules. The energy delivered through the chest wall, however, is determined by the patient's transthoracic impedance. An adequate amount of electrolyte gel must be applied to the paddles and a force of 10 to 12 kilograms (22 to 26.4 pounds) must be applied to each paddle in order to minimize this impedance. If hands-free therapy electrodes are used, make sure that they are properly applied. (Refer to the instructions on the electrode package).

## External Pacemaker

Propaq MD defibrillators include a transcutaneous pacemaker consisting of a pulse generator and ECG-sensing circuitry. Noninvasive transcutaneous pacing (NTP) is an established and proven technique. This therapy is easily and rapidly applied in both emergency and nonemergency situations when temporary cardiac stimulation is indicated.

The output current of the pacemaker is continuously variable from 10 to 140 mA (the output current is 0 mA when paused). The rate is continuously variable from 30 to 180 pulses per minute (ppm), by increments of 5 ppm (10 bpm when greater than 100 ppm).

The pacing output pulse is delivered to the heart via ZOLL hands-free defibrillation/pacing electrodes placed on the patient's back and the precordium.

Proper operation of the device, together with correct electrode placement, is critical to obtaining optimal results. Every operator must be thoroughly familiar with these operating instructions.

## ECG Monitoring

The patient's ECG is monitored by connecting the patient to the unit via a 3-, 5-, or 12-lead patient cable or hands-free therapy electrodes. The ECG waveform is presented on the display along with the following information:

- averaged heart rate, derived by measuring R to R intervals
- lead selection - I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6 (with ECG cable), PADDLES, or PADS.
- ECG size - 0.125, 0.25, 0.50, 1.0, 2.0, 4.0 cm/mV, AUTO
- status messages

The ECG bandwidth is user selectable.

## Electrodes

The Propaq MD will defibrillate, cardiovert, and monitor ECG using hands-free therapy electrodes. The Propaq MD will pace using ZOLL hands-free therapy electrodes.

**Energy Select, Charge and Shock** controls are located on the paddles and front panel. When using hands-free therapy electrodes, you must use the controls on the front panel of the unit. To switch between paddles and hands-free therapy electrodes, remove the multifunction cable (MFC) from the apex paddle and connect the hands-free therapy electrodes to the cable.

You should always check the expiration date on the electrode packaging. Do not use expired electrodes, which might result in false patient impedance readings and affect the level of delivered energy, or cause burns.



This symbol on the electrode package is accompanied by the expiration date.

For stat-padz® II, this symbol does not appear; the expiration date appears on the lower right corner of the label, below the lot number.

**Note:** ZOLL electrodes contain no hazardous materials and may be disposed of in general trash unless contaminated with pathogens. Use appropriate precautions when disposing of contaminated electrodes.

## Batteries

Propaq MD models use an easily replaced rechargeable lithium-ion battery pack (the *SurePower II* Battery Pack). A new, fully charged battery pack typically delivers more than 6 hours of ECG monitoring. Use of other functions (such as the defibrillator, printer, or pacemaker) reduces this time.

When a *LOW BATTERY* icon appears on the display and the unit emits three beeps in conjunction with the displayed battery icon, the battery must be replaced and recharged.

You can charge the battery by either of the following methods:

- **Internal charging** — plug the Propaq MD into an auxiliary power adapter to automatically begin charging the installed battery pack. The front panel battery indicator operates as follows:

| When the indicator is:       | It means:  |
|------------------------------|--|
| Steady yellow                | Battery is charging.   |
| Steady green                 | Battery is charged.  |
| Alternating yellow and green | The charge state cannot be determined or a battery charging fault has been detected. |
| Not lit                      | No battery in device.  |

**Note:** Upon power up, it takes approximately 45 seconds for the LEDs on the battery to accurately display run time.

- **External charging** — use the ZOLL SurePower™ Battery Charger with the Propaq MD battery adapter to charge the battery pack and test the battery's capacity. For details, refer to the *SurePower II Battery Pack Guide*.

The Recalibration LED icon (  ) lights for approximately 10 seconds (after you press and release the Display button) if the battery needs to be calibrated. If the Recalibration LED lights, the runtime indicator will not display run time for that battery. For best performance of the battery, you should recalibrate the battery as soon as possible.

To manually recalibrate the SurePower Battery Pack, you can insert the battery into the SurePower Charger Station and perform a Manual Test (for more information, see the ZOLL *SurePower Charger Station Operator's Guide*).

After you recalibrate the battery, the Recalibration LED will only flash when you press the Display button.

## Ready For Use (RFU) Indicator

The Propaq MD has an RFU indicator on the front panel that indicates if the device is ready for use. The RFU indicator has three states which are described in the following table.

| State  | Description  | Action   |
|--|--|--|
| Ready for Use<br> | <p>The device is ready for use. Patient monitoring, defibrillation, and pacing parameters are functional and the battery is above the low battery capacity.</p> <p><b>Note:</b> If the device is plugged into the auxiliary power adapter, the Ready for Use indicator may display even if the battery is depleted. Check the status of the battery before removing the device from the auxiliary power adapter.</p>   | None required.   |
| Flashing   | <p>One or more of the following has occurred:</p> <ul style="list-style-type: none"> <li>• The battery is not properly installed.</li> <li>• A low battery is installed.</li> <li>• A battery fault has occurred.</li> <li>• There is no battery installed while connected to auxiliary power.</li> <li>• One or more patient monitoring parameters have failed self-test (NIBP, SpO2, CO2, IBP, or Temp).</li> <li>• The front panel button self-test failed.</li> <li>• The speech database self-test failed.</li> </ul> | Install a fully charged battery in the unit and check the RFU indicator again. If the RFU indicator continues to flash, remove the unit from service and contact the appropriate technical personnel or the ZOLL Technical Service Department.                         |
| Do Not Use<br>  | <p>One or more of the following has occurred:</p> <ul style="list-style-type: none"> <li>• The battery is not properly installed.</li> <li>• No battery is installed and auxiliary power is not present.</li> <li>• A very low battery (below software shutdown limit) was installed.</li> <li>• ECG, defibrillator, or pacer self-tests have failed, or other critical self-tests have failed.</li> </ul>   | Install a fully charged battery in the unit and check the RFU indicator again. If the RFU indicator continues to display the Do Not Use symbol, remove the unit from service and contact the appropriate technical personnel or the ZOLL Technical Service Department. |

---

## Safety Considerations



All operators should review these safety considerations before using the Propaq MD unit.

Propaq MD units are high-energy defibrillators capable of delivering 200 joules. To completely deactivate the unit, press the power switch to turn the unit off.

To manually disarm a charged (or charging) defibrillator, do one of the following:

- Press the **Disarm** quick access key.
- Change the selected energy.
- Press the power switch to turn the unit off.

For safety, the Propaq MD automatically disarms if left charged for more than 60 seconds if the shock button () is not pressed.

## Warnings

### General

Federal (U.S.A.) law restricts this defibrillator to sale by or on the order of a physician.

Only appropriately trained, skilled personnel who are familiar with equipment operation should perform emergency defibrillation. The prescribing physician should determine what training, such as Advanced Cardiac Life Support (ACLS) or Basic Life Support (BLS) certification, is appropriate.

Only skilled personnel trained in Advanced Cardiac Life Support (ACLS) and who are familiar with equipment operation should perform synchronized cardioversion. The precise cardiac arrhythmia must be determined before attempting defibrillation.

These operating instructions describe the functions and proper operation of the Propaq MD products. They are not a substitute for a formal patient care training course. Operators should obtain formal training from an appropriate authority before using this defibrillator for patient care.

Proper operation of the unit and correct electrode placement is critical to obtaining optimal results. Operators must be thoroughly familiar with proper device operation.

The use of external pacing/defibrillation electrodes, accessories, or adapter devices from sources other than ZOLL is not recommended. ZOLL makes no representations or warranties regarding the performance or effectiveness of its products when used with pacing/defibrillation electrodes or adapter devices from other sources. Defibrillator failures attributable to the use of pacing/defibrillation electrodes or adapters not manufactured by ZOLL might void ZOLL's warranty.

At receipt of shipment, check pacing/defibrillation electrodes to ensure compatibility.

Allow ample slack in cables to make sure that cables do not tug at electrodes.

Do not disassemble the unit. A shock hazard exists. Refer all problems to authorized service personnel.

Follow all recommended maintenance instructions. If a problem occurs, obtain service immediately. Do not use the defibrillator until it has been inspected by appropriate personnel.

The Propaq MD unit might not perform to specifications when stored at the upper or lower extreme limits of storage temperature and then immediately put into use. The Propaq unit should not be stored or used outside of the environmental limits provided in Appendix A of this manual.

Avoid using the Propaq MD adjacent to, or stacked on, other equipment. If unavoidable, verify that the unit operates normally in this configuration before clinical use.

The Propaq MD should be installed and put into service according to the EMC information in Appendix A of this manual.

Do not use internal paddles while the Propaq MD auxiliary power source is connected to an aircraft AC power operating at a frequency of 400 Hz.

The use of accessories, transducers, and cables other than those specified in this manual and related Propaq MD option manual inserts may result in increased emissions or decreased immunity of the Propaq MD.

Perform functional test of internal paddles prior to use.

Do not use or place the unit in service if the Ready For Use indicator (at the upper right of the front panel) displays a red circle with a line through it.

Carefully route patient cables to avoid tripping over them, or inadvertently pulling the unit onto the patient.

Always inspect the unit for damage if it has been dropped.

Only authorized personnel should use the Supervisor menus.

If uncertain about the accuracy of any measurement, first check the patient's vital signs by alternate means, and then make sure the monitor is functioning correctly.

## ECG Monitoring

Implanted pacemakers might cause the heart rate meter to count the pacemaker rate during incidents of cardiac arrest or other arrhythmias. Dedicated pacemaker detection circuitry may not detect all implanted pacemaker spikes. Check the patient's pulse; do not rely solely on heart rate meters. Patient history and physical examination are important factors in determining the presence of an implanted pacemaker. Pacemaker patients should be carefully observed. See "Pacemaker Pulse Rejection:" on page A-15 of this manual for disclosure of the pacemaker pulse rejection capability of this instrument.

Use only ECG electrodes that meet the AAMI standard for electrode performance (AAMI EC-12). Use of electrodes not meeting this AAMI standard could cause the ECG trace recovery after defibrillation to be significantly delayed.

Prior to attempting synchronized cardioversion, ensure the ECG signal quality is good and that sync markers are displayed above each QRS complex.

Do not place electrodes directly over an implanted pacemaker.

The Propaq MD unit detects ECG electrical signals only. It does not detect a pulse (effective circulatory perfusion). Always verify pulse and heart rate by physical assessment of the patient. Never assume that the display of a nonzero heart rate means that the patient has a pulse.

Excessive artifact can result due to improper skin preparation of the electrode sites. Follow skin preparation instructions in Chapter 6: "Monitoring ECG."

Equipment such as electrocautery or diathermy equipment, RFID readers, electronic article surveillance (EAS) systems, or metal detectors that emit strong radio frequency signals can cause electrical interference and distort the ECG signal displayed by the monitor, thereby preventing accurate rhythm analysis. Ensure adequate separation between such emitters, the device, and the patient when performing rhythm analysis.

Shock Hazard: Use of accessories, other than those specified in the operating instructions, may adversely affect patient leakage currents.

Certain line-isolation monitors may cause interference on the ECG display and may inhibit heart rate alarms.

Monitoring ECG through the paddles may result in inaccurate heart rate display due to artifact.

## Defibrillation

The ZOLL Propaq MD can deliver 200 joules of electrical energy. If this electrical energy is not discharged properly, as described in this manual, the electrical energy could cause personal injury or death to the operator or bystander.

To avoid possible damage to the Propaq MD unit, turn off pacing before defibrillating the patient with a second defibrillator.

After a synchronized cardioversion, the SYNC mode may be cleared after each shock or disarm. The user may have to reselect (press) the SYNC button after each synchronized cardioversion shock performed on a patient. In Defib/Pacer Default settings in the Supervisor Setup menu, the Propaq MD can be configured to remain in the SYNC mode after each synchronized cardioversion.

Synchronized cardioversion can be performed in the paddle monitoring mode. However, it is possible that artifact can be produced by the moving paddles, which could cause the defibrillator to trigger on the artifact. It is recommended that monitoring in leads I, II or III be used during synchronized cardioversion. Paddle monitoring should not be used for elective cardioversions procedures.

To avoid stress to the defibrillator or the tester, never attempt to repeatedly charge and discharge the defibrillator in rapid succession. If a need for repetitive testing arises, allow a waiting period of at least 2 minutes after every third discharge.

In the SYNC mode, the defibrillator does not discharge without a command signal (R-wave detection) from the ECG monitor indicated by a SYNC marker on the trace and a flashing SYNC indicator.

If conductive gel forms a continuous path between the defibrillator electrodes, delivered energy may be dramatically reduced to zero. In this case, reposition the electrodes to eliminate the shunting path before attempting additional shocks.

Improper defibrillation technique can cause skin burns. To limit possible skin burns, use only ZOLL defibrillation gel on paddles, ensure the gel covers the entire paddle surface and press firmly against patient's chest.

If a new energy level is selected after the CHARGE button is pushed and while the defibrillator is charging or charged, the defibrillator will disarm. The CHARGE button will need to be pressed again to charge to the new energy level.

Prior to defibrillation, disconnect from the patient any medical electronic device that is not labeled "defibrillation protected."

Before charging the defibrillator, verify that the energy selected on the display is the desired output.

Defibrillation takes priority over external pacing. Should the defibrillator be charged during the administration of external pacing, the pacer turns off and the defibrillator charges to the selected energy.

## Pacing

Ventricular fibrillation does not respond to pacing and requires immediate defibrillation. Therefore, the patient's dysrhythmia must be determined immediately, so that you can employ appropriate therapy. If the patient is in ventricular fibrillation and defibrillation is successful but cardiac standstill (asystole) ensues, you should use the pacemaker.

Ventricular or supraventricular tachycardias can be interrupted with pacing, but in an emergency or during circulatory collapse, synchronized cardioversion is faster and more certain.

Pulseless electrical activity (PEA) can occur following prolonged cardiac arrest or in other disease states with myocardial depression. Pacing might then produce ECG responses without effective mechanical contractions, making other effective treatment necessary.

Pacing can evoke undesirable repetitive responses, tachycardia, or fibrillation in the presence of generalized hypoxia, myocardial ischemia, cardiac drug toxicity, electrolyte imbalance, or other cardiac diseases.

Pacing by any method tends to inhibit intrinsic rhythmicity. Abrupt cessation of pacing, particularly at rapid rates, can cause ventricular standstill and should be avoided.

Noninvasive temporary pacing can cause discomfort of varying intensity, which occasionally can be severe and preclude its continued use in conscious patients.

Similarly, unavoidable skeletal muscle contraction might be troublesome in very sick patients and might limit continuous use to a few hours. Erythema or hyperemia of the skin under the hands-free therapy electrodes often occurs; this effect is usually enhanced along the perimeter of the electrode. This reddening should lessen substantially within 72 hours.

There have been reports of burns under the anterior electrode when pacing adult patients with severely restricted blood flow to the skin. Prolonged pacing should be avoided in these cases and periodic inspection of the underlying skin is advised.

There are reports of transient inhibition of spontaneous respiration in unconscious patients with previously available units when the anterior electrode was placed too low on the abdomen.

The pacing rate determination can be adversely affected by artifact. If the patient's pulse and the heart rate display are significantly different, external pacing pulses may not be delivered when required.

Artifact and ECG noise can make R-wave detection unreliable, affecting the HR meter and the demand mode pacing rate. Always observe the patient closely during pacing operations. Consider using asynchronous pacing mode if a reliable ECG trace is unobtainable.

Transcutaneous pacing should not be used to treat V FIB (ventricular fibrillation). In cases of V FIB, immediate defibrillation is advised.

---

Transcutaneous pacing may cause discomfort ranging from mild to severe, depending on the patient's tolerance level, muscle contractions and electrode placement. In certain cases, discomfort may be decreased by slightly relocating the pacing pads.

It is important to monitor the patient closely to verify that both mechanical and electrical capture are occurring. Electrical capture can be verified by observing the presence of a large ectopic beat after the pacing pulse is delivered. The size and morphology of the beat are dependent on the patient. In some instances the beat may appear as a relatively normal looking QRS pulse. Mechanical capture can be verified by checking for signs of increased blood flow i.e., reddening of the skin, palpable pulses, increased blood pressure, etc. Continuously observe the patient during pacing administration, to insure capture retention. Do not leave the patient unattended when administering external pacing therapy.

---

**Warning!**    **This device can only be used for external pacing of patients and cannot be used for internal pacing. Do not connect internal pacing lead wires to the Propaq MD defibrillator.**

---

## Pulse Oximeter

Keep the ZOLL finger probe clean and dry.

SpO<sub>2</sub> measurements may be affected by certain patient conditions: severe right heart failure, tricuspid regurgitation or obstructed venous return.

SpO<sub>2</sub> measurements may be affected when using intravascular dyes, in extreme vasoconstriction or hypovolemia or under conditions where there is no pulsating arterial vascular bed.

SpO<sub>2</sub> measurements may be affected in the presence of strong EMI fields, electrosurgical devices, IR lamps, bright lights, improperly applied sensors; the use of non-ZOLL sensors, or damaged sensors; in patients with smoke inhalation, or carbon monoxide poisoning, or with patient movement.

Tissue damage can result if sensors are applied incorrectly, or left in the same location for an extended period of time. Move sensor every 4 hours to reduce possibility of tissue damage.

Do not use any oximetry sensors during MRI scanning. MRI procedures can cause conducted current to flow through the sensors, causing patient burns.

Do not apply SpO<sub>2</sub> sensor to the same limb that has an NIBP cuff. The SpO<sub>2</sub> alarm may sound when the arterial circulation is cut off during NIBP measurements, and may affect SpO<sub>2</sub> measurements.

In some instances, such as obstructed airway, the patient's breathing attempts may not produce any air exchange. These breathing attempts can still produce chest size changes, creating impedance changes, which can be detected by the respiration detector. It is best to use the pulse oximeter whenever monitoring respirations, to accurately depict the patient's respiratory condition.

## Noninvasive Blood Pressure

Only a physician can interpret pressure measurements.

Blood pressure measurement results may be affected by the position of the patient, his or her physiological condition and other factors.

Substitution of a component different from that supplied by ZOLL (e.g., cuff, hoses, etc.) may result in measurement error. Use only ZOLL-approved cuffs and hoses. To avoid the risk of intravenous line misconnection and possible introduction of air into a patient's blood, do not modify the NIBP system or hoses with Luer Lock adapters.

Do not use a blood pressure cuff on the limb being used for IV infusion or for SpO<sub>2</sub> monitoring.

Accurate pressure readings may not be achieved on a person experiencing arrhythmias, shaking, convulsions or seizures. Medication may also affect pressure readings. The correct size cuff is essential for accurate blood pressure readings.

Blood pressure hoses must be free of obstructions and crimps.

If the patient's cuff is not at heart level, an error in measurement may result.

When monitoring blood pressure at frequent intervals, observe the cuffed extremity of the patient for signs of impeded blood flow.

Do not monitor one patient's NIBP while monitoring another patient's ECG.

Blood pressure measurement may be inaccurate if taken while accelerating or decelerating in a moving vehicle.

If an NIBP measurement result is questionable or "motion" indication is displayed, repeat the measurement. If the repeated measurement result is still questionable, use another blood pressure measurement method.

Do not use the NIBP on cardiopulmonary bypass patients.

## IBP

To ensure compatibility and electrical safety, accessory pressure sensors should comply with ANSI/AAMI BP-22 and IEC 60601-2-34 for IBP or ANSI/AAMI NS28 for ICP.

Follow instructions supplied with any accessory pressure sensor regarding calibration and removal of trapped air.

Avoid touching metal parts of any transducer while it is in contact with the patient.

Do not reuse any components that are labeled for single use only.

Transducers should be rated to withstand an accidental drop of at least a meter onto a hard surface.

Transducers that are subject to immersion in liquids should be rated as watertight.

## CO<sub>2</sub>

During MRI scanning, the monitor must be placed outside the MRI suite. When the monitor is used outside the MRI suite, EtCO<sub>2</sub> monitoring can be implemented using a long FilterLine<sup>®</sup> which permits placement of the monitor outside the MRI suite.

When using the monitor with anesthetics, nitrous oxide or high concentrations of oxygen, connect the gas outlet to a scavenger system.

Use only Oridion Microstream CO<sub>2</sub> sampling lines.

Microstream CO<sub>2</sub> sampling lines are labeled for single patient use only. Do not reuse sampling lines.

If using the CO<sub>2</sub> Monitor for extended critical care, replace the airway adapter every 24 hours or when it becomes occluded.

CO<sub>2</sub> readings and respiratory rate can be affected by sensor application errors, certain ambient environmental conditions, and certain patient conditions.

## Respiration

Do not operate the Propaq MD with any other monitor with respiration measurements on the same patient. The two devices could affect the respiration accuracy.

The device should not be used as an apnea monitor.

## Ferromagnetic Equipment

Biomedical equipment and accessories, such as ECG electrodes, cables, and oximeter probes contain ferromagnetic materials. Ferromagnetic equipment must not be used in the presence of high magnetic fields created by magnetic resonance imaging (MRI) equipment.

The large magnetic fields generated by an MRI device can attract ferromagnetic equipment with an extremely violent force, which could cause serious personal injury or death to persons between the equipment and the MRI device.

## Battery

Although the device can operate with auxiliary power alone, ZOLL strongly recommends that you operate the unit with a battery installed at all times. Operating the unit with a battery provides a backup in case of ac power shortage, and results in faster charge time. The battery can be automatically recharged while it is installed in the unit. Keep a fully charged spare battery pack with the defibrillator at all times.

Test battery packs regularly. A battery that does not pass the ZOLL charger's capacity test might cause the Propaq MD unit to shut down unexpectedly.

If the Low Battery indication occurs at any time during operation, immediately replace the battery pack.

If the *LOW BATTERY* icon appears, plug the Propaq MD unit into a power source or install a fully charged battery pack. When the warning low battery shutdown prompt appears, immediately replace the battery pack with a fully charged pack or plug the Propaq MD unit into a power source, as unit shut down due to a low battery condition is imminent.

If mistreated, a battery pack might explode. Do not disassemble a battery pack or dispose of it in fire.

## Operator Safety



The Propaq MD can deliver 200 joules of electrical energy. If this electrical energy is not discharged properly, as described in this manual, the electrical energy could cause personal injury or death to the operator or bystanders.

Do not use the Propaq MD in the presence of oxygen-rich atmospheres, flammable anesthetics, or other flammable agents (such as gasoline). Using the unit in such environments might cause an explosion.

Do not use the unit near or within standing water. Electrical safety might be compromised when the defibrillator is wet.

Never discharge the unit with the defibrillation electrodes or paddles shorted together or in open air.

Do not discharge the defibrillator except as indicated in the instructions. Discharge the defibrillator only when defibrillation electrodes or paddles are properly applied to the patient.

To avoid risk of electrical shock, do not touch the gelled area of the hands-free therapy electrodes during pacing or defibrillation.

To avoid risk of electrical shock, do not allow electrolyte gel to accumulate on hands or paddle handles.

To avoid risk of electrical shock, do not allow patient connectors to contact other conductive parts, including earth.

For defibrillation using paddles, use only high-conductivity electrolyte gel specified for such use by the manufacturer.

When using paddles for defibrillation, use your thumbs to operate the **SHOCK** buttons. Doing so avoids inadvertent shock to the operator.

The use of accessory equipment that does not comply with the equivalent safety requirements of the Propaq MD defibrillator could reduce the level of safety of the combined system. When choosing accessory equipment, consider the following:

- Use of the accessory in the patient vicinity.
- Evidence that the safety certification of the accessory has been performed in accordance with the appropriate IEC (EN) 60601-1 and/or IEC (EN) 60601-1-1 harmonized national standards.

Always check that the equipment functions properly and is in proper condition before use.

Disconnect all electro-medical equipment that is not defibrillation-protected from the patient prior to defibrillation.

Before discharging the defibrillator, warn everyone to **STAND CLEAR** of the patient.

Do not touch the bed, patient, or any equipment connected to the patient during defibrillation. A severe shock can result. To avoid hazardous pathways for the defibrillation current, do not allow exposed portions of the patient's body to touch any metal objects, such as a bed frame.

To avoid risk of electrical shock, do not allow printer to come into contact with other conductive parts, such as equipment connected to the USB port.

---

## Patient Safety



Inappropriate defibrillation or cardioversion of a patient (for example, with no malignant arrhythmia) may precipitate ventricular fibrillation, asystole, or other dangerous arrhythmias.

Defibrillation without proper application of electrodes or paddle electrolyte gel might be ineffective and cause burns, particularly when repeated shocks are necessary. Erythema or hyperemia of the skin under the paddles, or electrodes often occurs; this effect is usually enhanced along the perimeter of the paddles or electrodes. This reddening should diminish substantially within 72 hours.

This equipment should be connected to only one patient at a time.

Neonatal and pediatric defibrillation energy level settings should be based on site-specific clinical protocols.

To ensure patient safety, do not place the monitor in any position that might cause it to fall on the patient.

To ensure patient safety, connect the Propaq MD only to equipment with circuits that are electrically isolated.

Use only high-quality ECG electrodes. ECG electrodes are for rhythm acquisition only; you cannot use ECG electrodes for defibrillation or pacing.

Do not use therapy or ECG electrodes if the gel is dried, separated, torn or split from the foil; patient burns may result from using such electrodes. Poor adherence and/or air pockets under therapy electrodes can cause arcing and skin burns.

Check the expiration date on the electrode packaging. Do not use electrodes after their expiration date.

Excessive body hair or wet, diaphoretic skin can inhibit electrode coupling to the skin. Clip excess hair and dry any moisture from the area where an electrode is to be attached.

Therapy electrodes should be replaced periodically during continuous pacing. Consult electrode directions for proper replacement instructions.

Prolonged pacing (more than 30 minutes), particularly in neonates or adults with severely restricted blood flow, may cause burns. Periodically inspect the skin under the electrodes.

Carefully route the patient cables away from the patient's neck to reduce the possibility of patient entanglement or strangulation.

To avoid electrosurgery burns at monitoring sites, ensure proper connection of the electrosurgery return circuit so that a return path cannot be made through monitoring electrodes or probes.

During electrosurgery, observe the following guidelines to minimize electrosurgery unit (ESU) interference and provide maximum operator and patient safety:

- Keep all patient monitoring cables away from earth ground, ESU knives, and ESU return wires.
- Use electrosurgical grounding pads with the largest practical contact area.

Always ensure proper application of the electrosurgical return electrode to the patient.

Check electrical leakage levels before use. Leakage current might be excessive if more than one monitor or other piece of equipment is connected to the patient.

## Cautions

If the unit is to be stored longer than 90 days, remove the battery pack.

Do not sterilize the defibrillator, or its accessories unless the accessories are labelled as sterilizable.

Do not immerse any part of the defibrillator in water.

Do not use the defibrillator if excessive condensation is visible on the device. Wipe only the outside with a damp cloth.

Do not use ketones (such as acetone or MEK) on the defibrillator.

Avoid using abrasives (including paper towels) on the display window.

To achieve the specified level of protection against spilled or splashed liquids, thoroughly dry all exposed surfaces of this device prior to operation or connections to auxiliary power.

If liquids enter the device connectors, remove all liquid from the connectors and allow the device to dry thoroughly prior to use.

Grounding reliability can be achieved only when the equipment is connected to a receptacle marked "HOSPITAL ONLY," "HOSPITAL GRADE," or equivalent. If the grounding integrity of the line cord or ac receptacle is questionable, operate the defibrillator using battery power only.

Do not connect to an electrical outlet controlled by a wall switch or dimmer.

To protect the unit from damage during defibrillation, for accurate ECG information, and to protect against noise and other interference, use only internal current-limiting ECG cables specified or supplied by ZOLL.

For continued safety and EMI performance, use only the line cord supplied by ZOLL.

Electrical installation of the room or the building in which the monitor is to be used must comply with regulations specified by the country in which the equipment is to be used.

Dispose of battery packs in accordance with national, regional and local regulations. Battery packs should be shipped to a reclamation facility for recovery of metal and plastic compounds as the proper method of waste management.

## Restarting the Defibrillator

Certain events require the Propaq MD products to be restarted after they shut off or become inoperative (for example, when the battery runs down and the unit shuts off).

In such a case, always try to restore defibrillator operation as follows:

1. Press the power switch on the top of the unit to turn it off.
2. If necessary, replace a depleted battery with a fully charged pack, or connect the defibrillator to auxiliary power.
3. Press the power switch on the top of the unit to turn it back on.

This sequence is necessary to restart the defibrillator and can also be used to clear some fault messages when immediate use of the defibrillator is required.

If the Propaq MD unit is powered off for less than 2 minutes, all patient monitoring parameter settings will be retained. If the unit has been powered off for at least two minutes, it will be

considered a New Patient and all of the patient-specific parameters (alarm limits, defibrillator energy, etc.) will be reset to their default values.

## FDA Tracking Requirements

U.S. Federal Law (21 CFR 821) requires the tracking of defibrillators. Under this law, owners of this defibrillator must notify ZOLL Medical Corporation if this product is

- received
- lost, stolen, or destroyed
- donated, resold, or otherwise distributed to a different organization

If any such event occurs, contact ZOLL Medical Corporation in writing with the following information:

1. Originator's organization – Company name, address, contact name, and contact phone number
2. Model number, and serial number of the defibrillator
3. Disposition of the defibrillator (for example, received, lost, stolen, destroyed, distributed to another organization), new location and/or organization (if known and different from originator's organization) – company name, address, contact name, and contact phone number
4. Date when the change took effect

Please address the information to:

**ZOLL Medical Corporation**  
**Attn: Tracking Coordinator**  
**269 Mill Road**  
**Chelmsford, MA 01824-4105**

Fax: (978) 421-0025

Telephone: (978) 421-9655

## Notification of Adverse Events

As a health care provider, you may have responsibilities under the Safe Medical Devices Act (SMDA), for reporting to ZOLL Medical Corporation, and possibly to the FDA, the occurrence of certain events.

These events, described in 21 CFR Part 803, include device-related death and serious injury or illness. In addition, as part of our Quality Assurance Program, ZOLL Medical Corporation requests to be notified of device failures or malfunctions. This information is required to ensure that ZOLL Medical Corporation provides only the highest quality products.

## Software License

**Note:** Read this Operator's Guide and License agreement carefully before operating any of the Propaq MD products.

Software incorporated into the system is protected by copyright laws and international copyright treaties as well as other intellectual property laws and treaties. This software is licensed, not sold. By taking delivery of and using this system, the Purchaser signifies agreement to and acceptance of the following terms and conditions:

1. **Grant of License:** In consideration of payment of the software license fee which is part of the price paid for this product ZOLL Medical Corporation grants the Purchaser a non-exclusive license, without right to sublicense, to use the system software in object-code form only.
2. **Ownership of Software/Firmware:** Title to, ownership of and all rights and interests in the system software and all copies thereof remain at all times vested in the manufacturer, and Licensors to ZOLL Medical Corporation and they do not pass to purchaser.
3. **Assignment:** Purchaser agrees not to assign, sublicense or otherwise transfer or share its rights under the license without the express written permission of ZOLL Medical Corporation.
4. **Use Restrictions:** As the Purchaser, you may physically transfer the products from one location to another provided that the software/firmware is not copied. You may not disclose, publish, translate, release or distribute copies of the software/firmware to others. You may not modify, adapt, translate, reverse engineer, decompile, crosscompile, disassemble or create derivative works based on the software/firmware.

### NO IMPLIED LICENSE

Possession or purchase of this device does not convey any express or implied license to use the device with replacement parts which would, alone, or in combination with this device, fall within the scope of one or more of the patents relating to this device.

## Service

The Propaq MD only requires recalibration of the CO<sub>2</sub> module. Service is required after 20,000 hours of use of the CO<sub>2</sub> module. Appropriately trained and qualified personnel should, however, perform periodic tests of the defibrillator functionality to verify proper operation.

If a unit requires service, contact the ZOLL Technical Service Department.

| For customers In the U.S.A.                 | For customers outside the U.S.A.   |
|---|--|
| Telephone: 1-800-348-9011<br>1-978-421-9655 | Call the nearest authorized ZOLL Medical Corporation representative.   |
| Fax: 1-978-421-0010                         | To locate an authorized service center, contact the International Sales Department at<br><br>ZOLL Medical Corporation<br>269 Mill Road<br>Chelmsford, MA 01824-4105<br><br>Telephone: 1-978-421-9655 |

When requesting service, please provide the following information to the service representative:

- Unit serial number
- Description of the problem
- Department using the equipment and name of the person to contact
- Purchase order to allow tracking of loan equipment
- Purchase order for a unit with an expired warranty
- Sample ECG or other stripcharts demonstrating the problem (if available and applicable), less any confidential patient information.

### Returning a unit for service

Before sending a unit to the ZOLL Technical Service Department for repair, obtain a service request (SR) number from the service representative.

Remove the battery pack from the unit. Pack the unit with its cables and battery in the original containers (if available) or equivalent packaging. Be sure the assigned service request number appears on each package.

| For customers      | Return the unit to  |
|--------------------|---|
| In the U.S.A.      | ZOLL Medical Corporation<br>269 Mill Road<br>Chelmsford, MA 01824-4105<br><br>Attention: Technical Service Department ( <i>SR number</i> )<br><br>Telephone: 1-800-348-9011   |
| In Canada          | ZOLL Medical Canada Inc.<br>1750 Sismet Road, Unit #1<br>Mississauga, ON L4W 1R6<br><br>Attention: Technical Service Department ( <i>SR number</i> )<br><br>Telephone: 1-866-442-1011   |
| In other locations | The nearest authorized ZOLL Medical Corporation representative.<br><br>To locate an authorized service center, contact the International Sales Department at<br><br>ZOLL Medical Corporation<br>269 Mill Road<br>Chelmsford, MA 01824-4105<br><br>Telephone: 1-978-421-9655 |

## The ZOLL Serial Number

Each ZOLL product displays a serial number that contains information about that product. From left to right, ZOLL serial numbers are structured as follows:

- A two-character product code
- A three-character date-of-manufacture code
- A product serial number of six or more alphanumeric characters

The first two characters of the date-of-manufacture code give the last two digits of the year (for example, “06” appears for products manufactured in 2006). The last character of the date-of-manufacture code gives the month in which the product was manufactured. The month appears in the form of a single alphanumeric character: “A” for January, “B” for February, “C” for March, and so on through “L” for December.


The product serial number is a unique set of alphanumeric characters that ZOLL assigns to each individual unit.

# Chapter 2

## Product Overview

---

### Defibrillator Controls and Indicators

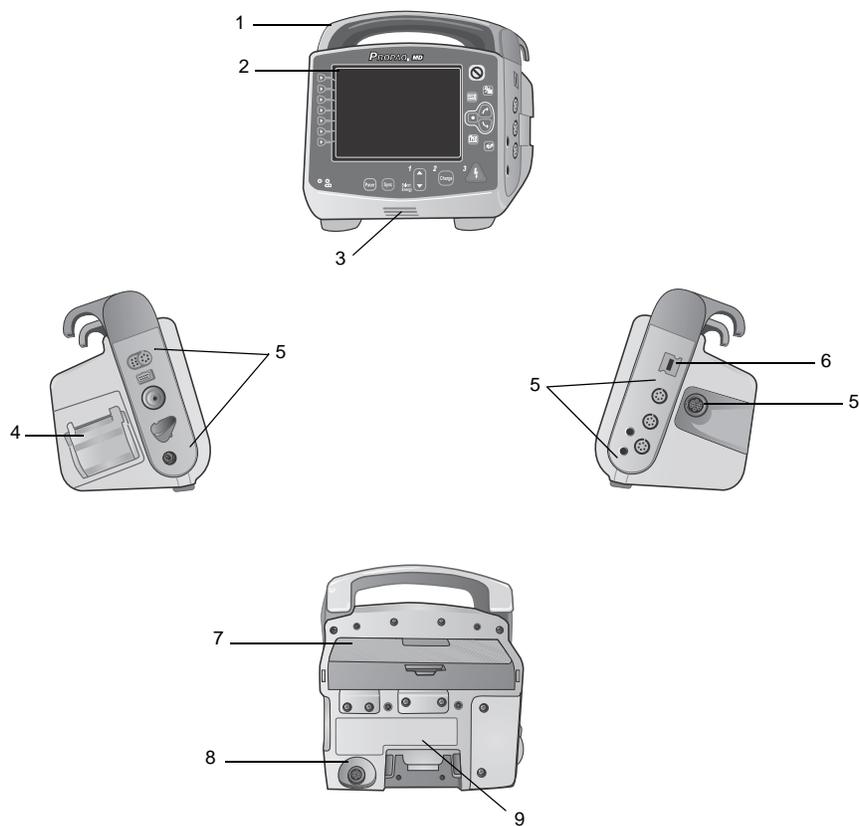


**Table 2-1. Propaq Unit Features**

|   | Item                      | Description   |
|---|---------------------------|---|
| 1 | Handle                    | Integrated carrying handle.   |
| 2 | Front panel               | Includes the display screen and primary controls.   |
| 3 | Speaker                   | Emits R-wave detection beeps and alarm tones.   |
| 4 | Paper Compartment         | Holds the paper for the printer.  |
| 5 | Patient connectors        | For details, refer to “Patient Cables and Connectors” on page 2-7.  |
| 6 | USB device connector      | For connecting the Propaq defibrillator to a USB device. For details, refer to “Transferring Data to a USB Device” on page 19-3”. |
| 7 | Battery compartment       | Holds a rechargeable lithium ion battery pack.  |
| 8 | Auxiliary power connector | For connecting the device to an auxiliary power adapter.  |
| 9 | Dock connector            | For connecting the device to a docking station.   |

## The Front Panel

The front panel of the Propaq MD device includes the display screen, quick access keys, battery and auxiliary power indicators, Ready For Use (RFU) indicator, and the defibrillation front panel buttons: **Pacer**, **Sync**, **Select Energy**, **Charge**, and Shock (⚡). See Figure 2-1. Refer to Table 2-2 on page 2-3 for information about the controls and indicators.

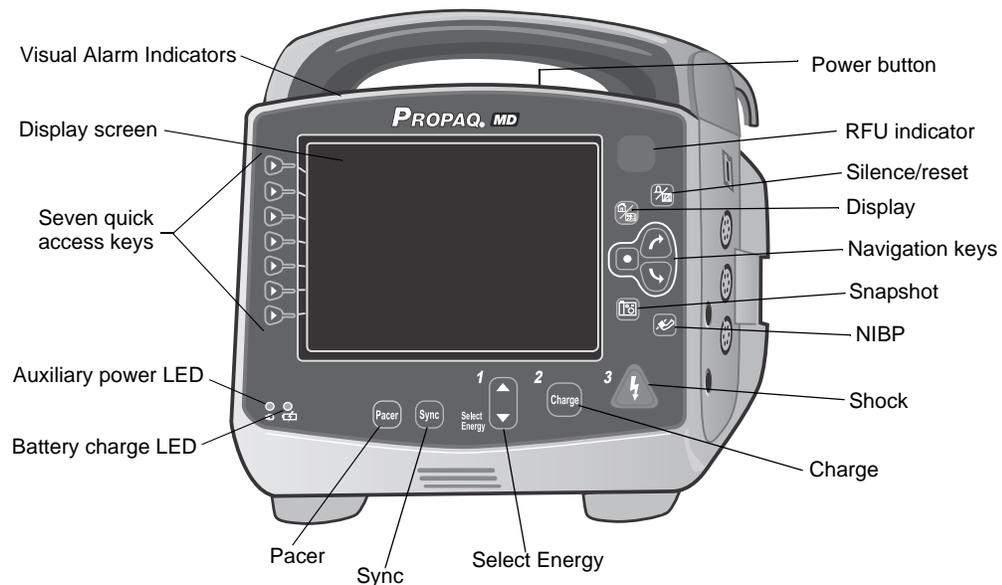

**Figure 2-1. Propaq MD Front Panel**

Table 2-2. Propaq Controls and Indicators

| Control or Indicator  | Description  |
|---|--|
| Display screen  | Shows therapeutic settings, physiological waveforms and other information for each monitored parameter, messages, time, and quick access key labels.   |
| Quick access keys   | Seven buttons control different functions of the unit. Labels for the quick access keys appear on the monitor display to the right of each key.  |
| Auxiliary power LED   | Illuminated when the unit is plugged in to an auxiliary power adapter.   |
| Battery charge LED  | Indicates battery status:<br>Steady yellow: Battery is charging.<br>Steady green: Battery is charged.<br>Alternating green and yellow: The charge state cannot be determined or a battery charging fault has been detected.<br><br>No light: Battery is not installed.   |
| Visual alarm indicators   | Red, yellow, and green lights located on the top of the unit that flash on and off when the unit is powered up and are used to indicate a patient alert, equipment alert, and data transfer.   |
| <b>Pacer</b> button<br>            | Displays pacer settings window to start/stop pacing activity or change the rate, output, or mode settings.   |
| <b>Sync</b> button<br>           | Activates the synchronized cardioversion mode.   |
| <b>Select Energy</b> buttons<br> | Two sets of up-down arrow buttons control the selection of defibrillator energy; one set is located on the front panel and the other set is located on the STERNUM paddle.   |
| <b>Charge</b> Button<br>         | Charges the defibrillator to the selected energy. In addition to the <b>Charge</b> button on the front panel, there is one located on the APEX paddle handle.  |
| <b>Shock</b> Button<br>          | The front panel Shock button is only active when using hands-free therapy electrodes or internal defibrillation paddles without a discharge button. The Shock button illuminates when the device is charged and ready.<br><br>To discharge the defibrillator when using paddles (internal or external) with discharge buttons, press and hold the <b>SHOCK</b> buttons on the paddles. |
| <b>NIBP</b> button<br>           | Starts/stops an NIBP measurement.  |
| <b>Snapshot</b> button<br>       | Records 24 seconds of numeric and waveform data.   |

**Table 2-2. Propaq Controls and Indicators (continued)**

| Control or Indicator   | Description   |
|--|---|
| Navigation keys<br>                         | <br> <p>The up (clockwise) arrow will cause the cursor to travel in an upward direction if the cursor is being used to navigate through a vertical list or in a clockwise direction if the cursor is being used to navigate around the full screen. Likewise, the down (counterclockwise) arrow will cause the cursor to travel in a downward direction if the cursor is being used to navigate through a vertical list or in a counterclockwise direction if the cursor is being used to navigate around the full screen. The up (clockwise) and down (counterclockwise) arrows may also be used to modify parameter settings.</p> <p> The Select button acts based on what is highlighted.</p> |
| Display/Home button<br>                     | Cycles through three available display modes or functions as a Home button when in a menu.  |
| Silence/Reset button<br>                    | Silences the current alarm tone for 90 seconds or resets a silenced alarm tone.   |
| RFU indicator<br><br>Ready      Do Not Use | Shows the status of the unit, based on its most recent readiness check. A red circle with a line through it indicates that the unit's readiness has been compromised and that it may not be ready for therapeutic use.  |
| Power button<br>                          | Located on the top of the unit, this button turns the unit on and off.  |
| Charge Indicator Light (not shown)   | Located on the APEX paddle, this light turns on when the defibrillator is charged and ready.  |

## Display Screen

The front panel includes a color display which shows:

- Date and time
- Patient mode
- Battery status indicator
- Time elapsed (since unit was turned on)
- Quick access keys
- Waveform source
- Color-coded waveforms and ECG lead identifiers
- SpO<sub>2</sub> numeric data
- Heart rate numeric data
- Respiration rate numeric data
- Temperature numeric data
- Non-invasive blood pressure numeric data

- EtCO<sub>2</sub> numeric data
- Invasive pressure numeric data
- The selected energy, charging status, and delivered energy for defibrillation and synchronized cardioversion
- The output current and stimulus rate for pacing
- Messages and prompts

Figure 2-2 shows the layout of parameter values, waveforms, system data, and quick access key labels.

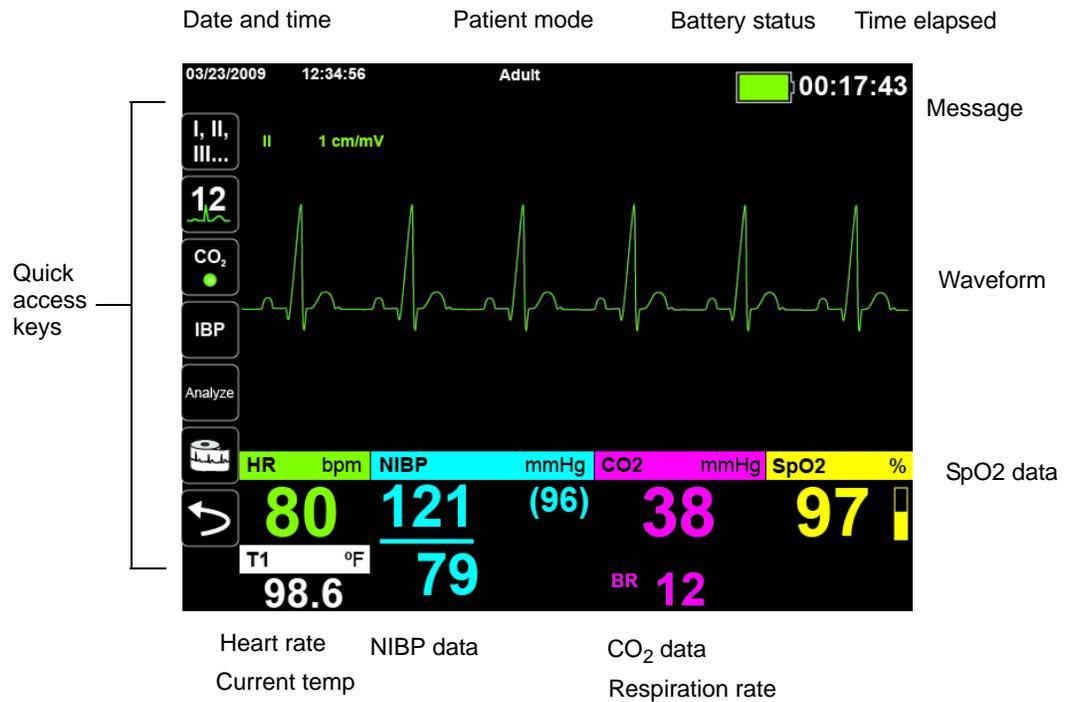


Figure 2-2. Propaq MD Display Screen


### Color coding

To differentiate information for various parameters, the unit displays each type of information in a specific user-configurable color.

## Battery Status and Auxiliary Power Indicators

The battery status indicator displays various battery icons to indicate the approximate remaining unit run time based on the charged state of the battery. Additionally, these icons provide indications of the status of the battery connection and communication with the unit. The auxiliary power indicator indicates that the unit is being powered by the auxiliary power adapter.

**Note:** Upon powering up the Propaq MD unit, the battery capacity will be displayed within approximately 15 seconds under normal conditions. Under some circumstances, such as activating the defibrillator immediately after the unit is turned on, the battery icon may display less than one hour battery capacity for up to two minutes after exiting the defibrillation mode.
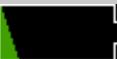
| Icon  | Status                               | Indication/Action   |
|---|--------------------------------------|---|
|    | Auxiliary power adapter is connected | The unit is being powered by the auxiliary power adapter.   |
|    | No battery detected                  | Either there is no battery in the unit while it is being powered by the auxiliary power adapter, or the device cannot detect that the battery is connected. |
|    | Low battery capacity                 | Replace the battery soon.   |
|   | Communication failure                | The unit is unable to establish communication with the battery and the battery capacity is unknown. Check the battery contacts.                             |
|  | Battery fault                        | A battery fault has been detected. Replace the battery.   |
|  | Battery Level 1                      | The battery has less than one hour of remaining battery capacity.   |
|  | Battery Level 2                      | The battery has greater than one hour of remaining battery capacity.  |
|  | Battery Level 3                      | The battery has greater than two hours of remaining battery capacity.   |
|  | Battery Level 4                      | The battery has greater than three hours of remaining battery capacity.   |
|  | Battery Level 5                      | The battery is fully charged.   |

## Patient Cables and Connectors

The left and right sides of the unit include sets of connectors for patient cables.

**Note:** The SPO<sub>2</sub>, NIBP, CO<sub>2</sub>, Temperature, and IBP functions are optional. If your unit does not include these options, it does not have the applicable connectors.

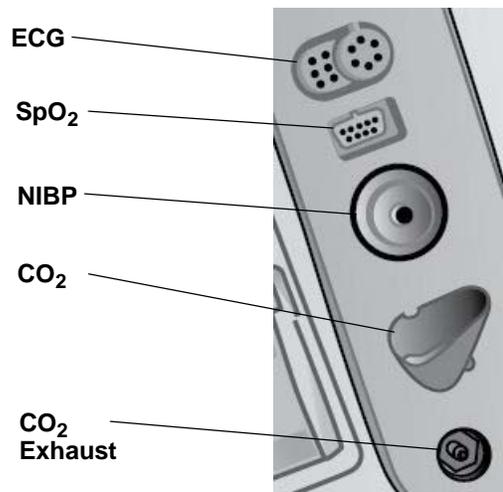


Figure 2-3. Patient Cable Connectors on Left Side of Unit

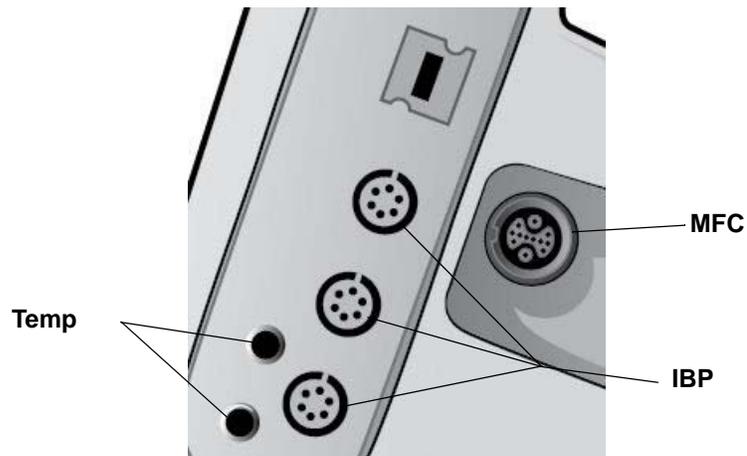


Figure 2-4. Patient Cable Connectors on Right Side of Unit

| Connector                 | Description  |
|---------------------------|--|
| ECG                       | For connecting 3- or 5-lead ECG cable (12-lead monitoring is optional).  |
| SpO <sub>2</sub>          | For connecting Nellcor pulse oximeter cable.                             |
| NIBP                      | For connecting NIBP hose.  |
| CO <sub>2</sub>           | For connecting CO <sub>2</sub> sampling line.                            |
| Temp                      | For connecting temperature probe(s).                                     |
| Multifunction Cable (MFC) | For connecting paddles or ZOLL hands-free therapy and pacing electrodes. |
| IBP                       | For connecting IBP cable(s).   |

### Multifunction Cable (MFC)

The unit ships with an MFC that is used to defibrillate the patient. Any other cables that ship with your unit depend on the options you have purchased.


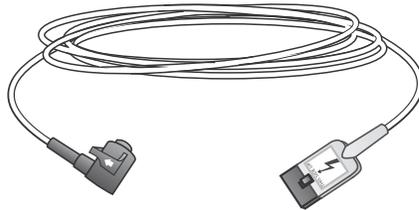


Figure 2-5. MFC

### Inserting MFC into Unit

Plug the MFC connector into the therapy input connector on the right side of the unit. Push the connector in with the arrows aligned. The connector will click when it locks into place.

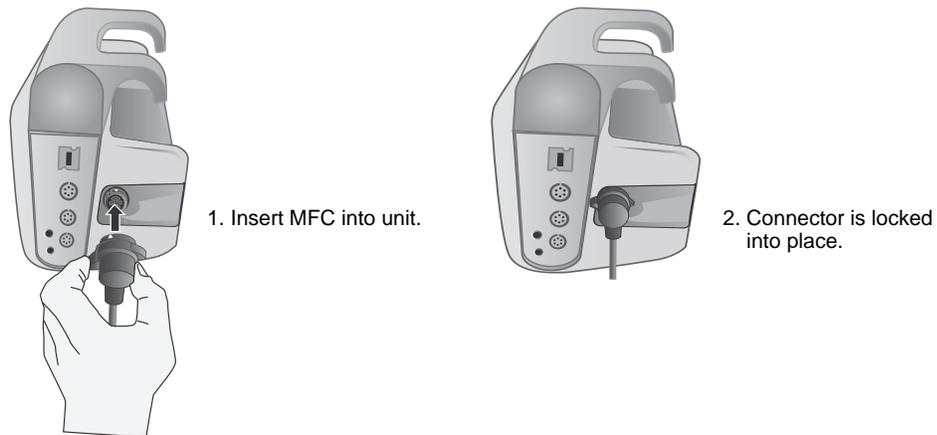

## Removing MFC from Unit

Twist the connector to the left to unlock it, and pull out the MFC connector.



## External Paddles



Paddles are defibrillation-proof Type BF equipment.

The external paddles on the Propaq device are used for defibrillation and synchronized cardioversion.

---


**Caution** You cannot use paddles for external transcutaneous pacing.

---

Attach the MFC from the Propaq unit to the connector at the base of the APEX paddle.




**Figure 2-6. Attaching the MFC to the APEX Paddle**

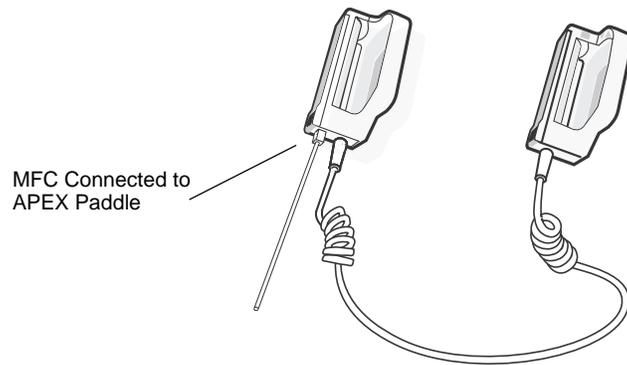

**Figure 2-7. MFC Connected to APEX Paddle**


If you need to detach the MFC from the APEX paddles, push the **RELEASE** button (see Figure 2-8) in the direction of the arrow and unplug the MFC.

Refer to Chapter 14, “Manual Defibrillation” before using paddles for defibrillation. The paddles include controls for selecting defibrillation energy, charging, and delivering a shock.

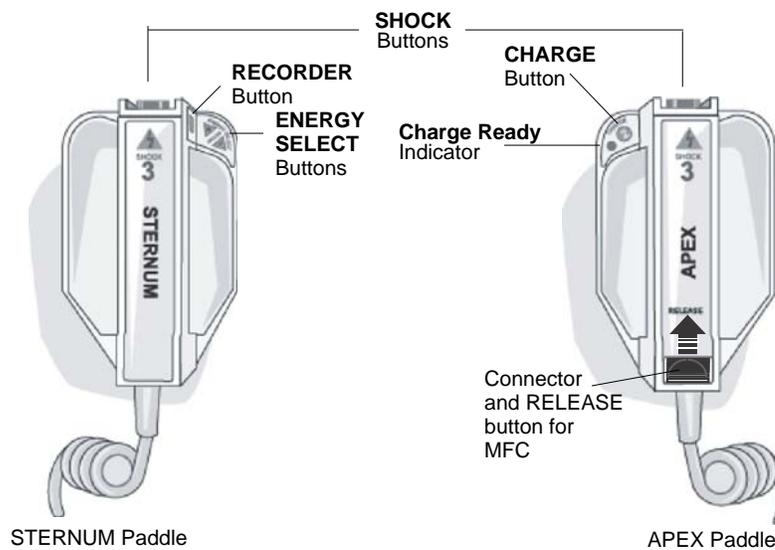

**Figure 2-8. External Paddles**

Pediatric-size electrodes are built in to the paddle assembly beneath the standard electrode plates. The user must manually adjust energy settings to pediatric levels consistent with their institution's protocols.

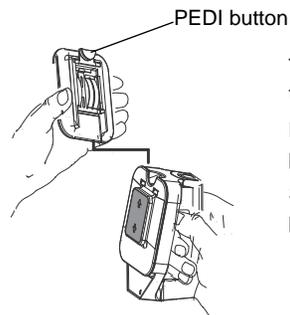


To expose the pediatric plate, press the PEDI button at the top of the paddle, then slide the Adult plate upward. Before replacing the Adult plate, be sure to clean the pediatric plate and surrounding area thoroughly. Slide the Adult plate onto the paddle until it locks into place.

**Figure 2-9. Pediatric Plate**

**Note:** The Propaq defibrillator also supports ZOLL autoclavable internal handles for use during open chest defibrillation procedures.

## Auxiliary Power Adapter

The auxiliary power adapter is used as backup power to operate the Propaq MD. When it is connected to the Propaq, it powers the unit and charges the battery that is installed in the Propaq. When the power cord is plugged in and the auxiliary power connector is inserted into the back of the Propaq unit, the auxiliary power LED on the front panel illuminates and the auxiliary power icon displays at the top of the display screen.

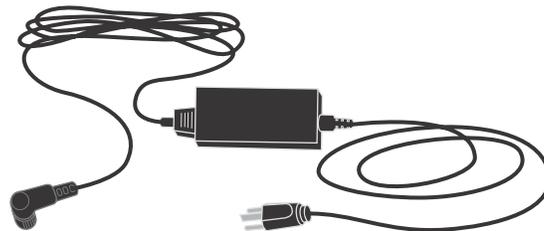

**Figure 2-10. Auxiliary Power Adapter**

To connect the auxiliary power adapter, align the white arrow of the auxiliary power connector with the arrow on the input connector on the back of the unit and push it in. To disconnect the auxiliary power adapter, grasp the connector and pull it out.

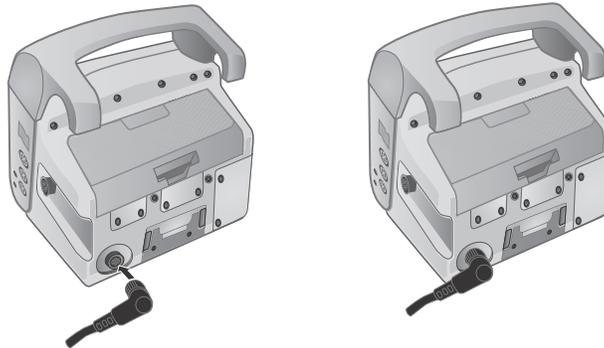


Figure 2-11. Connecting the Auxiliary Power Adapter

## Navigating the Display Screen

You can access the Propaq MD functions using the quick access keys that are located on the left side of the display screen, and the navigation keys that are located on the right side of the front panel.

### Quick Access Keys

The seven quick access keys on the left side of the display screen are an easy way to access the functionality of the Propaq MD. When you press the last key (left arrow), five more keys are displayed.

First level keys

Second level keys



**Table 2-3. Propaq MD Quick Access Keys**

| Quick access key   | Description   |
|--|---|
| Lead<br>                | Selects the ECG input source for the first waveform trace.  |
| 12 lead<br>             | Displays the 12-lead monitoring screen.   |
| CO <sub>2</sub><br>     | Turns CO <sub>2</sub> on and off.   |
| IBP<br>                 | Displays IBP setup and zero buttons.  |
| Brightness<br>          | Changes the brightness setting -- toggles through high contrast display (white background), color display (black background), and night vision goggle (NVG) friendly display. |
| Print<br>               | Starts or stops a continuous chart print.   |
| More/Back<br>          | Goes to the next or previous level of quick access keys.  |
| Treatment<br>         | Displays the current clinical treatment options.  |
| Alarms<br>            | Displays the Limits option to allow the user to view/set all parameter alarm limits and the alarm suspend button.   |
| Log<br>               | Opens the Log Control panel.  |
| Setup<br>             | Displays the Setup menu to allow the user to configure settings such as ECG, display/volume, printer, trends, operational checklist, and supervisor.                          |
| Analyze<br>           | Analyzes the patient's ECG rhythm.  |
| Treatment Summary<br> | Displays treatment summary cases, which you can print.  |
| Print Trends<br>      | Prints the trends that are displayed in the Trend Summary window.   |

**Table 2-3. Propaq MD Quick Access Keys**

| Quick access key   | Description   |
|--|---|
| Trend Settings<br>        | Displays settings for trend display format, trend on interval, and trend on alarm.            |
| Transfer Log<br>          | Transfers the current data in the log to a USB drive.   |
| Clear Log<br>             | Deletes the current data in the log.  |
| Acquire<br>               | Collects 10 seconds of 12-lead data for print.  |
| Stop Acquisition<br>      | Stops acquisition of 12-lead data.  |
| Patient Information<br> | Allows you to enter information to accompany 12-lead data: patient name, age, gender, and ID. |
| Row Up<br>              | Allows you to move to the previous row when entering patient information.                     |
| Row Down<br>            | Allows you to move to the next row when entering patient information.                         |
| 12-Lead Review<br>      | Reviews all your 12-lead captured data.   |
| 12-Lead Review Next<br> | Goes to the next page of the 12-lead snapshot you are reviewing.                              |
| Exit 12-Lead<br>        | Exits the 12-lead monitoring screen.  |

**Table 2-3. Propaq MD Quick Access Keys**

| Quick access key  | Description  |
|---|--|
| Stat Set<br>     | Sets all alarm limits relative to the patient's current vital signs.                   |
| Alarm Cancel<br> | Suspends the current alarm.  |
| Limits<br>       | Displays the current alarm settings.   |
| Disarm<br>       | Safely discharges the defibrillator internally. No energy is delivered to the patient. |
| IBP Setup<br>    | Brings up the IBP Control Panel for the corresponding channel (P1, P2, or P3).         |
| IBP Zero<br>     | Zeroes the IBP transducer for the corresponding channel (P1, P2, or P3).               |

## Navigation Keys

Use the navigation keys (up/clockwise arrow, down/counterclockwise arrow, and select button) to navigate through windows and make selections.

### Using Up/Clockwise and Down/Counterclockwise Arrows

Use the up/clockwise down/counterclockwise arrows to do the following:

- Move clockwise and counterclockwise through the main display windows.
- Move up and down in a window.
- Change parameter settings.

### Using the Select Button

Use the Select button to do the following:

- Display the settings window while a parameter is highlighted in the main window.
- Select options from a window.

## Display Brightness

The monitor can display in three different brightness modes:

- high contrast with white background (for optimal display in bright sunlight)
- color with black background (numerics and waveforms are easy to read)

- night vision goggle (NVG) friendly (display and alarm LEDs prevent interference with goggles)

## Common Tasks

The section contains procedures for the following tasks:

- “Changing the Display Brightness” on page 2-16.
- “Replacing a Battery Pack on the Propaq MD” on page 2-16.
- “Using Treatment Buttons” on page 2-17

### Changing the Display Brightness

The following procedure shows how to select the different brightness options.

1. Press the power switch to turn the unit on.
2. Press the Brightness quick access key () repeatedly to toggle through the brightness options until you find your selection.


**Note:** Selecting a higher brightness setting (such as 70%) will deplete the battery pack at a faster rate than when choosing a lower brightness setting (such as 30%). To select the brightness setting, go to the Setup>Display/Volume>Display Brightness menu to adjust the display percentage.

### Replacing a Battery Pack on the Propaq MD

This section describes how to replace a battery pack on the Propaq MD.

#### Replacing a Battery Pack on the Propaq MD

To remove a battery pack, use your fingers to grasp and raise the latch and pull the battery pack out of the compartment.

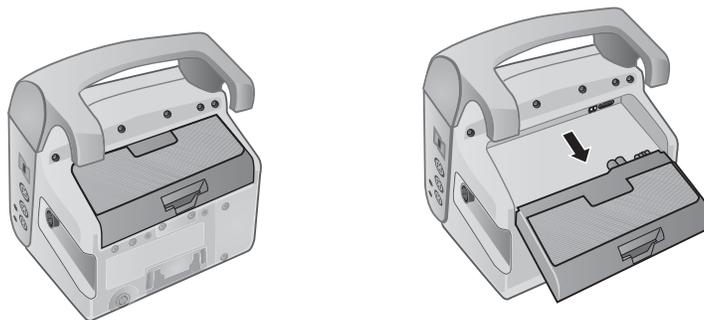


**Figure 2-12. Removing a Battery Pack**

To install a battery pack:

1. Line up the battery so it will slide into the battery well.

2. Push the battery into place.

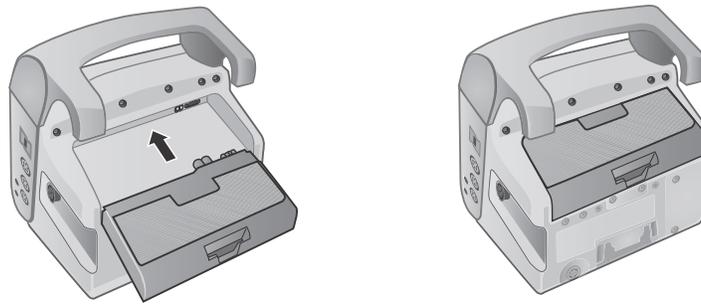

**Figure 2-13. Installing a Battery Pack**

## Using Treatment Buttons

Pressing the Treatment quick access key () causes the unit to display preconfigured buttons that contain clinical actions. These buttons allow you to add a treatment snapshot (which itemizes drugs or treatments administered to the patient) to a Treatment Summary Report. You can do this by selecting **Print on Treatment Snapshot** from Setup>Supervisor>Printer. The following is a list of preconfigured treatment buttons:

- O2
- ASA
- Nitro
- Morph
- IV
- B Block
- Lido
- MgSO4
- Valium
- Sedate

## Customizing Treatment Buttons

You can also customize up to 9 treatment buttons by pressing the Setup quick access key () , and then selecting Supervisor>Log>Treatment Options. Highlight **Define Custom Labels**, and then can customize up to 9 buttons.



# Chapter 3

## Monitoring Overview

---

This chapter provides an overview of the Propaq MD unit's monitoring functions. It describes the types of vital sign monitoring that Propaq MD provides, and the flexibility that the Propaq MD unit gives you in displaying a patient's vital signs information.

### Propaq MD Monitoring Functions

The Propaq MD unit provides an array of standard, and optional, monitoring functions, and allows you to view the vital signs measurements that these functions provide in a variety of formats. The Propaq MD unit also allows you to set alarm limits for each monitoring function. Should a patient's vital signs measurements go outside of these limits, the Propaq MD issues an audible alarm tone and displays visual alarm indications to alert you.

If the Propaq MD unit is powered off for less than 2 minutes, all patient monitoring parameter settings are retained. If the Propaq MD unit is powered off for 2 minutes or longer, the unit operates as if there is a New Patient and all patient-specific parameters (alarm limits, defibrillator energy, etc.) are reset to their default values.

The Propaq MD unit can monitor the following patient vital signs:

- ECG
- Heart Rate
- Respiration Rate
- Temperature
- Invasive Pressures (IBP)
- Non-invasive Blood Pressure (NIBP)
- Capnography (CO<sub>2</sub>)
- Pulse Oximetry (SpO<sub>2</sub>)

## ECG

An ECG waveform trace appears at the top of the display area. You can specify that the unit display the waveform trace of any available ECG source, such as **Pads**, ECG Leads **I, II, or III**, and so on, in this area. You can configure the Propaq MD unit to display up to four ECG waveform traces. In addition to being able to specify the ECG source for each waveform trace, you can adjust the display scale of those traces to make them easier to view.

## Heart Rate

A Heart Rate meter gives the patient's heart rate in Beats Per Minute (**bpm**). By default, the Propaq MD unit derives the heart rate from the patient's ECG, but can be configured to use other monitoring functions to derive the patient's heart rate.

## Respiration Rate

A Respiration Rate meter gives the patient's respiration rate in Breaths Per Minute (**br/min**). The Propaq MD unit can be configured to derive the respiration rate from the patient's ECG or from the optional CO<sub>2</sub> monitoring function.

## Temperature

The Temperature (**Temp**) meter can display temperature measurements from up to two temperature probes. The Propaq MD unit provides two separate temperature monitoring channels and, if both are used, displays the monitored temperatures, in degrees F or C, one after the other, followed by the difference between those temperatures.

## Invasive Pressures (IBP)

The Propaq MD unit provides three separate channels for monitoring arterial, venous, or intracranial pressure using internal probes. The pressure measurements for each pressure channel appear in a labeled (**P1, P2, P3**) numeric display.

## Non-Invasive Blood Pressure (NIBP)

The Propaq MD unit provides patented Smartcuf motion-tolerant technology for NIBP monitoring. NIBP monitoring measures the patient's systolic, diastolic, and mean blood pressure through an inflatable blood pressure cuff that the Propaq MD unit inflates/deflates. NIBP measurements can be taken automatically or on-demand by pressing the NIBP button (☞) on the front panel of the Propaq MD unit. The blood pressure measurements appear in a labeled (**NIBP**) numeric display. You can also specify that the Propaq MD unit display invasive pressure waveforms in the waveform trace area.

---

## Capnography (CO<sub>2</sub>)

CO<sub>2</sub> monitoring measures the CO<sub>2</sub> concentration in a patient's exhaled breath (End Tidal Carbon Dioxide --EtCO<sub>2</sub>). CO<sub>2</sub> monitoring can also measure a patient's breath rate and the CO<sub>2</sub> concentration in the gasses supplied to intubated patients (Fractional Inspired Carbon Dioxide -- FiCO<sub>2</sub>). Since FiCO<sub>2</sub> represents the amount of CO<sub>2</sub> present during inhalation, it also serves as an indicator for rebreathing in non-intubated patients. CO<sub>2</sub> monitoring can be used for both intubated and non-intubated patients.

The EtCO<sub>2</sub>, breath rate, and FiCO<sub>2</sub> measurements appear in a labeled (**EtCO<sub>2</sub>**) numeric display. The EtCO<sub>2</sub> and FiCO<sub>2</sub> measurements can appear as values given in millimeters of mercury (mmHg). You can also specify that the Propaq MD unit display a CO<sub>2</sub> capnogram in the waveform trace display area.

## Pulse Oximetry (SpO<sub>2</sub>)


Pulse Oximetry monitoring measures the oxygen saturation (SpO<sub>2</sub>) of arterial blood at a peripheral site such as a finger or toe. SpO<sub>2</sub> monitoring determines the ratio of oxygenated hemoglobin to total hemoglobin in arterial blood and displays this ratio as percent SpO<sub>2</sub> in a labeled (**SpO<sub>2</sub>**) numeric display. You can also specify that the Propaq MD unit display an SpO<sub>2</sub> plethysmograph in the waveform trace display area.

## Monitoring Display Options

The Propaq MD unit gives you great flexibility in how you display a patient's vital signs information. By pressing the Display/Home button () on the front panel, you can successively display the patient's vital signs information in these three windows:


- **Waveform Display window**, which initially displays an ECG waveform trace and numeric displays for each monitoring function.
- **Trends Status window**, which displays a report listing vital signs measurements that the Propaq MD unit logs automatically, and the primary ECG waveform trace.
- **Large Numerics Display window**, on which large numeric displays of all vital signs measurements appear.

The Waveform Display window appears when you power on the Propaq MD unit. Initially, the Waveform Display window displays a single ECG waveform trace. All other monitored values appear in numeric display areas at the bottom of the screen:

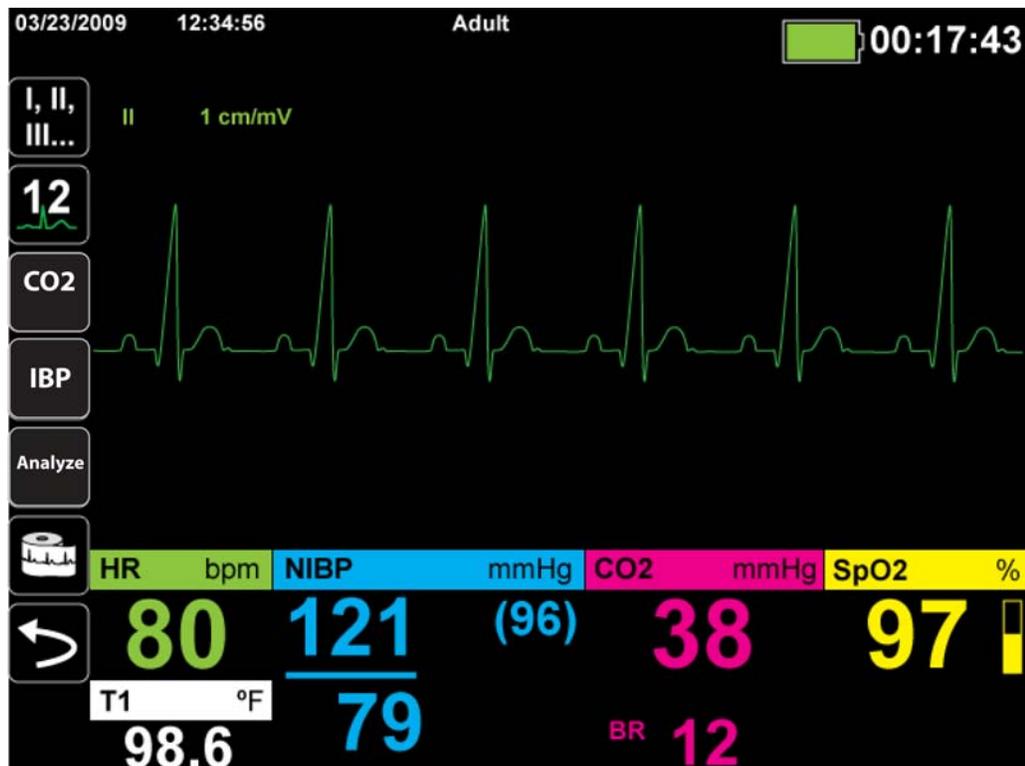


You can display up to four waveform traces that you specify on the Waveform Display window. You will determine how to add waveform traces to this window later in this chapter.

Press the Home/Display button when viewing the Waveform Display window, and the unit displays the Trends Status window. The Trends Status window reports the patient's vital sign measurements, which the Propaq MD logs automatically at a configurable interval (see the following chapter, *Trends*, for more detailed information about the Trends Status window). The primary ECG waveform trace appears above the Trends report:

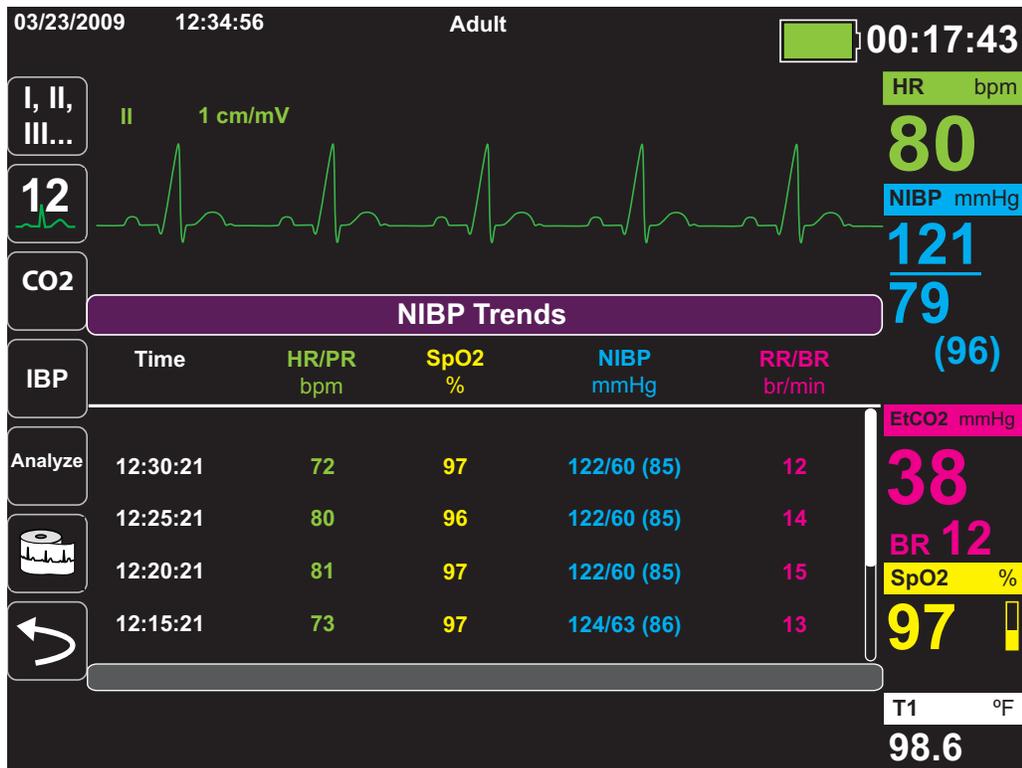

Press the Home/Display button when viewing the Trends Status window and the Large Numerics Display window appears. The patient's vital signs measurements appear in large labeled numeric displays; no waveform trace appears on this screen:

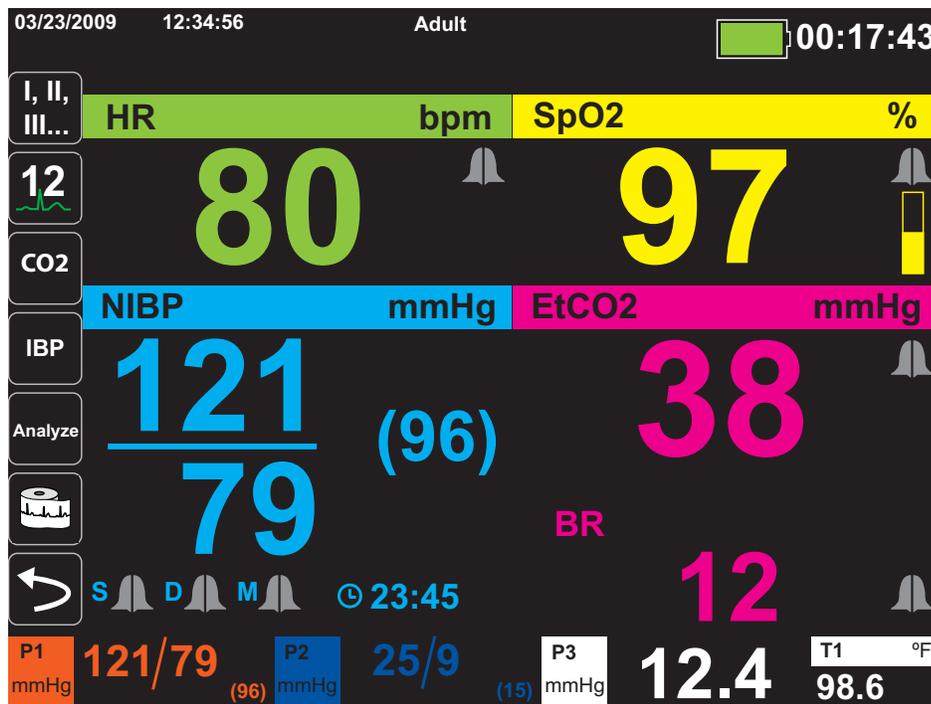

Press the Home/Display button to redisplay the Primary Display window.


**Note:** When the Propaq MD unit is displaying the Defibrillation or Pacing Control panels, the unit will not allow the display the Large Numerics Display window.

## Configuring the Waveform Display

You can display up to four waveform traces on the Waveform Display window. The first waveform trace always uses an ECG lead as its source (such **Pads** or Leads **I**, **II**, or **III**, and so on). As you insert the remaining three traces, you can specify that the traces use an ECG lead as the waveform source, or that the trace derive its waveform from other available monitoring functions (such as **Resp**, **CO2**, **SpO2** or IBP channels **P1**, **P2**, or **P3**).


The Propaq MD unit can also cascade a trace onto the adjoining trace area to double the duration of the trace display.

On the Waveform Display window, to insert a new trace (**Insert**) or cascade (**Cascade**) a displayed trace, highlight and select the trace label above the trace. In the following example, the unit is configured to cascade the ECG Lead I trace:

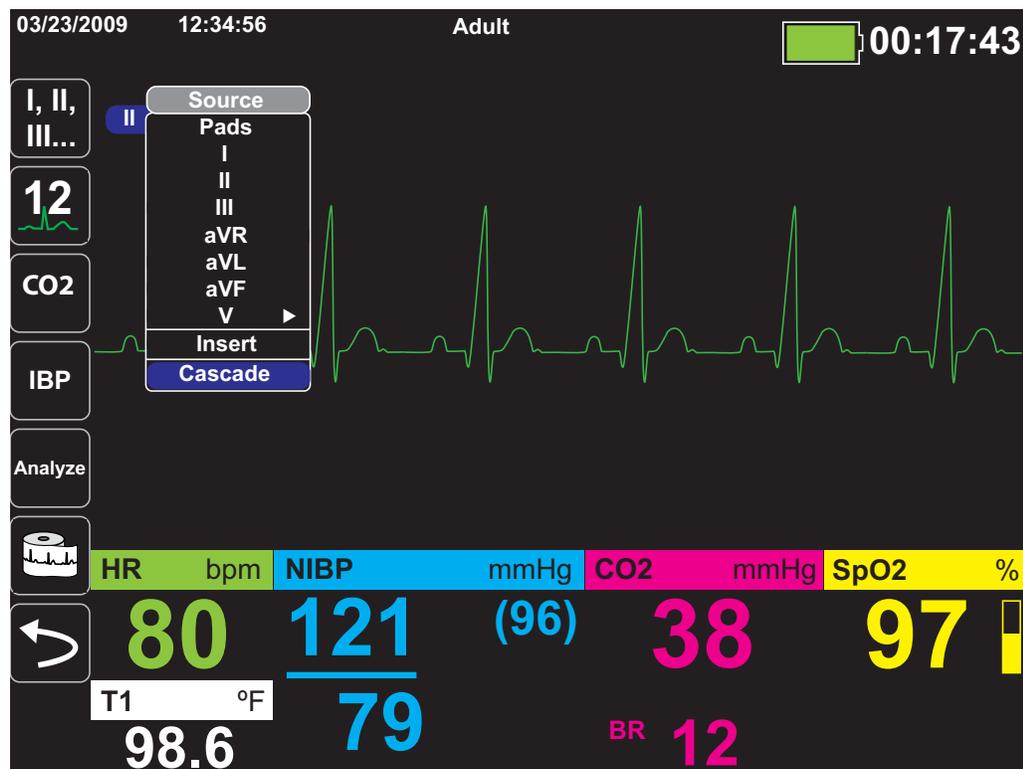



**Note:** The Propaq MD unit automatically inserts a new waveform when you turn on a parameter (CO2) or a new sensor signal is present (SPO2, IBP). The Propaq MD unit automatically removes a waveform when you turn off a parameter or remove a sensor and the unit displays the resulting equipment alert.

When the unit cascades the ECG Lead II trace, the Waveform Display window appears as follows:


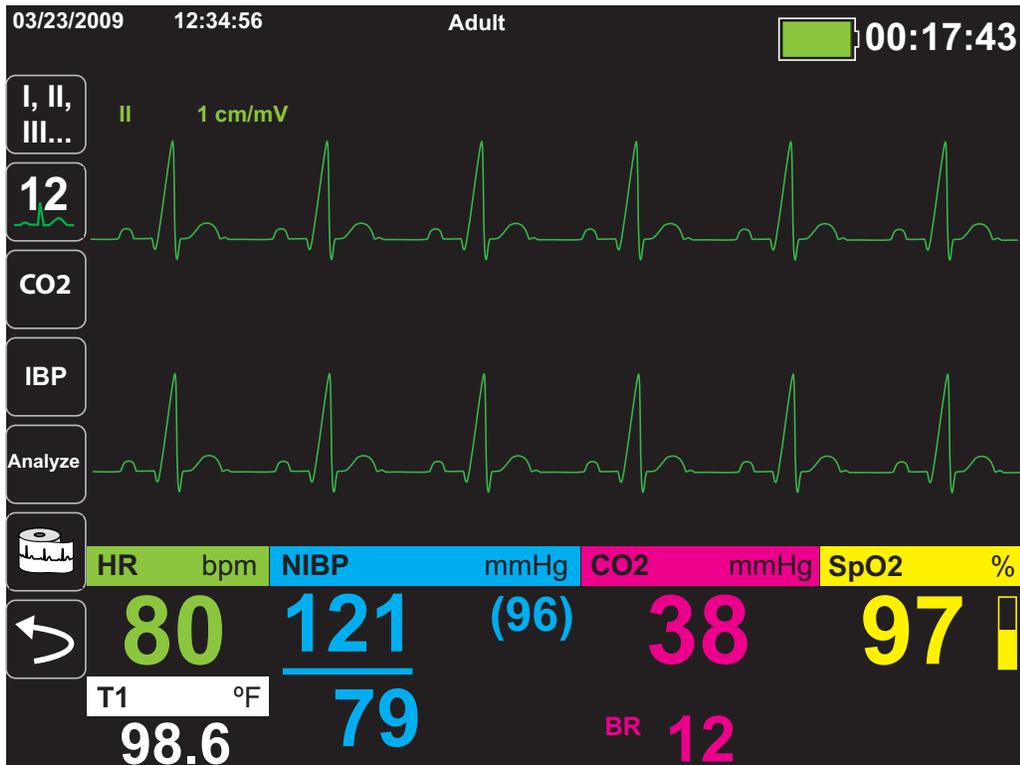


The following screens demonstrate how to insert two more waveform traces into the window. A third trace is inserted for ECG lead **aVR**, and fourth trace for **EtCO2** (a capnogram). Notice that when the third trace is inserted, the numeric displays move to the right side of the window to allow more room for the waveform traces.

Inserting a third waveform trace for ECG lead **aVR**:

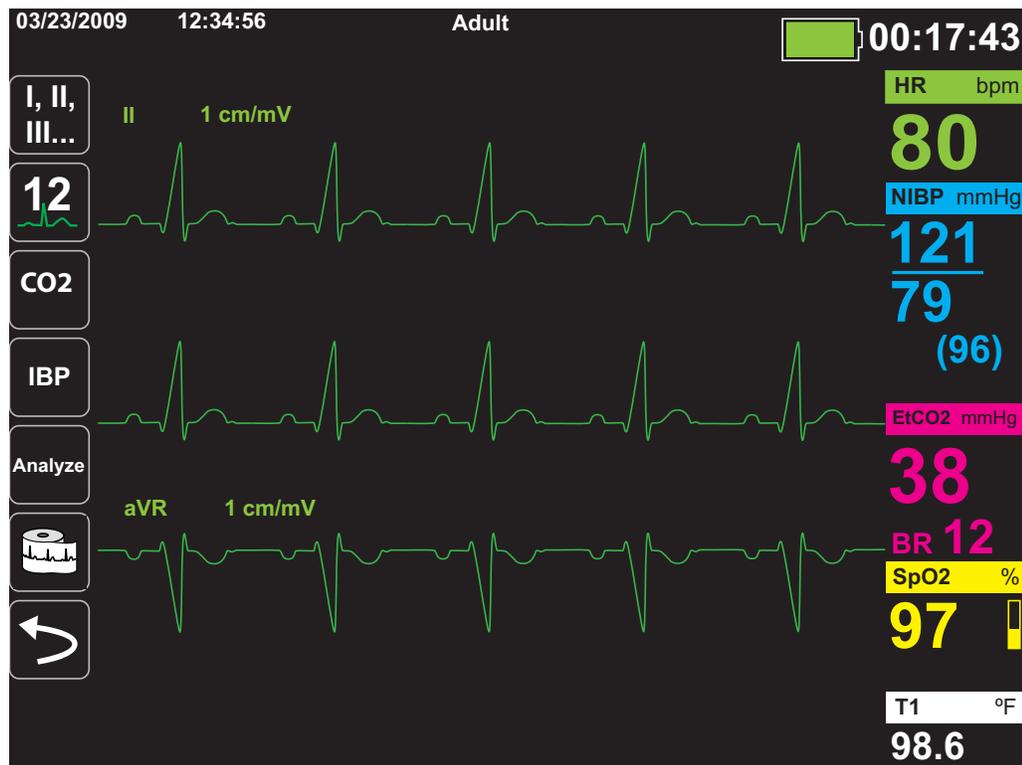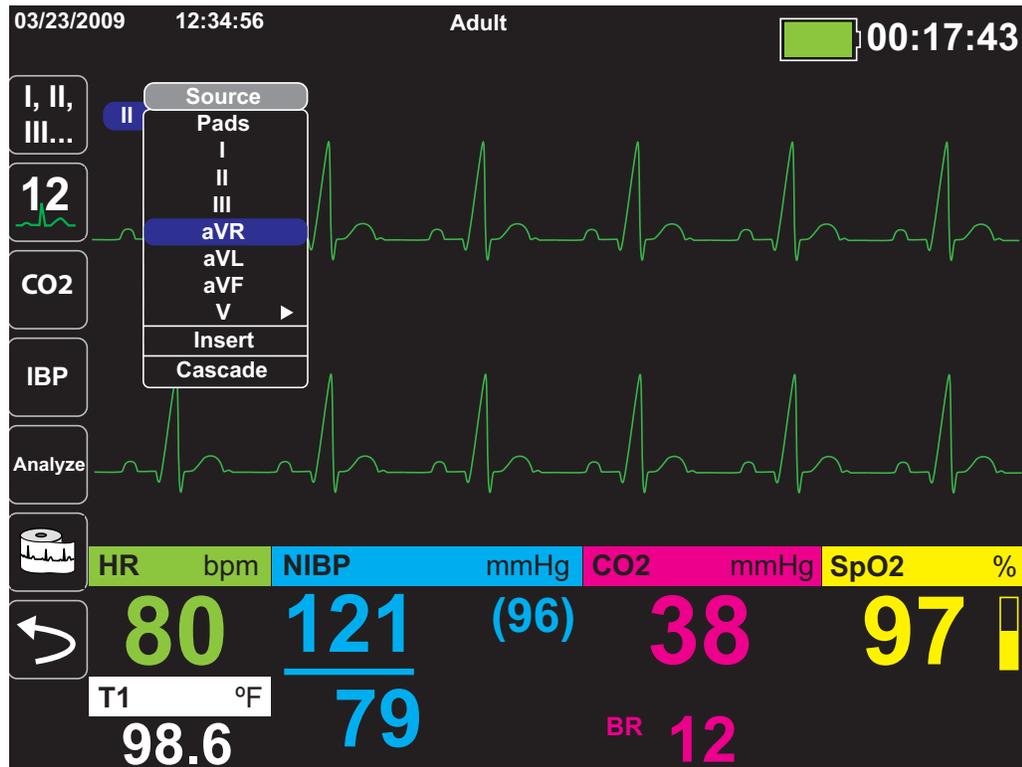

Inserting a capnogram (CO<sub>2</sub>) into the fourth trace area:

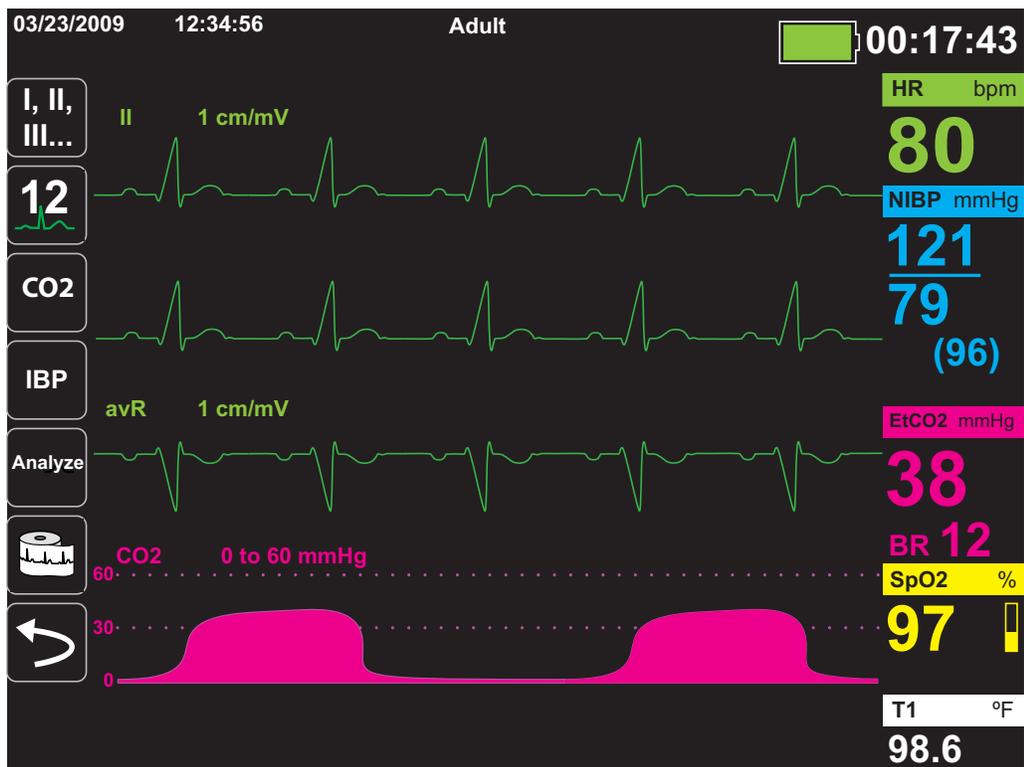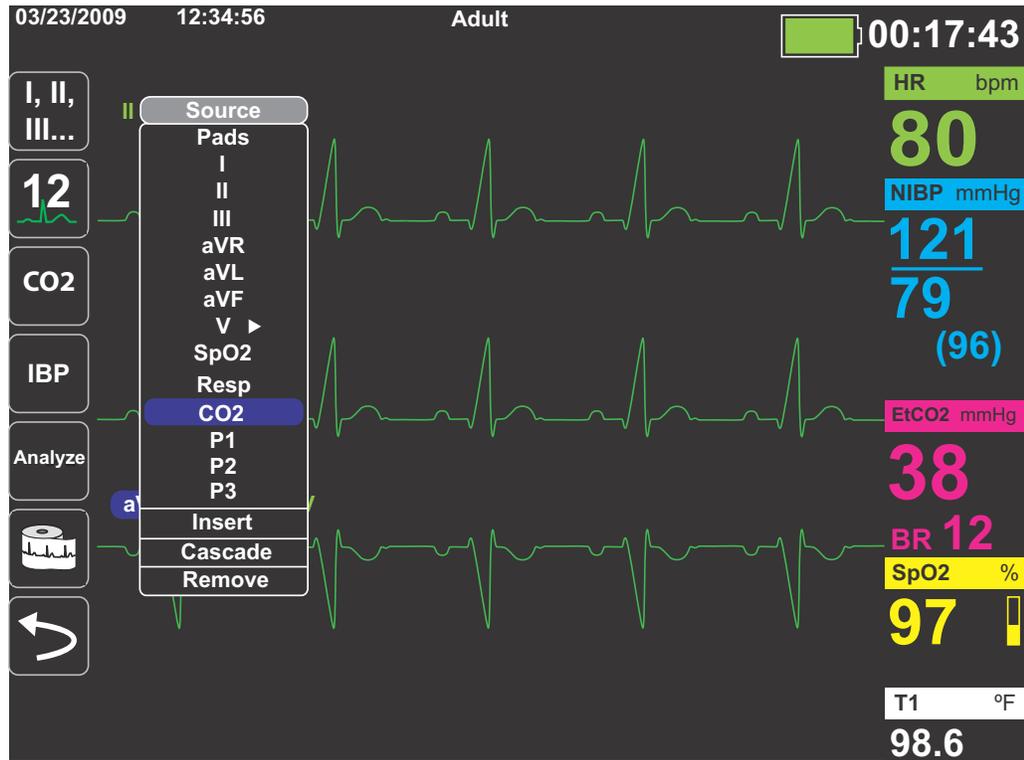

# Chapter 4

## Trends

---


The Propaq MD unit accumulates a patient's trend information by logging all monitored vital sign measurements to memory at a user-configurable interval. It also logs *all* monitored vital sign measurements when the following occurs:

- An NIBP measurement is taken and the **Trend on NIBP** option is on
- You press the Snapshot button (  ) on the front panel
- A patient alarm occurs and the **Trend on Alarm** option is on

The Propaq MD unit can store at least 24 hours of trend information when logged at a 1 minute trend interval. You can view, print, or save to external memory all logged trend information.

### Displaying the Trends Status Window

The Propaq MD unit displays the logged trend information in the Trends status window. Press the Display/Home button (  ) to display the Trends window, the primary ECG trace, and the small numeric displays for each monitoring function:

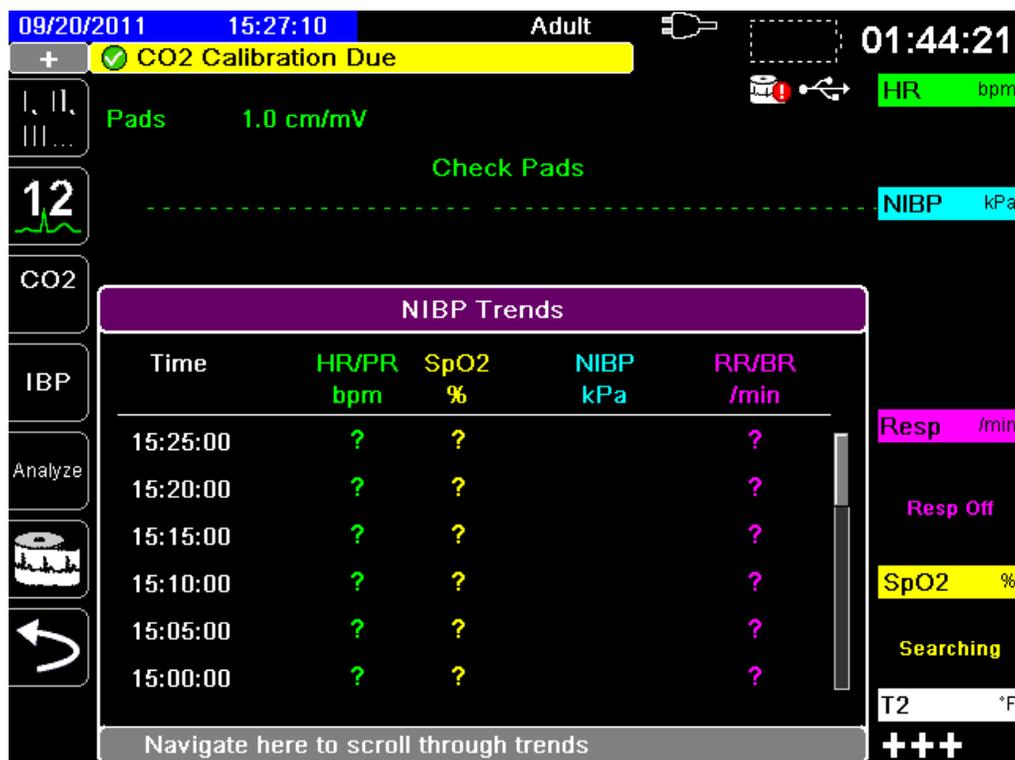

**Figure 4-1 Trends Status window**

The Trends status window displays the logged trend information and the time at which the trend measurements were logged. While trend measurements are logged to memory at a user-configurable interval, the Trends status window can display the logged information at an interval that you specify, with the exception of NIBP measurements, which are logged and reported at the times they are taken. The Trends status window reports the trends information at 5-minute intervals.

## Displaying and Printing Trend Information

To navigate in the Trends status window:

1. Use the navigation keys to highlight the **Navigate Here to Scroll Through Trends** bar, then press **Select**.
2. Press the Up/Down buttons ( $\uparrow/\downarrow$ ) to scroll through the trend information.
3. To print all trends for the current patient, highlight the **NIBP Trends** field, then press **Select**. Press the **Print Trend Summary** button in the Trend Settings menu.
4. To select which trends to print for the current patient, press the Log quick access key ( $\text{LOG}$ ), then press the **Print Trends** button ( $\text{Print Trends}$ ).

## Changing the Trends Status Window Display

By default, the Trends status window displays all logged trend information. It displays the numeric information for all monitoring functions, which the unit logs at a user-configurable interval, and when you take NIBP measurements, when a patient alarm occurs, and when you press .

To configure the display of the Trends status window, press () and press the Trend Settings quick access key () to display the Trends Settings control panel. On the Trends Settings control panel, select **Trend Display Format** to specify the following monitored vital signs that appear in the Trends status window:

| Trend Format | Vital Signs Displayed      |
|--------------|----------------------------|
| Resp         | HR, SpO2, RR, EtCO2, FiCO2 |
| NIBP         | HR, SpO2, NIBP, RR         |
| IBP1         | HR, SpO2, IBP1, RR         |
| IBP2         | HR, SpO2, IBP2 RR          |
| IBP3         | HR, SpO2, IBP3, RR         |
| Temp         | HR, SpO2, T1, T2, ΔT       |



# Chapter 5

## Alarms

---

The Propaq MD unit supports the detection and indication of *patient alarms* and *technical alerts*.

A patient alarm is any alarm condition that is caused by a monitored patient-related variable, such as a measured vital sign that falls outside of a configured alarm limit. *You can configure patient alarm limits for each of the physiologic monitoring functions.*

A technical alert is monitored equipment-related variable that the Propaq MD unit can detect, such as a disconnected sensor, internal diagnostics failures, and so on. *Technical alerts are always enabled and are not user-configurable.*

Patient alarms are always classified as high-priority alarms. Technical alerts are classified as medium or low priority alarms.

Alarm conditions from patient alarms and technical alerts are stored in the Event Log and retained with normal power down or total loss of power.

### Visual Alarm Indicators

In addition to status messages that appear on the display, the Propaq MD unit lights the red or yellow LED on the front panel to indicate the priority level of the highest-priority active alarm. The Propaq MD LEDs indicate the priority level of the highest-priority active alarm as shown in the following table.

| Active Alarm Priority              | Visual Alarm Indicator |
|------------------------------------|------------------------|
| High Priority -- Patient Alarm     | Flashing Red LED       |
| Medium Priority -- Technical Alert | Flashing Yellow LED    |
| Low Priority -- Technical Alert    | Continuous Yellow LED  |

## Audible Alarm Indicators

The Propaq MD unit sounds an audible alarm to indicate the priority level of highest-priority active alarm. The Propaq MD indicates the priority level of the highest-priority active alarm by sounding the audible alarm tones described in the following table.

| Active Alarm Priority                 | Audible Alarm Indicator  | Alarm/Alert Volume   |
|---------------------------------------|--|--|
| High Priority --<br>Patient Alarm     | Two sets of five short beep tones,<br>repeated at 15-second intervals  | Volume is adjustable, up to a<br>maximum sound pressure of at<br>least 70 dBA (measured at 1m) |
| Medium Priority --<br>Technical Alert | One set of three longer beep tones,<br>repeated at 30-second intervals | Volume is 3 to 12 dBA below high<br>priority alarm   |
| Low Priority --<br>Technical Alert    | A single short beep tone, not<br>repeated                              | Volume is 3 to 6 dBA below the<br>medium priority alert  |

Audible alarms can be silenced or suspended. More detailed information about how to silence and suspend audible alarms is included later in this chapter.


## Alarm Indicator Self-Test

The Propaq MD unit performs a self-test of the audio and visual alarm indicators upon power-up. To ensure that the alarms and alerts are functioning properly, verify that two alarm tones are heard and the green, yellow, and red LEDs are illuminated upon power up.

## Patient Alarm Display

When a patient's vital signs measurements trigger an alarm, in addition to sounding the patient alarm, the Propaq MD unit displays an alarm message, and changes the display characteristics of the monitoring function's numeric display (the alarming parameter appears in red against a white background).

In the following example, the EtCO<sub>2</sub> measurement (**22 mmHg**) has dropped below the lower alarm limit (**EtCO<sub>2</sub> Low Alarm**):

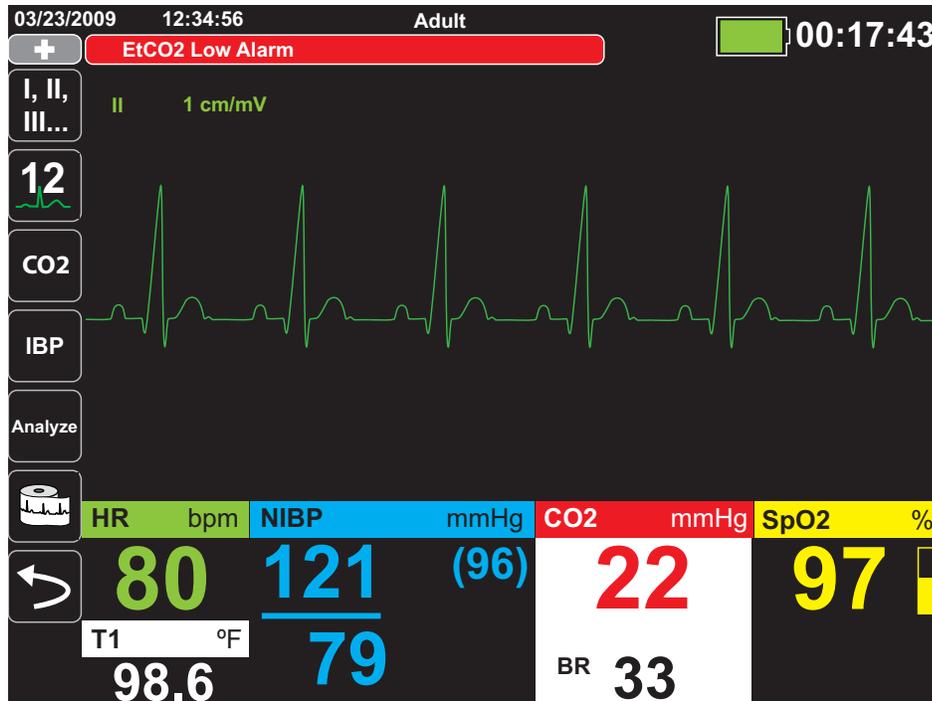

## Life Threatening Rhythm Alarms

When LTA monitoring is enabled, the Propaq MD will monitor for the following life threatening ECG rhythms: asystole, ventricular fibrillation, ventricular tachycardia, extreme bradycardia, and extreme tachycardia.

Depending upon configuration of the unit, it generates visible and audible alarms as needed.

**Note:** LTA monitoring is not available in AED mode.

## Equipment Alert Display


When a problem with the Propaq MD unit or an attached sensor triggers an alert, in addition to sounding an equipment alert, the Propaq MD unit displays an alert message (yellow background, black text).

---

**Warning!** Always respond immediately to a system alarm since the patient may not be monitored during certain alert conditions.

---

In the following example, an equipment alert message indicates that the SpO<sub>2</sub> sensor has become unattached (**SpO2 Check Sensor**) from the unit:

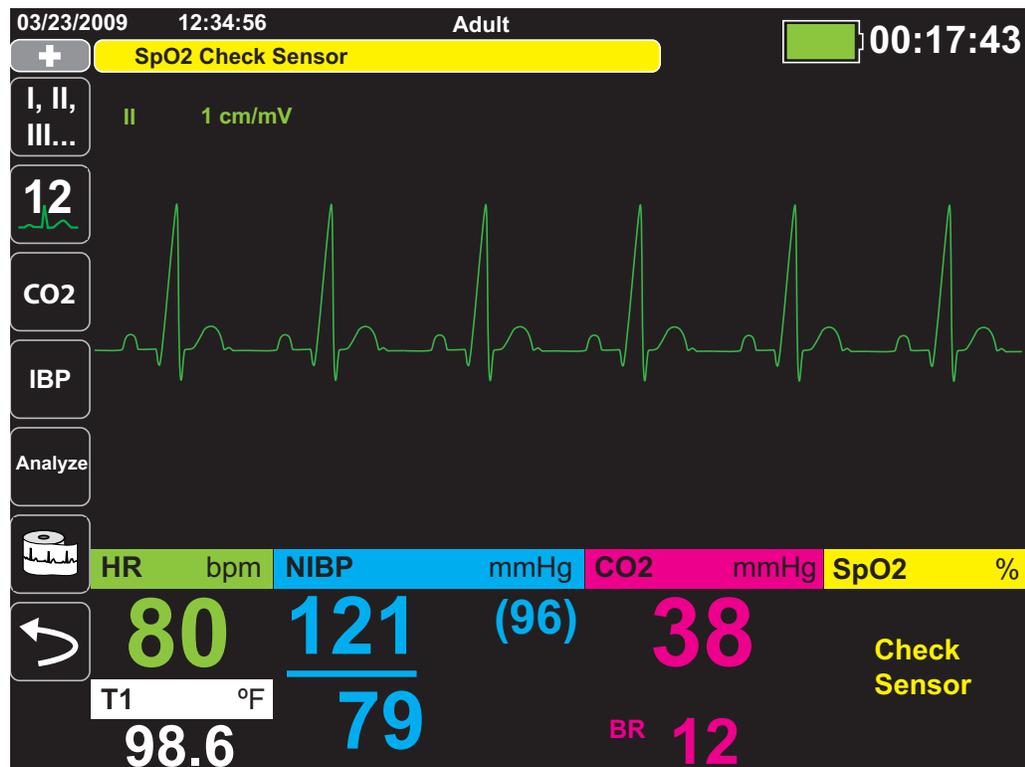

## Responding to Active Alarms -- Silencing the Alarm

When a patient alarm is triggered and the alarm tone sounds

1. Check the patient and provide appropriate care.
2. Press the Alarm Silence/Reset button () on the Propaq MD unit's front panel to silence the alarm briefly (90 seconds).
3. After caring for the patient, check that the appropriate alarms are set (for more information about setting and enabling alarms, see appropriate monitoring chapters later in this manual).

**Note:** Pressing  suspends the alarm tone for all active alarms. If the patient's vital signs measurements trigger another, different alarm, the patient alarm tone will sound, even if the first alarm silence period hasn't expired.

## Re-enabling an Alarm

To re-enable an alarm before the alarm silence period has expired, press the **Alarm Silence/Reset** button.

---

**Warning!** Do not silence the audible alarm if patient safety may be compromised.

---

## Suspending Alarms

When caring for a patient, you may want to suspend potential or current patient alarms and equipment alerts for a period of time. To suspend patient alarms

1. Press the More quick access key (  ) to access the second set of quick access keys and press the Alarms quick access key.
2. Press the Alarms Suspend quick access key (  ).

No alarms will sound while alarms are suspended; however, if an alarm occurs during the suspension period, the Propaq MD unit will display visual alarm indicators -- alarm messages in the message area (white text on a red background) and red/white numeric displays).

The duration of the alarm suspension can be configured to be for 2, 4, or 15 minutes, or for an indefinite period of time. The ability to suspend alarms can also be disabled.


---

**Warning!**      **When audible alarms are disabled, make sure that the patient is closely observed.**

---

## The Alarm Suspension Timer

During an alarm suspension, the window displays an alarm suspension timer at the top of the display next to the message area:

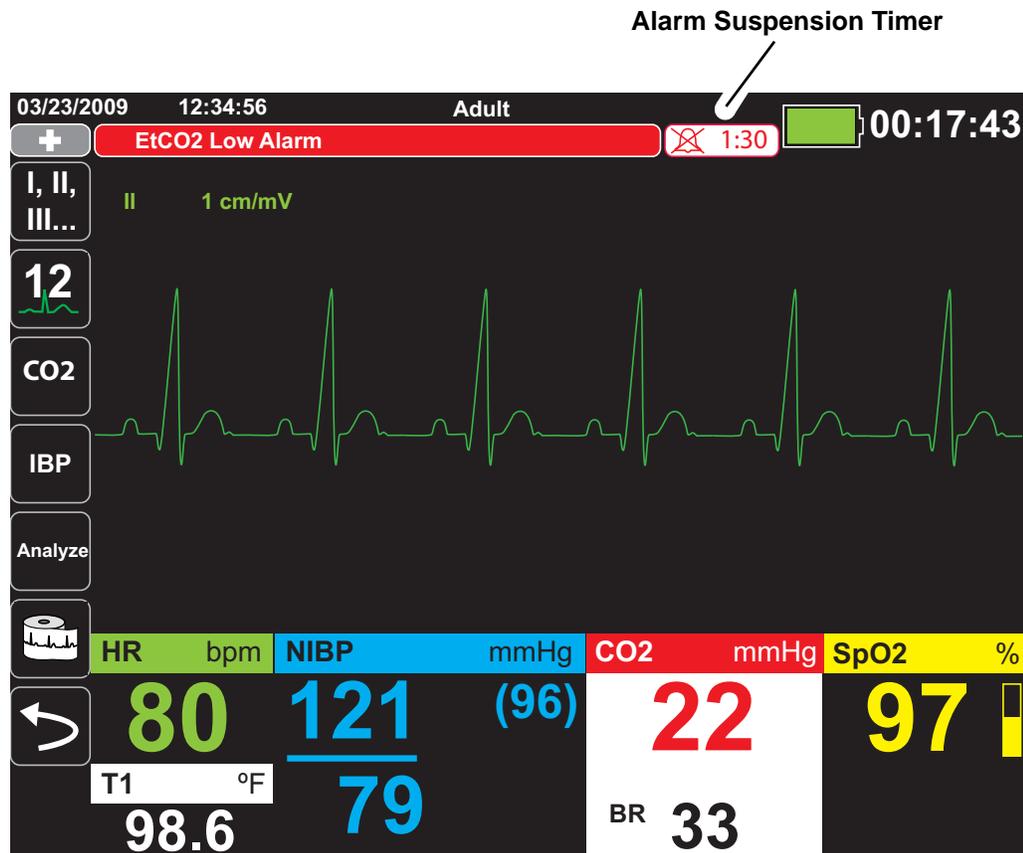

## Alarm Options

The Propaq MD unit provides alarm options that you can specify through the Supervisor parameter control panel (access to Supervisor is passcode-controlled).

Press the More quick access key (⏪), press the Setup quick access key (⚙️), and select **Supervisor**. Using the navigation keys, display and select the four digits in the Supervisor passcode:

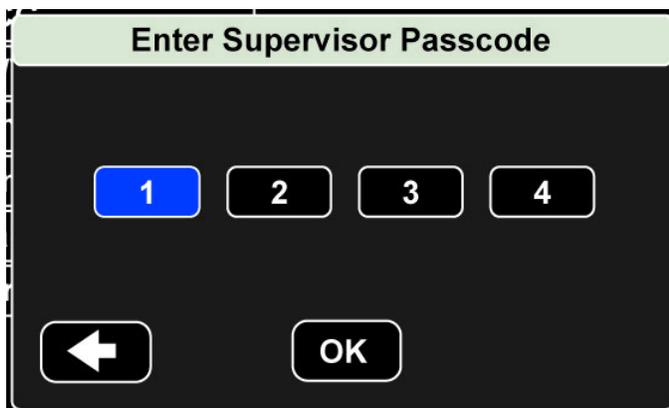


Figure 5-1 Supervisor Parameter Control Panel

Select **Alarms** to display the alarms parameter control panel:

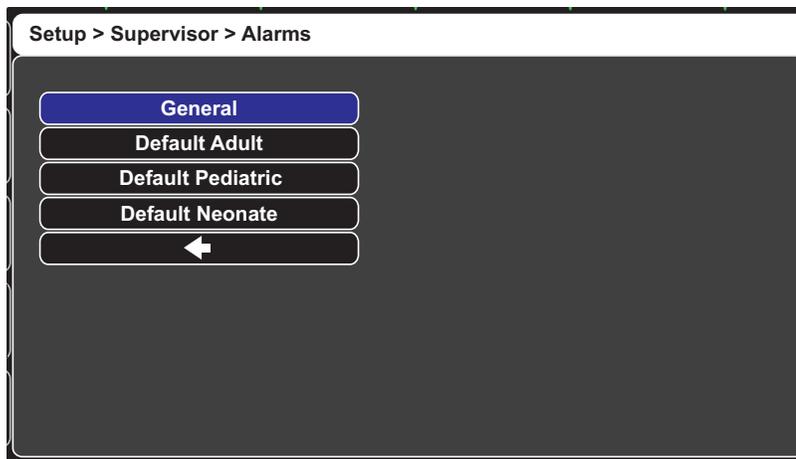


Figure 5-2 Alarms Parameter Control Panel

## Selecting Default Alarm Limits

The first three options -- **Default Adult**, **Default Pediatric**, **Default Neonate** -- allow you to set *all* alarm limits to the Propaq MD unit's factory-specified default values, by patient type.

- Warning!**
- **A potential hazard exists if different alarm limits are used for the same or similar equipment in any single area.**
  - **Confirm the alarm limits are appropriate for the patient each time there is a new patient case.**
  - **Do not set alarm limits to such extreme values that render the alarm system useless.**

## Setting Alarm Limits Relative to the Patient -- Stat Set Option

The Propaq MD unit also allows you to set all alarm limits relative to the patient's current vital signs measurements by performing the following actions:

1. Press .
2. Press the Alarm quick access key (.
3. Press the **Stat Set** quick access key (). The Propaq MD unit sets all parameters to a new value based on the current values as follows:

| Parameter (units)   | Range                              | Upper Limit Calculation | Lower Limit Calculation |
|---------------------|------------------------------------|-------------------------|-------------------------|
| <b>HR/PR (bpm)</b>  | Numeric < 26                       | Limit is unchanged      | Limit = 25              |
|                     | $26 \leq \text{Numeric} \leq 99$   | Limit = Numeric x 1.2   | Limit = Numeric x 0.8   |
|                     | $100 \leq \text{Numeric} \leq 250$ | Limit = Numeric + 20    | Limit = Numeric - 20    |
|                     | Numeric > 250                      | Limit = 250             | Limit is unchanged      |
| <b>IBP (mmHg)</b>   | Numeric < 26                       | Limit = Numeric + 5     | Limit = Numeric - 5     |
|                     | $26 \leq \text{Numeric} \leq 99$   | Limit = Numeric x 1.2   | Limit = Numeric x 0.8   |
|                     | Numeric > 99                       | Limit = Numeric + 20    | Limit = Numeric - 20    |
| <b>NIBP (mmHg)</b>  | Numeric < 26                       | Limit = Numeric + 5     | Limit = Numeric - 5     |
|                     | $26 \leq \text{Numeric} \leq 99$   | Limit = Numeric x 1.2   | Limit = Numeric x 0.8   |
|                     | Numeric > 99                       | Limit = Numeric + 20    | Limit = Numeric - 20    |
| <b>RR/BR (/min)</b> | Numeric < 26                       | Limit = Numeric + 5     | Limit = Numeric - 5     |
|                     | $26 \leq \text{Numeric} \leq 99$   | Limit = Numeric x 1.2   | Limit = Numeric x 0.8   |
|                     | Numeric > 99                       | Limit = Numeric + 20    | Limit = Numeric - 20    |

|                     |              |  |   |
|---------------------|--------------|--|---|
| <b>SpO2 (%)</b>     | Entire range | Limit = 100 (Adult and Pediatric)<br>Limit = Numeric + 5 (Neonate) | Limit = Numeric – 5   |
| <b>EtCO2 (mmHg)</b> | Entire range | Limit = Numeric + 10   | Limit = Numeric - 5 mmHg <i>unless</i> the numeric falls below the lower alarm limit range, in which case Stat Set sets the lower limit to 15 mmHg. |
| <b>FiCO2 (mmHg)</b> | Entire range | Limit = Numeric + 5  | N/A   |
| <b>Temp (°C)</b>    | Entire range | Limit = Numeric + 0.5  | Limit = Numeric – 0.5   |
| <b>Temp (°F)</b>    | Entire range | Limit = Numeric + 0.9  | Limit = Numeric – 0.9   |



# Chapter 6

## Monitoring ECG

---

This chapter describes how to use the Propaq MD unit to monitor ECG.

Propaq MD units can perform ECG monitoring through 3-, 5-, or 12-lead ECG patient cables, Multi-Function Pads, or standard defibrillation paddles. The use of an ECG patient cable and electrodes is required, however, to monitor ECG during pacing.

You can use a 3-lead, 5-lead, or 12-Lead wire configuration for ECG monitoring (see Chapter 13, *Monitoring 12-Lead ECG*, for information on 12-lead monitoring).

- 
- Warning!**
- **Excessive body hair or wet, sweaty skin may interfere with electrode adhesion. Remove the hair and/or moisture from the area where the electrode is to be attached.**
  - **Use only electrodes that are well within the expiration date indicated on the package.**
  - **Remove ECG electrodes from their sealed package immediately prior to use. Using previously opened or out-of- date electrodes may degrade the ECG signal quality.**
  - **Monitoring electrodes may become polarized during defibrillator discharge, causing the ECG waveform to briefly go off screen. ZOLL Medical Corporation recommends the use of high quality silver/silver chloride (Ag/AgCl) electrodes to minimize this effect; the circuitry in the instrument returns the trace to the monitor display within a few seconds.**
  - **To assure protection against the effects of defibrillator discharge, use only ZOLL-approved accessories.**

- **To avoid a shock hazard and interference from nearby electrical equipment, keep electrodes and patient cables away from grounded metal and other electrical equipment.**
  - **To avoid electrosurgery burns at monitoring sites, ensure proper connection of the electrosurgery return circuit so that the return paths cannot be made through monitoring electrodes or probes.**
  - **Check the operation and integrity of the Propaq MD unit and ECG cable regularly by performing the Daily Operational Verification Test.**
  - **Implanted pacemakers may cause the heart rate meter to count the pacemaker rate during incidents of cardiac arrest or other arrhythmias. Carefully observe pacemaker patients. Check the patient's pulse; do not rely solely on heart rate meters. Dedicated pacemaker detection circuitry may not detect all implanted pacemaker spikes. Patient history and physical exam are important in determining the presence of an implanted pacemaker.**
- 

## ECG Monitoring Setup

The proper application and placement of electrodes is essential for high quality ECG monitoring. Good contact between the electrode and skin minimizes motion artifact and signal interference.

The following procedure describes how to monitor a patient's ECG using 3- and 5-Lead ECG cables. For information on the application and use on MultiFunction Pads and External Paddles, which you can also use to monitor ECG, see Chapter 14, *Manual Defibrillation*.

To monitor a patient's ECG using 3- and 5-Lead ECG cables, perform the following steps:

1. Prepare the patient's skin for electrode application:
2. Apply the electrode pads to the patient.
3. Connect each lead of the ECG cable to the appropriate electrode.
4. Insert the patient cable plug into the ECG input connector on the Propaq MD unit.
5. Select the ECG waveforms to be displayed on the waveform trace display screen.
6. Observe the patient's electrocardiogram on the display, and adjust size of the ECG waveform trace, as necessary.

## Preparing the Patient for Electrode Application

The proper application of electrodes is essential for high quality ECG monitoring. Good contact between the electrode and skin minimizes motion artifact and signal interference.

Before applying electrodes, prepare the patient's skin, as necessary:

- Shave or clip excess hair at electrode site.
- Clean oily skin with an alcohol pad.
- Rub site briskly to dry.

## Applying Electrodes to the Patient

The following sections show where to place electrodes when using 3- and 5-Lead cables to perform ECG monitoring. For 3-Lead ECG cables, apply electrodes as in Figure 4-1, *3-Lead Electrode Placement*. For 5-Lead ECG cables, apply electrodes as in Figure 4-2, *5-Lead Electrode Placement*.


Avoid placing electrodes over tendons and major muscle masses.

Make sure that the ECG electrodes are placed to allow defibrillation, if necessary.

### 3-Lead Electrode Placement

Depending upon local usage, the ECG leads are marked either RA, LA, and LL (or R, L, and F). The following table shows the markings and color codes for the different lead sets.

| AHA Color Coding   | IEC Color Coding   | Placement of Electrodes  |
|--------------------|--------------------|--|
| RA/White Electrode | R/Red Electrode    | Place near patient's right mid-clavicular line, directly below clavicle.           |
| LA/Black Electrode | L/Yellow Electrode | Place near patient's left mid-clavicular line, directly below clavicle.            |
| LL/Red Electrode   | F/Green Electrode  | Place between 6th and 7th intercostal space on patient's left mid-clavicular line. |

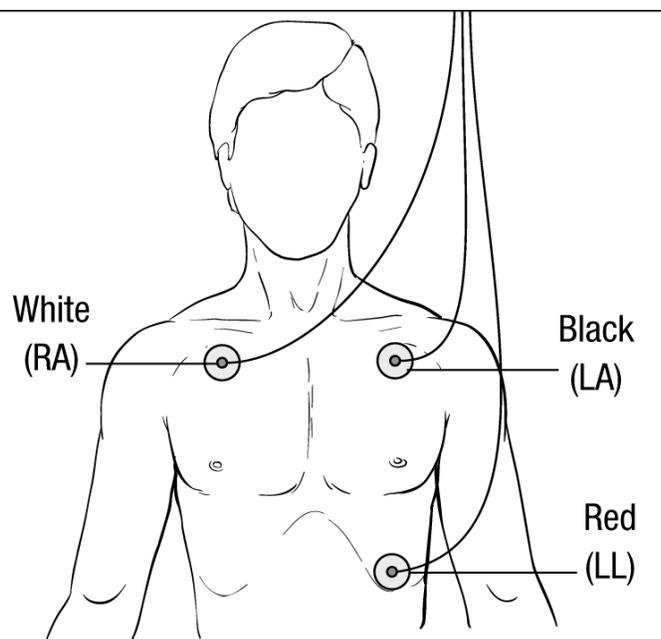

**Figure 6-1 3-Lead Electrode Placement**

### 5-Lead Electrode Placement

Depending upon local usage, the ECG leads are marked either RA, LA, LL, RL, and V or R, L, F, N and C. The following table shows the markings and color codes for the different lead sets.

| AHA Color Coding    | IEC Color Coding   | Placement of Electrodes   |
|---------------------|--------------------|---|
| RA/White Electrode  | R/Red Electrode    | Place near patient's right mid-clavicular line, directly below clavicle.  |
| LA/Black Electrode  | L/Yellow Electrode | Place near patient's left mid-clavicular line, directly below clavicle.   |
| LL/Red Electrode    | F/Green Electrode  | Place between 6th and 7th intercostal space on patient's left mid-clavicular line.  |
| RL/Green* Electrode | N/Black* Electrode | Place between 6th and 7th intercostal space on patient's right mid-clavicular line.   |
| V/Brown* Electrode  | C/White* Electrode | <p>Single movable chest electrode. Place this electrode in one of the positions, V1 - V6, as shown in the following figure.</p> <p>V1 -- 4th intercostal space at right sternal margin.</p> <p>V2 -- 4th intercostal space at left sternal margin.</p> <p>V3 -- Midway between V2 and V4 leads.</p> <p>V4 -- 5th intercostal space at mid-clavicular line.</p> <p>V5 -- Same transverse level as V4 at left anterior-axillary line.</p> <p>V6 -- Same transverse level as V4 at left mid-axillary line.</p> |

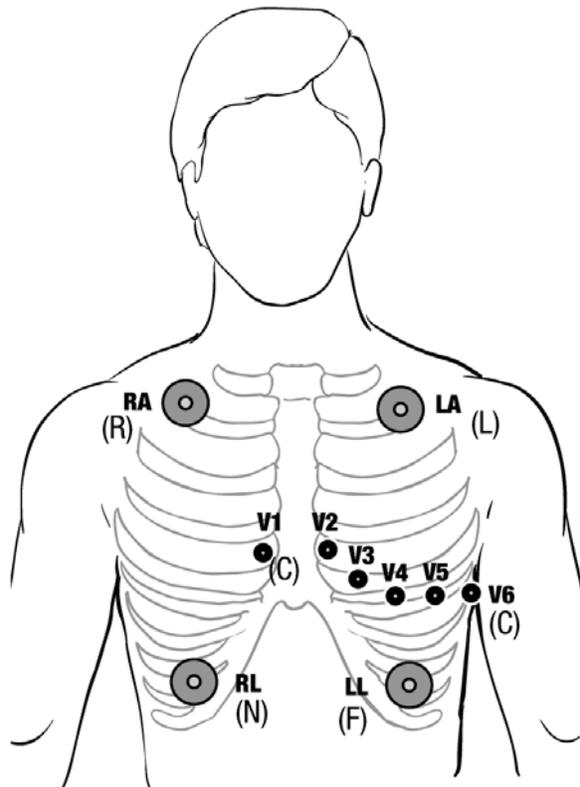


Figure 6-2 5-Lead Electrode Placement

## Connecting the ECG Cable To the Propaq MD Unit


The Propaq MD unit accepts Welch Allyn Propaq ECG cables as well as ZOLL Propaq MD ECG cables. Connect the ECG cable to the ECG connector on the left side of the Propaq MD unit as follows:



Figure 6-3 Connecting ECG Cable to Propaq MD Unit


## Selecting ECG Waveforms for Display

You can fit up to four waveforms on the Propaq MD display. The first waveform at the top of the display is always an ECG waveform. In the following example, Lead II (RA-LL), is the source of the ECG waveform trace:

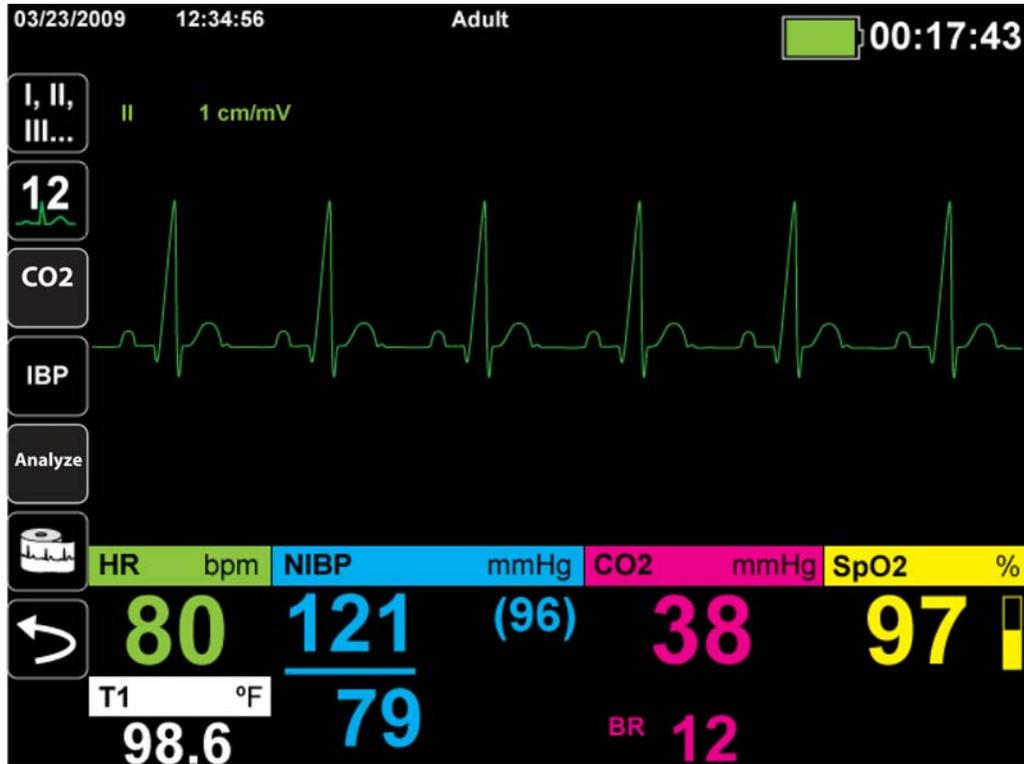

There are two ways to specify which ECG lead is the source of the primary waveform trace. One way is to press the ECG lead selection quick access key  to display the available ECG waveform sources. The available waveform sources are determined by the type of ECG cable connected to the unit.

The other way to specify the source of the primary waveform trace is to navigate to and select the source label for the primary ECG waveform (Lead I in the screen below). The Propaq MD unit then displays the available ECG waveform sources. The following example illustrates the waveform source list that the Propaq MD unit displays when a 5-lead ECG cable is connected to it. The list of available ECG waveform sources includes Leads I, II, III, aVR, aVL, aVF, and V. You can select Lead II (the default), or use the navigation keys to highlight and select another displayed ECG lead as the source for the waveform trace.

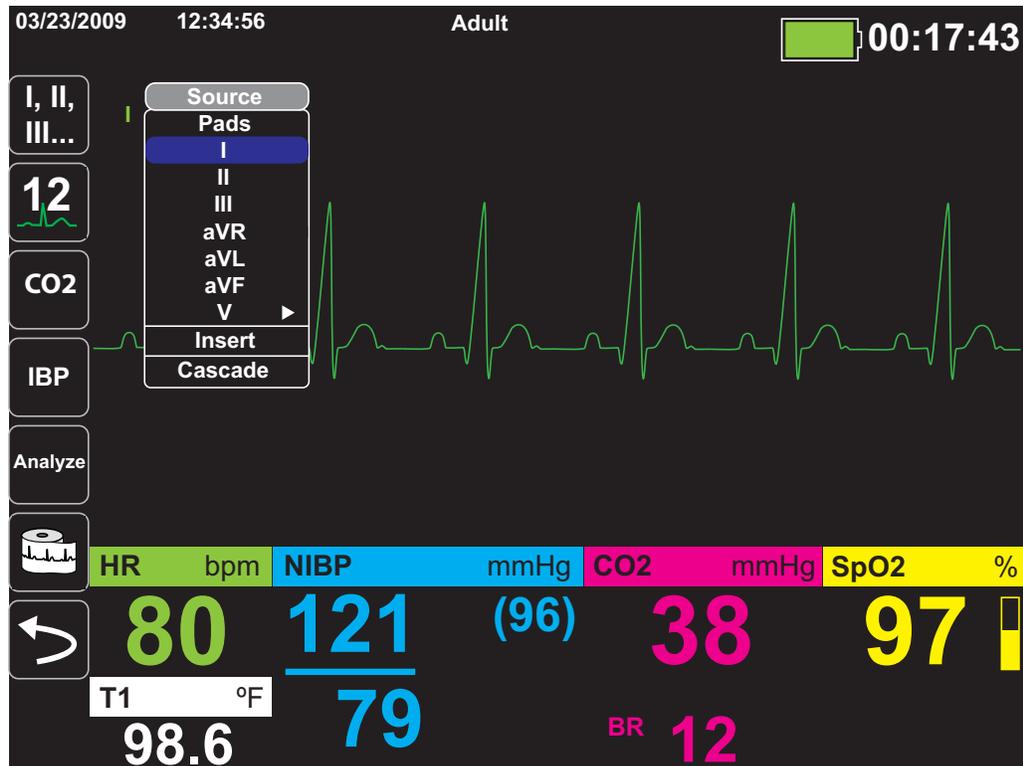


Select a current waveform source, and the unit displays the waveform immediately. If you select a waveform source that is not currently available, the unit displays the message, *LEAD FAULT*.

For more information on how to configure the display of waveforms on the Propaq MD unit, see Chapter 3, *Monitoring Overview*.

## Selecting the Waveform Trace Size

The Propaq MD unit allows you to select the waveform trace size to adjust the size of displayed the ECG waveform.

To select the waveform size, use the navigation keys to highlight and select the trace size that appears to the right of the electrode label:

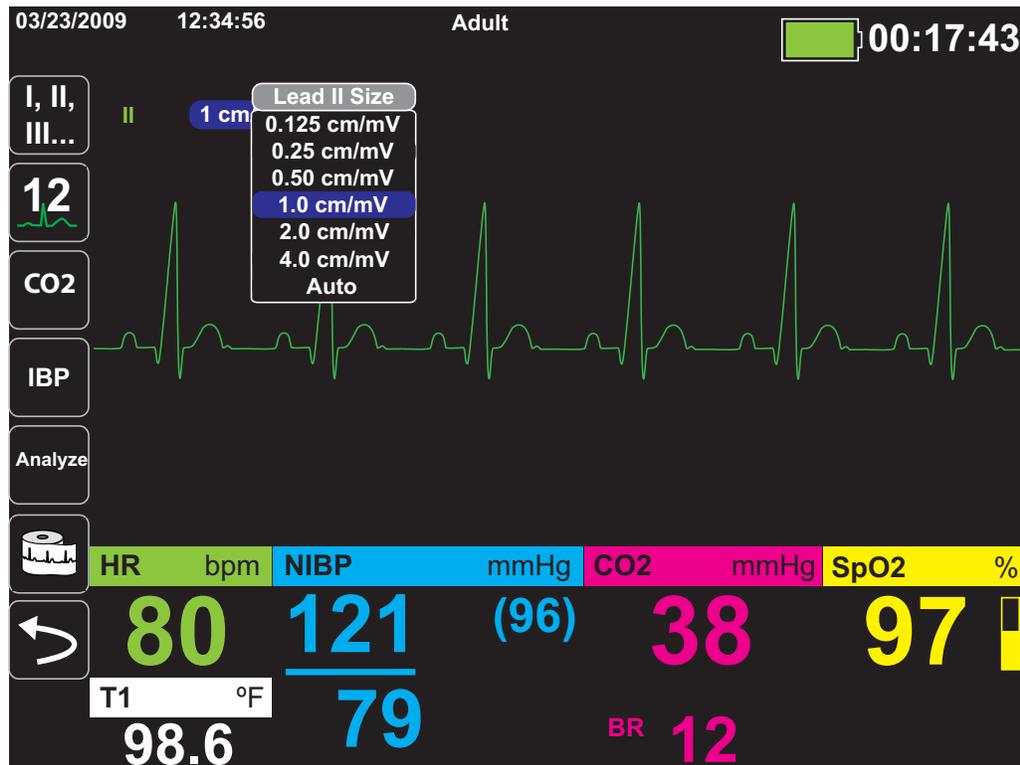

The default trace size is **1cm/mV**. You can select a larger (**2.0, 4.0 cm/mV**) or smaller (**0.125, 0.25, 0.50 cm/mV**) trace size. You can also specify that the Propaq MD unit select a best-fit trace size (**AUTO**).

## ECG Monitoring and Pacemakers

When the unit performs ECG monitoring on a patient with an implantable pacemaker, the unit's Pacer indicator feature can indicate the occurrence of pacemaker signals.

If the Pacer Indicator setting is **ON**, the Propaq MD performs the following actions:

- detects the implantable pacemaker pulses
- blanks the pacemaker pulses from the waveform—preventing them from disturbing the ECG waveform and allowing for an accurate QRS detection
- displays and prints vertical dashed lines to indicate the detected pacemaker signals

If the Pacer Indicator setting is **OFF**, the Propaq MD does not perform the following actions:

- detect the implantable pacemaker pulses
- blank the pacemaker pulses from the waveform
- display or print the vertical dashed line pacer markers

You can turn the Pacer Indicator **ON/OFF** from the Setup>ECG menu.

There are situations where ECG artifact could simulate pacemaker signals which could cause false pacemaker detection and blanking. These situations may cause inaccurate QRS detection and it may be desirable to turn the Pacer Indicator off. Inversely, when the Pacer Indicator setting is **OFF**, implantable pacemaker signals may cause inaccurate QRS detection and it may be desirable to turn the Pacer Indicator on.

## ECG System Messages

When monitoring ECG, the Propaq MD unit may display the following messages:

| System Message    | Cause   |
|-------------------|---|
| <i>LEAD FAULT</i> | <p>The current ECG source lead is defective (check lead and replace, if necessary).</p> <p style="text-align: center;">-- OR --</p> <p>An unavailable waveform source has been specified for the trace display (check specified waveform source and correct, if necessary).</p> |




# Chapter 7

## Monitoring Respiration (Resp) and Heart Rate (HR)

This chapter describes how to use the Propaq MD unit to monitor Respiration (**Resp**) and Heart Rate (**HR**).

The Propaq MD unit displays Respiration (**Resp**) and Heart Rate (**HR**) meters. The Respiration and Heart Rate meters display values that the Propaq MD unit derives from measurements taken by other Propaq MD monitoring functions.

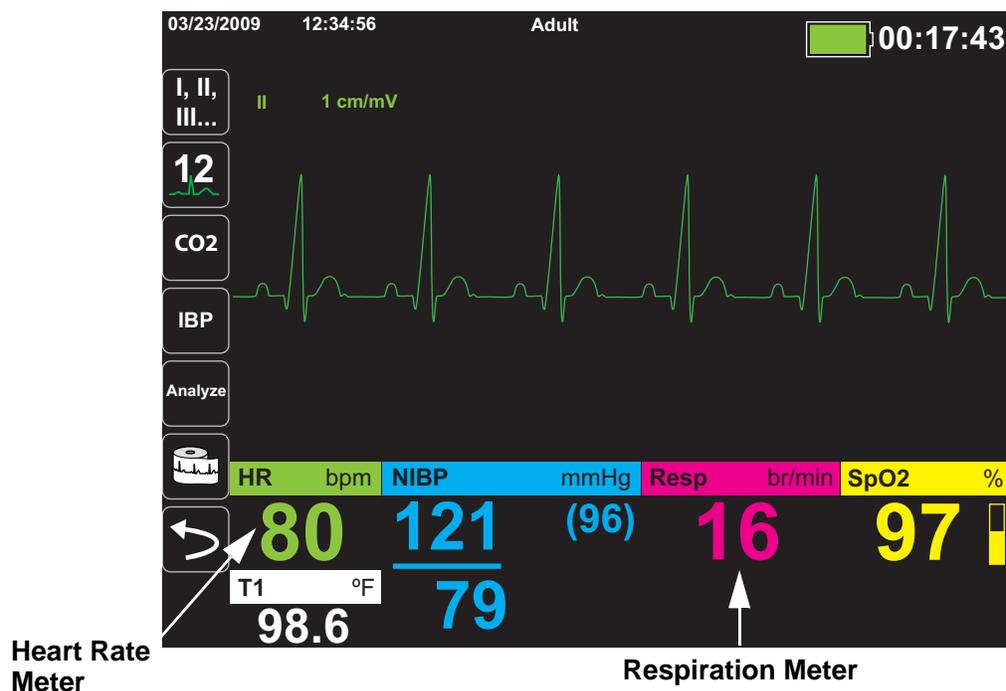

## Respiration/Breath Rate Meter

If enabled, the Propaq MD unit displays the patient's respiration in the Respiration/Breath Rate Meter.



The respiration meter displays the respiration rate that it derives, by default, from the unit's CO<sub>2</sub> monitoring function. If CO<sub>2</sub> monitoring is not available, the unit derives the respiration rate through *impedance pneumography*, using a specified ECG electrode configuration. If ECG monitoring isn't functioning, the **Resp/BR** meter will not display a respiration rate.

### Using Impedance Pneumography to Measure Respiration

Impedance pneumography detects respiration by applying a high-frequency, low-current AC signal to the patient and measuring the changes in impedance through ECG electrode Lead I (RA-LA) or Lead II (RA-LL). As the patient inhales and chest volume expands, impedance increases; as the patient exhales, impedance decreases.

- 
- Warning!**
- **Impedance pneumography detects respiratory effort through changes in chest volume. However, No Breath episodes with continued respiratory effort may go undetected. Always monitor and set alarms for SpO<sub>2</sub> when using impedance pneumography to monitor respiratory function.**
  - **With any monitor that detects respiratory effort through impedance pneumography, artifact due to patient motion, apnea mattress shaking, or electrocautery use may cause apnea episodes to go undetected. Always monitor and set alarms for SpO<sub>2</sub> when using impedance pneumography to monitor respiratory function.**
  - **When using impedance pneumography, don't use the Propaq MD unit with another respiration monitor on the same patient, because the respiration measurement signals may interfere with one another.**
  - **Impedance pneumography is *not* recommended for use on paced patients, because pacemaker pulses may be falsely counted as breaths.**
  - **Impedance pneumography is not recommended for use with high frequency ventilation.**
  - **Since impedance pneumography uses the same leads as the ECG channel, the Propaq MD unit determines which signals are cardiovascular artifact and which signals are the result of respiratory effort. If the breath rate is within five percent of the heart rate, the monitor may ignore breaths and trigger a respiration alarm.**
-

## Configuring Respiration (RR/BR) Alarms and Settings

The Propaq MD unit allows you to enable and disable the Respiration (**RR/BR**) Rate alarm, to set alarm limits, and to specify the ECG monitoring source for the Respiration rate.

### Enabling/Disabling RR/BR Alarms and Setting Alarm Limits

When enabled, the Propaq MD unit sounds alarms whenever the patient's respiration rate is above or below the specified respiration rate alarm limits.

To enable (or disable) Resp alarms and set Upper and Lower alarm limits, you can either do so through the Alarms quick access key () or the **Resp Parameter Control Panel**.

To configure **RR/BR** alarms through the **Alarms** quick access key:

1. Press .
2. Press .
3. Press the **Limits** quick access key () . Use the navigation keys to highlight and select the **RR/BR Alarm** menu selection.
4. On the **RR/BR Alarm Settings** menu, use the navigation keys to select the fields that you want to change:
  - **Upper Limit Enable**
  - **Lower Limit Enable**
  - **Upper Limit**
  - **Lower Limit**
5. When you are finished changing values on the alarm menu, navigate to the Backarrow key () to confirm your choices and exit the menu.


## Respiration Rate Alarm Limits

Initially, the **Resp Alarm Settings** menu specifies that Resp alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower respiration rate alarm limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default respiration rate alarm limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | Respiration Rate Default       | Respiration Rate Range                     |
|--------------|--------------------------------|--|
| Adult        | Lower: 3 BPM<br>Upper: 50 BPM  | Lower: 0 to 145 BPM<br>Upper: 5 to 150 BPM |
| Pediatric    | Lower: 38 BPM<br>Upper: 50 BPM | Lower: 0 to 145 BPM<br>Upper: 5 to 150 BPM |
| Neonate      | Lower: 12 BPM<br>Upper: 80 BPM | Lower: 0 to 145 BPM<br>Upper: 5 to 150 BPM |

## Using the Resp Parameter Control Panel

To configure alarms through the **Resp** Parameter Control Panel, use the navigation keys to highlight and select the Respiration Rate meter and display the **Resp** Parameter Control Panel:

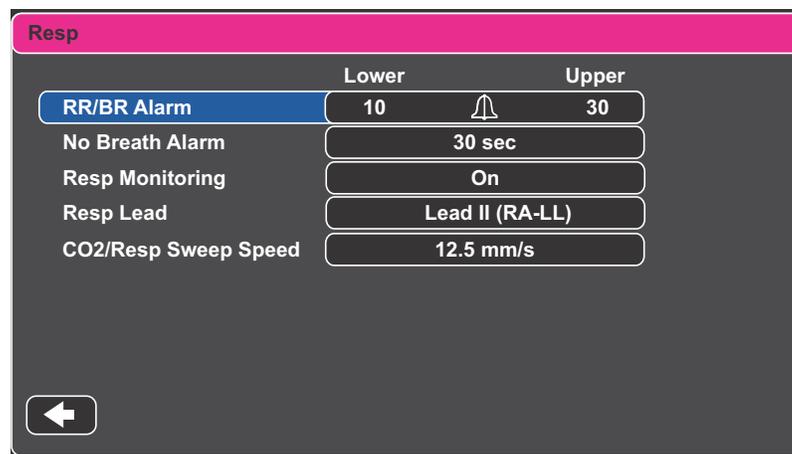

**Figure 7-1 Respiration Parameter Control Panel**

The **Resp** Parameter Control Panel allows you to set the following parameters:

- **RR/BR Alarm** -- enable/disable the Resp alarm and set high/low alarm limits.
- **No Breath Alarm** -- sets the duration of the No Breath alarm.
- **Resp Monitoring** -- enable/disable respiration monitoring.
- **Resp Lead** -- selects the Resp lead, Lead I (RA-LA) or Lead II (RA-LL), from which the Propaq MD unit calculates the respiration rate. Resp Lead selection is independent of ECG Lead selection.
- **CO2/Resp Sweep Speed** -- sets the respiratory sweep speed on the display.

### Enabling/Disabling Resp Monitoring

Select the Resp Monitoring prompt to enable or disable **Resp** monitoring. When Resp Monitoring is set to On (the default), the Propaq MD unit displays the Respiration Rate Meter. When set to off, Propaq MD removes the Respiration Rate meter from the display.

---

**Warning!** When using impedance pneumography, the Propaq MD unit automatically rejects cardiovascular artifact (CVA). This function requires the accurate ECG R-wave detection. Therefore, always select the ECG lead with the most prominent QRS complex when using impedance pneumography to monitor respiration.

---

## Heart Rate Meter

The Heart Rate meter displays the heart rate that it derives from the ECG monitoring function (by default) or from a monitoring function that you specify. If the ECG (or user-specified monitor function) measurements are not available, the Heart Rate meter derives the heart rate from the following monitoring functions, if they are available, in this order: User-selected default source, ECG, IBP channel 1, SpO<sub>2</sub>, IBP channel 2, IBP channel 3, and NIBP. The Heart Rate meter is labeled **HR** (as in the following example) if the source is ECG, and **PR** if any other source is used.



## Configuring Heart Rate (HR) Meter Alarms

The Propaq MD unit allows you to enable and disable the Heart Rate (HR) alarm, to set alarm limits, and to select a Heart Rate tone.

## Enabling/Disabling HR Alarms and Setting Alarm Limits

When enabled, the Propaq MD unit sounds alarms whenever the patient’s heart rate is above or below the specified heart rate alarm limits.

To enable (or disable) HR alarms and set Upper and Lower alarm limits, you can either do so through the Alarms quick access key () or the **HR/PR Parameter Control Panel**.

To configure HR alarms through the **Alarms** quick access key:

1. Press .
2. Press .
3. Press .
4. Use the navigation buttons to highlight and select **HR/PR Alarm**.
5. On the HR/PR Alarm Settings menu, use the navigation keys to select the fields that you want to change:
  - **Upper Limit Enable**
  - **Lower Limit Enable**
  - **Upper Limit**
  - **Lower Limit**

 When you are finished changing values on the alarm menu, navigate to the Backarrow key to confirm your choices and exit the menu.

### Heart Rate (HR/PR) Alarm Limits


Initially, the HR/PR Alarm Settings menu specifies that alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower heart rate alarm limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default HR alarm limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | HR Default                       | HR Rate Range                                |
|--------------|----------------------------------|--|
| Adult        | Lower: 50 BPM<br>Upper: 120 BPM  | Lower: 20 to 298 BPM<br>Upper: 22 to 300 BPM |
| Pediatric    | Lower: 50 BPM<br>Upper: 150 BPM  | Lower: 20 to 298 BPM<br>Upper: 22 to 300 BPM |
| Neonate      | Lower: 100 BPM<br>Upper: 200 BPM | Lower: 20 to 298 BPM<br>Upper: 22 to 300 BPM |

## Life Threatening Rhythm Alarms

When LTA monitoring is enabled, the Propaq MD unit will monitor for the following life threatening ECG rhythms: asystole, ventricular fibrillation, ventricular tachycardia, extreme bradycardia, and extreme tachycardia. These events are also displayed on Treatment Summary Reports, Alarm Snapshots, and Continuous Waveform printouts.

The following example shows an Asystole alarm event:

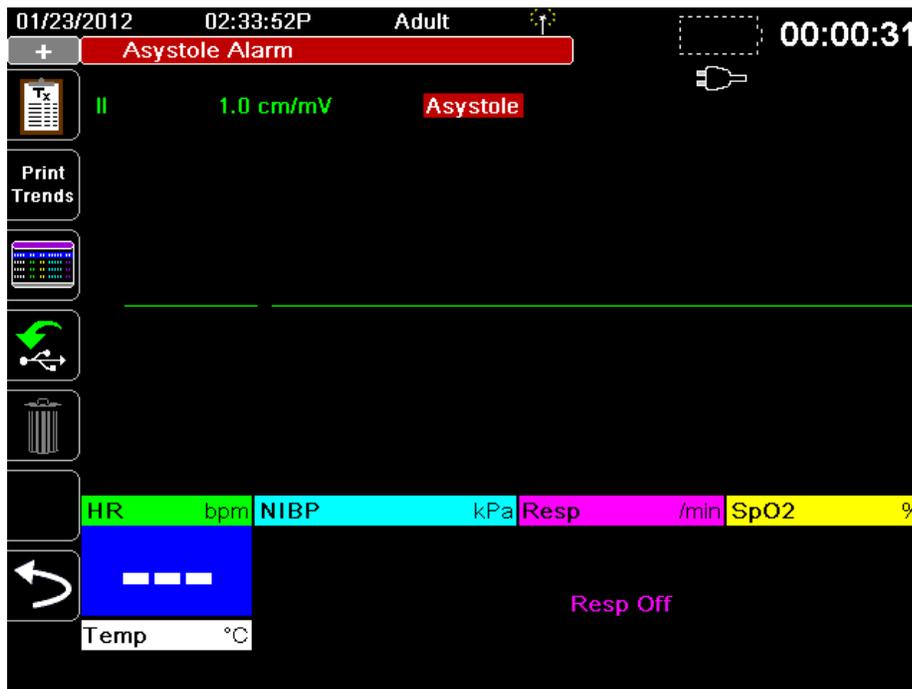

To enable or disable LTA alarms and set Extreme Brady/Tachy alarm limits, you can either do so through the Alarms quick access key (🔔), or the **HR/PR Parameter Control Panel**.

To configure LTA alarms through the Alarms quick access key:

1. Press .
2. Press .
3. Press . Use the navigation keys to highlight and select the fields that you want to change:
  - **LTA Monitoring**
  - **Extreme Brady/Tachy Alarms**

 When you are finished changing values on the alarm menu, navigate to the Backarrow key to confirm your choices and exit the menu.

### Extreme Bradycardia and Extreme Tachycardia Alarm Limits

The Extreme Brady/Tachy Alarm Settings menu specifies the configured delta values that are applied to the HR lower limit alarm and HR upper limit alarm for the Extreme Bradycardia and Extreme Tachycardia LTA alarm detection, respectively. For example, if the HR lower limit alarm is configured to 50 and the Extreme Brady delta value is configured to -10, the Extreme Bradycardia LTA detection threshold will be at 40 bpm. Likewise, if the HR upper limit alarm is configured to 120 and the Extreme Tachy delta value is configured to +20, the Extreme Tachycardia LTA detection threshold will be at 140 bpm. The HR lower and upper limit alarms are not required to be enabled for the Extreme Bradycardia and Extreme Tachycardia LTA alarm detection to be enabled. The Extreme Bradycardia and Extreme Tachycardia LTA alarms will not activate until 20 seconds after the alarm limit threshold is crossed.

The following tables list the default Extreme Bradycardia and Extreme Tachycardia alarm limits for adult, pediatric, and neonate patients, and the ranges in which you can set these limits:

#### Extreme Bradycardia Alarm Limits

| Patient Type | Default | Delta Value Range            |
|--------------|---------|------------------------------|
| Adult        | -10     | -5 to -30 in increments of 5 |
| Pediatric    | -10     | -5 to -30 in increments of 5 |
| Neonate      | -10     | -5 to -30 in increments of 5 |

#### Extreme Tachycardia Alarm Limits

| Patient Type | Default | Delta Value Range            |
|--------------|---------|------------------------------|
| Adult        | +20     | +5 to +30 in increments of 5 |
| Pediatric    | +20     | +5 to +30 in increments of 5 |
| Neonate      | +20     | +5 to +30 in increments of 5 |


### Acknowledging LTA Alarms

LTA alarms are latching, which means that the alarm indication will persist until both of the following conditions are met:

1. The alarm has been acknowledged by pressing the Alarm Silence/Reset button on the front panel of the unit.
2. The alarming condition no longer exists.

## Using the Heart Rate Parameter Control Panel

To configure alarms through the Heart Rate Parameter Control Panel, use the navigation keys to highlight and select the Heart Rate meter and display the HR/PR Parameter Control Panel:

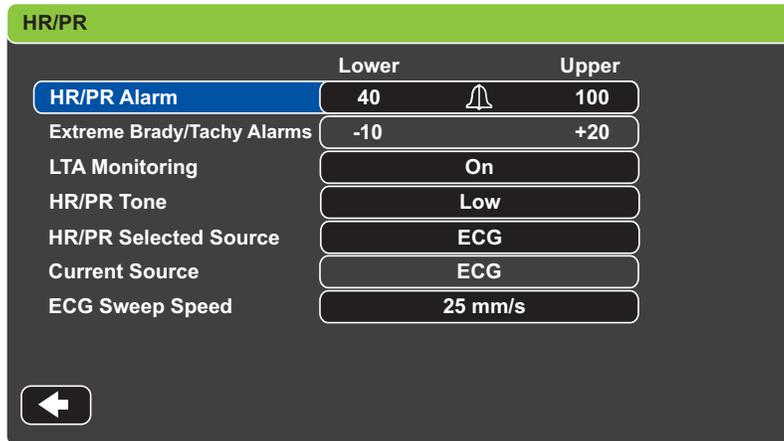

On the **HR/PR Parameter Control Panel**, you can enable/disable alarms and set high and low alarm limits (as described previously).

You can also configure these HR/PR options:

- **HR/PR Selected Source** -- sets the source of the HR/PR (ECG, IBP channel 1, SpO2, IBP channel 2, or IBP channel 3).
- **Current Source** -- shows the selected HR/PR source.
- **ECG Sweep Speed** -- sets the ECG sweep speed on the display (25 mm/s or 50 mm/s).



# Chapter 8

## Monitoring Non-Invasive Blood Pressure (NIBP)

---

The Propaq MD NIBP input is Type CF defibrillator proof.

This chapter describes how to use the Propaq MD unit to perform Non-Invasive Blood Pressure (NIBP) measurements using an inflatable cuff to measure arterial pressure.

The Propaq MD uses the enhanced NIBP features of Welch Allyn's SureBP and patented Smartcuf motion-tolerant technology.

The Propaq MD unit must perform ECG monitoring when using Smartcuf.

The SureBP monitoring software allows the Propaq MD unit to take an NIBP measurement as the cuff is inflating, which saves time -- the measurement takes about 15 seconds -- and improves patient comfort.

The Smartcuf monitoring software enables the Propaq MD unit to make accurate NIBP measurements in the presence of extreme artifact, weak pulses, and some dysrhythmias by synchronizing the NIBP measurements with the patient's R-wave.

- 
- Warning!**
- **Observe the patient's limb periodically to ensure that circulation is not impaired for a prolonged period of time.**
  - **Never use the Propaq MD to monitor NIBP on one patient while simultaneously monitoring ECG on another patient.**
  - **If a non-invasive blood pressure measurement is suspect, repeat the measurement. If you are still uncertain about the measurement, use another method.**

- **Do not use NIBP without proper training. Check that the correct patient mode has been selected to ensure that the initial inflation pressure is set correctly. Patient movement, very low pulse volume, or vibration from outside sources can influence the accuracy of blood pressure measurements.**
  - **Do not attempt to take NIBP measurements on patients during cardiopulmonary bypass procedures.**
  - **Some or all NIBP safety functions are disabled when performing the NIBP test in the Service menu. Do not conduct NIBP tests when the cuff is attached to a patient.**
- 

## How does NIBP Work?

The blood pressure cuff and hose connect to the Propaq MD unit through the NIBP connector on the side panel of the unit. The NIBP button on the front panel of the unit allows you to initiate and terminate blood pressure measurements, which are displayed in the NIBP area of the monitor.

The Propaq MD unit's NIBP module measures the oscillometric pulses transmitted through the blood pressure cuff and hose, and calculates the blood pressure measurements accordingly. The pressure measurement proceeds as follows:

1. The cuff inflates to a preconfigured pressure (determined by patient type) above the patient's systolic blood pressure. As the cuff inflates, the Propaq MD unit measures the oscillometric pulses transmitted from the cuff through the hose, and the SureBP monitoring software calculates the patient's systolic, diastolic, and mean blood pressure. Once these measurements are taken, the cuff deflates. The Propaq MD unit takes these measurements in about 15 seconds.

The SureBP feature requires the use of a cuff and dual lumen hose.


2. If high artifact prevents an accurate measurement during cuff inflation, or if you use a cuff and single lumen hose, the cuff inflates to its target pressure to occlude blood flow through the arteries in the monitored limb. The cuff deflates incrementally, allowing blood to flow through the cuff and into the monitored limb. As blood flows through the partially deflated cuff, it produces pressure oscillations that are transmitted to the Propaq MD unit through the hose. The Propaq MD unit measures the oscillometric pulses and uses them to calculate the corresponding systolic, diastolic, and mean blood pressure. This measurement takes about 30 seconds.

- The NIBP option automatically adjusts the blood pressure measurement procedure in response to certain error conditions such as:

| Condition   | Adjustment/Response   |
|---|---|
| The unit cannot detect systolic pressure                                      | The unit automatically increases the cuff inflation pressure and completes the blood pressure measurement |
| The unit cannot detect systolic, diastolic or mean pressure after 180 seconds | The unit aborts the blood pressure measurement and deflates the cuff                                      |
| The unit detects a fault  | The unit displays a corresponding error message on the monitor and aborts the measurement                 |

## The NIBP Numeric Display

When NIBP monitoring has been set up and the Propaq MD unit has begun taking NIBP measurements, the systolic, diastolic, and mean blood pressure measurements appear on the NIBP numeric display as follows:

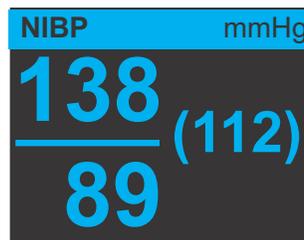

The following sections describe how to set up NIBP monitoring.

## NIBP Setup and Use

To take safe and accurate NIBP measurements using the Propaq MD unit, you must perform the following steps, each of which corresponds to a section in this chapter. Read each section carefully before you perform NIBP measurements.

- Select the correct size cuff.
- Apply the cuff to the patient.
- Connect the inflation hose to the Propaq MD unit and to the cuff.
- Configure NIBP alarms and settings (if the current NIBP alarms and settings are not appropriate).
- Press the NIBP button on the Propaq MD unit's front panel and take the blood pressure measurement.

## Selecting the NIBP Cuff

To take accurate measurements, you must use the proper sized cuff: the cuff's bladder length should be at least 80 percent of the limb circumference, while the cuff width should be equal to 40 percent of the limb circumference.

**Caution** Use only hoses and cuffs that are approved by ZOLL Medical Corporation. See Appendix B, *Accessories*, for a listing of the approved hoses and cuffs. Use the following guidelines when selecting the appropriate hose and cuff:

|                                       | <b>Adult Mode</b>                             | <b>Pediatric Mode</b>                            | <b>Neonate Mode</b>  |
|---------------------------------------|---|--|--|
| <b>Cuffs</b> (typical cuff labeling)  | Adult, Large Adult, Small Adult, Child, Thigh | Child, Small Child, Small Adult, Infant, Newborn | Neonate #1 to #5 -- disposable<br>Newborn (#6), Infant (#7) -- reusable                                    |
| <b>Recommended Limb Circumference</b> | 15 cm or greater.                             | 7.7 to 25 cm                                     | 15 cm or less  |
| <b>Hoses</b>                          | Adult   | Adult  | Neonate/Infant (Disposable Cuff only)<br>* Reusable Newborn (#6) and Infant (#7) cuffs use the Adult Hose. |

**Caution** Selection of the correct cuff is critical to the accuracy of NIBP measurements. Using a cuff that is too small results in measurements higher than the patient's actual blood pressure. Using a cuff that is too large results in measurements lower than the patient's actual blood pressure.

---

The Propaq MD unit uses the same definitions of Neonates, Pediatrics, and Adults as defined in the AAMI SP10:2002 standard:


|   |   |
|---|---|
| <b>Neonate or Newborn</b>                         | Children 28 days or less of age if born at term (37 weeks gestation or more); otherwise, up to 44 gestational weeks |
| <b>Pediatric or Child</b><br>(other than newborn) | Individuals between 29 days and 12 years of age   |
| <b>Adult</b>                                      | Individuals greater than 12 years of age  |

## Connecting the NIBP Cuff

Connecting the NIBP cuff requires you to attach the inflation hose to the Propaq MD unit and the NIBP cuff. To use the SureBP feature, which enables the Propaq MD unit to measure blood pressure on cuff inflation, you must use the FlexiPort cuff and dual lumen (two-tube) adaptor and hose. You can also use a single lumen (one-tube) hose and adaptor with the Propaq MD unit, but the unit will only measure blood pressure on cuff deflation.


To connect the NIBP cuff to the Propaq MD unit:

1. Attach the inflation hose's threaded metal connector to the NIBP connector on the side panel of the Propaq MD unit. Mesh the threads carefully so that the connector turns easily and then turn the connector in a clockwise direction until it is snug. Attach a double lumen hose as follows:

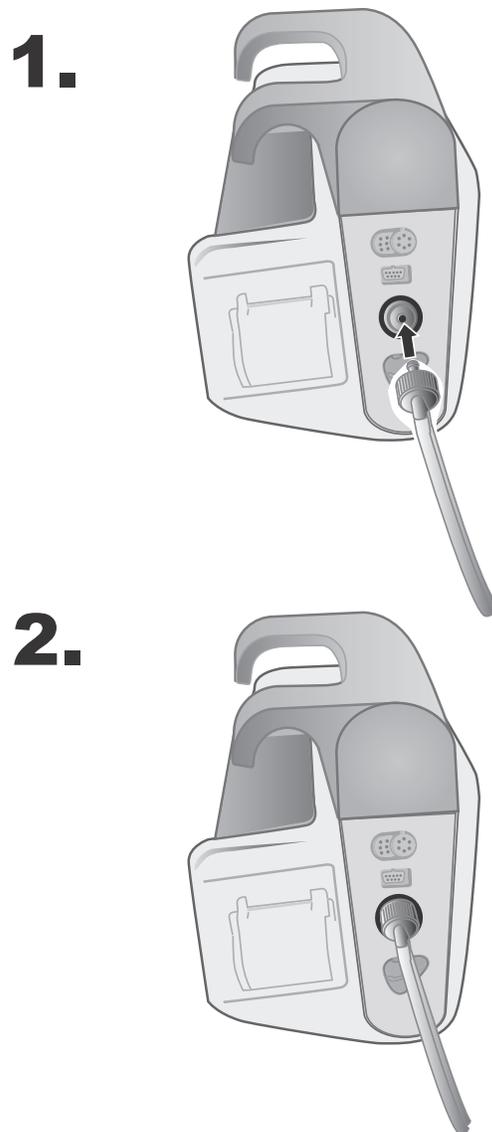



**Figure 8-1 Attaching a Dual Lumen Hose to the Propaq MD Unit**

Attach a single lumen hose as follows:

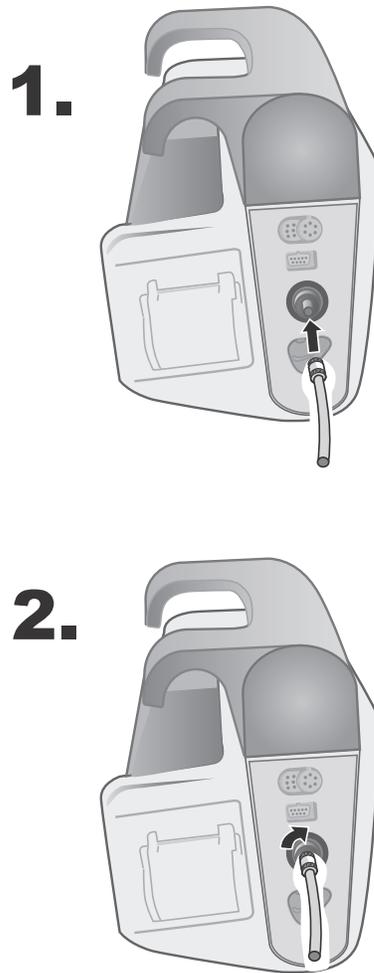

**Figure 8-2 Attaching Single Lumen Hose to the Propaq MD Unit**

2. Insert the plastic connectors on the NIBP hose into the cuff hose connector, and twist the connectors until they lock.

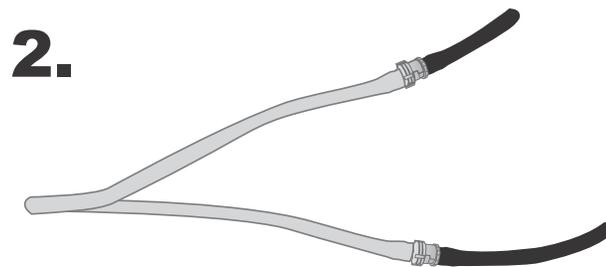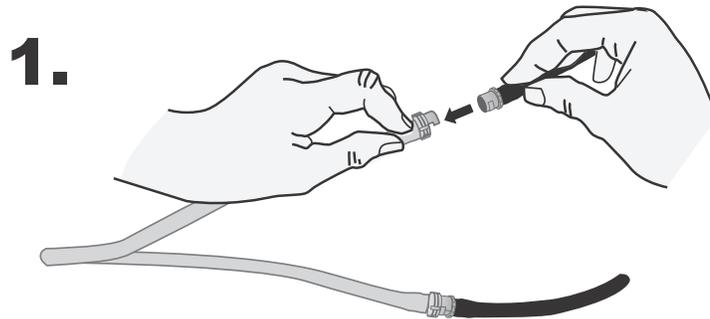

## Applying the Cuff to the Patient

To apply the cuff to the patient:

1. Ensure the patient is sitting or lying down with the limb relaxed, extended, and placed on a smooth surface for support.
2. Squeeze as much air from the cuff as possible before placing it on the patient.
3. Place the cuff 2 to 5 cm above the elbow crease or 5 to 10 cm (1.9 to 3.9 in.) above the knee crease.

---


**Warning!** Do not place the NIBP cuff on the same arm or leg as an SpO<sub>2</sub> sensor. Inflation of the cuff causes the SpO<sub>2</sub> monitor to read incorrectly. Also, do not attach the cuff to a limb being used for IV infusion. Cuff inflation might block the infusion, causing harm to the patient.

---

4. Adjust the cuff so that the artery marker on the cuff is over the artery, pointing to the hand or foot.
5. Check that the cuff ends between the range lines marked on the cuff.
6. If they do not line up, use a different size cuff.
7. Wrap the deflated cuff snugly around the limb without impeding blood flow.
8. Ensure that the hose is routed to avoid kinking or compression.

- Caution**
- Using a cuff that is loosely applied or too small results in measurements higher than the patient’s actual blood pressure.
  - Using a cuff that is too large results in values lower than the patient’s actual blood pressure.
  - Ideally, the cuff should be at the same level as the heart. Cuff placement substantially above or below heart level will result in blood pressure measurements that are erroneously low or high.

The following illustrates one possible cuff placement for adult/pediatric patients and, to the right, possible cuff placements for neonates:

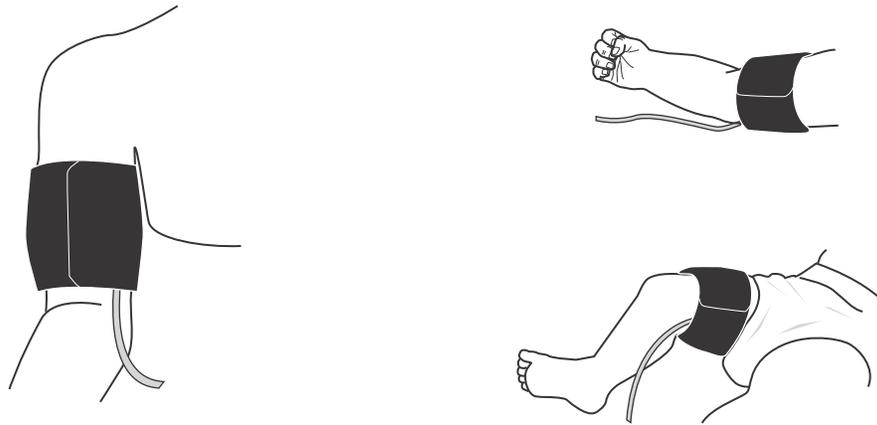

**Figure 8-3 Applying Cuff to the Patient**

## Ensuring Correct Cuff Inflation Settings

Before taking an NIBP measurement for a new patient, ensure that the cuff inflation settings are appropriate for that patient.

Check that the correct patient mode is selected. The initial Cuff Inflation Target (CIT) pressure is dependent on the patient type and the configured CIT preset. The default and configurable CIT presets for each patient type are indicated in the following table (default values are in bold).

**Note:** You can configure the CIT pressure from the Setup>NIBP menu.

| Adult           | Pediatric       | Neonate        |
|-----------------|-----------------|----------------|
| 120 mmHg        | 80 mmHg         | 60 mmHg        |
| 140 mmHg        | 90 mmHg         | 70 mmHg        |
| <b>160 mmHg</b> | 100 mmHg        | 80 mmHg        |
| 180 mmHg        | 110 mmHg        | <b>90 mmHg</b> |
| 200 mmHg        | <b>120 mmHg</b> | 100 mmHg       |
| 220 mmHg        | 130 mmHg        | 110 mmHg       |

| Adult    | Pediatric | Neonate  |
|----------|-----------|----------|
| 240 mmHg | 140 mmHg  | 120 mmHg |
| 260 mmHg | 150 mmHg  | 130 mmHg |

In order to accurately measure systolic pressure, the CIT pressure must be high enough to occlude the underlying artery. However, setting the CIT pressure too high may unnecessarily increase the reading determination time and patient discomfort.

After each NIBP measurement, the Propaq MD unit adjusts the cuff inflation pressure to optimize the next NIBP measurement.

**Note:** The maximum cuff inflation pressure for neonates is 153 mmHg.

---

**Warning!** Before using the Propaq MD unit to monitor a new patient, power down the unit for *at least 2 minutes* to reset all patient parameters and eliminate all adjustments made for the previous patient.

---

## Configuring NIBP Alarms and Settings

The last step in preparing to perform NIBP measurements is to ensure that the necessary alarms are enabled (or disabled), that alarm limits are appropriate, and that the NIBP settings are correct.

### Enabling/Disabling NIBP Alarms and Setting Alarm Limits

When enabled, the Propaq MD unit sounds alarms whenever measurements are outside set limits for the following:

- High and Low Systolic Pressure
- High and Low Diastolic Pressure
- High and Low Mean Arterial Pressure (MAP)

To enable (or disable) NIBP alarms and set upper and lower alarm limits, you can do so through the Alarms quick access key () or the **NIBP Parameter Control Panel**.

To configure NIBP alarm limits through the Alarms quick access key:

1. Press the More quick access key ()
2. Press the Alarms quick access key ()
3. Press the **Limits** quick access key ()
4. Use the navigation keys to highlight and select the appropriate alarm menu selection. For NIBP, the alarm menu selections are **NIBP Systolic Alarm**, **NIBP Diastolic Alarm**, and **NIBP MAP Alarm**.

5. On the selected NIBP alarm settings menu, use the navigation keys to select the fields that you want to change. The fields are
  - **Upper Limit Enable**
  - **Lower Limit Enable**
  - **Upper Limit**
  - **Lower Limit**



6. When you are finished changing values on the alarm settings menu, select the back arrow to exit the menu.

### Setting Upper and Lower Systolic Alarm Limits

Initially, the **NIBP Systolic Alarm Settings** menu specifies that the NIBP systolic pressure alarms are enabled (**ON**) or disabled (**OFF**), and displays the default upper and lower systolic limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default NIBP Systolic limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | NIBP Systolic Limit Default       | NIBP Systolic Limit Range                |
|--------------|-----------------------------------|--|
| Adult        | Lower: 75 mmHg<br>Upper: 220 mmHg | Lower: 30-258 mmHg<br>Upper: 32-260 mmHg |
| Pediatric    | Lower: 75 mmHg<br>Upper: 145 mmHg | Lower: 30-158 mmHg<br>Upper: 32-160 mmHg |
| Neonate      | Lower: 50 mmHg<br>Upper: 100 mmHg | Lower: 20-118 mmHg<br>Upper: 22-120 mmHg |

### Setting Upper and Lower Diastolic Alarm Limits

Initially, the **NIBP Diastolic Alarm Settings** menu specifies that the NIBP diastolic pressure alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower diastolic limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default diastolic limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | Diastolic Limit Default           | Diastolic Limit Range                    |
|--------------|-----------------------------------|--|
| Adult        | Lower: 35 mmHg<br>Upper: 110 mmHg | Lower: 20-218 mmHg<br>Upper: 22-220 mmHg |
| Pediatric    | Lower: 35 mmHg<br>Upper: 100 mmHg | Lower: 20-128 mmHg<br>Upper: 22-130 mmHg |
| Neonate      | Lower: 30 mmHg<br>Upper: 70 mmHg  | Lower: 10-108 mmHg<br>Upper: 12-110 mmHg |

### Setting Upper and Lower NIBP MAP Alarm Limits

Initially, the **NIBP MAP Alarm Settings** menu specifies that **NIBP MAP** alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower MAP limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default MAP alarm limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | MAP Default                       | MAP Range                                |
|--------------|-----------------------------------|--|
| Adult        | Lower: 50 mmHg<br>Upper: 120 mmHG | Lower: 25-230 mmHg<br>Upper: 23-228 mmHg |
| Pediatric    | Lower: 50 mmHg<br>Upper: 110 mmHg | Lower: 25-140 mmHg<br>Upper: 23-138 mmHg |
| Neonate      | Lower: 35 mmHg<br>Upper: 80 mmHg  | Lower: 15-110 mmHg<br>Upper: 13-108 mmHg |

### Using the NIBP Parameter Control Panel

Use the navigation keys to highlight and select the NIBP numeric display to display the **NIBP Parameter Control Panel**:

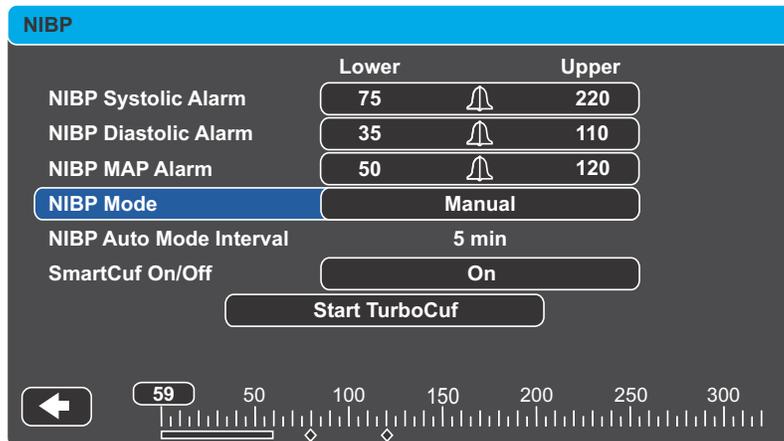


Figure 8-4 NIBP Parameter Control Menu

On the NIBP Parameter Control Panel, you can select an alarm configuration option (**SYS**, **DIA**, or **MAP**) to enable/disable alarms and set high and low alarm limits (as described previously).

You can also configure these NIBP options:

- NIBP Mode (**Mode**)
- NIBP Automatic Measurement Interval (**Auto Interval**)
- Enable/Disable Smartcuf motion artifact filter (**Smartcuf**)
- Start/Stop Turbocuf Measurements (**Turbocuf**)

### Specify NIBP Mode

You can specify that the Propaq MD unit operate in either *Manual* or *Automatic Mode*.

In Manual Mode, the Propaq MD unit takes a single NIBP measurement when you press the NIBP button on the front panel (☞). To repeat the NIBP measurement, you must press the NIBP key again (select **Manual**).

In Automatic Mode, the Propaq MD unit takes the first of a series of NIBP measurements when the **Auto Interval** timer expires, and then repeats the NIBP measurement at this specified interval.

**Note:** You can press the NIBP button and manually initiate an NIBP reading at any time while in Automatic Mode. Manually initiated NIBP readings will not affect the timing of subsequent NIBP readings in Automatic Mode.

### Specify the Automatic NIBP Measurement Interval

You can specify the time interval between NIBP measurements in Automatic Mode. The default interval between measurements is 5 minutes.

You can specify intervals of **1, 2, 3, 5, 10, 15, 30, and 60** minutes between NIBP measurements.

### Enable/Disable Smartcuf Motion Artifact Filter


Enabling the use of the Smartcuf motion artifact filter increases the accuracy of NIBP measurements when motion artifact or weak pulses are present.

NIBP measurements can be adversely affected by many factors, such as cardiac arrhythmias, sudden changes in blood pressure, body motions such as shivering and convulsions, bumping the cuff, vibration, vehicle motion, or weak pulses. The Smartcuf filter synchronizes the NIBP reading with the R-wave of the patient's ECG to eliminate noise created by patient motion or vibration.

**Note:** The Propaq MD unit must perform ECG monitoring when using Smartcuf.

If artifact is severe, a special symbol appears on the display and printouts:

**Figure 8-5 High Artifact Symbol**

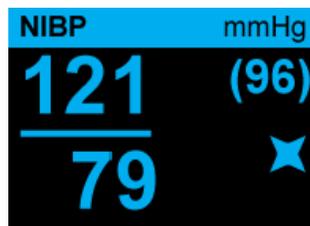

By default, Smartcuf is **Enabled**.

There are situations where you may choose to disable Smartcuf. These may include

- Very extreme motion artifact
- Certain types of arrhythmias
- Situations that prevent getting a ECG signal.

NIBP measurements can still be performed with Smartcuf disabled.

### **Start/Stop TurboCuf**

Selecting **Start TurboCuf** starts Short-term Automatic (STAT) NIBP measurements. The Propaq MD unit begins its first NIBP measurement, after which it continues to perform as many NIBP measurements as possible over a 5-minute period.

Select **Stop TurboCuf** to immediately stop STAT measurements.

---

**WARNING!** **Repeated use of STAT measurements on the same patient over a short time interval can affect blood pressure readings, limit circulation to the limb, and cause injury to the patient.**

---

### **Setting the NIBP Display Format**

You can specify the appearance of the NIBP numeric display format. The display format allows you to choose whether or not you will display MAP measurement (**M**) with the Systolic (**S**) and Diastolic (**D**) measurements in the in one of the following formats:

- **S/D**
- **S/D (M)** (default display)
- **(M) S/D**

**Note:** If a MAP alarm is active when the **S/D** format is selected, the display format will be **S/D (M)** until you clear the MAP alarm.

## NIBP System Messages

When monitoring NIBP, the Propaq MD unit may display the following messages:

| <b>System Message</b>             | <b>Cause</b>  |
|-----------------------------------|---|
| <i>READING IN PROGRESS</i>        | The unit is taking an NIBP measurement and functioning normally.  |
| <i>READING STOPPED</i>            | The unit has stopped an NIBP measurement, because the operator has pressed the NIBP button and cancelled the measurement.   |
| <i>READING FAILED</i>             | The patient's pulse is too weak to for NIBP measurement, or the cuff requires adjustment.   |
| <i>ARTIFACT</i>                   | Excessive artifact is preventing the NIBP measurement.  |
| <i>NEONATE ALERT</i>              | The unit has detected a neonate cuff when in Adult mode. Replace cuff or correct patient mode, as appropriate.  |
| <i>AIR LEAK</i>                   | A major air leak is preventing cuff inflation. Check hose and cuff connections, replace a defective hose or cuff, as necessary, and reattempt NIBP measurement.   |
| <i>KINKED HOSE</i>                | An air obstruction is preventing the correct operation of the unit. Check hose for kinks. Eliminate kinks or replace hose, if necessary.  |
| <i>NIBP DISABLED</i>              | A system error has occurred, and the Propaq MD unit should be serviced.   |
| <i>NIBP CALIBRATION CHECK DUE</i> | The annual calibration check is due. You can continue to use the device, however, we recommend that you contact the ZOLL Technical Service Department or a ZOLL authorized service representative as soon as possible to perform the recommended annual NIBP calibration check. |



# Chapter 9

## Monitoring Pulse Oximetry (SpO<sub>2</sub>)

---

The Propaq MD SpO<sub>2</sub> input is Type CF defibrillator proof.

This chapter describes how to use the Propaq MD unit to monitor Pulse Oximetry (SpO<sub>2</sub>).

The Propaq MD pulse oximeter continuously and noninvasively measures the oxygen saturation (SpO<sub>2</sub>) of arterial blood at a peripheral site, such as the foot, toe, or finger. It is used to monitor patients at risk of developing hypoxemia. This option is widely used because it is noninvasive, continuous, easily applied, and painless.

The oximeter sensor contains two emitters that transmit red and infrared light through the monitored site. This light passes through the patient's tissues and is received by a photodetector in the sensor. Because oxygen-saturated blood absorbs light differently than unsaturated blood, the amount of red and infrared light absorbed by blood flowing through the monitored site can be used to calculate the ratio of oxygenated hemoglobin to total hemoglobin in arterial blood. The monitor displays this ratio as percent SpO<sub>2</sub>. Normal values typically range from 95% to 100% at sea level.

The quality of SpO<sub>2</sub> measurements depends on the correct size and application of the oximetry sensor, adequate blood flow through the sensor site, and the sensor's exposure to ambient light. For correct placement and location of the sensors, refer to the *Directions for Use* contained in all oximetry sensor packages.

## Warnings -- SpO<sub>2</sub>, General

- SpO<sub>2</sub> style pulse oximeter measurements may be affected by numerous factors such as bright lights, improperly applied probes, use of non-ZOLL approved sensors, patient conditions/movements. The clinician should use good clinical judgment when interpreting SpO<sub>2</sub> measurements. Should the clinician question an SpO<sub>2</sub> measurement, an arterial blood gas oxygen saturation measurement should be obtained.
- Interfering substances: Carboxyhemoglobin and methemoglobin can erroneously alter SpO<sub>2</sub> readings. The level of change is approximately equal to the amount of carboxyhemoglobin or methemoglobin present. Dyes or any substance containing dyes that alter arterial pigmentation might cause erroneous readings.
- Do not use the Propaq MD pulse oximeter or oximeter sensors during magnetic resonance imaging (MRI). Induced current could cause burns. The pulse oximeter might affect the MRI image and the MRI unit might interfere with the accuracy of oximetry measurements.
- Carefully arrange patient cabling to reduce the possibility of patient entanglement or strangulation.
- Inaccurate measurements may be caused by venous pulsations.
- The pulse oximeter can be used during defibrillation, but the readings may be inaccurate for a short time.

## Warnings -- SpO<sub>2</sub>, Oximeter Sensor

- Use only ZOLL-approved Nellcor oximeter sensors. Other manufacturers' sensors might not perform properly with the Propaq MD oximeter.
- Tissue damage can result from incorrect application or use of a sensor (for example, wrapping the sensor too tightly, applying supplemental tape, failing to periodically inspect the sensor site). To ensure skin integrity, correct positioning, and sensor adhesion, inspect the sensor site as directed in the *Directions for Use* provided with the sensor.
- Do not use damaged sensors or cables.
- Do not use a sensor with exposed optical components.
- Do not sterilize a sensor by irradiation, steam, or ethylene oxide. Refer to the cleaning instructions in the *Directions for Use* for reusable Nellcor sensors.
- Do not allow the sensor to remain on the same site for a prolonged period, especially when monitoring neonates. Check the application site at regular intervals (at least every 2 hours) and change the site if any compromise in skin quality occurs.
- Do not attach the oximeter sensor to a limb being monitored with a pressure cuff or with restricted blood flow.
- A poorly applied sensor might give incorrect saturation readings. A weak pulse signal on the display might indicate a poorly applied sensor or a poorly chosen monitoring site.
- Choose a site with sufficient perfusion to ensure accurate oximetry values.
- Certain nail aberrations, nail polish, fungus, and so on might cause inaccurate oximetry readings. Remove any nail polish or move the sensor to an unaffected digit.
- Exposure to high ambient light from surgical lamps (especially those with a xenon light source), bilirubin lamps, fluorescent lights, infrared heating lamps, or direct sunlight can affect the accuracy of oximetry readings. To prevent interference from ambient light, ensure that the sensor is properly applied. If necessary, cover the sensor with opaque material.

## SpO<sub>2</sub> Setup and Use

To take accurate SpO<sub>2</sub> measurements using the Propaq MD unit, you must perform the following steps, each of which corresponds to a section in this chapter.

1. Select the correct SpO<sub>2</sub> sensor.
2. Apply the SpO<sub>2</sub> sensor to the patient.
3. Connect the sensor to the Propaq MD unit.
4. Configure SpO<sub>2</sub> alarms and settings (if the current SpO<sub>2</sub> alarms and settings are not appropriate).

Pulse oximetry measurements begin as soon as the sensor is applied to the patient and connected to the Propaq MD unit.

### Selecting the SpO<sub>2</sub> Sensor

When selecting the SpO<sub>2</sub> sensor, consider the patient's weight, the adequacy of perfusion, the available sensor sites, and the anticipated duration of monitoring. For more information, refer to the *Accessories* section of this chapter, which provides a list of ZOLL-approved reusable and single-use SpO<sub>2</sub> sensors for adult, pediatric, and neonate patients. Before applying the sensor, always familiarize yourself with the *Directions for Use* that the manufacturer provides with the sensor.

### Applying the SpO<sub>2</sub> Sensor

Choose a site that is well perfused and restricts a conscious patient's movements the least. The ring finger or middle finger of the nondominant hand is preferred.

Alternatively, you can use the other digits on the nondominant hand. Be sure the sensor's detector is fully covered by flesh. You can use the great toe or long toe (next to the great toe) on restrained patients or patients whose hands are unavailable.

To prevent interference from ambient light, ensure that the sensor is properly applied, and cover the sensor site with opaque material, if required. Failure to take this precaution in high ambient light conditions may result in inaccurate measurements.

Do not select an SpO<sub>2</sub> sensor site on the same arm/leg as an NIBP cuff. Inflation of the cuff will cause the SpO<sub>2</sub> values to read incorrectly.



## Applying a Reusable SpO<sub>2</sub> Sensor

The following instructions describe how to apply a reusable Nellcor DS-100A SpO<sub>2</sub> sensor. For all other reusable sensors, refer to the sensor packaging and the manufacturer's

*Directions for Use* for instructions on how to apply the sensor.

**Note:** A reusable sensor is not intended for use on the thumb or across a child's hand or foot. After selecting a monitoring site, apply the reusable DS-100A sensor as follows:

1. Place the selected digit over the sensor window, making sure that the sensor cable runs over the top of the patient's hand.
2. On finger sites, make sure the tip of the finger touches the raised digit stop inside the sensor. If the fingernail is long, it may extend over and past the finger stop.
3. Check the sensor position to ensure that the top and bottom halves of the sensor are parallel. To ensure accurate data, you must have complete coverage of the detector window (see previous figure).

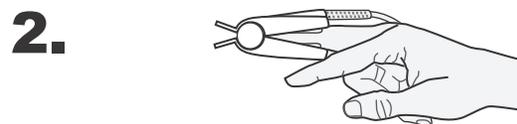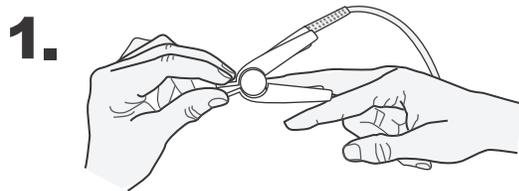


**Note:** With smaller digits, the digit may not need to be pushed all the way to the stop to completely cover the detector window.

## Applying a Single-Use Sensor


When applying a single-use SpO<sub>2</sub> sensor, be careful not to wrap the adhesive too tightly as this can cause venous pulsations that could lead to inaccurate saturation measurements.

1. Remove plastic backing from the sensor and locate transparent windows on the adhesive side. The transparent windows cover the optical components. An index finger is the preferred sensor location. Alternatively, apply the sensor to a small thumb, smaller finger, or great toe.

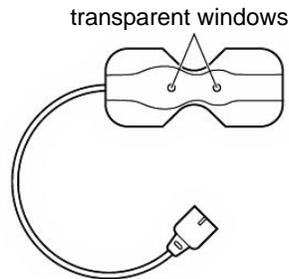


**Note:** When selecting a sensor site, priority should be given to an extremity free of an arterial catheter, blood pressure cuff, or intravascular infusion line.

2. Orient the sensor so the dashed line in the middle of the sensor is centered on the tip of the digit. Wrap adhesive flaps on non-cable end around the digit. Note that the cable must be positioned on the top of the hand.

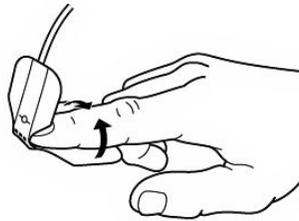

3. Fold cable end over top of digit so that windows are directly opposite each other. Wrap adhesive securely around sides of digit.

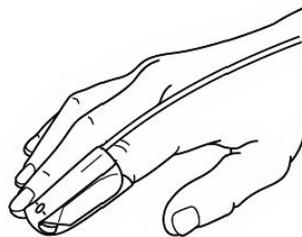

**Note:** If the sensor does not track the pulse reliably, it may be incorrectly positioned or the sensor site may be too thick, thin, or deeply pigmented, or otherwise deeply colored (for example, as a result of externally applied coloring such as nail polish, dye, or pigmented cream) to permit appropriate light transmission. If any of these situations occurs, reposition the sensor or choose an alternate sensor for use on a different site.

## Connecting the SpO<sub>2</sub> Sensor


To connect the SpO<sub>2</sub> sensor to the Propaq MD unit:

1. When using a sensor extension cable, inspect the cable before use. Replace the cable if it shows any signs of wear, breakage, or fraying. Plug the sensor extension cable into the SpO<sub>2</sub> socket on the side of the Propaq MD unit:




**Figure 9-1 Connecting the SpO<sub>2</sub> Sensor to the Propaq MD Unit**

2. Lift the clear plastic protective cover from the female end of the extension cable, then plug the sensor cable's male connector into the extension cable connector -- make sure that the plug is all the way into the connector:

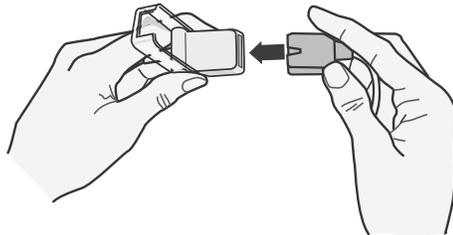

**Figure 9-2 Plug the Sensor Connectors Together**

3. Lower the clear plastic cover over the connection to secure it:

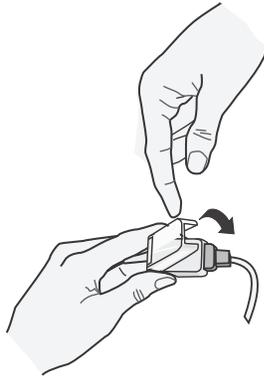

**Figure 9-3 Lower Plastic Cover Over Cable Connection**

## Displaying SpO<sub>2</sub> Measurements

When the connection is made between the SpO<sub>2</sub> sensor and the unit, the message, *INITIALIZING*, appears in the SpO<sub>2</sub> display. After a brief delay, the unit displays the SpO<sub>2</sub> measurement. If the message, *SENSOR FAILURE*, appears, the sensor is either incompatible with the Propaq MD unit, or it is not working, and you will need to replace the sensor.

A blip bar appears on the right side of the SpO<sub>2</sub> numeric display window. The blip bar indicates the relative pulsatile strength and quality of the SpO<sub>2</sub> data.

## Enabling/Disabling SpO<sub>2</sub> Alarms and Setting Alarm Limits

When enabled, the Propaq MD unit sounds alarms whenever measurements are outside set limits for the High and Low SpO<sub>2</sub> values.

You can enable (or disable) SpO<sub>2</sub> alarms and set Upper and Lower alarm limits through the Alarms quick access key (🔔) or through the SpO<sub>2</sub> Parameter Control Panel.

To configure SpO<sub>2</sub> alarms through the Alarms quick access key:

1. Press the **More** quick access key (⏮).
2. Press 🔔.
3. Press the **Limits** quick access key (Limits).
4. Use the navigation buttons to highlight and select the appropriate alarm menu selection. For NIBP, the alarm menu selection is SpO<sub>2</sub> Alarm.
5. On the selected SpO<sub>2</sub> Alarm Settings menu, use the navigation keys to select the fields that you want to change. The fields are
  - **Upper Limit Enable**
  - **Lower Limit Enable**
  - **Upper Limit**
  - **Lower Limit**



- When you are finished changing values on the alarm menu, navigate to the Backarrow key to exit the menu.

## Setting Upper and Lower Systolic Alarm Limits

Initially, the SpO<sub>2</sub> Alarm Settings menu specifies whether the SpO<sub>2</sub> alarms are enabled (**ON**) or disabled (**OFF**), and displays the default upper and lower systolic limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default SpO<sub>2</sub> limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits.

| Patient Type | SpO <sub>2</sub> Limit Default | SpO <sub>2</sub> Limit Range        |
|--------------|--------------------------------|-------------------------------------|
| Adult        | Lower: 85%<br>Upper: 100%      | Lower: 50 - 98%<br>Upper: 52 - 100% |
| Pediatric    | Lower: 85%<br>Upper: 100%      | Lower: 50 - 98%<br>Upper: 52 - 100% |
| Neonate      | Lower: 85%<br>Upper: 95%       | Lower: 50 - 98%<br>Upper: 52 - 100% |

## Using the SpO<sub>2</sub> Parameter Control Panel

Use the navigation keys to highlight and select the SpO<sub>2</sub> numeric display to display the SpO<sub>2</sub> Parameter Control Panel, on which you can select options that optimize the use of SpO<sub>2</sub> for the patient:



Figure 9-4 SpO<sub>2</sub> Parameter Control Menu

## Selecting the Heart Rate/ Pulse Rate (HR/PR) Tone

The unit allows you enable or disable the tone that the monitor uses to indicate detection of the patient's pulse: **On** or **Off** (no tone sounds). The default tone is **Off**.

## SpO<sub>2</sub> System Messages

The Propaq MD unit may display the following system messages when monitoring SpO<sub>2</sub>:

| System Message        | Cause  |
|-----------------------|--|
| <i>INITIALIZING</i>   | The SpO <sub>2</sub> pulse oximeter is initializing.   |
| <i>SEARCHING</i>      | The unit is searching for a pulse.   |
| <i>CHECK SENSOR</i>   | The SpO <sub>2</sub> sensor has become disconnected from the unit, or the sensor is no longer on the patient. Check sensor and then reconnect it to the unit or reapply it to the patient. |
| <i>SENSOR FAILURE</i> | The SpO <sub>2</sub> sensor is defective. Replace the sensor.  |
| <i>LOW PERFUSION</i>  | Perfusion has dropped below 20%.   |
| <i>SPO2 DISABLED</i>  | A system error has occurred. The Propaq MD unit cannot take SpO <sub>2</sub> measurements and should be serviced.  |

## Functional Testers and Patient Simulators

Some models of commercially available bench top functional testers and patient simulators can be used to verify the proper functionality of Nellcor pulse oximeter sensors, cables and monitors. See the individual testing device's operator's manual for the procedures specific to the model of tester that you are using.

While such devices may be useful for verifying that the pulse oximeter sensor, cable, and monitor are functional, they are incapable of providing the data required to properly evaluate the accuracy of a system's SpO<sub>2</sub> measurements.

Fully evaluating the accuracy of the SpO<sub>2</sub> measurements requires, at a minimum, accommodating the wavelength characteristics of the sensor and reproducing the complex optical interaction of the sensor and the patient's tissue. These capabilities are beyond the scope of known bench top testers, including known devices which claim to measure sensor LED wavelength.

SpO<sub>2</sub> measurement accuracy can only be evaluated in vivo by comparing pulse oximeter readings with values traceable to SpO<sub>2</sub> measurements obtained from simultaneously sampled arterial blood using a laboratory CO-oximeter.

Many functional testers and patient simulators have been designed to interface with the pulse oximeter's expected calibration curves and may be suitable for use with Nellcor monitors and/or sensors. Not all such devices, however, are adapted for use with the Nellcor OxiMax™ digital calibration system. While this will not affect use of the simulator for verifying system functionality, displayed SpO<sub>2</sub> measurement values may differ from the setting of the test device.

For a properly functioning monitor, this difference will be reproducible over time and from monitor to monitor within the performance specifications of the test device.

# Chapter 10

## Monitoring CO<sub>2</sub>

---

This chapter describes how to use the Propaq MD unit to monitor End Tidal Carbon Dioxide (EtCO<sub>2</sub>), breath rate, and Fractional Inspired Carbon Dioxide (FiCO<sub>2</sub>). These options use the same connector on the Propaq MD unit and may be used interchangeably.

### Overview

The Propaq MD unit uses the Oridion Microstream FilterLine<sup>®</sup> and Smart CapnoLine<sup>®</sup> sidestream systems to monitor CO<sub>2</sub>.

The FilterLine and Smart CapnoLine sidestream systems draw small samples of gas from the patient's airway via a nasal/oral cannula or airway adapter, and pass these gases through a solid state infrared sensor (located away from the patient's airway) that measures CO<sub>2</sub>. The Microstream system can be used for CO<sub>2</sub> measurements on intubated and non-intubated infant, pediatric, and adult patients.

The Microstream CO<sub>2</sub> sensor generates infrared light, and beams it through the sampled breathing gases and determines CO<sub>2</sub> concentration by measuring the amount of light absorbed by gases.

The Propaq MD unit displays EtCO<sub>2</sub> (the concentration of carbon dioxide detected at the end of each exhalation) and FiCO as a numerical value in millimeters of mercury (mmHg). In addition, the unit can display a capnogram. The capnogram is a valuable clinical tool that can be used to assess patient airway integrity and proper endotracheal (ET) tube placement. The unit calculates respiration rate by measuring the time interval between detected peaks of the CO<sub>2</sub> waveform. The technology differentiates between waveforms caused by breathing and those caused by cardiogenic oscillations and artifact.

The Propaq MD is equipped with automatic barometric pressure compensation.

- Warning!**
- **When using a sampling line for intubated patients with a closed suction system, do not place the airway adapter between the suction catheter and endotracheal tube. This is to ensure that the airway adapter does not interfere with the functioning of the suction catheter.**
  - **Do not cut or remove any part of the sample line. Cutting the sample line could lead to erroneous readings**
  - **If too much moisture enters the sampling line (i.e., from ambient humidity or breathing of unusually humid air), the message *Clearing FilterLine* will appear in the message area. If the sampling line cannot be cleared, the message *FilterLine Blockage* will appear in the message area. Replace the sampling line once the *FilterLine Blockage* message appears.**
  - **Do NOT use the Microstream sidestream CO<sub>2</sub> accessories in the presence of flammable anesthetics or other flammable gases.**
  - **Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.**
  - **Always ensure the integrity of the patient breathing circuit after insertion of the airway adapter by verifying a proper CO<sub>2</sub> waveform (capnogram) on the monitor display.**
- 

**Caution** Microstream<sup>®</sup> EtCO<sub>2</sub> sampling lines are designed for single patient use, and are not to be reprocessed. Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Before use, carefully read the Microstream EtCO<sub>2</sub> sampling line's *Directions for Use*.

---

## CO<sub>2</sub> Monitoring Setup and Use

Follow these steps to set up CO<sub>2</sub> monitoring:

1. Select the correct CO<sub>2</sub> sampling line for the patient.
2. Attach the sampling line to the unit's CO<sub>2</sub> inlet port.
3. Apply the Filterline airway adaptor or Smart CapnoLine Nasal or Nasal/Oral cannula to the patient.
4. Check that the Propaq MD unit is set up for the correct patient type -- Adult, Pediatric, or Neonate.
5. Configure alarms (if the current alarm settings are not appropriate) and other CO<sub>2</sub> features.
6. Press the CO<sub>2</sub> quick access key () to initiate CO<sub>2</sub> monitoring.

## Selecting the CO<sub>2</sub> Sampling Line

To select the correct CO<sub>2</sub> sampling line, you must determine the following:

- Is the patient adult, pediatric, or neonate?
- Is the patient intubated (ventilated) or non-intubated (non-ventilated)?

You can use the following Oridion Microstream accessories for sidestream CO<sub>2</sub> monitoring with the Propaq MD unit:

**Table 7-1. Oridion Microstream CO<sub>2</sub> Sampling Lines for use with Propaq MD units.**

| Accessory   | Type   | Part Number  |
|---|--|--------------|
| FilterLine Set (Adult/Pediatric), box of 25                     | Intubated sampling line and airway adapter for short term monitoring | 8300-0520-01 |
| FilterLine H Set (Adult/Pediatric), box of 25                   | Intubated sampling line and airway adapter for humid environments    | 8300-0521-01 |
| FilterLine H Set (Infant/Neonate), box of 25                    | Intubated sampling line and airway adapter for humid environments    | 8300-0522-01 |
| VitaLine H Set (Adult/Pediatric), box of 25                     | Intubated sampling line and airway adapter for high ambient humidity | 8300-0523-01 |
| Smart Capnoline Plus with O <sub>2</sub> (Adult), box of 25     | Non-Intubated Oral/ Nasal sampling with O <sub>2</sub> delivery      | 8300-0524-01 |
| Smart Capnoline Plus with O <sub>2</sub> (Pediatric), box of 25 | Non-Intubated Dual Purpose   | 8300-0525-01 |

---


**Warning!** Use only CO<sub>2</sub> Microstream accessories authorized by ZOLL Medical Corporation. ZOLL Medical Corporation does not guarantee the use of non-authorized accessories.

---

## Connecting the CO<sub>2</sub> Sampling Lines

To connect the FilterLine or Smart CapnoLine:

1. Slide open the Propaq MD unit's CO<sub>2</sub> inlet port cover.
2. Put the fitting at the end of the sidestream tubing over the CO<sub>2</sub> inlet port connector.
3. Turn the fitting clockwise to tighten.

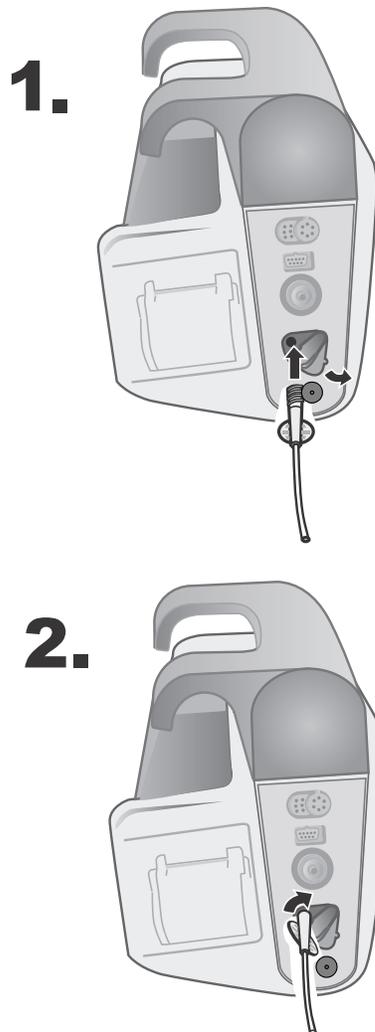

**Figure 10-1** Connecting the Sampling Line Connector to the CO<sub>2</sub> Inlet Port

- 
- WARNING!**
- The exhaust port for the Microstream sidestream CO<sub>2</sub> apparatus is an output for only expired gases from the patient and any connected breathing apparatus. The exhaust port is intended only for connection to gas collection equipment, such as gas scavenger devices -- *there should be no other connections to the exhaust port.*
  - When connecting the Microstream sidestream CO<sub>2</sub> accessory to patients who are receiving or have recently received anesthetics, connect the CO<sub>2</sub> exhaust port to a scavenging system, or to the patient's anesthetic machine or ventilator to prevent exposing medical staff to anesthetics.
  - Do not lift the monitor by the FilterLine, as it could disconnect from the monitor, causing the monitor to fall on the patient.
  - The FilterLine may ignite in the presence of high O<sub>2</sub> concentrations when directly exposed to laser or ESU devices. Use caution when performing these procedures.
- 

**Note:** If you use a gas scavenging system, ensure that it is installed according to the manufacturer's instructions. The gas scavenging system should comply with ISO 8835-3:1997 (E).

**Note:** In order to avoid moisture buildup and sampling line occlusion during nebulization or suction for intubated patients, remove the sampling line luer connector from the monitor.

## Applying a FilterLine Set

The FilterLine set is intended for the CO<sub>2</sub> monitoring of intubated patients.


Before attaching the airway adapter to the breathing circuit, verify that the adapter is clean, dry, and undamaged. Replace if necessary.

---

**Caution** The disposable FilterLine set is intended for single patient use. Do NOT reuse or sterilize any part of the FilterLine Set, as the monitor may be damaged by cleaning or reuse.

---

1. Place the airway adapter at the proximal end of the airway circuit between the elbow and the ventilator circuit wye. Do NOT place the airway adapter between the ET tube and the elbow as this may allow patient secretions to accumulate in the adapter.
2. If pooling does occur, the airway adapter may be removed from the circuit, rinsed with water and reinserted into the circuit. To prevent moisture from draining into the sample tubing, ensure that the sampling tube exits from the top of the airway adapter, not its bottom or sides. See the following figure.

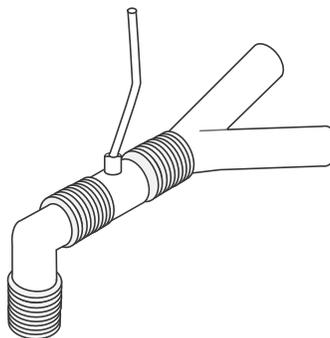

## Applying a Smart CapnoLine Nasal or Nasal/Oral Cannula

The nasal and nasal/oral cannulas are intended for monitoring CO<sub>2</sub> in non-intubated patients.

Oral/nasal sampling cannulas are especially valuable for patients who are prone to mouth breathing, since most (if not all) of the CO<sub>2</sub> is exhaled through the mouth. If a standard nasal CO<sub>2</sub> sampling cannula is used on such patients, the EtCO<sub>2</sub> values and capnogram displayed will be substantially lower than the actual CO<sub>2</sub> levels present in the patient's expired breath.

- 
- WARNING!**
- **The disposable Smart CapnoLine nasal and nasal/oral cannula sets are intended for single patient use. Do NOT reuse or sterilize any part of this product, as the monitor may be damaged by reuse of the sampling line.**
  - **If oxygen is being delivered while using sidestream CO<sub>2</sub>, be sure to use a CO<sub>2</sub> sampling with O<sub>2</sub> delivery. Using a different type of cannula will not enable oxygen delivery.**
- 

Remove the cannula from the package. Verify that the cannula is clean, dry, and undamaged. Replace if necessary.

### Placing the Cannula onto the Patient

Place the oral/nasal cannula onto the patient as follows:



---

**Caution** Dispose of Microstream EtCO<sub>2</sub> consumables according to standard operating procedures or local regulations for the disposal of contaminated medical waste.


---

## Measuring CO<sub>2</sub>

Once setup is complete, press  to begin monitoring CO<sub>2</sub>. The numeric CO<sub>2</sub> display appears on the screen and displays the message, *INITIALIZING*. The CO<sub>2</sub> display gives the current EtCO<sub>2</sub> value, and after a delay of approximately 1 minute, the patient's Respiration Rate (in Breaths/Minute), identified as **BR**:



Check that connections have been made correctly by verifying the display a proper capnogram (the waveform is inserted automatically on the waveform display window).

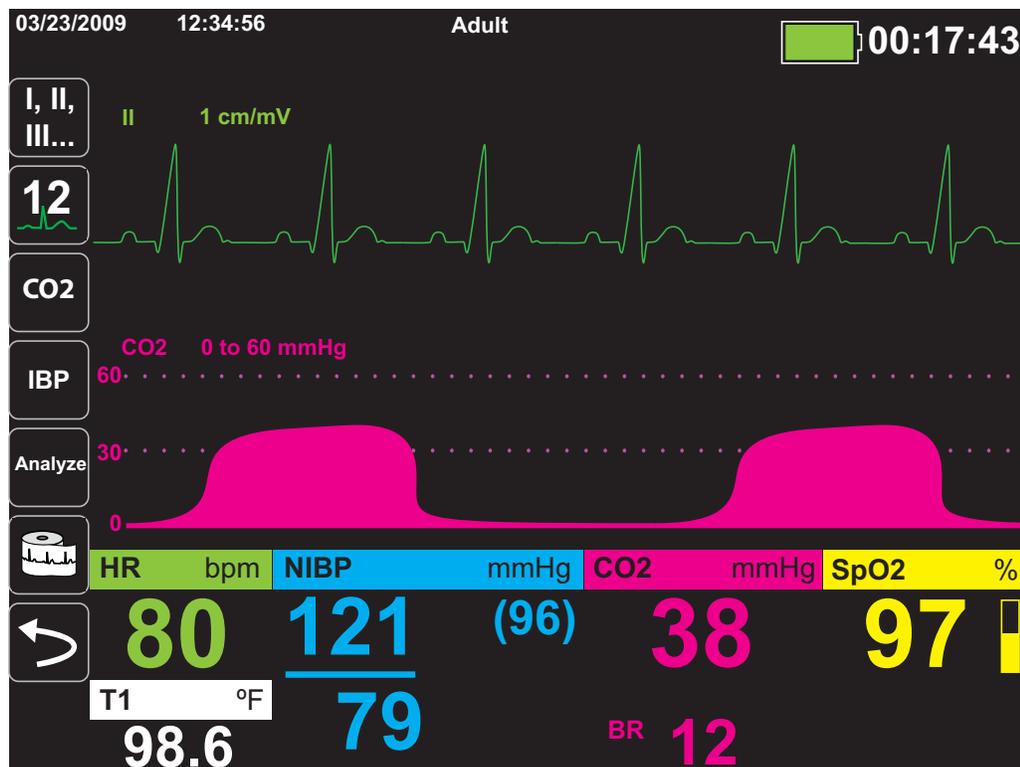

## Setting CO<sub>2</sub> and Respiration Rate Alarms

The Propaq MD unit sounds alarms whenever measurements are outside set limits for the following:

- High and Low EtCO<sub>2</sub>
- High and Low Respiration Rate (in Breaths/Minute)
- High FiCO<sub>2</sub>
- No Breath Alarm

### Enabling/Disabling Alarms and Setting CO<sub>2</sub> Alarm Limits

You can enable (or disable) CO<sub>2</sub> alarms and set upper and lower alarm limits through the Alarms quick access key () or the CO<sub>2</sub> Parameter Control Panel.

To configure CO<sub>2</sub> alarms through the Alarms quick access key:

1. Press the More quick access key ()
2. Press .
3. Press the **Limits** quick access key ()
4. Use the navigation buttons to select appropriate alarm menu. For CO<sub>2</sub>, the alarm menus are: EtCO<sub>2</sub> Alarm, FiCO<sub>2</sub> Alarm, or RR/BR Alarm.
5. On the alarm settings menu, use the navigation keys to select the fields that you want to change. The fields are
  - **Upper Limit Enable**
  - **Lower Limit Enable (only EtCO<sub>2</sub> and RR/BR Alarms)**
  - **Upper Limit**
  - **Lower Limit (only EtCO<sub>2</sub> and RR/BR Alarms)**
6.  When you are finished changing values on the alarm menu, select the Backarrow key to exit the menu.

### Setting Upper and Lower EtCO<sub>2</sub> Limits

Initially, the EtCO<sub>2</sub> Alarm menu specifies that the EtCO<sub>2</sub> alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower EtCO<sub>2</sub> Limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default EtCO<sub>2</sub> limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | EtCO <sub>2</sub> Limit Default | EtCO <sub>2</sub> Limit Range          |
|--------------|---------------------------------|--|
| Adult        | Lower: 8 mmHg<br>Upper: 60 mmHg | Lower: 0-145 mmHg<br>Upper: 5-150 mmHg |
| Pediatric    | Lower: 8 mmHg<br>Upper: 60 mmHg | Lower: 0-145 mmHg<br>Upper: 5-150 mmHg |
| Neonate      | Lower: 8 mmHg<br>Upper: 60 mmHg | Lower: 0-145 mmHg<br>Upper: 5-150 mmHg |

**Caution**

In high-altitude environments, EtCO<sub>2</sub> values may be lower than values observed at sea level, as described by Dalton's law of partial pressures. When using the Propaq MD unit in high-altitude environments, it is advisable to adjust EtCO<sub>2</sub> alarm settings accordingly.

### Setting Upper FiCO<sub>2</sub> Limits

Initially, the FiCO<sub>2</sub> Alarm menu specifies that the FiCO<sub>2</sub> alarms are enabled (**ON**) or disabled (**OFF**), and displays the default upper limit. The upper limit can be **ON** or **OFF** (default is **OFF**). The following table lists the default FiCO<sub>2</sub> upper limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | Upper FiCO <sub>2</sub> Limit Default | Upper FiCO <sub>2</sub> Limit Range |
|--------------|---------------------------------------|-------------------------------------|
| Adult        | 8 mmHg                                | 2-98 mmHg                           |
| Pediatric    | 8 mmHg                                | 2-98 mmHg                           |
| Neonate      | 8 mmHg                                | 2-98 mmHg                           |


### Setting Upper and Lower Respiratory Rate (RR/BR) Limits

Initially, the RR/BR Alarm menu specifies that RR/BR alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower respiratory limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default Respiratory limits for adult, pediatric, and neonate patients in Breaths/Minute, and gives the range in which you can set these limits:

| Patient Type | Respiration Rate Default       | Respiration Rate Range                     |
|--------------|--------------------------------|--|
| Adult        | Lower: 3 BPM<br>Upper: 50 BPM  | Lower: 0 to 145 BPM<br>Upper: 5 to 150 BPM |
| Pediatric    | Lower: 3 BPM<br>Upper: 50 BPM  | Lower: 0 to 145 BPM<br>Upper: 5 to 150 BPM |
| Neonate      | Lower: 12 BPM<br>Upper: 80 BPM | Lower: 0 to 145 BPM<br>Upper: 5 to 150 BPM |

### Using the CO<sub>2</sub> Parameter Control Panel

Use the navigation keys to highlight and select the **CO<sub>2</sub>** numeric display to display the CO<sub>2</sub> Parameter Control Panel:

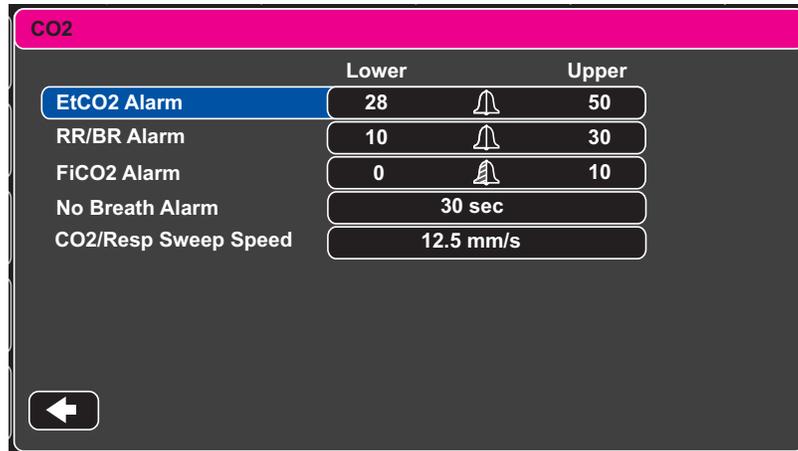

**Figure 10-2 CO<sub>2</sub> Parameter Control Menu**

On the CO<sub>2</sub> Parameter Control Panel, you can select a CO<sub>2</sub> alarm (EtCO<sub>2</sub>, Breath Rate, or FiCO<sub>2</sub>). On the selected alarm settings menu, you can enable/disable alarms and set alarm limits (as described previously). You can also set the CO<sub>2</sub> sweep speed.

## Setting the CO<sub>2</sub> Sweep Speed

The EtCO<sub>2</sub> sweep speed determines the X-axis scale of the capnogram. For patients with slower respiration rates, a slower sweep speed will make the capnogram easier to view. You can specify sweep speeds of 3.13, 6.25, and 12.5 mm/second. The default sweep speed is 6.25 mm/second.

## System Messages

When monitoring CO<sub>2</sub>, the Propaq MD unit may display the following messages:

| System Message              | Cause  |
|-----------------------------|--|
| <i>INITIALIZING</i>         | The unit is initializing the CO <sub>2</sub> monitoring function measurement, is functioning normally, and will display CO <sub>2</sub> measurement after a brief delay.   |
| <i>CHECK FILTERLINE</i>     | The sampling line is not connected. Check sampling line connection. Replace airway adaptor or cannula, if defective.   |
| <i>FILTERLINE OCCLUSION</i> | The sampling or exhaust line is blocked. Check sampling and exhaust lines. Make sure that the sampling line and any inputs to the patient breathing apparatus are not connected to the exhaust port. If the current FilterLine is correctly attached, replace the FilterLine.  |
| <i>AUTO ZERO</i>            | The Propaq MD unit displays <i>AUTO ZERO</i> when the CO <sub>2</sub> module performs periodic self-maintenance. During self-maintenance, the CO <sub>2</sub> module performs one or more of the following tests: ambient pressure measurement, auto zero, and a flow test. The CO <sub>2</sub> module completes its self-maintenance tests in approximately 10 seconds.   |
| <i>PURGING</i>              | The CO <sub>2</sub> module performs a purging whenever it detects an occlusion in the line or airway adaptor, and displays <i>PURGING</i> while it tries to clear the occlusion. If the CO <sub>2</sub> module cannot clear the occlusion within 30 seconds, the Propaq MD unit issues an equipment alert and displays the message <i>FILTERLINE OCCLUSION</i> . At this point, if the current FilterLine is correctly attached, replace the FilterLine. |
| <i>VALUE OVER RANGE</i>     | The CO <sub>2</sub> value exceeds the specified range.   |
| <i>CO2 DISABLED</i>         | An error has occurred with the CO <sub>2</sub> module, and the Propaq MD unit has disabled the module for the duration of the unit's power cycle. If the problem persists, the unit may require servicing.   |

## Patents

The capnography component of the Propaq MD unit is covered by one or more of the following US patents: 6,428,483; 6,997,880; 5,300,859; 6,437,316; 7,488,229; and their foreign equivalents. Additional patent applications pending.

### **NO IMPLIED LICENSE**

Possession or purchase of this device does not convey any express or implied license to the device with unauthorized consumable CO<sub>2</sub> sampling consumable products, which would, alone, or in combination with this device, fall within the scope of one or more of the patents relating to this device and/or CO<sub>2</sub> sampling consumable products.

# Chapter 11

## Monitoring Invasive Pressures (IBP)

---

The Propaq MD IBP inputs are Type CF defibrillator proof.

This chapter describes how to use the Propaq MD unit to monitor invasive pressures (IBP).

The Propaq MD unit has three invasive pressure channels: P1, P2, and P3. You can use these channels to measure arterial, venous, or intracranial pressures using invasive transducers with 5uV/V/mmHg sensitivity. Each channel requires its own connector, cabling, and pressure transducer.

### Invasive Pressure Transducers

The Propaq MD unit is compatible with many types of invasive pressure transducers, including non disposable, disposable dome, and fully disposable transducers. See Appendix B, *Accessories*, for a list of compatible transducers. Do not use light-sensitive disposable transducers.

Use the invasive pressure transducers according to your organization's established clinical protocol and follow the manufacturer's recommendations. Always refer the manufacturer's Directions for Use before using a transducer.

- Warning!**
- **If electrocautery is used, always avoid using any transducer with a conductive (metal) case connected to its ground shield. Using a conductive transducer case that is connected to its cable shield risks high-frequency burns at the ECG electrodes if the transducer case becomes grounded to earth.**
  - **Normal alarm functions will detect complete disconnections of invasive pressure transducers; however, the alarm functions will not detect a partial disconnection or the use of some incompatible transducers. Use only approved transducers and ensure that the transducers are connected properly.**
  - **Before you use the Propaq MD unit on a new patient, always turn it off for at least 2 minutes. This clears the previous patient's trend values, alarm limit settings, and NIBP cuff inflation pressure.**
  - **Use *only* ZOLL-approved IBP sensors. Use of non-approved sensors may result in inaccurate IBP measurements.**
- 

## IBP Setup

To take safe and accurate IBP measurements using the Propaq MD, you must perform the following steps, each of which corresponds to a section in this chapter. Read each section carefully before you perform IBP measurements.

1. Attach the invasive pressure transducer to the Propaq MD unit.
2. Zero the transducer.
3. Set the invasive pressure alarms (according to your organization's standards).
4. Select a label for the invasive pressure channel.

## Attaching the Invasive Pressure Transducer

Follow these steps when attaching the invasive pressure transducer:

1. Inspect the transducer cable. If the cable shows any signs of wear, breakage, or fraying, replace the cable. Replace the transducer dome, if necessary.
2. Apply the transducer according to your organization's procedures. Always refer the manufacturer's Directions for Use before using a transducer.
3. If the transducer is a disposable unit with a separate cable, connect the transducer to the transducer cable.
4. Plug the transducer cable into one of the three six-pin IBP cable connectors on the side of the Propaq MD unit.



**Figure 11-1 Plugging the Transducer into the Propaq MD**


When you plug the transducer cable into the unit, the message *ZERO PROBE* appears in the numeric display window for that IBP channel.

## Zeroing the Transducer

To ensure that the Propaq MD unit measures pressure accurately, you must zero the transducer before each use. If you change or disconnect a transducer, you must zero the new transducer before use. If you move a transducer to a different monitor, you must zero the transducer again, even if you have already done so on another unit. In addition to the procedure below, follow the transducer manufacturer's *Directions for Use* and your organization's established clinical protocol.

Follow these steps when zeroing a transducer:

1. Place the transducer at the same height as the patient's left atrium.
2. Close the transducer stopcock to the patient.
3. Open the transducer's venting stopcock to atmospheric air.
4. Allow the transducer a few seconds to settle.
5. Use the navigation keys to highlight and select the transducer's IBP channel and display the IBP channel's parameter control panel:

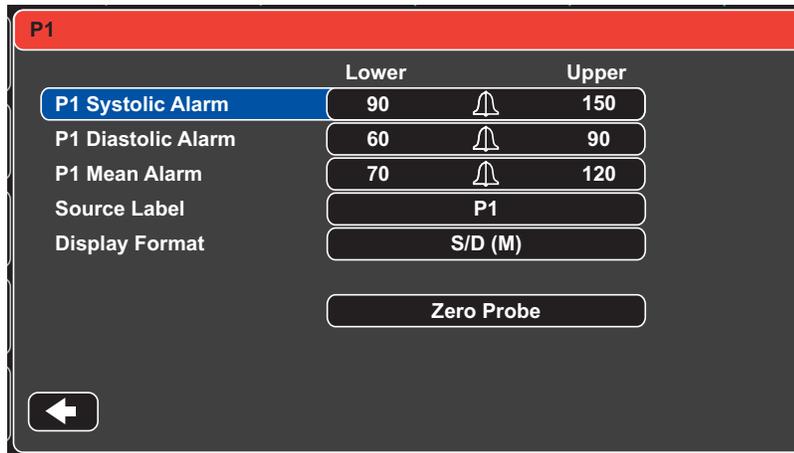

**Figure 11-2 IBP Channel Parameter Control Panel**

6. Select **Zero Probe**. The unit displays the message *ZEROING* in the IBP Channel's numeric display.
7. The message *ZEROED* appears in the IBP channel's numeric display.
8. Close the transducer's stopcock.
9. If the unit was unable to zero the transducer, the message *ZERO REJECTED* appears in the IBP channel's numeric display. The unit will not display pressure values for the IBP channel until it has successfully zeroed the transducer and established an acceptable zero reference.

Check that the unit is open to atmospheric air and that it is properly connected to the unit, then try zeroing the transducer again. The Propaq MD unit will not zero the transducer if it detects pulsation in the pressure channel, if there is too much noise in the signal, or if transducer's offset is too great.

If you cannot zero the transducer after several attempts, replace the transducer or the transducer cable.

## Rezeroing a Transducer

You can re-zero a transducer at any time by opening the transducer stopcock to atmospheric air.

If the unit accepts the new zero reference value, the unit displays pressure values that are based on that value and adjusts the waveform according to the new scale.

---

**Warning!** If you select an IBP channel after the channel has been successfully zeroed and is currently monitoring a pressure waveform, the unit will display the message, **ZERO REJECTED**, in the IBP channel's numeric window. This message will preempt the display of the valid invasive pressure numerics.

---

---

## Displaying IBP Measurements

After attaching and zeroing a transducer, the Propaq MD unit displays the invasive pressure's systolic, diastolic, and MEAN values in the IBP channel's numeric display and, optionally (if enabled through the unit's Waveform Select menu) the waveform for that IBP channel:



The Propaq MD unit allows you to specify a label that identifies the channel's IBP measurement, and to select a display format for the numeric display.

The unit displays the waveform scales after you zero the transducer. When the unit accepts the zero reference value, it determines the waveform scales and displays them.

## Conditions Affecting IBP Measurements

When reading blood pressure measurements on the IBP numeric display, keep in mind that the following conditions can affect the accuracy of IBP measurements:

- Catheter placement in the vasculature. Artifact such as catheter whip should be handled per your established clinical protocols.
- Position of the transducer stopcock, catheter, and flush port.
- Saline line flushes which will temporarily interrupt accurate pressure measurement.
- Position of the transducer relative to the patient's phlebostatic axis or catheter tip.
- Patient movement.
- Catheter clogging.
- Air bubbles in catheter or transducer dome.

---

**Caution** Flush catheter regularly while taking IBP measurements. Always view the IBP waveform to ensure that pressure measurements are based on a physiological waveform.

---

## Enabling/Disabling IBP Alarms and Setting Alarm Limits

When enabled, the Propaq MD unit sounds alarms whenever IBP measurements are outside set limits for the following:

- High and Low Systolic Pressure
- High and Low Diastolic Pressure
- High and Low Mean Arterial Pressure (MEAN)

To enable (or disable) IBP alarms and set Upper and Lower alarm limits and IBP channel, you can either use the navigation buttons to highlight and select the IBP channel display or:

1. Press the More quick access key ()
2. Press .
3. Press the **Limits** quick access key ()
4. Use the navigation buttons to highlight and select the appropriate alarm menu selection. For each IBP channel, there are alarm menus for systolic (Systolic Alarm), diastolic (Diastolic Alarm) and mean arterial pressure (Mean Alarm) alarm settings. On the selected Alarm Settings menu, use the navigation keys to select the fields that you want to change. The fields are
  - **Upper Limit Enable**
  - **Lower Limit Enable**
  - **Upper Limit**
  - **Lower Limit**
5. When you are finished changing values on the alarm menu, navigate to the Backarrow key  to exit the menu.

### Setting Upper and Lower Systolic (SYS) Alarm Limits

Initially, the IBP Systolic Alarm Settings menu specifies that the IBP systolic pressure alarms are enabled (**ON**) or disabled (**OFF**), and displays the default upper and lower systolic limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default NIBP Systolic limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | IBP Systolic Limit Default        | IBP Systolic Limit Range                         |
|--------------|-----------------------------------|--|
| Adult        | Lower: 75 mmHg<br>Upper: 220 mmHg | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |
| Pediatric    | Lower: 75 mmHg<br>Upper: 145 mmHg | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |
| Neonate      | Lower: 50 mmHg<br>Upper: 100 mmHg | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |

## Setting Upper and Lower Diastolic (DIA) Alarm Limits

Initially, the IBP Diastolic Alarm Settings menu specifies that the IBP diastolic pressure alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower diastolic limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default diastolic limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | IBP Diastolic Limit Default       | IBP Diastolic Limit Range                        |
|--------------|-----------------------------------|--|
| Adult        | Lower: 35 mmHg<br>Upper: 110 mmHg | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |
| Pediatric    | Lower: 35 mmHg<br>Upper: 100 mmHg | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |
| Neonate      | Lower: 30 mmHg<br>Upper: 70 mmHg  | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |

## Setting Upper and Lower Mean Arterial Pressure (MEAN) Alarm Limits

Initially, the IBP MEAN Alarm Settings menu specifies that IBP MEAN alarms are enabled (**ON**) or disabled (**OFF**), and displays the default Upper and Lower MEAN limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default MEAN alarm limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | IBP Mean Limit Default            | IBP Mean Limit Range                             |
|--------------|-----------------------------------|--|
| Adult        | Lower: 50 mmHg<br>Upper: 120 mmHg | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |
| Pediatric    | Lower: 50 mmHg<br>Upper: 110 mmHg | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |
| Neonate      | Lower: 35 mmHg<br>Upper: 80 mmHg  | Lower: -30 to 298 mmHg<br>Upper: -28 to 300 mmHg |

## Setting IBP Source Label

In the IBP channel menu, you can select a label that identifies the source of the channel's IBP measurement. By default, the channels are labeled P1, P2, and P3.

At the Source Label prompt, you can specify that the unit display one of the following labels to identify the IBP channel:

| Source Label | Description               | Source Label | Description               |
|--------------|---------------------------|--------------|---------------------------|
| <b>ABP</b>   | Abdominal Aorta Pressure  | <b>AO</b>    | Aorta                     |
| <b>ART</b>   | Arterial Pressure         | <b>BAP</b>   | Brachial Artery Pressure  |
| <b>CVP</b>   | Central Venous Pressure   | <b>FAP</b>   | Femoral Artery Pressure   |
| <b>ICP</b>   | Intracranial Pressure     | <b>LAP</b>   | Labial Artery Pressure    |
| <b>PAP</b>   | Pulmonary Artery Pressure | <b>RAP</b>   | Radial Artery Pressure    |
| <b>UAP</b>   | Umbilical Artery Pressure | <b>UVP</b>   | Umbilical Venous Pressure |

In the following example, source labels are specified for all three IBP channels:



---

## IBP System Messages

The Propaq MD unit may display the following messages when monitoring IBP:

| <b>System Message</b>          | <b>Cause</b>   |
|--------------------------------|--|
| <i>TRANSDUCER FAILURE</i>      | The IBP probe is damaged and needs to be replaced.   |
| <i>INCOMPATIBLE TRANSDUCER</i> | The IBP probe is not compatible. See the Appendix B, <i>Accessories</i> , for a list of ZOLL-approved IBP probes.        |
| <i>CHECK PROBE</i>             | The IBP probed has become disconnected.  |
| <i>ZERO PROBE</i>              | The IBP probe is connected and needs to be zeroed.   |
| <i>ZEROING</i>                 | The IBP probe is zeroing.  |
| <i>IBP DISABLED</i>            | A system error has occurred, and the Propaq MD unit should be serviced.  |
| <i>ZERO REJECTED</i>           | The IBP probe was not removed due to a pulsatile pressure signal, excessive IBP artifact or excessive transducer offset. |



# Chapter 12

## Monitoring Temperature

---

The Propaq MD Temperature inputs are Type CF defibrillator proof.

This chapter describes how to use the Propaq MD unit to monitor temperature.

The Propaq MD unit provides two temperature channels. When both channels are in use, the unit displays each channel's temperature successively, followed by the difference between the temperatures (labeled  $\Delta T$ ).

### Temperature Monitoring Setup

To monitor temperature using the Propaq MD unit, perform the following steps:

1. Select the temperature probe and apply it to the patient.
2. Connect the temperature probe to the Propaq MD unit.
3. Configure Temperature alarms and settings (if the current Temperature alarms and settings are not appropriate).

### Selecting and Applying Temperature Probes

You should use only temperature probes that are approved for use with the Propaq MD unit. See Appendix B, *Accessories* for a list of ZOLL-approved temperature probes. The use of other probes that do not match the performance specifications of the ZOLL-approved probes may produce incorrect temperature readings.

To apply the temperature probe to the patient, follow your organization's standard procedures. Always refer to probe manufacturer's Directions for Use before using the probe.

- Warning!**
- The application and use of metal-jacketed temperature probes that come in contact with conductive objects or clinical personnel during electrocautery may cause burns at the point of contact between the patient and the temperature probe.
  - Use *only* ZOLL-approved temperature probes. The use of non-approved probes may result in accurate measurements.
- 

## Connecting the Temperature Probe

To connect the temperature probe, insert the probe's 1/4" plug into one of the two connection jacks on the side of the Propaq MD unit.



Figure 12-1 Connecting the Temperature Probe to the Propaq MD Unit

## Displaying Temperature

When you connect the probe, the unit displays the temperature after a brief pause. The Propaq MD unit displays temperature as a numeric value in the Temperature window. You can specify that the unit display the temperature in °C or °F.

## Enabling/Disabling Temperature Alarms and Setting Alarm Limits

When enabled, the Propaq MD unit sounds alarms whenever temperature measurements are outside set limits.

You can enable (or disable) temperature alarms and set the upper and lower alarm limits through the Alarms quick access key or the Temp Parameter Control Panel.

To configure temperature alarms through the **Alarms** quick access key:

1. Press the More quick access key ()
2. Press .
3. Press the **Limits** quick access key ()
4. Use the navigation keys to highlight and select the **T1 Alarm, T2 Alarm, or ΔT Alarm** menu selection. On the Alarm Settings menu, use the navigation keys to select the fields that you want to change. The fields are
  - **Upper Limit Enable**
  - **Lower Limit Enable**
  - **Upper Limit**
  - **Lower Limit**
5. When you are finished changing values on the alarm menu, navigate to the Backarrow key  to exit the menu.

## Setting Upper and Lower Temperature Alarm Limits

Initially, the Temperature Alarm Settings menu specifies that the Temperature alarms are enabled or disabled and displays the default upper and lower limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default Temperature limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | Temperature Limit Default                             | Temperature Limit Range  |
|--------------|---|--|
| Adult        | Lower: 35 ° C (95.0 ° F)<br>Upper: 37.8 ° C (100 ° F) | Lower: 0.0 - 48.0 ° C (32.0 - 120.0 ° F)<br>Upper: 2.0 - 50.0 ° C (34.0 - 122.0 ° F) |
| Pediatric    | Lower: 35 ° C (95.0 ° F)<br>Upper: 37.8 ° C (100 ° F) | Lower: 0.0 - 48.0 ° C (32.0 - 120.0 ° F)<br>Upper: 2.0 - 50.0 ° C (34.0 - 122.0 ° F) |
| Neonate      | Lower: 35 ° C (95.0 ° F)<br>Upper: 37.8 ° C (100 ° F) | Lower: 0.0 - 48.0 ° C (32.0 - 120.0 ° F)<br>Upper: 2.0 - 50.0 ° C (34.0 - 122.0 ° F) |

## Setting Upper and Lower $\Delta$ Temperature Alarm Limits

Initially, the  $\Delta$  Temperature Alarm Settings menu specifies that the  $\Delta$  Temperature alarms are enabled (**ON**), and displays the default upper and lower limits. The upper and lower limits can be **ON** or **OFF** (default is **OFF**). The following table lists the default  $\Delta$  Temperature limits for adult, pediatric, and neonate patients, and gives the range in which you can set these limits:

| Patient Type | $\Delta$ Temperature Limit Default               | $\Delta$ Temperature Limit Range   |
|--------------|--|--|
| Adult        | Lower: 0.0 °C (5.0 °F)<br>Upper: 2.8 °C (2.0 °F) | Lower: 0.0 - 49.8 °C (0.0 - 89.8 °F)<br>Upper: 0.2 - 50.0 °C (0.2 - 90.0 °F) |
| Pediatric    | Lower: 0.0 °C (5.0 °F)<br>Upper: 2.8 °C (2.0 °F) | Lower: 0.0 - 49.8 °C (0.0 - 89.8 °F)<br>Upper: 0.2 - 50.0 °C (0.2 - 90.0 °F) |
| Neonate      | Lower: 0.0 °C (5.0 °F)<br>Upper: 2.8 °C (2.0 °F) | Lower: 0.0 - 49.8 °C (0.0 - 89.8 °F)<br>Upper: 0.2 - 50.0 °C (0.2 - 90.0 °F) |

## Selecting the Temperature Label

Through the Temp Parameter Control Panel, you can configure alarms (as described previously) and select descriptive Temperature channel labels that will appear on the numeric display and on the trend report:

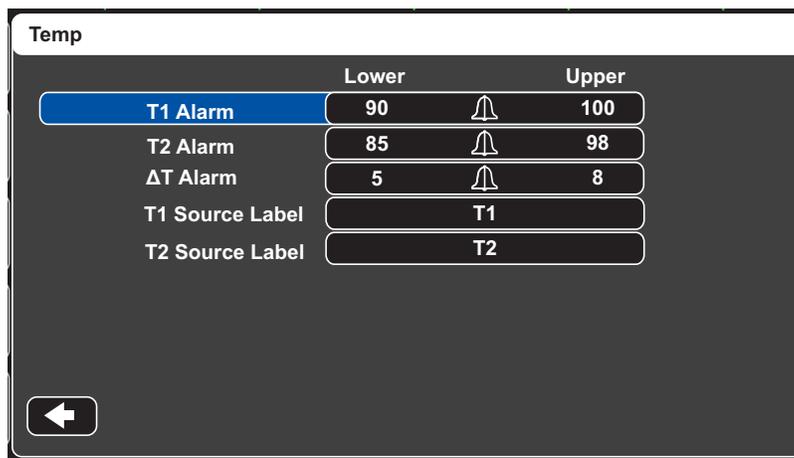


Figure 12-2 Temp Parameter Control Panel

You can select one of the following labels for each temperature channel:

| Source Label | Description   |
|--------------|---|
| <b>ART</b>   | Arterial temperature probe                                      |
| <b>CORE</b>  | Central body temperature or tympanic membrane temperature probe |
| <b>ESOPH</b> | Esophageal temperature probe                                    |
| <b>RECT</b>  | Rectal temperature probe  |
| <b>SKIN</b>  | Skin temperature probe (surface application)                    |
| <b>VEN</b>   | Ventilator airway temperature probe                             |
| <b>NASO</b>  | Nasopharynx or nasal/oral temperature probe                     |

If you don't select a label, the Temperature channels appear with the default labels of T1 and T2.

## Temperature System Messages

The Propaq MD unit may display the following messages when monitoring Temperature.

**Note:** The temperature function performs a self test when initially powered on and also performs system tests automatically, every 10 seconds, while this function is active.

| System Message       | Cause  |
|----------------------|--|
| <i>CHECK PROBE</i>   | The temperature probe is disconnected. Check probe and reconnect it.   |
| <i>PROBE FAILURE</i> | The temperature probe is defective. Replace the Temperature probe.   |
| <i>TEMP DISABLED</i> | A system error has occurred. The Propaq MD unit cannot take temperature measurements and should be serviced. |



# Chapter 13

## Automated External Defibrillator (AED) Operation

---



ZOLL hands-free therapy electrodes are a defibrillation-protected Type BF patient connection.

This chapter describes the recommended method of operation in AED Mode. The Propaq MD is configured to operate in compliance with the American Heart Association and European Resuscitation Council Guidelines for Adult Basic Life Support and Use of Automated External Defibrillators.<sup>1,2,3,4</sup> If your local protocol requires a different procedure, follow that protocol.

The AED Mode is Supervisor configurable, and when the unit is configured to start up in AED Mode, it powers up in the Analysis/CPR Protocol, and guides you through a cardiac event by performing ECG analysis, preparing the device for a shock (if needed), and leading you through a CPR interval.

This chapter also describes how to switch the AED unit to Manual mode (see “Switching to Manual Mode Operation” on page 13-9).

The Propaq MD defibrillator is capable of analyzing a patient’s ECG rhythm in two different ways. The first mode of analyzing is automatic; the other mode of analyzing is user activated analysis and is initiated by pressing the **Analyze** key.

- 
1. *Circulation*, 2005; 112; IU-19 — IU-34
  2. *Resuscitation (2005)*; 671S, S7-S23
  3. *Circulation 2010*; 122; S640-S656
  4. *Resuscitation (2010)*; 1219-1276

**Warning!** Use only pediatric electrodes to defibrillate patients under 8 years of age in AED mode, and make sure the patient mode is set to pediatric. Use of adult electrodes or adult mode with pediatric patients can result in the delivery of excessive energy doses.

---

Automatic and user activated analysis of a patient's ECG can be performed only when:

- Hands-free therapy electrodes are connected, and making good contact with the patient.
- The defibrillator is on.

This analysis normally consists of up to three consecutive 3-second ECG rhythm analyses. If at least two of the three analyses determine that the patient has a shockable rhythm, the unit automatically prompts the operator to shock the patient at the preconfigured energy level. If two or more of the three 3-second ECG analyses do not detect a shockable rhythm, the unit alerts the operator that no shock is advised.

If the first two segments are in agreement with each other, the final shock or no shock rhythm decision is made and a third segment is not processed.

If the shock button is pressed and a shock is successfully delivered, the shock count increments by one and is displayed on the screen.

## AED Operation

### Determine Patient Condition Following Medical Protocols

Verify:

- Unconsciousness.
- Absence of breathing.
- Absence of pulse.

### Begin CPR Following Medical Protocols

Request additional assistance.

### Prepare Patient

Remove all clothing covering the patient's chest. Dry chest if necessary. If the patient has excessive chest hair, clip or shave it to ensure proper adhesion of the electrodes.

Attach hands-free therapy electrodes according to instructions on the electrode packaging.

Ensure that the electrodes are making good contact with the patient's skin and are not covering any part of the ECG electrodes.


Connect the hands-free therapy electrodes to the multi-function cable (MFC) if not already connected.

The message *ATTACH PADS* or *CHECK PADS* will be displayed and energy will not be delivered if the therapy electrodes are not making good contact with the patient. The message *SHORT DETECTED* is displayed if a short circuit exists between the therapy electrodes.

## Therapy Electrode Application

**Warning!** Poor adherence and/or air under the therapy electrodes can lead to the possibility of arcing and skin burns.

1. Apply one edge of the pad securely to the patient.
2. Roll the pad smoothly from the applied edge to the other, being careful not to trap any air pockets between the gel and skin.

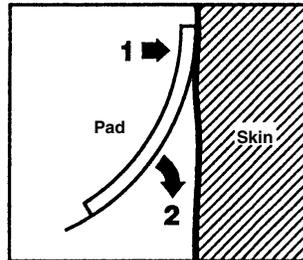

**Note:** If it is not possible to place the “BACK” pad on the patient’s back, the pads should be placed in the standard apex-sternum positions. Effective defibrillation will result, but pacing will usually be less effective.

### 1 Turn on unit

Press the green power switch located on the top of the unit. The green, yellow, and red lights on the top of the unit flash on and off, and then displays the message *SELF TEST PASSED*.

If no hands-free therapy electrodes have been attached to the patient and connected to the Propaq MD, the *ATTACH PADS* message and voice prompt will be issued.



## Energy Select

The default energy selections for adult patients are:

Shock 1 - 120 joules

Shock 2 - 150 joules

Shock 3 - 200 joules

When the device is configured in Pediatric mode, the default energies are:

Shock 1 - 50 joules

Shock 2 - 70 joules

Shock 3 - 85 joules

**Note:** Pediatric defibrillator energy levels should be selected based on site specific protocols.

**Note:** If the Propaq MD has been configured to begin CPR upon start up, it will automatically begin with the CPR interval.

## 2 Analyze

---


**Warning!** Do not analyze the patient ECG during patient movement. A patient must be motionless during ECG analysis. Do not touch the patient during analysis. Cease all movement via stretcher or vehicle before analyzing the ECG.

---

The Propaq MD automatically begins the analysis of the patient's ECG rhythm, displays an ANALYZING ECG message for 5 seconds, then announces and displays a *STAND CLEAR* message. If therapy electrodes have not been properly connected to the patient, an *ATTACH PADS* or *CHECK PADS* message is displayed and analysis will be inhibited.

**Note:** You can press **Pause** to suspend the rescue cycle. While the rescue cycle is paused, ECG analysis continues in the background. The trends, log, and 12-lead features are only accessible when the rescue cycle is suspended. Press **Analyze** to resume the analysis process.

**Note:** If the Propaq MD has been configured to perform extra CPR, it displays a *CHECK PULSE* voice prompt and the message appears for 10 seconds. Then it displays an *PERFORM CPR* message along with a voice prompt for the configured duration before analysis begins. You can start an ECG analysis during the CPR interval by pressing the **ANALYZE** key.

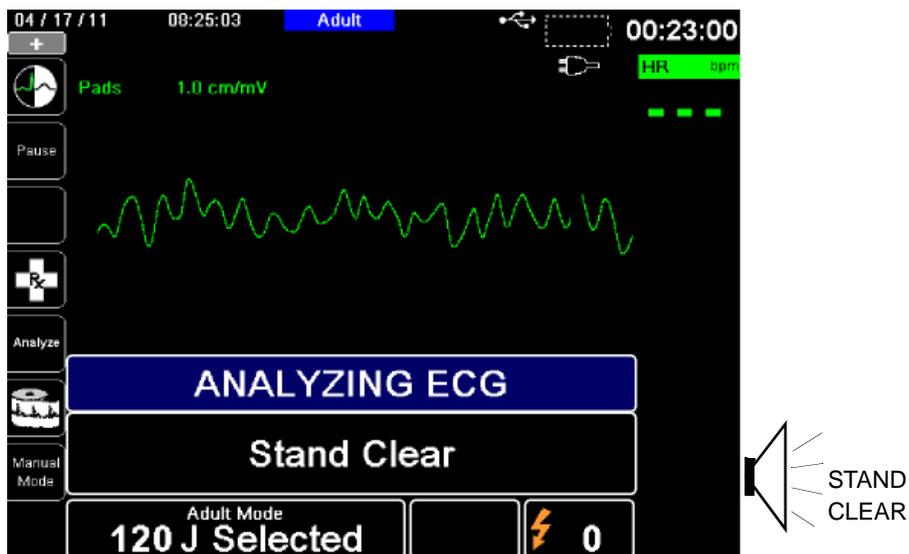


An ANALYZING ECG message is displayed while the patient's ECG is analyzed. Once the analysis is completed, the unit indicates whether or not a shock is advised.

---

**Warning!** ECG rhythm analysis does not warn of patient asystole, which is not a shockable rhythm.

---

When a nonshockable rhythm is detected, the unit displays a *NO SHOCK ADVISED* message. Immediately begin chest compressions and continue other treatment per protocol.

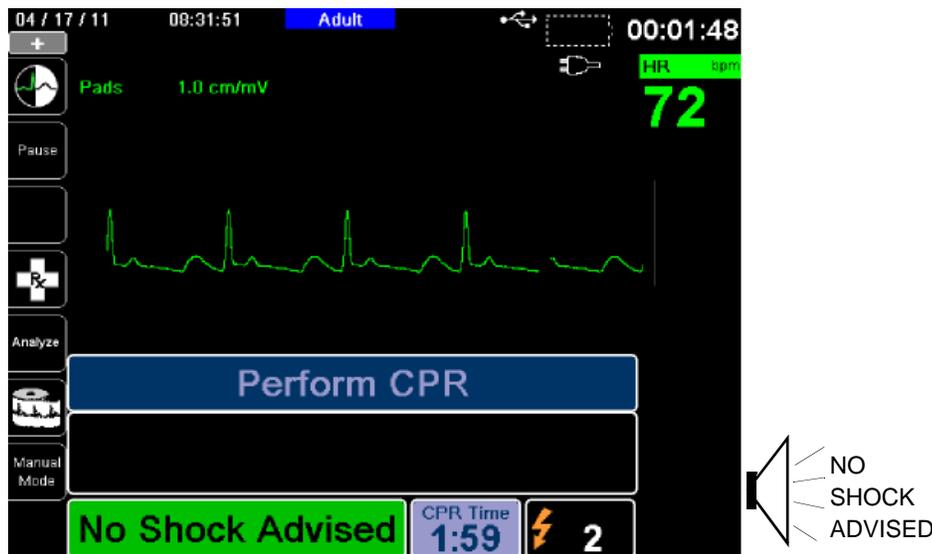

If the patient's rhythm is shockable, the unit displays the *SHOCK ADVISED* and *PRESS SHOCK* messages. The defibrillator automatically prompts the operator to shock the patient at the preconfigured energy level and the **SHOCK** button illuminates.

A continuous tone sounds for 20 seconds, followed by an intermittent tone for 10 seconds. You must deliver the shock within this 30 second interval, or the defibrillator will disarm itself.



### 3 Press SHOCK

---

**Warning!** Warn all persons in attendance of the patient to **STAND CLEAR** prior to defibrillator discharge.


**Do not touch the bed, patient, or any equipment connected to the patient during defibrillation. A severe shock can result. Do not allow exposed portions of the patient's body to come in contact with metal objects, such as a bed frame, as unwanted pathways for defibrillation current may result.**

---

Press and hold the illuminated **SHOCK** button on the front panel until energy is delivered to the patient.

Observe the patient or ECG response to verify that the shock has been delivered.

The delivered energy level and the shock number (1) are displayed in the panel at the bottom of the screen.

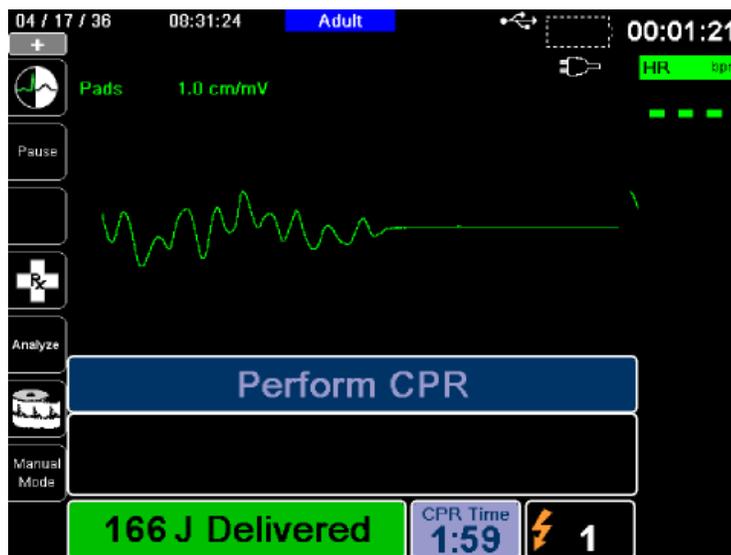

### Perform CPR

Begin chest compressions and rescue breathing per local protocol as prompted by the unit.

### Repeat Analysis

After performing CPR for the configured period of CPR, the unit will restart ECG analysis automatically.

**Note:** Reanalysis of the ECG rhythm is inhibited for 3 seconds after a shock.

### Stop CPR

After performing CPR for the configured period of CPR, the unit will issue a *STOP CPR* prompt while it restarts ECG analysis.

### Continue Patient Care

Continue patient care according to medical protocols.

## Operating Messages

The unit uses both audio and visual prompts to present critical information to operators. The following information describes the unit default configuration. If your device has been custom configured, some of the information may be different.

There are 10 voice prompts used in AED mode. Most of these prompts are accompanied by a message displayed on the monitor. The voice prompts are given only once, but the monitor continues to display the message until new action is taken by the operator or the device status changes.

The unit will alternately display two different messages in the same field of the display when two conditions are detected at the same time. For example, a *LOW BATTERY* message may alternately display on the same line of the monitor as the *CHECK PADS* message.

## Audio and Display Messages

The display messages and voice prompts that can occur during AED operation are described below.

### **ATTACH PADS**

If the unit is powered on without therapy pads connected to the patient, the *ATTACH PADS* message is announced and displayed.

### **ANALYZING ECG/STAND CLEAR**

The *ANALYZING ECG* message is displayed and the *STAND CLEAR* message is displayed and announced when the ECG analysis starts automatically or after pressing the **ANALYZE** key. They indicate that an active ECG analysis is in progress.

### **SHOCK ADVISED**

A shockable rhythm has been detected and defibrillation is advised. The selected energy level is displayed.

### **PRESS SHOCK**

This message is displayed and announced when the ECG analysis has determined that a shock is advised and the selected energy is ready to be delivered.

### **SHOCKS: XX**

This message displays the number of shocks that have been delivered by the unit since it was powered on. It resets to 0 after the unit has been off for more than two minutes.

### **NO SHOCK ADVISED**

When ECG analysis detects a nonshockable rhythm, this message is announced and displays for 10 seconds following completion of the analysis.

### **CHECK PULSE**

If configured to do so, this message is displayed and announced in the following situations:

- After a No Shock Advised analysis result
- During the CPR interval after a No Shock Advised analysis result
- After the final shock is delivered

### **IF NO PULSE, PERFORM CPR**

If configured to do so, this message is displayed and announced in the following situations:

- During the CPR interval after a No Shock Advised analysis result
- During the start of extra CPR interval

### **PERFORM CPR**

If configured to do so, this message is displayed and announced during the CPR interval after a No Shock Advised analysis result.

### **STOP CPR**

After performing CPR for the configured period, the unit will issue a *STOP CPR* prompt while it restarts ECG analysis.

### **CHECK PADS**

This message is displayed and announced when the therapy pads have been disconnected from the patient.

## Switching to Manual Mode Operation

Press the **Manual Mode** quick access key on the front panel of the unit to enter the Manual mode of operation.

Using the navigation keys, select the four digits in the Manual Mode pass code. Press **SAVE** when you are finished. Once you have entered your pass code, you will be able to enter Manual mode.

**Note:** If the unit has not been configured to enter a pass code, the message *Exit to Manual Mode* is displayed. Use the navigation keys to select **Yes** to enter the Manual mode of operation. If you do not press **Yes** within 10 seconds, the unit will revert back to AED operation.

When changing from AED mode to Manual mode, the current selected energy level is maintained.

**Note:** To transfer back to AED mode from Manual mode, power down the unit for more than 30 seconds and less than two minutes, then power it back up. If you wait longer than two minutes, the unit will reset the settings to the defaults and treat the case as a new patient.



# Chapter 14

## 12-Lead ECG Interpretive Analysis

---

The Propaq MD 12-Lead input is Type CF defibrillator proof.

This chapter describes how to use the Propaq MD unit to monitor 12-Lead ECG for adult and pediatric patients, and how to display 12-Lead ECG Interpretive Analysis information for adult patients.


Propaq MD 12-Lead ECG Monitoring provides simultaneous acquisition and storage of 12-Lead information for adult and pediatric patients, and post-acquisition Interpretive Analysis for adult patients.

- 
- Warning!**
- **12-Lead ECG monitoring is intended for the recording 12-lead ECG signals from adult and pediatric patients in the supine, resting position -- always ensure that the patient is kept motionless during 12-lead ECG signal acquisition and analysis. Use of the device to acquire ECG signals from moving or shaking patients may produce noisy signals that are difficult to interpret.**
  - **12-Lead Interpretive Analysis is for use with *only* adult patients.**
  - **Excessive body hair or wet, sweaty skin may interfere with electrode adhesion. Remove the hair and/or moisture from the area where the electrode is to be attached.**
  - **Remove ECG electrodes from their sealed package immediately prior to use. Using previously opened or out-of- date electrodes may degrade the ECG signal quality.**
  - **Monitoring electrodes may become polarized during defibrillator discharge, causing the ECG waveform to briefly go off screen. ZOLL Medical Corporation recommends the use of high quality silver/silver chloride (Ag/AgCl) electrodes to minimize this effect; the circuitry in the instrument returns the trace to the monitor display within a few seconds.**

- **Wait 15 seconds after defibrillator discharge before attempting a 12-lead acquisition. Electrode polarization subsequent to defibrillator discharge may result in excessive noise on the 12-lead ECG printout.**
- **When not in use, cover the patient cable's V-lead connector with the supplied plastic cap. Failure to do so may result in a shock hazard during defibrillation attempts.**
- **To assure protection against the effects of defibrillator discharge, use only 12-lead cables supplied by ZOLL Medical Corporation.**
- **Check the operation and integrity of the Propaq MD unit and 12-lead cable regularly by performing the Daily Operational Verification Test.**
- **When attempting to interpret subtle ECG changes (such as ST segments) use only the Diagnostic frequency response setting. Other frequency response settings may cause misinterpretation of the patient's ECG.**
- **Use only ZOLL-approved accessories to ensure the Type CF defibrillator proof rating of the 12-Lead input.**
- **Implanted pacemakers may cause the heart rate meter to count the pacemaker rate during incidents of cardiac arrest or other arrhythmias. Carefully observe pacemaker patients. Check the patient's pulse; do not rely solely on heart rate meters. Dedicated pacemaker detection circuitry may not detect all implanted pacemaker spikes. Patient history and physical exam are important in determining the presence of an implanted pacemaker.**

## Entering Patient Information

To enter patient information, press the 12-Lead quick access key (12), then the Patient Info quick access key (Patient Info icon). The screen displays the Patient Info parameter panel, in which you can enter the patients's name, age, gender, and identification number:

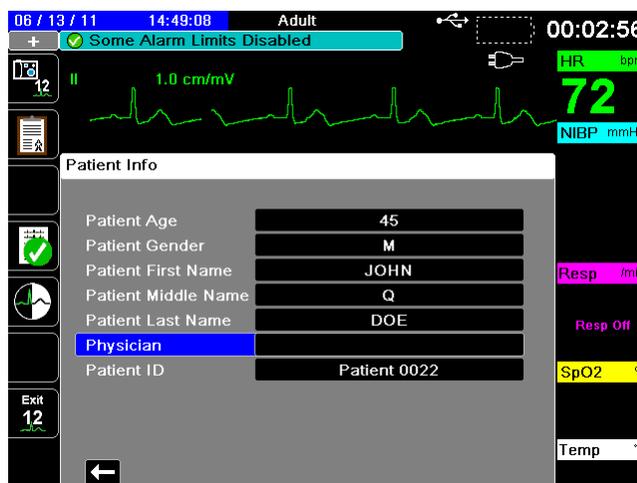

**Figure 14-1 Patient Info Control Panel**

The Propaq MD unit uses the name that you enter in the Patient Info panel to label the 12-lead ECG monitoring snapshots that it saves.

To enter patient information, use the navigation keys to highlight and select a parameter on the Patient Info panel, then press the Select key.

## Entering the Patient Name and ID

When you select the **Patient Last Name** field (or the Patient First/Middle Name, ID, or Physician fields), the screen displays an information entry panel:

The screenshot shows a dark-themed information entry panel titled "Patient Last Name". At the top is a white text input field. Below it is a grid of character keys arranged in rows: Row 1: A, B, C, D, E, F, G, H, I; Row 2: J, K, L, M, N, O, P, Q, R; Row 3: S, T, U, V, W, X, Y, Z, [blank]; Row 4: Backspace, [blank], Clear; Row 5: 0, 1, 2, 3, 4, 5, 6, 7, 8, 9; Row 6: ., -, #, \*, /, ?, &, ', -; Row 7: SAVE, Cancel.

To enter a character into the parameter, highlight the character, then press Select. The screen displays the selected character in the area below the parameter name.

To navigate through the information entry panel, use the following keys:

- Use the Row Up quick access key (  ) to move to the previous row on the panel.
- Use the Row Down quick access key (  ) to move down one row on the panel.

Use the navigation keys on the front panel to highlight the next or previous character on the information entry panel.

You can also select the following function keys on the information entry panel:

- **Backspace** Erases the last character entered.
- **Clear** Erases all characters entered.
- **SAVE** Saves the characters entered for that parameter and returns you to the Patient Info panel.
- **Cancel** Returns you to the Patient Info panel without saving the characters entered.

## Entering Patient Age and Gender

The Patient Info parameter panel provides default values for the Patient Age and Patient Gender parameters. To change a default value, highlight and select the parameter, and then specify a new value as follows:

To change Patient Age, use the navigation keys on the front panel to decrease or increase the default value (45), then press Select.

To change Patient Gender, use the navigation keys on the front panel to toggle between the default value, M (male), and F (female), then press Select.

## 12-Lead ECG Monitoring Setup

The proper application and placement of electrodes is essential for high quality 12-Lead ECG monitoring. Good contact between the electrode and skin minimizes motion artifact and signal interference.

To set up 12-lead ECG monitoring, perform the following steps:

1. Prepare the patient's skin for electrode application.
2. Apply the electrodes to the patient.
3. Connect each lead of the ECG cable to the appropriate electrode.
4. Connect the 12-Lead cable to the Propaq MD unit.
5. Observe the patient's electrocardiogram on the display, and adjust size of the 12-Lead ECG waveform traces, as necessary.

## Preparing the Patient for Electrode Application

The proper application of electrodes is essential for high quality ECG monitoring. Good contact between the electrode and skin minimizes motion artifact and signal interference.

Before applying electrodes, prepare the patient's skin, as necessary:

- Shave or clip excess hair at electrode site.
- Clean oily skin with an alcohol pad.
- Rub site briskly to dry.

## Applying Electrodes to the Patient

Depending on local usage, ECG lead wires are marked with certain labels. Refer to the following table for labels and color codes for the different lead sets


| Location  | AHA <sup>1</sup> Labels | IEC <sup>2</sup> Labels |
|-----------|-------------------------|-------------------------|
| Right Arm | RA (white)              | R (red)                 |
| Left Arm  | LA (black)              | L (yellow)              |
| Right Leg | RL (green)              | N (black)               |
| Left Leg  | LL (red)                | F (green)               |
| Chest     | V1                      | C1                      |
| Chest     | V2                      | C2                      |
| Chest     | V3                      | C3                      |
| Chest     | V4                      | C4                      |
| Chest     | V5                      | C5                      |
| Chest     | V6                      | C6                      |


<sup>1</sup> American Heart Association

<sup>2</sup> International Electrotechnical Commission

Patients should be in a resting, supine position when performing 12-Lead ECG monitoring. ZOLL Medical Corporation recommends placing the limb electrodes anywhere along the ankles and wrists.

When it is difficult for the patient to remain motionless due to shivering, muscle tremors, or transport vehicle movement, place limb electrodes on the patient's thorax for better results. (Refer to the following two diagrams for limb electrode placement).

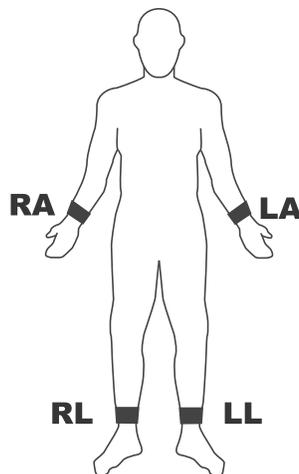

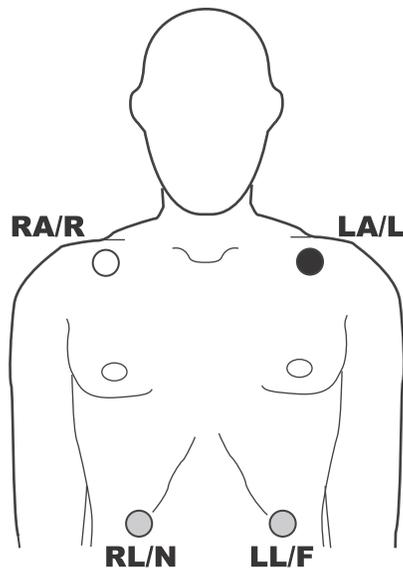

Avoid placing electrodes over tendons and major muscle masses.


Make sure that the ECG electrodes are placed to allow defibrillation, if necessary.

Place the precordial electrodes across the chest in the following locations:

| Electrode | Placement   |
|-----------|---|
| V1/C1     | Fourth intercostal space, at the patient's right sternal margin.            |
| V2/C2     | Fourth intercostal space, at the patient's left sternal margin.             |
| V3/C3     | Midway between V2/C2 and V4/C4.   |
| V4/C4     | Fifth intercostal space, on the patient's midclavicular line.               |
| V5/C5     | Patient's left anterior axillary line, at the horizontal level of V4.       |
| V6/C6     | Patient's left midaxillary line, at the same horizontal level as V4 and V5. |

Locating the V1/C1 position (fourth intercostal space) is critically important because it is the reference point for locating the placement of the remaining V-leads. To locate the V1/C1 position:

1. Place your finger on top of the jugular notch (see figure below).
2. Move your finger slowly downward about 1.5 inches (3.8 centimeters) until you feel a slight horizontal ridge or elevation. This is the “Angle of Louis,” where the manubrium joins the body of the sternum.

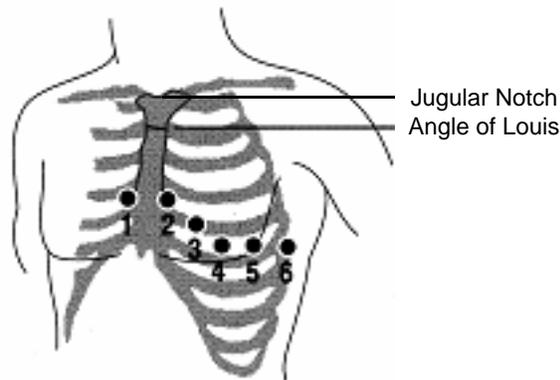

3. Locate the second intercostal space on the patient’s right side, lateral to and just below the “Angle of Louis.”
4. Move your finger down two more intercostal spaces to the fourth intercostal space which is the V1 position.

**Note:** When placing electrodes on female patients, always place leads V3-V6 under the breast rather than on the breast.

## Connecting the 12-Lead Cable


Connect the 12-Lead ECG cable to the ECG input connector on the left side of the unit as follows:



**Figure 14-2** Connecting the 12-Lead ECG Cable

## Observing the 12-Lead Waveform Traces

To observe the 12-Lead waveform traces, press **12**. The screen displays all twelve waveform traces, with the size displayed above the waveform traces:

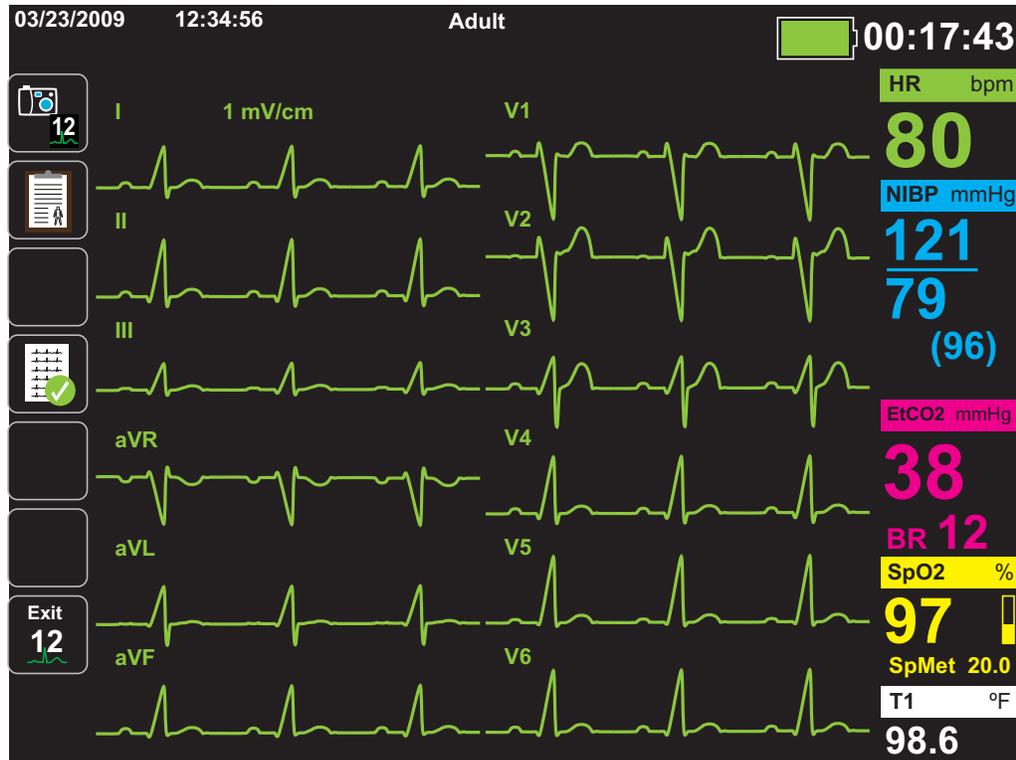

## 12-Lead Interpretive Analysis

After observing the patient's ECG and determining that all 12-lead traces display correctly, you can initiate 12-Lead Interpretive Analysis.

**Note:** 12-Lead Interpretive Analysis works for *only* Adult patients.


To begin 12-Lead Interpretive Analysis, press the **Acquire** quick access key (**12**). The Propaq MD unit displays the *Acquiring 12-Lead* Status bar as it collects 10 seconds of 12-Lead ECG data:



After acquiring the ECG data, the unit saves the data and displays the *Saving 12-Lead Status Bar* in the following manner:




After saving the data, the unit performs the post-acquisition Interpretive Analysis and displays the first page of 12-Lead Interpretive Analysis information:

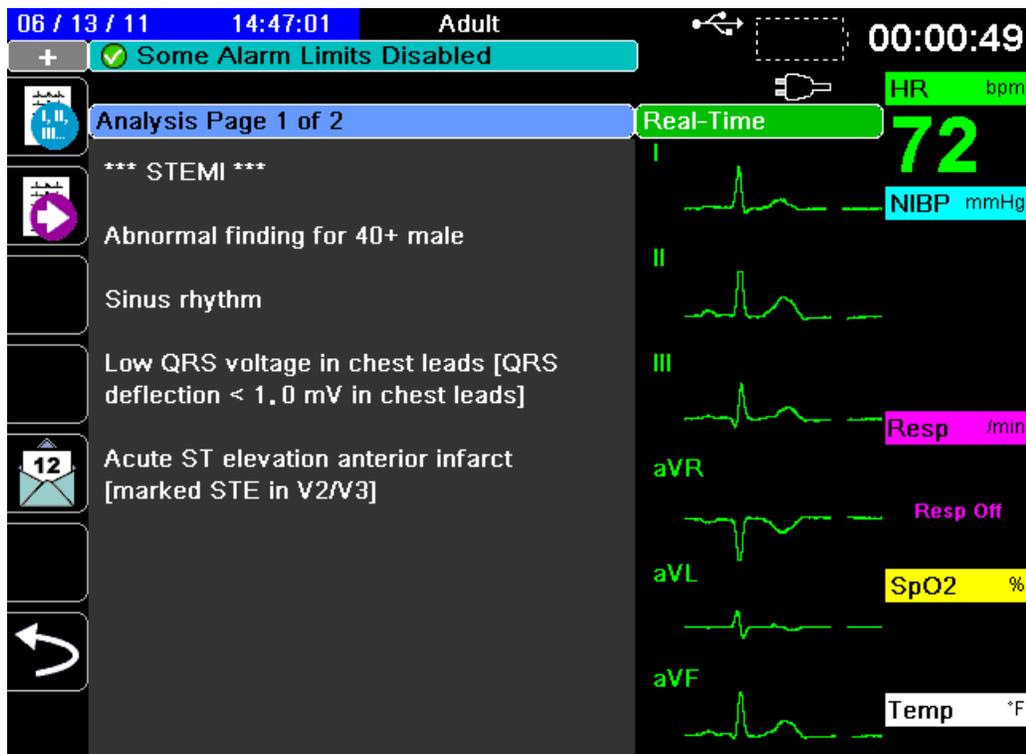


In the example above, the interpretive statement, **\*\*\*STEMI\*\*\***, indicates the occurrence of ST-Elevation Myocardial Infarction. The interpretive statements that the Propaq MD unit displays are produced by the Audicor software of Inovise Medical, Inc. For more information on these interpretive statements, see the *Audicor 200 Physician's Guide, Software Version 1.00*.

To display *Analysis Page 2*, press the 12-Lead Review Next quick access key (  ).

*Analysis Page 2* displays the identifying information that has been entered for the patient and additional analysis statements:

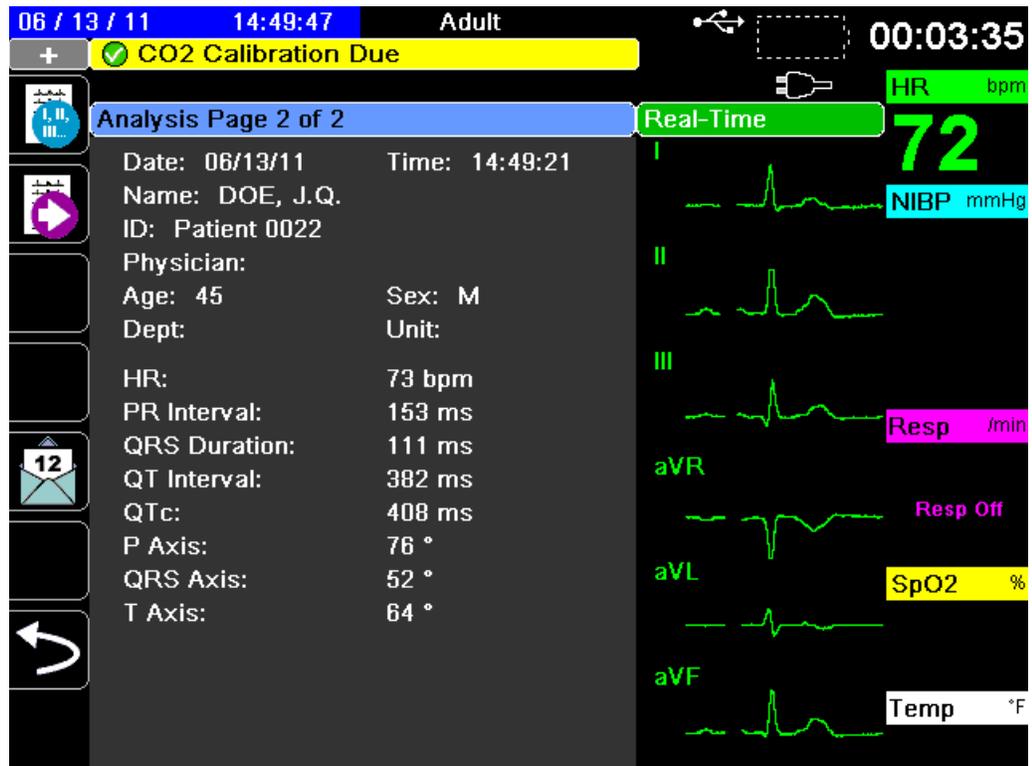

You can then view four 12-Lead snapshot pages by pressing  to step through them sequentially. For example, the unit displays the following 12-Lead snapshot after the *Analysis Page 2*:

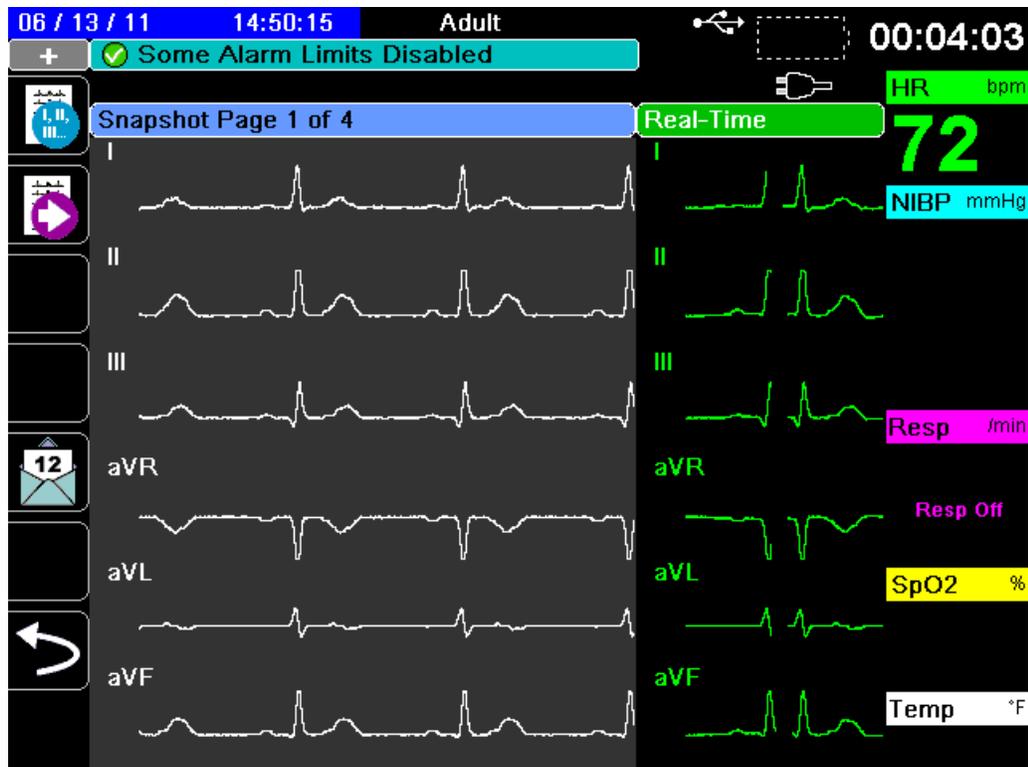

## Fault Conditions Affecting 12-Lead Interpretive Analysis

The Propaq MD unit will not perform Interpretive Analysis if it detects one of the following fault conditions when attempting to acquire 12-Lead data:

- A pacer signal is present.
- A lead fault is detected in the ECG cable.
- An invalid cable is in use.

If the Propaq MD unit detects one of these fault conditions, *Analysis Page 1* indicates that there is *No data available* for Interpretive Analysis and lists the fault condition; all measurements on *Analysis Page 2* appear as *N/A*.


Once you have corrected the fault condition, press  to confirm the correction and perform the 12-Lead Interpretive Analysis.

## Printing 12-Lead Waveform Traces


Once you have determined that the Propaq MD unit is correctly setup for 12-Lead ECG monitoring, you can print the 12-Lead ECG traces for review and analysis.

Press  to collect 10 seconds of 12-Lead data for print.

Press the Print quick access key () to print a 12-lead snapshot, which begins with a header that gives the date, time, and patient information, followed by 3.3 second samples of all twelve waveform traces. The waveform traces print in either a 3 x 4 format (the default).

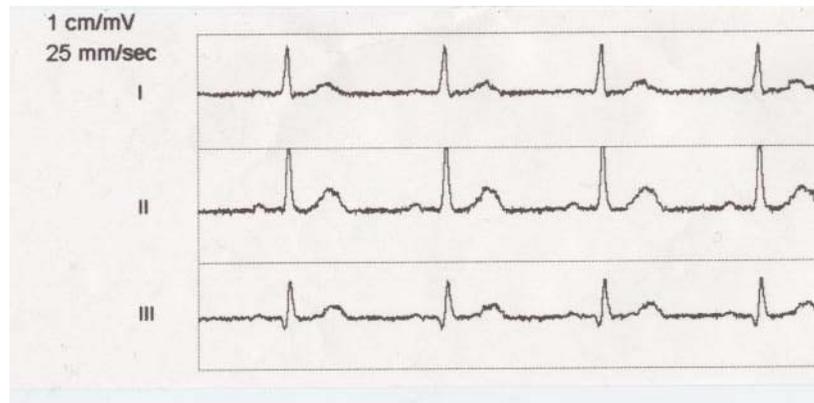

or a 2 x 6 format:

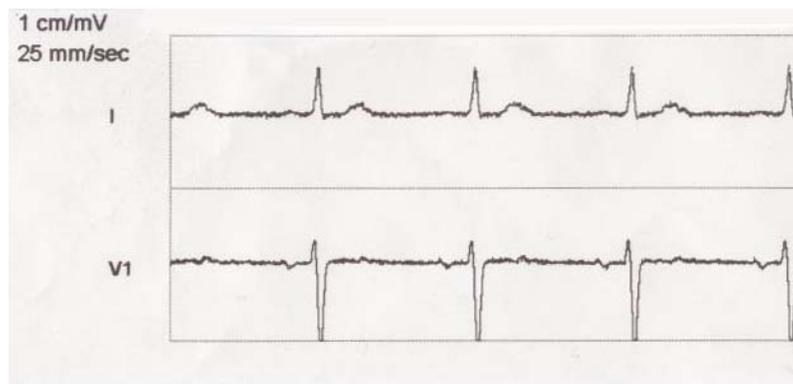

The Propaq MD unit stores a minimum of 32 12-Lead snapshots in a separate log. Once 32 12-lead snapshots are stored, the oldest snapshot in the log is overwritten by subsequent snapshots. Note that if a patient data log is full, you will not be able to store 12-lead snapshots. The 12-lead snapshots are erased when you clear the log.


When you are done viewing and printing the 12-Lead waveform traces, press the Exit 12 quick access key () to restore the display of other monitoring functions.

## 12-Lead Print and Display Options

The Propaq MD unit provides additional 12-Lead print and display options that you can specify through the Supervisor parameter control panels (access to Supervisor is passcode-controlled).

Press the Setup quick access key () , then select Supervisor. Using the navigation keys, select the four digits in the Supervisor passcode. Press **SAVE** when you are finished. Once you have entered your supervisor passcode, you will be able to access the configurable options in the Supervisor menu.

Select the Supervisor> ECG> 12-Lead option to display the 12-Lead parameter control panel:

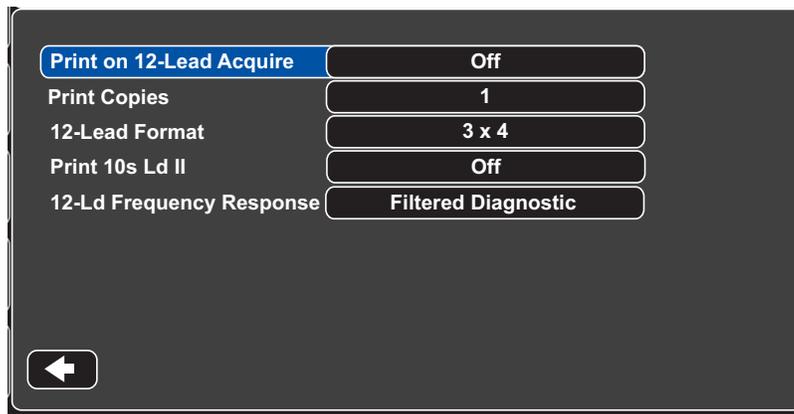

### Selecting 12-Lead Acquire

When enabled, the unit automatically prints the 12-Lead report when you press  . By default, this feature is not enabled.

### Specifying the Number of 12-Lead Print Copies

This option allows you to specify that the Propaq MD unit print up to five copies of the 12-lead waveform trace set after pressing  . By default, the unit prints only one 12-lead snapshot.

### Specifying the 12-Lead Print Format

This option allows you to specify the print format for the 12-lead waveform traces: 3 x 4 (the default) or 2 x 6.

### Printing 10 Seconds of the Lead II Waveform Trace

This option allows you to specify that the unit print 10 seconds of the Lead II waveform trace after printing a 12-Lead snapshot.

## Specifying the 12-Lead Frequency Response

This option allows you to specify the frequency response of the 12-lead waveform display.

You can specify the following trace display ranges:

| Display Type        | Frequency Response |
|---------------------|--------------------|
| Diagnostic          | 0.05 to 150 Hz     |
| Filtered Diagnostic | 0.25 to 40 Hz      |

**Note:** The display and the recording present the waveform according to the filter setting (Diagnostic or Filtered Diagnostic), but the 12-Lead Interpretive Analysis is always performed using the diagnostic bandwidth with a 50/60 Hz notch filter.

# Chapter 15

## Manual Defibrillation

---



Paddles are a defibrillation-protected Type BF patient connection.



ECG leads are a defibrillation-protected Type CF patient connection.

### Emergency Defibrillation Procedure with Paddles

---

**Warning!** To avoid risk of electrical shock, do not allow electrolyte gel to accumulate on hands or paddle handles.

When defibrillating with paddles, use your thumbs to operate the **SHOCK** buttons in order to avoid inadvertent operator shock. No portion of the hands should be near the paddle plates.

Be sure to use the proper paddles/electrodes based on the size of the patient (adult - large, pediatric - small).

---

## Determine the Patient's Condition Following Local Medical Protocols

Verify the following:

- Unconsciousness
- Absence of breathing
- Absence of pulse

## Begin CPR Following Local Medical Protocols

Request additional assistance.

## Turn Unit On

Press the green power switch located on the top of the unit. The green, yellow, and red lights at the top of the unit flash on and off, and the unit displays the message *SELF TEST PASSED*.

## 1 Select Energy Level


Press the **Select Energy** arrows up or down to select the desired energy level. These buttons are located either on the front of the unit or on the STERNUM paddle.

**Note:** The initial defibrillator pad/external paddle energy selection can be set in Defib/Pacer default settings in the Setup>Supervisor menu.


The following are the default energy selections:

- Adult - 120 joules
- Pediatric- 50 joules
- Neonate - 50 joules

**Note:** Neonatal and pediatric defibrillator energy levels should be selected based on site-specific protocols.

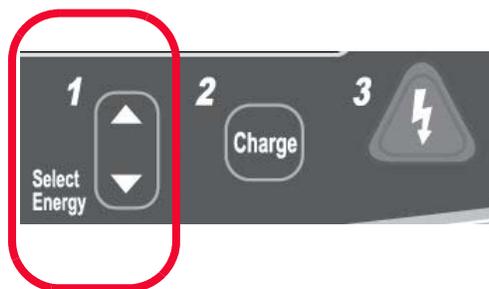

or

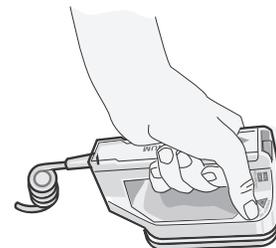

The selected energy level is displayed at the bottom of the display screen.


## Prepare Paddles

Ensure that the paddles are connected to the multi-function cable (MFC) and that the MFC is connected to the Propaq MD unit. Apply a liberal amount of electrolyte gel to the electrode surface of each paddle, and rub the electrode surfaces together to evenly distribute the applied gel. (You can substitute electrode gel patches for the gel.)

### Apply Paddles to Chest

Apply the paddles firmly to the anterior wall of the chest. Place the STERNUM paddle to the right of the patient's STERNUM (patient's right), just below the clavicle.

Place the APEX paddle on the chest wall, just below and to the left of the patient's left nipple, along the anterior-axillary line.

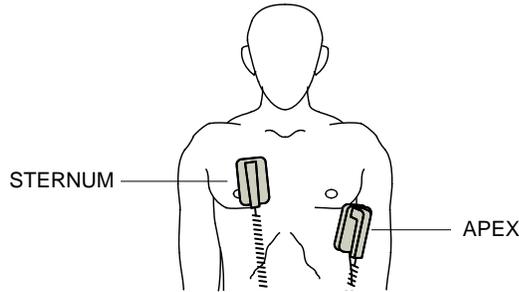

Rub the paddles against the skin to maximize the paddle-to-patient contact.

---

**Warning!** Do not permit gel to accumulate between the paddle electrodes on the chest wall (gel bridge). This could cause burns and reduce the amount of energy delivered to the heart.


If using defibrillator gel pads, make sure that the size of the pad is large enough to cover the entire paddle electrode area.

---

The paddles may be used for ECG monitoring in emergency situations when time does not allow connection of standard ECG monitoring electrodes.

## 2 Charge Defibrillator

Press the **CHARGE** button on the APEX handle or on the front panel.

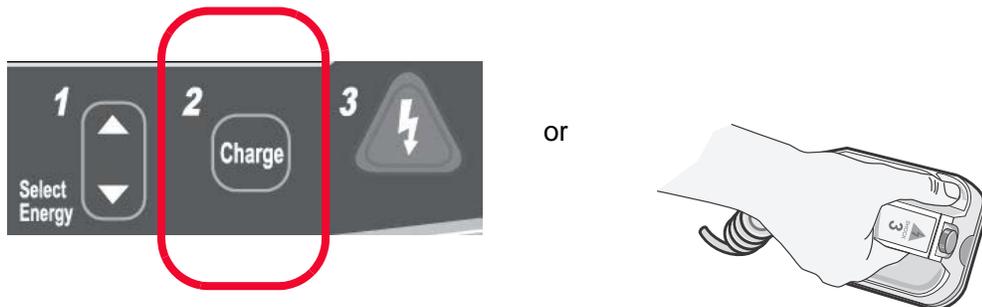

To increase or decrease the selected energy after you have pressed the **Charge** button, use the defibrillator **Select Energy** buttons on either the STERNUM paddle or the defibrillator front panel.

---

**Caution** Changing the selected energy while the unit is charging or charged causes the defibrillator to disarm itself. Press the **Charge** button again to charge the unit to the newly selected energy level.

---

A charging message displays at the bottom of the screen, and a distinctive charging tone sounds indicating that the unit is charging. The energy range bar graph on the right side of the display highlights the charge level until it reaches the selected energy. When the unit is fully charged, the tone changes to a continuous charge ready tone, the highlighted energy bar graph includes the selected energy, and the charge indicator on the APEX paddle lights up.



### 3 Deliver Shock


**Warning!** Warn all persons in attendance of the patient to **STAND CLEAR** prior to defibrillator discharge.

**Do not touch the bed, patient, or any equipment connected to the patient during defibrillation. A severe shock can result. Do not allow exposed portions of the patient's body to come into contact with metal objects, such as a bed frame, as unwanted pathways for defibrillation current may result.**

Apply a force of 10 - 12 kilograms (22 - 26.4 pounds) to each paddle in order to minimize patient impedance and achieve optimal results.

Using your thumbs, simultaneously press and hold both **SHOCK** buttons (one on each paddle) until energy is delivered to the patient.

**Note:** The front panel Shock button (⚡) is inactive when using external paddles. Pressing this button instead of the paddle **SHOCK** buttons will produce an audible invalid operation tone.

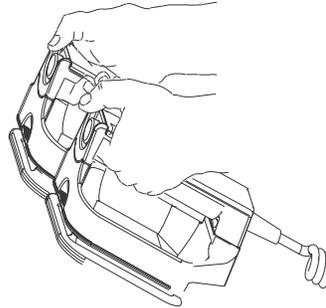

The delivered energy level is displayed at the bottom of the screen and the shock number (1) displays at the top of the screen and in the Defib Control panel at the bottom of the screen.



**Note:** If at any time you want to cancel the defibrillation, press the **Disarm** quick access key.

If the defibrillator is not discharged within 60 seconds after reaching the selected energy level, the unit automatically disarms itself.

If additional countershocks are needed, follow steps 1 through 3 of this procedure starting on page 15-2, to readjust the energy settings, charge the unit, and deliver the shock.

## Emergency Defibrillation Procedure with Hands-Free Therapy Electrodes



ZOLL hands-free therapy electrodes are a defibrillation-protected Type BF patient connection.



ECG leads are a defibrillation-protected Type CF patient connection.

### Determine the Patient's Condition Following Local Medical Protocols

Verify:

- Unconsciousness
- Absence of breathing
- Absence of pulse

## Begin CPR Following Medical Protocols

Request additional assistance.

## Prepare Patient

Remove all clothing covering the patient's chest. Dry chest if necessary. If the patient has excessive chest hair, clip or shave it to ensure proper adhesion of the electrodes.

Attach hands-free therapy electrodes according to instructions on the electrode packaging.

Ensure that the therapy electrodes are making good contact with the patient's skin and are not covering any part of the ECG electrodes.


## Therapy Electrode Application

---

**Warning!** Poor adherence and/or air under the therapy electrodes can lead to the possibility of arcing and skin burns.

---

1. Apply one edge of the pad securely to the patient.
2. Roll the pad smoothly from the applied edge to the other, being careful not to trap any air pockets between the gel and skin.

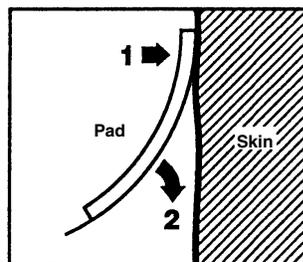

**Note:** If it is not possible to place the “BACK” electrode on the patient's back, place the electrodes in the standard apex-sternum positions. Effective defibrillation results, but pacing will usually be less effective.

## Turn Unit On

Press the green power switch located on the top of the unit. The green, yellow, and red lights at the top of the unit flash on and off, and the unit displays the message *SELF TEST PASSED*.

If defibrillation electrodes are not making good contact with the patient's skin and the pad selection is ECG Lead, the unit issues the message *CHECK THERAPY ELECTRODES* and does not allow the delivery of energy.


## 1 Select Energy Level

Press the **Select Energy** arrows up or down to select the desired energy level. These buttons are located on the front panel of the unit.

The following are the default energy selections:

- Adult - 120 joules
- Pediatric- 50 joules
- Neonate - 50 joules


**Note:** Neonatal and pediatric defibrillator energy levels should be selected based on site-specific protocols.

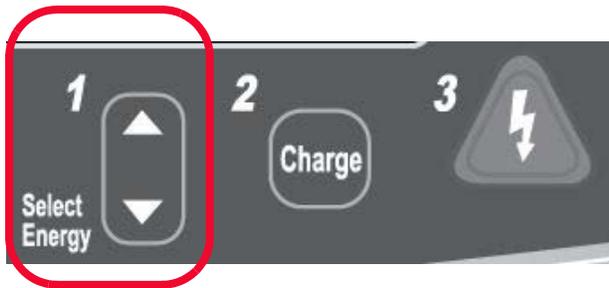

The selected energy level is shown on the display.

## 2 Charge Defibrillator

Press the **Charge** button on the front panel.

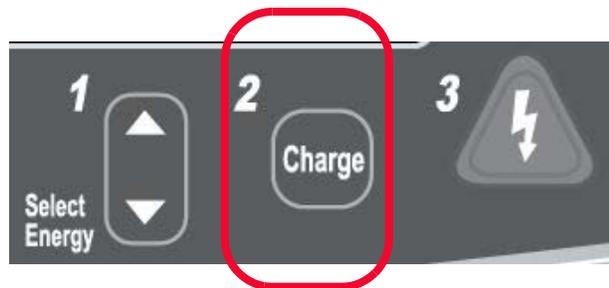

To increase or decrease the selected energy after you have pressed the **Charge** button, use the defibrillator **Select Energy** arrows on the front panel.


---

### Caution

Changing the selected energy while the unit is charging or charged causes the defibrillator to disarm itself. Press the **CHARGE** button again to charge the unit to the newly selected energy level.

---

A charging message displays at the bottom of the screen, and a distinctive charging tone sounds indicating that the unit is charging. The energy range bar graph on the right side of the display highlights the charge level until it reaches the selected energy. When the unit is fully charged, the tone changes to a continuous charge ready tone, the highlighted energy bar graph includes the selected energy, and the shock button lights up.

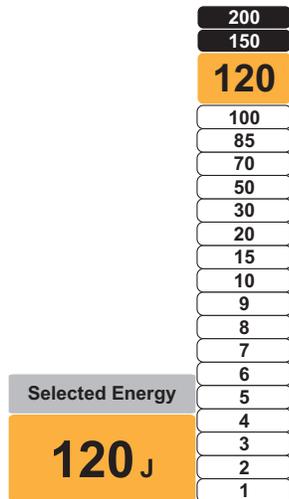


### 3 Deliver Shock

**Warning!** Warn all persons in attendance of the patient to **STAND CLEAR** prior to defibrillator discharge.

**Do not touch the bed, patient, or any equipment connected to the patient during defibrillation. A severe shock can result. Do not allow exposed portions of the patient's body to come into contact with metal objects, such as a bed frame, as unwanted pathways for defibrillation current may result.**

Press and hold the  button on the front panel until energy is delivered to the patient.

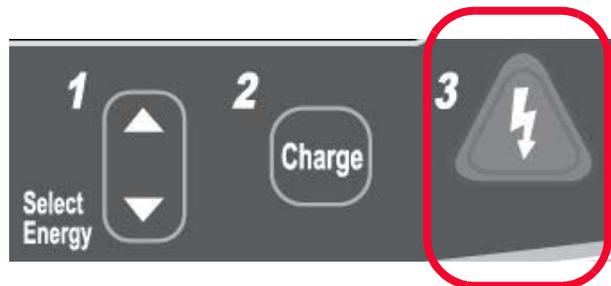

The delivered energy level is displayed at the bottom of the screen and the shock number (1) displays at the top of the screen and in the Defib Control panel at the bottom of the screen.



**Note:** If at any time you want to cancel the defibrillation, press the **Disarm** softkey.

If the defibrillator is not discharged within 60 seconds after reaching the selected energy level, the unit automatically disarms itself.

If additional countershocks are needed, follow steps 1 through 3 of this procedure starting on page 15-7, to readjust the energy settings, charge the unit, and deliver shock.

## Internal Paddles

ZOLL internal paddles are designed for use with the ZOLL Propaq MD defibrillator to defibrillate the heart during open chest procedures. Two types of Autoclavable Internal Handles are available:

- Molded Autoclavable Internal Handles with integrated electrodes
- Autoclavable Internal Handles with removable internal defibrillation electrodes

When an internal handle set is connected to the Propaq MD, it automatically limits defibrillator energy output to a maximum of 50 joules.

For step-by-step procedures for open chest defibrillation as well as important cleaning and sterilization information regarding the autoclavable electrodes, refer to the *Autoclavable Internal Handle and Electrode Operator's Guide*.

## Verification Prior to Use

Before each use with the Propaq MD, verify the proper operation of the ZOLL internal paddles using the following procedure. This procedure requires a second person if you are using internal handles without a Discharge button.

---

**Warning!** When performing internal paddle verification, keep hands away from the electrode plates while pressing the Shock button.

---

1. Inspect the connector contact sockets for damage or corrosion. If you observe damage or corrosion in the connector contact sockets, remove the handle set from use.
2. Connect the Autoclavable Internal Handles to the Propaq MD unit. Use the Lead quick access key  to verify that the Propaq MD unit correctly identifies the Internal Handle and Electrode set by displaying **Int. Pdis**.
3. Before charging the defibrillator, press the **Discharge** button on the handle set (if present) and verify that there is an audible click and that the button springs back upon release. Verify that the defibrillator window displays the message *APPLY PADDLES TO PATIENT*. This message verifies that the **Discharge** button located on the right handle is operating correctly.
4. Press the Energy Select arrows (located on the front panel of the Propaq MD unit) up or down to select 30 Joules.
5. Press the **Charge** button on the defibrillator front panel to charge the unit to the selected energy level. Wait for the READY tone.
6. Press the electrode plate surfaces firmly together and away from any person or object.

7. Discharge the energy in the following manner.
  - For internal handles with a Discharge button:  
Press and hold the **Discharge** button on the apex handle to deliver the test energy to the electrodes.
  - For internal handles without a Discharge button:  
Have a second person press and hold  on the defibrillator front panel to deliver the test energy to the electrodes.

The Propaq MD device unit discharges and displays the message *DEFIB SHORT TEST PASSED*.


## Synchronized Cardioversion

**Warning!** Only skilled personnel trained in Advanced Cardiac Life Support and familiar with equipment operation should perform synchronized cardioversion. The precise cardiac arrhythmia must be determined before attempting defibrillation or cardioversion.

**Before attempting synchronized cardioversion, ensure that ECG signal quality is sufficient to minimize the risk of synchronizing on artifact.**

Certain arrhythmias, such as ventricular tachycardia, atrial fibrillation, and atrial flutter, require synchronizing the defibrillator discharge with the ECG R-wave to avoid the induction of ventricular fibrillation. In this case, a synchronizing (SYNC) circuit within the defibrillator detects the patient's R-waves. When the **Shock** button (or buttons, if using paddles) is pressed and held, the unit discharges with the next detected R-wave, thus avoiding the vulnerable T-wave segment of the cardiac cycle.

When in the SYNC mode, the unit displays markers (**S**) above the ECG trace to indicate the points in the cardiac cycle (R waves) where discharge can occur.

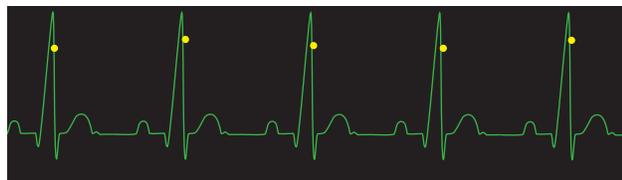

S marker indicates each detected R wave during synchronization.

Verify that markers are clearly visible on the monitor and their location is appropriate and consistent from beat to beat.

The synchronized cardioversion procedure for ZOLL hands-free therapy electrodes is identical to that for paddles with the exception of the **SHOCK** button location.

---

# Synchronized Cardioversion Procedure

## Determine the Patient's Condition and Provide Care Following Local Medical Protocols

### Prepare Patient

Remove all clothing covering the patient's chest. Dry chest if necessary. If the patient has excessive chest hair, clip or shave it to ensure proper adhesion of the electrodes.

Attach ECG electrodes (see Chapter 6 for instructions on attaching ECG electrodes to the patient).

A standard ECG cable and ECG electrodes are recommended for use during cardioversion. Hands-free therapy electrodes may be used as an ECG source. Signal quality will be equal to that of standard leads except immediately following a discharge when there may be more noise due to muscle tremors, especially if an electrode is not in complete contact with the skin.

Attach hands-free therapy electrodes according to instructions on the electrode packaging and as described in "Therapy Electrode Application" on page 15-6.

Ensure that the therapy electrodes are making good contact with the patient's skin and are not covering any part of any other electrodes.

If paddles are being used for synchronized cardioversion, refer to "Emergency Defibrillation Procedure with Paddles" on page 15-1 for preparing paddles, applying paddles, charging the defibrillator, and delivering a shock. Note, however, that synchronized discharge with paddles as an ECG source is discouraged since the artifact induced by moving the paddles may resemble an R-wave and trigger defibrillator discharge at the wrong time.

### Turn the Unit On

Press the green power switch located on the top of the unit. The green, yellow, and red lights at the top of the unit flash on and off, and the unit displays the message *SELF TEST PASSED*.

### Press the Sync Key

Press the **Sync** button on the front panel. The system will now be in SYNC mode. A sync marker (S) appears on the monitor above each detected R-wave to indicate where discharge will occur. A Sync indicator appears at the top of the display screen, and a green light next to the indicator flashes with each Sync marker.



**Note:** If the marker does not appear over the R wave, select a different ECG lead. If the sync marker doesn't display, the defibrillator will not discharge.


Unless otherwise configured, the unit automatically exits Sync mode after each shock. To reactivate SYNC mode, press the **Sync** button on the front panel again. Changing the selected energy levels does not cause the unit to leave SYNC mode.

**Note:** The unit can be configured to stay in SYNC mode after defibrillation in Defib/Pacer default settings in the Setup>Supervisor menu.

## 1 Select Energy Level


Press the **Select Energy** arrows up or down to select the desired energy level. These buttons are located either on the front of the unit or on the STERNUM paddle.

**Warning!** When used with pedi-padz, defibrillator energies must be set manually based on site-specific institutional protocols for pediatric defibrillation.

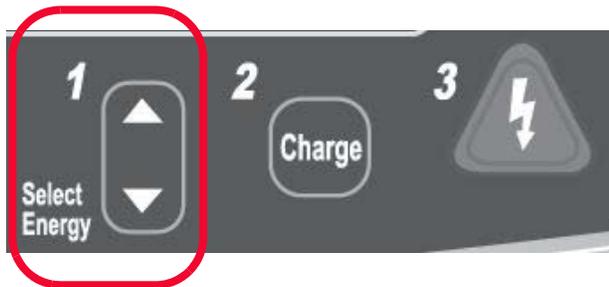

## 2 Charge Defibrillator

Press the **CHARGE** button on the front panel or on the APEX paddle handle.

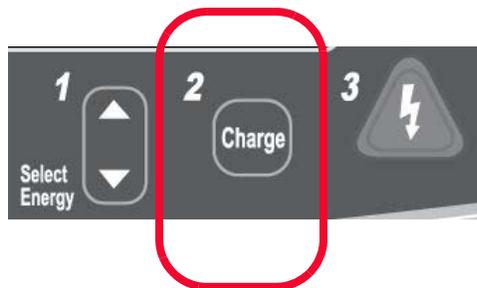

or



To increase or decrease the selected energy after you have pressed the **Charge** button, use the defibrillator **Select Energy** arrows on the front panel or sternum panel.

**Caution** Changing the selected energy while the unit is charging or charged causes the defibrillator to disarm itself. Press the **CHARGE** button again to charge the unit to the newly selected energy level.

A charging message displays at the bottom of the screen, and a distinctive charging tone sounds indicating that the unit is charging. The energy range bar graph on the right side of the display highlights the charging progress until it reaches the selected energy. When the unit is fully charged, the tone changes to a continuous charge ready tone, the highlighted energy bar graph includes the selected energy, and the shock button lights up.

### 3 Deliver Shock

---

**Warning!** Warn all persons in attendance of the patient to *STAND CLEAR* prior to defibrillator discharge.

Verify that no one is in contact with the patient, monitoring cable or leads, bed rails, or any other potential current pathways.

---

Verify that the ECG waveform is stable and that sync markers appear over each R-wave.

Press and hold the illuminated **Shock** button on the front panel, (or simultaneously press and hold both paddle **Shock** buttons) until energy is delivered to the patient. The defibrillator will discharge with the next detected R wave.

The delivered energy level is displayed at the bottom of the screen and the shock number (1) displays at the top of the screen.

**Note:** If at any time you want to cancel the defibrillation, press the **Disarm** quick access key.

If the defibrillator is not discharged within 60 seconds after reaching the selected energy level, the unit automatically disarms itself.

If additional countershocks are needed (and the **Sync after Cardioversion** setting is disabled), press the **Sync** softkey again and follow steps 1 through 3 of this procedure starting on page 15-12, to readjust the energy settings, charge the unit, and deliver shock.

You can configure the **Sync after Cardioversion** setting through the Setup>Supervisor>Defib/Pacer>Default Settings menu.



# Chapter 16

## Advisory Defibrillation

---



ZOLL hands-free therapy electrodes are a defibrillation-protected Type BF patient connection.

With Advisory defibrillation, the Propaq MD can identify shockable rhythms using its built in ECG analysis capability. You must read the advisory messages, charge the defibrillator to the preconfigured or user-selected energy level (if automatic charge is disabled), and deliver treatment to the patient when required by protocol and patient condition.

The advisory function can be activated only when:

- The unit is on and in manual mode.
- Hands-free therapy electrodes are properly connected to the patient.
- Valid impedance is detected and pacer is off.
- Patient mode is not set to neonate.

---

**Warning!** Use only pediatric electrodes to defibrillate patients under 8 years of age in Advisory mode, and make sure the patient mode is set to pediatric. Use of adult electrodes or adult mode with pediatric patients can result in the delivery of excessive energy doses.

---

## Advisory Defibrillation Procedure

### Determine the Patient's Condition Following Local Medical Protocols

Verify:

- Unconsciousness.
- Absence of breathing.
- Absence of pulse.

### Begin CPR Following Local Medical Protocols

Request additional assistance.

### Prepare Patient

Remove all clothing covering the patient's chest. Dry chest if necessary. If the patient has excessive chest hair, clip or shave it to ensure proper adhesion of the electrodes.

Attach hands-free therapy electrodes according to instructions on the electrode packaging and as described in "Therapy Electrode Application" on page 15-6.

Ensure that the electrodes are making good contact with the patient's skin and are not covering any part of the ECG electrodes.

If therapy electrodes are not making good contact with the patient's skin, the unit issues the message *CHECK PADS* and does not allow delivery of energy.

### 1 Turn on unit

Press the green power switch located on the top of the unit. The green, yellow, and red lights on the top of the unit flash on and off, and the unit displays the message *SELF TEST PASSED*.

#### If Unit is in AED Mode

1. Press the **Manual Mode** quick access key on the front panel of the unit to enter the Manual mode of operation.
2. Using the navigation keys, select the four digits in the Manual Mode pass code. Press **SAVE** when you are finished. Once you have entered your pass code, you will be able to enter Manual mode.

**Note:** If the unit has been not been configured to enter a pass code, the message *Exit to Manual Mode* is displayed. Use the navigation keys to select **Yes** to enter the Manual mode of operation. If you do not press **Yes** within 10 seconds, the unit will revert back to AED operation.

**If Unit is in Manual Mode** – No additional steps are necessary to proceed.

If no hands-free therapy electrodes have been attached to the patient and connected to the Propaq MD, the *ATTACH PADS* message and voice prompt will be issued.

### Energy Select

The default energy selections for adult patients are:

Shock 1 - 120 joules

Shock 2 - 150 joules

Shock 3 - 200 joules

When the device is configured in Pediatric mode, the default energies are:

Shock 1 - 50 joules

Shock 2 - 70 joules

Shock 3 - 85 joules

**Note:** Pediatric defibrillator energy levels should be selected based on site specific protocols.

---

**Warning!** Use only pediatric electrodes to defibrillate patients under 8 years of age in Advisory mode, and make sure the patient mode is set to pediatric. Use of adult electrodes or adult mode with pediatric patients can result in the delivery of excessive energy doses.

---

If medical protocols allow, you may select a different energy level using the energy select up and down arrow buttons on the front panel. The new energy setting displays on the monitor.

## 2 Press ANALYZE Key


---

**Warning!** Keep patient motionless during ECG analysis. Do not touch the patient during analysis. Cease all movement via stretcher or vehicle before analyzing the ECG.

---

Press the **ANALYZE** key to begin the analysis of the patient's ECG rhythm and to determine if a shockable rhythm is present.

An *ANALYZING* message is displayed at the top of the screen for 6 to 12 seconds while the patient's ECG is analyzed. Once the analysis is completed, the unit indicates whether or not a shock is advised.

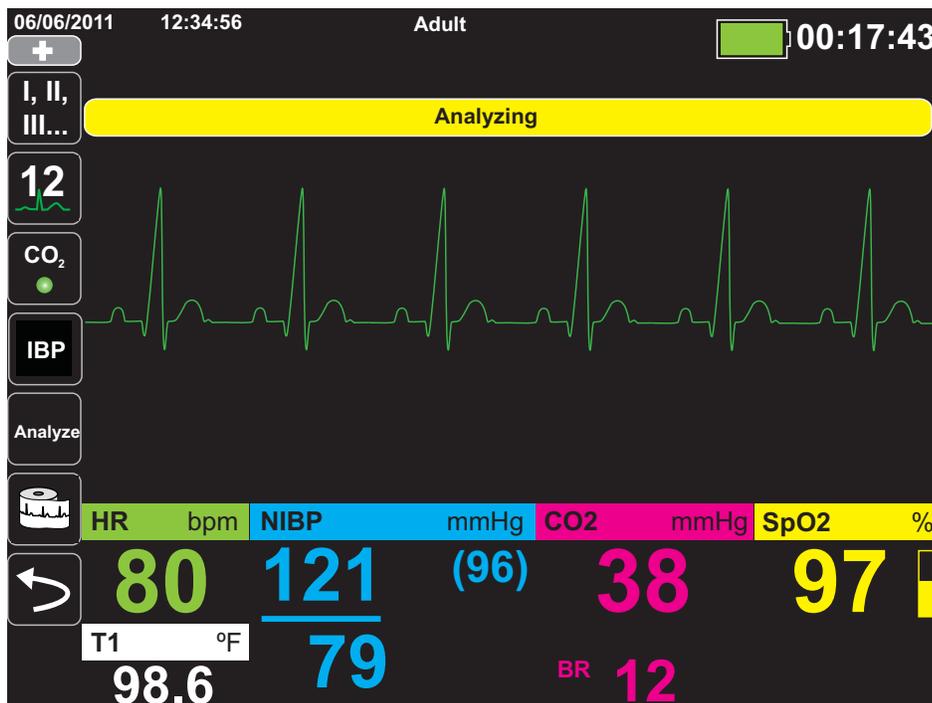

This analysis normally consists of up to three consecutive 3-second ECG rhythm analyses. If at least two of the three analyses determine that the patient has a shockable rhythm, the unit automatically prompts the operator to shock the patient at the preconfigured energy level. If two or more of the three 3-second ECG analyses do not detect a shockable rhythm, the unit alerts the operator that no shock is advised.

If the first two segments are in agreement with each other, the final shock or no shock rhythm decision is made and a third segment is not processed.

---

**Warning!** ECG rhythm analysis does not warn of patient asystole, which is not a shockable rhythm.

---

When a nonshockable rhythm is detected, the unit displays a *NO SHOCK ADVISED* message. Follow the local protocols to continue CPR or other life support, and re-analyze the ECG at appropriate intervals.

**Note:** When a nonshockable rhythm is detected, the Propaq MD does not prevent the user from manually defibrillating the patient.

When a shockable rhythm is detected (ventricular fibrillation or wide-complex tachycardia with heart rate > 150), the unit will display the *SHOCK ADVISED* message. Press the **CHARGE** button, and press the **Select Energy** arrows up or down to select the desired energy level if needed. Press **CHARGE** again to confirm.

Regardless of the analysis result, the user can control the defibrillator manually. For example, the user can defibrillate the patient even if the advisory function issues a *NO SHOCK ADVISED* message.

---

## 3 Press SHOCK

---

**Warning!** Warn all persons in attendance of the patient to **STAND CLEAR** prior to defibrillator discharge.

**Do not touch the bed, patient, or any equipment connected to the patient during defibrillation. A severe shock can result. Do not allow exposed portions of the patient's body to come in contact with metal objects, such as a bed frame, as unwanted pathways for defibrillation current may result.**

---

Press and hold the **SHOCK** button  on the front panel until energy is delivered to the patient.

The delivered energy level is displayed at the bottom of the screen and the shock number (1) displays at the top of the screen and in the Defib Control panel at the bottom of the screen.



**Note:** If at any time you want to cancel the defibrillation, press the **Disarm** softkey.

If the defibrillator is not discharged within 60 seconds after reaching the selected energy level, the unit automatically disarms itself.

If additional countershocks are needed, follow steps 1 through 3 of this procedure starting on page 2, to readjust the energy settings, charge the unit, and deliver shock.

### Perform CPR

Begin chest compressions and rescue breathing per local protocol.

### Repeat Analysis

Press the **ANALYZE** key to restart an ECG analysis and determine if additional shocks are required.

**Note:** Reanalysis of the ECG rhythm is inhibited for 3 seconds after each shock.

### Continue Patient Care

Continue patient care according to medical protocols.



# Chapter 17

## Advisory/CPR Protocol Defibrillation

---



ZOLL hands-free therapy electrodes are a defibrillation-protected Type BF patient connection.

When the Propaq MD unit is configured for Advisory/CPR Protocol defibrillation, the unit guides you through a cardiac event by performing ECG analysis, preparing the device for a shock (if needed), and leading you through a CPR interval. This cycle is repeated as long as Advisory/CPR Protocol is active and pads are attached to the patient. If pads become detached from the patient or shorted during rescue protocol, the protocol halts until the pads are reattached or continues through the CPR interval and then halts and waits for the pads to be reattached.

If you press the Energy Select Up/Down arrow keys or **Charge** button while in Advisory/CPR Protocol mode, the unit transitions to Manual mode. If the unit is charged when you transition from Advisory/CPR Protocol to Manual mode, the unit disarms and halts any analysis that is taking place.

The Advisory/CPR Protocol function can be activated only when:

- The unit is configured for Analysis/CPR Protocol.
- The unit is on and in manual mode.
- Hands-free therapy electrodes are properly connected to the patient.
- Valid impedance is detected and pacemaker is off.
- Patient mode is not set to neonate.

**WARNING!** Use only pediatric electrodes to defibrillate patients under 8 years of age in Advisory/CPR Protocol mode, and make sure the patient mode is set to pediatric. Use of adult electrodes or adult mode with pediatric patients can result in the delivery of excessive energy doses.

---

## Advisory/CPR Protocol Defibrillation Procedure

### Determine the Patient's Condition Following Local Medical Protocols

Verify:

- Unconsciousness.
- Absence of breathing.
- Absence of pulse.

### Begin CPR Following Local Medical Protocols

Request additional assistance.

### Prepare Patient

Remove all clothing covering the patient's chest. Dry chest if necessary. If the patient has excessive chest hair, clip or shave it to ensure proper adhesion of the electrodes.

Attach hands-free therapy electrodes according to instructions on the electrode packaging and as described in "Therapy Electrode Application" on page 15-6.

Ensure that the electrodes are making good contact with the patient's skin and are not covering any part of the ECG electrodes.

If therapy electrodes are not making good contact with the patient's skin, the unit issues the message *CHECK PADS* and does not allow delivery of energy.

## 1 Turn on unit

Press the green power switch located on the top of the unit. The green, yellow, and red lights on the top of the unit flash on and off, and the unit displays the message *SELF TEST PASSED*.

### If Unit is in AED Mode

1. Press the **Manual Mode** quick access key on the front panel of the unit to enter the Manual mode of operation.
2. Using the navigation keys, select the four digits in the Manual Mode pass code. Press **SAVE** when you are finished. Once you have entered your pass code, you will be able to enter Manual mode.

**Note:** If the unit has been not been configured to enter a pass code, the message *Exit to Manual Mode* is displayed. Use the navigation keys to select **Yes** to enter the Manual mode of operation. If you do not press **Yes** within 10 seconds, the unit will revert back to AED operation.

**If Unit is in Manual Mode** – No additional steps are necessary to proceed.

If no hands-free therapy electrodes have been attached to the patient and connected to the Propaq MD unit, the *ATTACH PADS* message and voice prompt will be issued.

### Energy Select

The default energy selections for adult patients are:

Shock 1 - 120 joules

Shock 2 - 150 joules

Shock 3 - 200 joules

The default energy selections for pediatric patients are:

Shock 1 - 50 joules

Shock 2 - 70 joules

Shock 3 - 85 joules

**Note:** Pediatric defibrillator energy levels should be selected based on site specific protocols.

---

**WARNING!** Use only pediatric electrodes to defibrillate patients under 8 years of age in Advisory/CPR Protocol mode, and make sure the patient mode is set to pediatric. Use of adult electrodes or adult mode with pediatric patients can result in the delivery of excessive energy doses.

---


## 2 Press ANALYZE Button

**WARNING!** Keep patient motionless during ECG analysis. Do not touch the patient during analysis. Cease all movement via stretcher or vehicle before analyzing the ECG.

Press the **ANALYZE** button to begin the analysis of the patient's ECG rhythm and to determine if a shockable rhythm is present.

The *ANALYZING ECG* and *STAND CLEAR* messages alternate at the bottom of the screen while the patient's ECG is analyzed. Once the analysis is completed, the unit indicates whether or not a shock is advised.

**Note:** You can terminate the Advisory/CPR Protocol mode and return to Manual mode at any time by pressing the **Exit** quick access key.

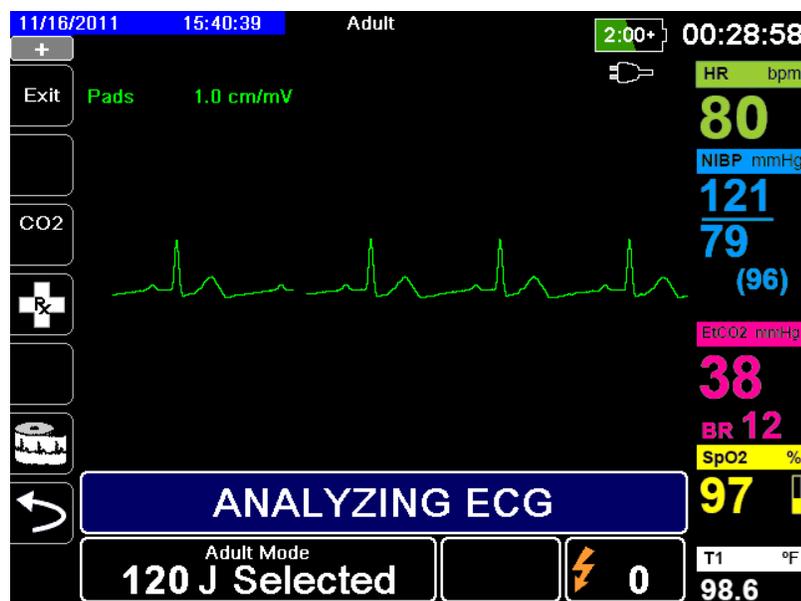

This analysis normally consists of up to three consecutive 3-second ECG rhythm analyses. If at least two of the three analyses determine that the patient has a shockable rhythm, the unit automatically prompts the operator to shock the patient at the preconfigured energy level. If two or more of the three 3-second ECG analyses do not detect a shockable rhythm, the unit alerts the operator that no shock is advised.

If the first two segments are in agreement with each other, the final shock or no shock rhythm decision is made and a third segment is not processed.

**WARNING!** ECG rhythm analysis does not warn of patient asystole, which is not a shockable rhythm.

When you press the Energy Select Up/Down arrow keys or **Charge** button while in Advisory/CPR Protocol mode, the unit transitions to Manual mode. If the unit is charged when you transition from Advisory/CPR Protocol to Manual mode, the unit disarms and halts any analysis that is taking place.

### Non-Shockable Rhythm

When a nonshockable rhythm is detected, the unit displays a *NO SHOCK ADVISED* message. The unit then leads you through a CPR interval and then restarts ECG analysis automatically. The Advisory/CPR Protocol mode repeats the Analysis and CPR intervals as long as Advisory/CPR Protocol is active. You can press the **Exit** quick access key at any time to return to Manual mode.

**Note:** When a nonshockable rhythm is detected, the Propaq MD unit does not prevent the user from manually defibrillating the patient.

### Shockable Rhythm

If the patient's rhythm is shockable, the unit displays the *SHOCK ADVISED* and *PRESS SHOCK* messages. The defibrillator automatically prompts the operator to shock the patient at the preconfigured energy level and the **SHOCK** button illuminates.

A continuous tone sounds for 20 or 50 seconds (depending on configuration), followed by a higher pitch tone for 10 seconds. You must deliver the shock within this 30 or 60 second interval (depending on configuration), or the defibrillator will disarm itself

## 3 Press SHOCK button

---

**WARNING!** Warn all persons in attendance of the patient to **STAND CLEAR** prior to defibrillator discharge.

**Do not touch the bed, patient, or any equipment connected to the patient during defibrillation. A severe shock can result. Do not allow exposed portions of the patient's body to come in contact with metal objects, such as a bed frame, as unwanted pathways for defibrillation current may result.**

---

Press and hold the illuminated **SHOCK** button  on the front panel until energy is delivered to the patient.

The delivered energy level is displayed at the bottom of the screen and the shock number (1) displays at the top of the screen and in the Defib Control panel at the bottom of the screen.



The unit then leads you through a CPR interval and then restarts ECG analysis automatically. The Advisory/CPR Protocol mode repeats the Analysis and CPR intervals as long as Advisory/CPR Protocol is active. You can press the **Exit** quick access key at any time to return to Manual mode.



# Chapter 18

## External Pacing

---



When ZOLL hands-free therapy electrodes are used, the patient connection is considered to be defibrillation-protected Type BF.



ECG leads are a defibrillation-protected Type CF patient connection.

---

**Warning!** Pacing is intended for use on adult patients and on adolescent, child, and infant pediatric patients.

To avoid risk of electrical shock, do not touch the gelled area of the hands-free therapy electrodes while pacing.

Therapy electrodes should be replaced periodically. Consult electrode directions for specific recommendations.

Prolonged pacing (in excess of 30 minutes), particularly in adolescent, child, and infant pediatric patients or adults with severely restricted blood flow, may cause burns. Periodic inspection of the underlying skin is recommended.

When pacing in demand mode, the pacer may be adversely affected from EMI, RFI, or ESU. Move patient away from any potential sources of interference.

---

## External Pacing

Propaq MD defibrillators contain a pacemaker for the emergency treatment of hemodynamically compromising bradycardia, bradycardia with escape rhythms that are unresponsive to pharmacological therapy, refractory tachycardia (supraventricular or ventricular) and bradysystolic cardiac arrest.

Proper demand pacing requires a reliable, high quality surface ECG signal. For best results apply both standard ECG monitoring electrodes and hands-free pacing therapy electrodes to the patient.

### Pacer Modes

The Propaq MD has two pacer mode settings: Demand and Fixed. The default factory mode setting is Demand.

In Demand mode, pacing pulses are inhibited by the patient's QRS complexes that occur during an interval that is dependent on the setting of the rate control. If during that interval no QRS complexes are detected, a pacing pulse is delivered to the patient. In the demand mode, the pacer supplies the required number of pacing pulses to maintain the patient's heart rate at approximately the rate selected in the pacing rate window. See the procedure below, "Pacing in Demand Mode" for more information.

In Fixed mode, pacing pulses are not dependent on the patient's cardiac activity. Fixed pacing should be performed only in an emergency when no alternative is available. The pacer delivers pacing pulses at the selected pacing rate. See "Pacing in Fixed Mode" on page 18-5 for more information.

## Pacing in Demand Mode

### Determine Patient Condition and Provide Care Following Local Medical Protocols.

#### Prepare the Patient

Remove all clothing covering the patient's chest. Dry chest if necessary. If the patient has excessive chest hair, clip it to ensure proper adhesion of the electrodes.

#### 1 Turn Unit On

Press the green power switch located on the top of the unit. The green, yellow, and red lights at the top of the unit flash on and off, and the unit displays the message SELF TEST PASSED.

#### 2 Apply ECG Electrodes/Hands-Free Therapy Electrodes


Apply ECG electrodes, attach lead wires, and connect the ECG cable to the Propaq MD side panel (see Chapter 6, "Monitoring ECG" for instructions on attaching ECG electrodes to the

patient). Attach hands-free therapy electrodes according to instructions on the electrode packaging. Connect these therapy electrodes to the multifunction cable (MFC).

### Therapy Electrode Application

**Warning!** Poor adherence and/or air under the therapy electrodes can lead to the possibility of arcing and skin burns.

1. Apply one edge of the pad securely to the patient.
2. Roll the pad smoothly from the applied edge to the other, being careful not to trap any air pockets between the gel and skin.

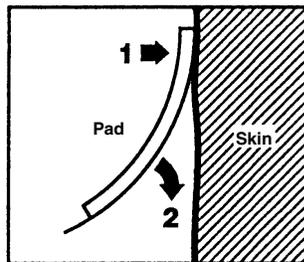


3. Ensure that hands-free therapy electrodes are making good contact with the patient's skin and are not covering any part of any other ECG electrodes.
4. Press the Lead quick access key and select I, II, or III to provide the largest amplitude QRS complex.
 

**Note:** When the Pacer is on, the lead selection is restricted to Leads I, II, or III.
5. Verify that R-waves are being properly detected by confirming that a QRS tone occurs with each displayed R wave or by verifying that the Propaq MD unit's heart rate display accurately reflects the patient's pulse rate.

### 3 Press Pacer button

Press the **Pacer** button on the front panel of the unit. The Pacer Settings window displays.

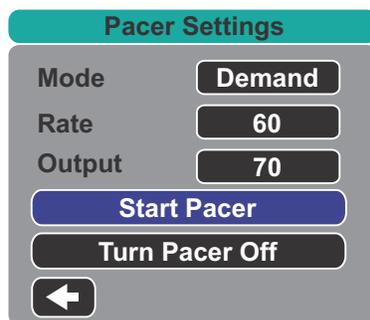

### 4 Set Mode

Use the arrow keys to navigate to Mode, press the Select button, and then use the arrow keys and the Select button to set the Pacer Mode to Demand.

**Note:** The initial pacing mode can be set in Defib/Pacer Default settings in the Supervisor Setup menu.

## 5 Set Pacer Rate

Use the arrow keys to navigate to Rate, press the Select button, and then use the arrow keys and the Select button to set the Pacer Rate to a value 10-20 ppm higher than the patient's intrinsic heart rate. If no intrinsic rate exists, use 100 ppm. You can increase or decrease the pacer rate by a value of 5 ppm for rates below 100, and by 10 ppm for rates above 100.

**Note:** The initial pacing rate can be set in Defib/Pacer Default settings in the Supervisor Setup menu.

## 6 Turn Pacer On

Use the arrow keys to navigate to Start Pacer, then press the Select button to select it. The Pacing window displays behind the Pacer Settings window.

| ● PACING |        |        |
|----------|--------|--------|
| Rate     | Output | Mode   |
| 80 ppm   | 95 mA  | Demand |

## 7 Set Pacer Output

In the Pacer Settings window, use the arrow keys and the Select button to adjust the pacer output. The pacer output is adjustable in 10 mA increments when increasing the output, and in 5 mA increments when decreasing the output. Observe the ECG for evidence of electrical capture. Select the lowest output current that achieves both electrical and mechanical capture.

**Note:** If the Pacer Settings window disappears before you have set the output current, press the Pacer button again to display the settings window.

## 8 Determine Capture

It is important to recognize when pacing stimulation has produced a ventricular response (capture). Determination of capture must be assessed both electrically and mechanically in order to ensure appropriate circulatory support of the patient.

Electrical capture is determined by the presence of a widened QRS complex, the loss of any underlying intrinsic rhythm, and the appearance of an extended, and sometimes enlarged, T-wave.

Ventricular response is normally characterized by suppression of the intrinsic QRS complex.

**Warning!** Determination of electrical capture should only be performed by viewing the ECG trace on the Propaq MD display with its ECG connection directly attached to the patient. Use of other ECG monitoring devices might provide misleading information due to the presence of pacer artifacts.

---

Mechanical capture is assessed by palpation of the peripheral pulse.

To avoid mistaking muscular response to pacing stimuli for arterial pulsations, use ONLY the following locations for palpating pulse during pacing:

- femoral artery
- right brachial or radial artery

### **Effective pacing**

Changing ECG leads and size can sometimes be helpful in determining capture.

**Note:** The shape and size of the paced ECG waveforms can vary depending on the ECG lead configuration chosen; variation from patient to patient can be expected.

## **9 Determine Optimum Threshold**

The ideal pacer current is the lowest value that maintains capture — it is usually about 10% above threshold. Typical threshold currents range from 40 to 80 mA. Location of the hands-free or therapy electrodes affects the current required to obtain ventricular capture. Typically the lowest threshold is obtained when the position of the electrodes provides the most direct current pathway through the heart while avoiding large chest muscles. Lower stimulation currents produce less skeletal muscle contraction and are better tolerated.

## **Pacing in Fixed Mode**

If ECG electrodes are not available or there is some circumstance that prevents or interferes with the surface ECG, the Propaq MD delivers pacemaker pulses at a fixed rate.

Fixed pacing should be performed only in an emergency when no alternative is available.

### **1 Turn Unit On**

Press the green power switch located on the top of the unit. The green, yellow, and red lights at the top of the unit flash on and off, and the unit displays the message SELF TEST PASSED.


### **2 Apply ECG Electrodes/Hands-Free Therapy Electrodes**

Apply ECG electrodes, attach lead wires, and connect the ECG cable to the Propaq MD side panel (see Chapter 6 for instructions on attaching ECG electrodes to the patient). Attach hands-free therapy electrodes according to instructions on the electrode packaging. Connect these therapy electrodes to the MFC.

## Therapy Electrode Application

**Warning!** Poor adherence and/or air under the therapy electrodes can lead to the possibility of arcing and skin burns.

1. Apply one edge of the pad securely to the patient.
2. Roll the pad smoothly from the applied edge to the other, being careful not to trap any air pockets between the gel and skin.

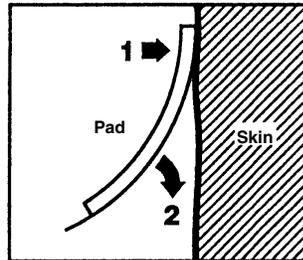


3. Ensure that hands-free therapy electrodes are making good contact with the patient's skin and are not covering any part of any other ECG electrodes.
4. Press the Lead quick access key and select I, II, or III to provide the largest amplitude QRS complex.

**Note:** When the Pacer is on, the lead selection is restricted to Leads I, II, or III.

5. Verify that R-waves are being properly detected by confirming that a QRS tone occurs with each displayed R wave or by verifying that the Propaq MD unit's heart rate display accurately reflects the patient's pulse rate.

### 3 Press Pacer button

Press the **Pacer** button on the front panel of the unit. The Pacer Settings window displays.

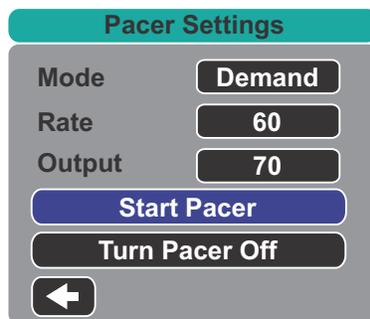

### 4 Set Mode

Use the arrow keys to navigate to Mode, press the Select button, then use the arrow keys and Select button to set the Pacer Mode to Fixed. The selected mode is displayed in the pacing window.

**Note:** The initial pacing mode can be set in Defib/Pacer Default settings in the Supervisor Setup menu.

## 5 Set Pacer Rate

Use the arrow keys to navigate to Rate, press the Select button, then use the arrow keys and the Select button to set the Pacer Rate to a value 10-20 ppm higher than the patient's intrinsic heart rate. If no intrinsic rate exists, use 100 ppm. You can increase or decrease the pacer rate by a value of 5 ppm for rates below 100, and by 10 ppm for rates above 100.

**Note:** The initial pacing rate can be set in Defib/Pacer Default settings in the Supervisor Setup menu.

## 6 Turn Pacer On

Use the arrow keys to navigate to Start Pacer, and press the Select button to select it. The Pacing window displays behind the Pacer Settings window.



## 7 Set Pacer Output

In the Pacer Settings window, use the arrow keys and the Select button to adjust the pacer output. The pacer output is adjustable in 10 mA increments when increasing the output, and in 5 mA increments when decreasing the output. Observe the ECG for evidence of electrical capture. Select the lowest output current that achieves both electrical and mechanical capture.

**Note:** If the Pacer Settings window disappears before you have set the output current, press the Pacer button again to display the settings window.

## 8 Determine Capture

It is important to recognize when pacing stimulation has produced a ventricular response (capture). Determination of capture must be assessed both electrically and mechanically in order to ensure appropriate circulatory support of the patient.

Electrical capture is determined by the presence of a widened QRS complex, the loss of any underlying intrinsic rhythm, and the appearance of an extended, and sometimes enlarged, T-wave.

Ventricular response is normally characterized by suppression of the intrinsic QRS complex.

---

**Warning!** **Determination of electrical capture should only be performed by viewing the ECG trace on the Propaq MD display with its ECG connection directly attached to the patient. Use of other ECG monitoring devices might provide misleading information due to the presence of pacer artifacts.**

---

Mechanical capture is assessed by palpation of the peripheral pulse.

To avoid mistaking muscular response to pacing stimuli for arterial pulsations, use **ONLY** the following locations for palpating pulse during pacing:

- femoral artery
- right brachial or radial artery

### **Effective pacing**

Changing ECG leads and size can sometimes be helpful in determining capture.

**Note:** The shape and size of the paced ECG waveforms can vary depending on the ECG lead configuration chosen; variation from patient to patient can be expected.

## **9 Determine Optimum Threshold**

The ideal pacer current is the lowest value that maintains capture — it is usually about 10% above threshold. Typical threshold currents range from 40 to 80 mA. Location of the hands-free or therapy electrodes affects the current required to obtain ventricular capture. Typically the lowest threshold is obtained when the position of the electrodes provides the most direct current pathway through the heart while avoiding large chest muscles. Lower stimulation currents produce less skeletal muscle contraction and are better tolerated.

## Pediatric Pacing

Noninvasive pacing of pediatric patients is performed in an identical manner to adult pacing. Smaller size pediatric therapy electrodes are available for patients weighing less than 33 lbs/15 kg. If it is necessary to pace for more than 30 minutes, periodic inspection of the underlying skin is strongly advised. Carefully follow all instructions on electrode packaging.

### Pace Fault

The Propaq MD unit may display the following messages when pacing.

| System Message                          | Description  |
|---|--|
| <i>PAUSED</i>                           | The pacer has paused pacing the patient.   |
| <i>PACING</i>                           | The pacer is pacing the patient.   |
| <i>PACING: CHECK THERAPY ELECTRODES</i> | Therapy pads are not connected or are not making good contact with the patient. Apply therapy pads to the patient. |
| <i>ECG FAULT: PACING IN FIXED MODE</i>  | The pacer is pacing in Fixed mode because the ECG lead displayed in the primary waveform is faulted.               |
| <i>PACING: SHORT DETECTED</i>           | The pacer output is short circuited due a test plug connection or a device/MFC fault.                              |



# Chapter 19

## Patient Data

---

This chapter describes procedures for storing, viewing, and transferring patient data from the Propaq MD to an external system, such as a personal computer or handheld device.

**Note:** Before deleting log files from the Propaq MD, view the files on a personal computer to verify that they have been successfully transferred.

**Note:** Data transfers will not be successful if the USB flash storage device is full or does not have sufficient memory available. Always use a USB flash storage device that has at least 128 MB available.

---

**WARNING!** Do not connect non-isolated equipment to the USB port while monitoring a patient.

---

### Storing Data

The Propaq MD unit stores a combination of 24 hours (at one-minute intervals) of patient trend information, one thousand time-stamped events, and 32 snapshots (24 second duration) for the patient being monitored. The actual number of trends, events, or snapshots stored could be more or less depending on the use profile. When the data storage is at capacity, the log becomes full and the patient data must be cleared or transferred before the Propaq MD unit can store additional patient data. Data will not be lost if the Propaq MD unit is turned off, or if the battery or auxiliary power adapter is removed.

## Capturing a Data Snapshot

Press the Snapshot button () on the front panel to capture a 24-second period of numeric and waveform patient data. The unit captures 12 seconds proceeding and 12 seconds following the button press.

The Propaq MD unit can store a minimum of 32 snapshots, including

- Monitor snapshots
- Defibrillator snapshots
- Treatment snapshots
- Pacer snapshots
- Alarm snapshots

## Reviewing and printing snapshots

You can perform this procedure from any main display screen.

1. Press the Home/Display button () repeatedly until the Trends window displays.
2. Use the navigation keys to get to the Trends window.
3. Press Select button to scroll through the list of trends.

**Note:** The snapshots have a snapshot icon next to the time stamp.

4. Select the desired snapshot from the Trends list and press Select. The snapshot numeric data displays.
5. To print the data and the snapshot waveforms, highlight **Print this Snapshot** and press Select.

## Treatment Summary Report

The Treatment Summary Report is a report that shows all of the treatment events related to a patient such as defibrillation/pacer events, alarm events, and treatment snapshots. It is helpful to print out this report at the end of a case.

| TREATMENT SUMMARY REPORT  |                     |          |                            |
|---------------------------|---------------------|----------|----------------------------|
| Name: John Doe            |                     | 09:14:14 | New Case ID                |
| ID: Patient 0015          | Patient Mode: Adult | 09:14:14 | System On                  |
| Start Time: 09/14/10      | 09:14:14            | 09:14:14 | Patient Mode Adult         |
| Last Event: 09/14/10      | 09:43:40            | 09:14:14 | Some Alarm Limits Disabled |
| Elapsed Time: 00:29:48    | # Events:           | 09:14:20 | Self Test Passed           |
| Total Shocks: 1           |                     | 09:14:43 | Treatment Snapshot: ASA    |
| Total Pace Time: 00:00:00 |                     |          |                            |
| Total 12 Leads: 0         |                     |          |                            |
| Dept: ICU                 |                     |          |                            |
| Unit:                     |                     |          |                            |
| S/N: 0 SW: 00.00.00.00    |                     |          |                            |

## Printing Treatment Summary Report


To print a treatment summary report, do the following:

1. Press the More quick access key ().
  1. Press the Log quick access key ().
  2. Press the Treatment Summary quick access key (). The Treatment Summary window displays.
  3. Use the navigation keys to highlight and select the treatment summary you want to print.
- Note:** The Treatment Summary Report that you selected has a check mark next to it.
4. Use the navigation keys to highlight and select **Print Treatment Summary**.

## Transferring Data to a USB Device

You can transfer patient data from the unit using a USB transfer device.

Before you begin, insert a USB device into the Propaq MD USB device port.

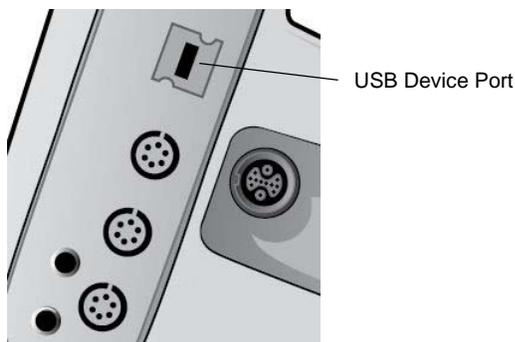

**Figure 19-1. USB Port**

To transfer data through the USB port:

1. Press the power switch to turn the unit on.
2. Press .
3. Press .
4. Press the Transfer Log quick access key () (ensure that the USB drive is connected to the device).

**Note:** Do not remove the USB data drive from the Propaq MD unit during transfer.

5. Use the navigation keys to select **Transfer** from the Transfer the Log menu.
6. The green LED on the top of the device turns on while data is transferred to the USB device.

**Note:** Wait for the log transfer to complete and for the green light on the top of the Propaq MD to turn off before removing the USB drive.

**Note:** Printing and logging snapshots is disabled during the log transfer.

After transferring data, you must remove the USB drive and reinsert it before attempting another transfer. If the USB device does not establish communication with the Propaq MD unit, try powering off the unit and then on again to establish communication.

## Clearing the Log

You should clear the patient data log after transferring data to the USB device or when the log is full.

**Note:** Clearing the log during patient treatment results in the loss of all patient data and events recorded prior to clearing the log. Clearing the log creates a New Patient record and all patient-specific parameters (alarm limit, defibrillation energy, etc.) are set to their default values.

To clear the log:

1. Press .
2. Press .
3. Press the Clear Log quick access key (.
3. Use the navigation keys to select **Yes**.

---

**Warning!** To avoid a possible shock hazard, do NOT make any electrical connections to the USB port except to connect a USB flash drive while in the vicinity of the patient.

---

# Chapter 20

## Printing

---

This chapter describes procedures for printing permanent records of pertinent patient care data and events.

---

**WARNING!** To avoid risk of electrical shock, do not touch the patient and internal parts of the printer simultaneously while the printer door is open.

---

### Printing Patient Data

The printing option allows you to print the following patient information:

- Waveforms
- Reports
- Trends

**Note:** A date that contains question marks (??/??/??) indicates that the Propaq MD was not able to determine the date and time on power up. Power cycling the unit may correct the problem. If the problem persists, set the date and time by highlighting the **Date and Time** display field and pressing Select.

## Printer Setup

To configure the printer functions, press the More quick access key () , press the Setup quick access key () , highlight **Printer**, and press Select. Use the navigation keys to select the desired print configuration. The Setup>Printer window contains the following print configuration options:

| Print configuration    | Options                   |
|------------------------|---------------------------|
| Print Speed            | 25 mm or 50 mm per second |
| Print Number of Traces | 1, 2, or 3                |
| Print Grid             | On or Off                 |
| Print on Snapshot      | On or Off                 |
| Print on Patient Alarm | On or Off                 |
| Print on NIBP          | On or Off                 |
| Print on Trend         | On or Off                 |

## Automatic Prints

You can print any following events by setting the monitor to print automatically in the Setup>Supervisor>Printer window. To do this, press () , then press () . Using the navigation buttons, select **Supervisor**, and then **Printer**.

- Defib/Pacer Events
- Event
- Treatment Snapshot
- Treatment Summary Snapshots

## Printing Waveforms

You can print waveforms by pressing the Print quick access key () . The Propaq MD unit will print the displayed waveforms as configured in the Number of Traces option. The Print Number of Traces option can be set to **1**, **2**, or **3**. Patient vital sign numerics are printed above the waveforms.

**Note:** Waveforms that have invalid data print a dashed line for that trace.

### The ECG Waveform

ECG waveforms are printed on a grid with major divisions (dotted lines) every 5mm and minor divisions (single dots) every 1 mm. The ECG waveform is always printed if ECG is monitored.

### The Invasive Pressure Waveform

Pressure waveforms are printed on a grid with major divisions (vertical dotted lines) every 5 mm, and the pressure scale grids are printed horizontally.

### The SpO<sub>2</sub> Plethysmograph

The SpO<sub>2</sub> plethysmograph is printed with major divisions (vertical dotted lines) every 5 mm.

### CO<sub>2</sub> Waveform

CO<sub>2</sub> waveforms are printed on a grid with major divisions (vertical dotted lines) every 5 mm, and the pressure scale grids are printed horizontally.

### RESP Waveform

Respiration waveforms are printed with major divisions (vertical dotted lines) every 5 mm.

## Printing Reports

### Print on Snapshot

Prints snapshot, including numerics and waveform each time a snapshot is acquired.

### Print on Patient Alarm

You can print a report each time an alarm occurs. This report is called the Print on Patient Alarm Report. The **Print on Patient Alarm** setting must be turned on in the Setup>Print window.

### Print on NIBP

You can print the results of an NIBP measurement each time one occurs. This report is called the NIBP Ticket Report. The **Print on NIBP** setting must be turned on in the Setup>Print window.

### Print on Trend

Prints numeric trend values each time a trend data snapshot is taken.

### Printing Treatment Summary Report

To print a treatment summary report, do the following:

1. Press the More quick access key (.
  1. Press the Log quick access key (.
  2. Press the Treatment Summary quick access key (). The Treatment Summary window displays.
  3. Use the navigation keys to highlight and select the treatment summary you want to print.
- Note:** The Treatment Summary Report that you selected has a check mark next to it.
4. Use the navigation keys to highlight and select **Print Treatment Summary**.

## Printing Trends

Printed trends are useful for reviewing the patient's vital signs over the last several minutes to the last five hours. The Propaq MD unit enables you to print vital signs data at one selected time or a trend summary showing vital signs values acquired during the current case (up to the last 24 hours).

### Printing Trend Summary

Follow the steps below:

1. Use the navigation keys to highlight **Trends** and press Select.
2. Highlight **Print Trend Summary** and press Select. The Trend Summary Report begins printing and Print Summary field changes to Cancel Report.
3. To cancel the report, highlight **Cancel Report** and press Select.

### Printing Individual Trend Snapshots

Follow the steps below:

1. Highlight Press the Select Button to Start Scrolling at the bottom of the Trends window, then press Select.
2. Using the navigation keys, select the trend snapshot you want to print and press Select.
3. Highlight **Print this Trend** and press Select. The trend snapshot begins printing.

### Printing Trend Data for Current or Older Cases

You can print a trend summary for one or more cases by doing the following:

1. Press .
2. Use the navigation keys to select the case or cases you want to print.
3. Press Select. The case(s) appears with a check mark.
4. Highlight the **Print Trend Summary** field and press Select. The trend data for all the selected cases begins printing.

# Chapter 21

## Maintenance

---

Resuscitation equipment must be maintained to be ready for immediate use. To ensure the readiness and optimum working condition of the Propaq MD unit, you should perform the following inspections and tests daily or at each shift change.

In addition to the daily check, authorized personnel should complete performance and calibration testing at regularly scheduled intervals, which should not exceed one year.

A maintenance log is an important part of a successful maintenance program in which you record information on a regular basis. This allows for verification of necessary maintenance and for scheduling periodic requirements such as calibration and certification.

In accordance with the recommendations of the Defibrillator Working Group of the Food and Drug Administration, an operator's shift check list is included in this chapter, which you can copy for use as needed.

# Daily/Shift Check Procedure

## Inspection

### Equipment and Accessories

- Ensure that the Propaq MD unit is clean (with no fluid spills) and free of visible damage.
- Inspect all cables, cords, and connectors for signs of damage or excessive wear (no cuts in insulation, fraying, broken wires, or bent connector pins). Replace if damaged.
- Inspect the battery, ECG patient cable, multifunction cable (MFC), pulse oximetry sensor, blood pressure cuff, and hose and temperature sensors for signs of damage or excessive wear. Replace if damaged.
- Inspect the defibrillator paddles for damage or excessive wear. Check that paddle surfaces are clean and free of electrolyte gel or other contaminants. Verify that all paddle controls operate freely when pressed and released.
- Inspect the external AC adapter and its power cords for signs of damage. Replace if damaged.

### Supplies

- Verify the presence, proper condition, and appropriate quantities of all disposable supplies (hands-free therapy electrodes, ECG monitoring electrodes, defibrillator gel, recorder paper, alcohol swabs, razors/scissors, etc.).
- Verify that two sets of hands-free therapy pads are available.
- Ensure that the therapy and monitoring electrodes are sealed within the packages and within the expiration dates printed on the packages.
- Open the recorder door on the left side of the Propaq MD unit and verify that an adequate supply of paper is available in the unit.

### Batteries

- Verify that a charged battery is fully inserted into the battery compartment of the Propaq MD unit.
- Ensure that a fully charged spare battery pack is available.

### Ready For Use Indicator

- Check the Ready For Use (RFU) indicator on the Propaq MD front panel. Verify that the RFU symbol is displayed.

If the RFU indicator is flashing or displaying the Do Not Use symbol, then either an equipment fault has been detected or the unit's battery is low or not properly installed. Install a fully charged battery in the unit and check the RFU indicator again. If the RFU indicator continues to flash or display the Do Not Use symbol, remove the unit from service and contact the appropriate technical personnel or the ZOLL Technical Service Department.

# Defibrillator/Pacing Test with Hands-Free Therapy Electrodes

**Note:** If a low battery message appears during any of this testing, the battery is close to depletion and should be replaced or recharged.

**Warning!** **Keep hands, fingers, and other conductive materials away from paddle electrode plates when discharging the defibrillator, or pacing.**

|   | Function   | Response   |
|---|--|--|
| 1 | Connect the external AC adapter to a working AC outlet and to the Propaq MD rear panel.  | Verify that the green Auxiliary Power LED illuminates on the Propaq MD front panel.  |
| 2 | Insert a battery into the unit (if a battery is not already in the slot).  | Verify that the Battery Charge LED on the Propaq MD front panel illuminates yellow. When the battery pack has been fully charged, the battery status indicator light is green.   |
| 3 | Press the unit's power on switch.  | <p>Verify the unit issues two audio beeps and that the red, yellow, and green Visual Alarm indicators illuminate for 2 to 3 seconds.</p> <p><b>Note:</b> The yellow LED may remain lit after the unit powers on.</p> <p>The Propaq MD unit performs a number of power on self tests.</p> <p>Verify that the unit displays <i>SELF TEST PASSED</i>.</p> <p>If the message <i>SELF TEST FAILED</i> appears on the display, contact the appropriate technical personnel or the ZOLL Technical Service Department.</p> |
| 4 | Disconnect the external AC adapter from the Propaq MD rear panel.  | <p>Verify that the unit continues to operate without interruption on battery power and that the displayed battery icon indicates that a fully charged battery is installed.</p> <p>Verify that no error messages are displayed.</p>  |
| 5 | With no ECG cables attached to the unit, select Lead I, II, or III as the waveform display source.   | Verify that a <i>LEAD FAULT</i> message displays in the waveform window.   |
| 6 | Select <b>Pads</b> as the waveform display source and connect the multifunction cable to the unit. (Do not connect the test connector to the MFC.) | Verify that a <i>CHECK THERAPY ELECTRODES</i> message displays in the waveform window.   |
| 7 | Connect the multi-function cable (MFC) to the test connector.  | Verify that the <i>CHECK THERAPY ELECTRODES</i> message is replaced by a <i>SHORT DETECTED</i> message.  |

|    | Function  | Response   |
|----|---|--|
| 8  | Press the Select Energy arrow (up or down).   | Verify that the defibrillation window displays and the following message appears:<br><i>PADS SHORT DETECTED</i><br><i>SELECT 30J TO TEST</i>   |
| 9  | Press the Select Energy arrow (up or down) to select 30 Joules on the Propaq MD unit.   | Verify that the defibrillator window shows 30 J as the selected energy.  |
| 10 | Press the <b>Charge</b> button on the front panel.  | Verify that a periodic tone sounds while the defibrillator is charging. At the completion of the charge cycle, the tone is continuous. The charge time should be less than 7 seconds.<br><br>Verify that the Shock button (  ) on the front panel illuminates when the defibrillator is charged.                    |
| 11 | Press and hold  on the front panel until the unit discharges.  | Verify that the defibrillator discharges and that a <i>DEFIB SHORT TEST PASSED</i> message displays.<br><br>If the message <i>DEFIB SHORT TEST FAILED</i> displays, contact the appropriate technical personnel or the ZOLL Technical Service Department.  |
| 12 | After the defibrillator display panel has been removed, press the <b>Pacer</b> button on the front panel to display the Pacer Settings dialog box.  | Verify that the Pacer Settings dialog displays with the pacer status bar behind it.  |
| 13 | Perform the following steps: <ul style="list-style-type: none"> <li>• Set the Pacer mode to Fixed.</li> <li>• Set the Pacer Rate to 60 ppm.</li> <li>• Set the Output to 100 mA.</li> <li>• Select the Start Pacer button.</li> </ul> | Verify that the green pace indicator on the pacer status bar flashes once each second.<br><br>Verify that pace markers are printed every 25 mm on the chart recorder printout.<br><br><b>Note:</b> If a printout does not automatically begin after starting the pacer, select the Print quick access key to start a continuous printout. Press the Print quick access key again to stop the printout. |
| 14 | Press the <b>Pacer</b> button on the front panel button again to close the Pacer Settings dialog box.   | Verify that the pacer status bar indicates <i>PACING: SHORT DETECTED</i> .<br><br>Verify that the green pace indicator continues to flash once each second.<br><br>Verify that no pacer-related error messages display.  |
| 15 | Disconnect the test connector from the MFC.   | Verify that the pacer status bar displays:<br><i>PACING: CHECK THERAPY ELECTRODES</i>  |
| 16 | Press the <b>Pacer</b> button on the front panel to display the Pacer Settings dialog. Select <b>Turn Pacer Off</b> to turn off the pacer.  | Verify that the Pacer Settings dialog is closed and that the pacer status bar is removed from the display.   |
| 17 | Reconnect the external AC Adapter to the Propaq MD rear panel.  | Verify that the green Auxiliary Power indicator light on the Propaq MD front panel is illuminated.   |

## Defibrillator Testing with External Paddles

Prior to testing external defibrillator paddles with the Propaq MD unit, complete the testing described in “Defibrillator/Pacing Test with Hands-Free Therapy Electrodes” on page 21-3.

**Note:** If a low battery message appears during any of this testing, the battery is close to depletion and should be replaced or recharged.

---

**Warning!** **Keep hands, fingers, and other conductive materials away from paddle electrode plates when discharging the defibrillator, or pacing.**

---

|    | Function   | Response   |
|----|--|--|
| 1  | Press the Propaq MD power switch to turn the unit on.  | Verify that the unit displays the <i>SELF TEST PASSED</i> message.   |
| 2  | Connect the MFC to the defibrillator and disconnect the patient end of the MFC from any attached accessory (paddles or test connector). Select <b>Pads</b> as the waveform display source. | Verify that a <i>CHECK THERAPY ELECTRODES</i> message displays in the waveform window  |
| 3  | Connect the paddles set to the Propaq MD MFC cable. Do not place the paddles electrodes in contact with each other.  | Verify that the <i>CHECK THERAPY ELECTRODES</i> message changes to <i>APPLY PADDLES TO PATIENT</i> .   |
| 4  | Press the <b>RECORDER ON/OFF</b> button on the Sternum paddle.   | Verify that the recorder begins printing.  |
| 5  | Press the <b>RECORDER ON/OFF</b> button again.   | Verify that the recorder stops printing.   |
| 6  | Press the Apex and Sternum paddle electrodes together (face to face) maximizing the contact area between the electrodes.   | Verify that the <i>APPLY PADDLES TO PATIENT</i> message changes to <i>SHORT DETECTED</i> .   |
| 7  | Press the <b>ENERGY SELECT (+)</b> button on the Sternum paddle.   | Verify that the defib window displays and the following message appears:<br><br><i>PADDLE SHORT DETECTED</i><br><br><i>SELECT 30 J TO TEST</i>   |
| 8  | Press the <b>ENERGY SELECT (+)</b> button again.   | Verify that the selected energy increases to the next highest level.   |
| 9  | Repeatedly press the <b>ENERGY SELECT (-)</b> button on the Sternum paddle until 30 Joules is selected.  | Verify that the defibrillator window shows 30 J as the selected energy.  |
| 10 | Press the <b>CHARGE</b> button on the Apex paddle. (Press the <b>CHARGE</b> button twice if the defibrillator window has disappeared.)   | Verify that a periodic tone sounds while the defibrillator is charging. At the completion of the charge cycle, the tone is continuous. The charge time should be less than 7 seconds.<br><br>Verify that the <b>CHARGE</b> light on the Apex handle illuminates when the defibrillator is charged and ready to deliver energy. |
| 11 | While the defibrillator is charged, press the paddles electrodes together and then press and hold the <b>SHOCK</b> button on the Apex paddle (only).                                       | Verify that the defibrillator does not discharge.  |

|    | <b>Function</b>   | <b>Response</b>   |
|----|---|---|
| 12 | While the defibrillator is still charged, press the paddles electrodes together then press and hold the <b>SHOCK</b> button on the Sternum paddle (only).           | Verify that the defibrillator does not discharge.   |
| 13 | While the defibrillator is charged, press the paddles electrodes together then simultaneously press and hold both the Apex and Sternum paddle <b>SHOCK</b> buttons. | Verify that the defibrillator discharges and that a <i>DEFIB SHORT TEST PASSED</i> message displays.<br><br>If the message <i>DEFIB SHORT TEST FAILED</i> displays, contact the appropriate technical personnel or the ZOLL Technical Service Department. |

## Recommended Minimum Preventive Maintenance Schedule

Operational Tests should be performed at regular intervals. The Operational Tests augment the automated self-tests that the Propaq MD unit performs to help ensure readiness. For more information, see the Operational Tests in the *Propaq MD Service Manual*.

### Annually

In addition to Daily/Shift Check Procedure and Operational Tests, the following is recommended:

- Perform NIBP calibration check.
- Perform CO<sub>2</sub> calibration check.
- The NIBP and CO<sub>2</sub> calibration checks should be performed annually or according to local requirements (to be performed by a qualified Biomedical Equipment Technician (BMET)) as described in the *Propaq MD Service Manual*.

## Guidelines for Maintaining Peak Battery Performance

- Each battery should be identified with a number or letter. An identification mark is useful in tracking battery performance.
- Keep extra batteries in the *SurePower* Charging Station where their status can be quickly determined. Illumination of the Ready light is the most positive indication of a fully charged battery.
- Always carry at least one fully charged spare battery. If no other source of back-up power is available, two spare batteries are advisable.
- Rotate spare batteries routinely. The charge level gradually diminishes in a battery after it is removed from the charger even if it is not used. Regular battery rotation helps to avoid incidents where a low battery condition is encountered because the battery has not been recharged or used in more than 30 to 90 days.
- Whenever possible, recharge a partially depleted battery. This can be accomplished following any incident that involves patient monitoring. It ensures maximum operating time for each use, without reliance on spares. The need for a spare can then serve as an alert when an aging battery fails to provide normal operating time.
- Keep discharged batteries separated from spare batteries that are charged. When removing a discharged battery from the monitor, never place it in the location intended to carry a charged spare.

---

**Caution** DO NOT leave Propaq MD battery packs in a depleted state. Damage to the battery packs can occur if they are left in a depleted state for more than 30 days.

---

## Cleaning instructions

### Cleaning the Propaq MD unit

To clean the Propaq MD unit, use a nearly dry cloth containing one of the mild cleaning agents listed below. DO NOT allow cleaning agent or water to run into the crevices or connector openings at any time. Thoroughly wipe off any excess cleaning solution from the Propaq MD unit with a dry cloth.

Always check monitor and connector opening for unusual wear, damage or dampness while cleaning.

Use only these recommended cleaning agents:

- Warm water
- Hydrogen peroxide solution
- Liquid soap
- Wex-cide<sup>®</sup>
- Windex<sup>®</sup>
- T.B.Q.<sup>®</sup>

**Never** use these cleaning agents:

- Butyl alcohol
- Denatured ethanol
- Freon
- Mild chlorine bleach solution
- Isopropyl alcohol
- Trichloroethane, trichloroethylene
- Acetone
- Vesphene II
- Enviroquat
- Staphene
- Misty
- Glutaraldehyde

### Cleaning the NIBP Blood Pressure Cuff

Clean the cuff with common hospital disinfectants, including Clorox<sup>®</sup> (1:10 solution), isopropyl alcohol, Lysol<sup>®</sup> solution, PhisoHex<sup>®</sup>, Quatricide<sup>®</sup>, Virex<sup>®</sup> and Vesphene<sup>®</sup>. Wash gently with the solution, then rinse. DO NOT allow solution to enter cuff tubes. Allow the cuff and hose to completely dry before patient use.

## Cleaning SpO<sub>2</sub> Sensors

Clean the SpO<sub>2</sub> sensors with a cloth that has been slightly dampened with one of the recommended agents listed above. DO NOT submerge the probe or its connector in any liquids or cleaning agents. Thoroughly wipe off any excess cleaning solution with a dry cloth. Allow the sensor to completely dry before patient use.

## Cleaning Cables and Accessories

Cables, cuff tubing, paddles and other accessories can be wiped clean with a damp cloth moistened in a mild detergent solution.


## Loading Recorder Paper

The unit displays the message *PRINTER OUT OF PAPER* when the printer is activated without recorder paper or if the supply runs out during printing.

Use ZOLL recorder paper (part number 001739-U).


To load the recorder paper into the printer:

1. Insert your fingers into the indentations on the sides of the printer door, pull the door open, then remove any paper from the spool.

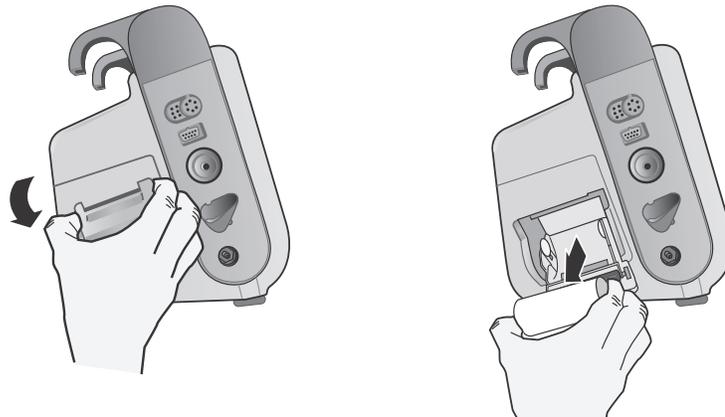

**Figure 21-1. Opening the Printer Door and Removing Paper**

2. Refer to the illustration inside of the paper compartment for proper paper orientation, and place a new roll of recorder paper on the empty spool.
3. Pull enough paper off the spool so that paper extends out of the unit when the printer cover is closed.
4. Close the printer door. Be sure the cover is flush with the side of the unit.

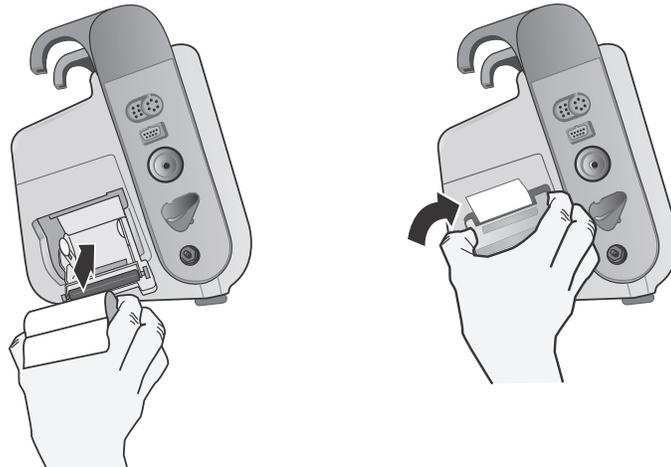

**Figure 21-2. Inserting the Paper and Closing Printer Door**

5. After the paper is loaded, press the Print quick access key (  ) to resume printing.

## Cleaning the Print Head

To clean the recorder print head, perform the following steps:

1. Pull the printer door open (see Fig. 21-1), then remove any paper.
2. Locate the print head along the front floor of the printer compartment.
3. Gently wipe the print head with a cotton swab moistened with isopropyl alcohol, and dry any residual alcohol with another dry cotton swab.



**Figure 21-3. Cleaning the Print Head**

4. Place the paper back into the unit and close the cover (see Fig. 21-2).

### Propaq MD OPERATOR'S SHIFT CHECKLIST

Date: \_\_\_\_\_ Shift: \_\_\_\_\_ Location: \_\_\_\_\_  
Mfr/Model No.: \_\_\_\_\_ Serial No. or Facility ID No.: \_\_\_\_\_

At the beginning of each shift, inspect the unit. Indicate whether all requirements have been met.  
Note any corrective actions taken. Sign the form.

|   | Okay as found | Corrective Action/Remarks |
|---|---------------|---------------------------|
| <b>1. Defibrillator Unit</b>  |               |                           |
| Clean, no spills, and casing undamaged  |               |                           |
| <b>2. Cables/Connectors</b>   |               |                           |
| a. Inspect for damaged insulation, frayed/broken wires, or bent connector pins<br>b. Connectors engage securely   |               |                           |
| <b>3. Sensors</b> (pulse oximetry, NIBP cuff and hose, temperature sensors)   |               |                           |
| a. Inspect for signs of damage or excessive wear<br>b. Connectors engage securely   |               |                           |
| <b>4. Paddles</b>   |               |                           |
| a. Clean, not pitted, or damaged.<br>b. Switches operate freely<br>c. Cables and connectors free of damage and engage securely  |               |                           |
| <b>5. Supplies and Accessories</b>  |               |                           |
| a. Therapy pads in sealed pouches (2 sets, not expired)<br>b. Defib gel or gel pads<br>c. ECG monitoring electrodes<br>d. Alcohol wipes<br>e. Razors/scissors<br>f. Recorder paper                                |               |                           |
| <b>6. Batteries</b>   |               |                           |
| a. Fully charged battery installed in unit<br>b. Fully charged spare battery available  |               |                           |
| <b>7. Ready For Use Indicator shows Ready</b> - No flashing or Do Not Use symbol  |               |                           |
| <b>8. Operational Checks</b>  |               |                           |
| <b>A. Power Up Sequence</b>   |               |                           |
| a. Battery charge and AC indicators illuminate with AC power<br>b. Audio beeps heard and Visual Alarm indicators briefly illuminate<br>c. Self Test passed  |               |                           |
| <b>B. Hands Free Defibrillation (Test with only battery power)</b>  |               |                           |
| a. <i>CHECK THERAPY ELECTRODES</i> and <i>PADS SHORT DETECTED</i> messages display<br>b. Charge time < 7 seconds<br>c. <i>DEFIB SHORT TEST</i> passed at 30 Joules  |               |                           |
| <b>C. Pacer Test (Test with only battery power)</b>   |               |                           |
| a. Printer prints Pace markers every 25 mm at 60 ppm<br>b. <i>PACING: SHORT DETECTED</i> message displays-- no error messages at 100 mA<br>c. <i>PACING: CHECK THERAPY ELECTRODES</i> message displays            |               |                           |
| <b>D. Paddles</b>   |               |                           |
| a. <i>APPLY PADDLES TO PATIENT</i> message displays when paddles are connected to MFC<br>b. Paddle switches functional (Recorder, Energy Select, Charge, Shock)<br>c. <i>DEFIB SHORT TEST</i> passed at 30 Joules |               |                           |
| <b>E. Reconnect Unit to AC Power</b>  |               |                           |
| _____ <b>Major problem(s) identified (OUT OF SERVICE)</b>   |               |                           |

Signature \_\_\_\_\_



# Appendix A

# Specifications

---

This chapter provides specification information for the Propaq MD Monitor/Defibrillator.

- “Defibrillator” on page A-2.
- “Monitor/Display” on page A-14
- “Impedance Pneumography” on page A-15
- “Alarms” on page A-15
- “Recorder” on page A-16
- “Battery ” on page A-17
- “General” on page A-17
- “Pacer” on page A-18
- “CO2” on page A-19
- “Pulse Oximeter” on page A-20
- “Non-Invasive Blood Pressure” on page A-20
- “Invasive Pressures” on page A-22
- “Temperature” on page A-22
- “Electromagnetic Compatibility Guidance and Manufacturer’s Declaration” on page A-27

## Defibrillator

### Charge Time:

- Less than 7 seconds with a new, fully charged battery (first 15 charges to 200 joules).
- For the sixteenth discharge at maximum energy, the charge time is less than 10 seconds. Depleted batteries result in a longer defibrillator charge time.
- Less than 15 seconds when operating without a battery, using AC power alone at 90% of the rated mains voltage.
- Less than 25 seconds from the initial power on, with a new, fully charged battery pack (depleted by up to fifteen 200 joule discharges) or when operating without a battery, using AC power alone at 90% of the rated mains voltage.

### Rhythm Analysis and Charge Time in AED Mode

- Less than 30 seconds with a new, fully charged battery (first 15 charges to 200 joules).
- For the sixteenth discharge at maximum energy, the analysis and charge time is less than 30 seconds. Depleted batteries result in a longer defibrillator charge time.
- Less than 30 seconds when operating without a battery, using AC power alone at 90% of the rated mains voltage.
- Less than 40 seconds from the initial power on, with a new, fully charged battery pack (depleted by up to fifteen 200 joule discharges) or when operating without a battery, using AC power alone at 90% of the rated mains voltage.

**Patient Impedance Range:** 10- 300 ohms

**Synchronized Mode:** Synchronizes defibrillator discharge to the patient's R wave. SYNC is indicated on the display with R wave markers above the ECG waveform on the screen and stripchart. When ECG is monitored by the device, meets the DF-80:2003 requirement of 60ms maximum time delay between the peak of the R wave and the delivery of energy.

Table A-1 shows the characteristics of the Propaq MD Rectilinear Biphasic™ waveform when discharged into 25 ohm, 50 ohm, 100 ohm, 125 ohm, 150 ohm and 175 ohm loads at the maximum energy setting of 200 joules.

**Table A-1. Propaq MD Rectilinear Biphasic Waveform Characteristics**

|   | 200 J discharged into |        |        |        |        |        |
|---|-----------------------|--------|--------|--------|--------|--------|
|   | 25Ω                   | 50Ω    | 100Ω   | 125Ω   | 150Ω   | 175Ω   |
| <b>First phase</b>  |                       |        |        |        |        |        |
| Maximum initial current   | 31.4 A                | 30.4 A | 19.7 A | 19.4 A | 16.7 A | 15.6 A |
| Average current   | 27.1 A                | 24.9 A | 17.5 A | 16.2 A | 14.4 A | 13.2 A |
| Duration  | 6 ms                  | 6 ms   | 6 ms   | 6 ms   | 6 ms   | 6 ms   |
| <b>Interphase duration</b><br>(between first and second phases) |                       |        |        |        |        |        |
|   | 200 μs                | 200 μs | 200 μs | 200 μs | 200 μs | 200 μs |
| <b>Second phase</b>   |                       |        |        |        |        |        |
| Initial current   | 29.2 A                | 18.8 A | 15.1 A | 13.2 A | 12.1 A | 11 A   |
| Average current   | 14.7 A                | 13 A   | 12.5 A | 11.3 A | 10.7 A | 9.9 A  |
| Duration  | 4 ms                  | 4 ms   | 4 ms   | 4 ms   | 4 ms   | 4 ms   |

**Table A-2. Delivered Energy at Every Defibrillator Setting into a Range of Loads**

| Selected Energy | Load  |       |       |       |       |       |       | Accuracy* |
|-----------------|-------|-------|-------|-------|-------|-------|-------|-----------|
|                 | 25Ω   | 50Ω   | 75Ω   | 100Ω  | 125Ω  | 150Ω  | 175Ω  |           |
| 1 J             | 1 J   | 1 J   | 1 J   | 1 J   | 1 J   | 1 J   | 1 J   | ±15%      |
| 2 J             | 1 J   | 2 J   | 2 J   | 2 J   | 2 J   | 2 J   | 2 J   | ±15%      |
| 3 J             | 2 J   | 3 J   | 3 J   | 3 J   | 3 J   | 3 J   | 3 J   | ±15%      |
| 4 J             | 3J    | 4 J   | 4 J   | 5 J   | 5 J   | 5 J   | 4 J   | ±15%      |
| 5 J             | 3 J   | 5 J   | 6 J   | 6 J   | 6 J   | 6 J   | 6 J   | ±15%      |
| 6 J             | 4 J   | 6 J   | 7 J   | 7 J   | 7 J   | 7 J   | 7 J   | ±15%      |
| 7 J             | 5 J   | 7 J   | 8 J   | 8 J   | 8 J   | 8 J   | 8 J   | ±15%      |
| 8 J             | 5 J   | 8 J   | 9 J   | 9 J   | 10 J  | 9 J   | 9 J   | ±15%      |
| 9 J             | 6 J   | 9 J   | 10 J  | 11 J  | 11 J  | 11 J  | 10 J  | ±15%      |
| 10 J            | 7 J   | 10 J  | 12 J  | ±15%      |
| 15 J            | 10 J  | 16 J  | 17 J  | 18 J  | 18 J  | 18 J  | 17 J  | ±15%      |
| 20 J            | 14 J  | 21 J  | 23 J  | 24 J  | 24 J  | 24 J  | 23 J  | ±15%      |
| 30 J            | 21 J  | 32 J  | 35 J  | 36 J  | 37 J  | 36 J  | 35 J  | ±15%      |
| 50 J            | 35 J  | 54 J  | 59 J  | 61 J  | 62 J  | 61 J  | 59 J  | ±15%      |
| 70 J            | 49 J  | 76 J  | 83 J  | 85 J  | 87 J  | 86 J  | 83 J  | ±15%      |
| 85 J            | 60 J  | 92 J  | 101 J | 104 J | 106 J | 104 J | 101 J | ±15%      |
| 100 J           | 71 J  | 109 J | 119 J | 122 J | 125 J | 123 J | 119 J | ±15%      |
| 120 J           | 85 J  | 131 J | 143 J | 147 J | 150 J | 147 J | 143 J | ±15%      |
| 150 J           | 107 J | 164 J | 180 J | 183 J | 188 J | 184 J | 179 J | ±15%      |
| 200 J           | 142 J | 230 J | 249 J | 253 J | 269 J | 261 J | 260 J | ±15%      |

\* For all energy levels, accuracy is equal to either ±15% or 3 joules, whichever is greater.

The Propaq MD Rectilinear Biphasic waveform employs the same first and second phase timing, similar first and second phase currents/voltages, and essentially the same mechanisms for controlling defibrillation waveshape as the ZOLL R Series<sup>®</sup> defibrillator. The R Series and Propaq MD defibrillation waveforms are considered substantially equivalent.

Figures A-1 through A-20 show the Rectilinear Biphasic waveforms that are produced when the Propaq MD defibrillator is discharged into loads of 25, 50, 75, 100, 125, 150, and 175 ohms at each energy setting.

The vertical axis shows the current in amperes (A); the horizontal axis shows the duration of time in milliseconds (ms).

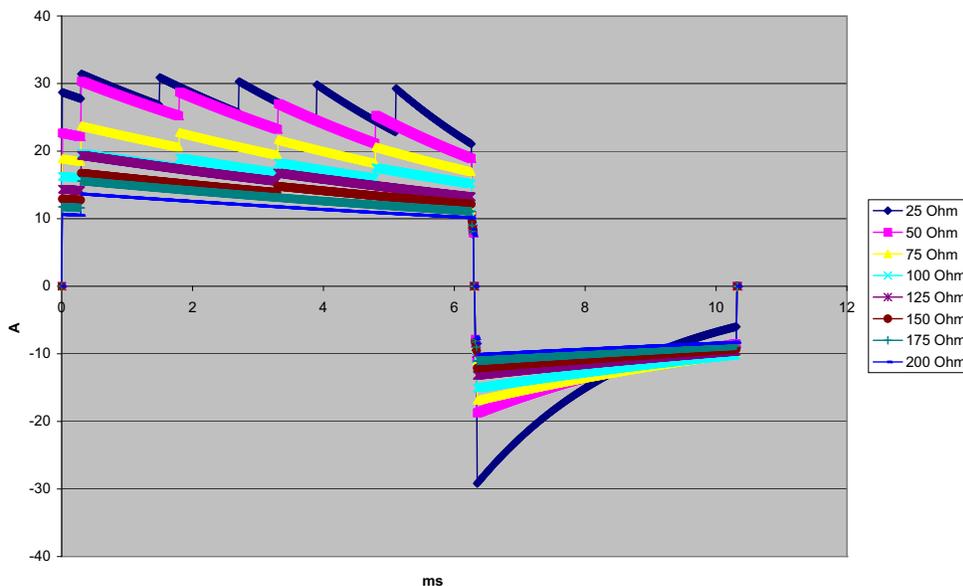


Figure A-1. Rectilinear Biphasic Waveform at 200 Joules

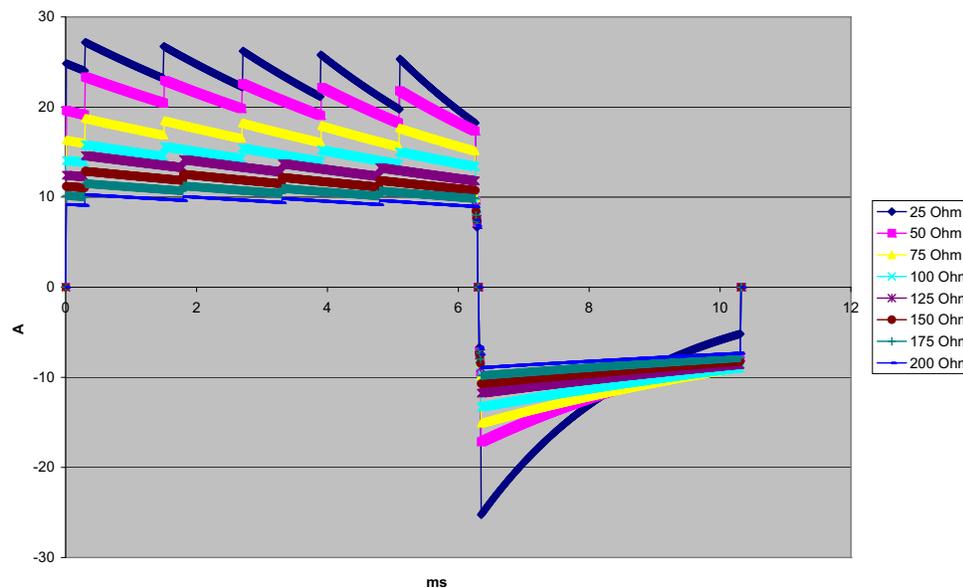


Figure A-2. Rectilinear Biphasic Waveform at 150 Joules

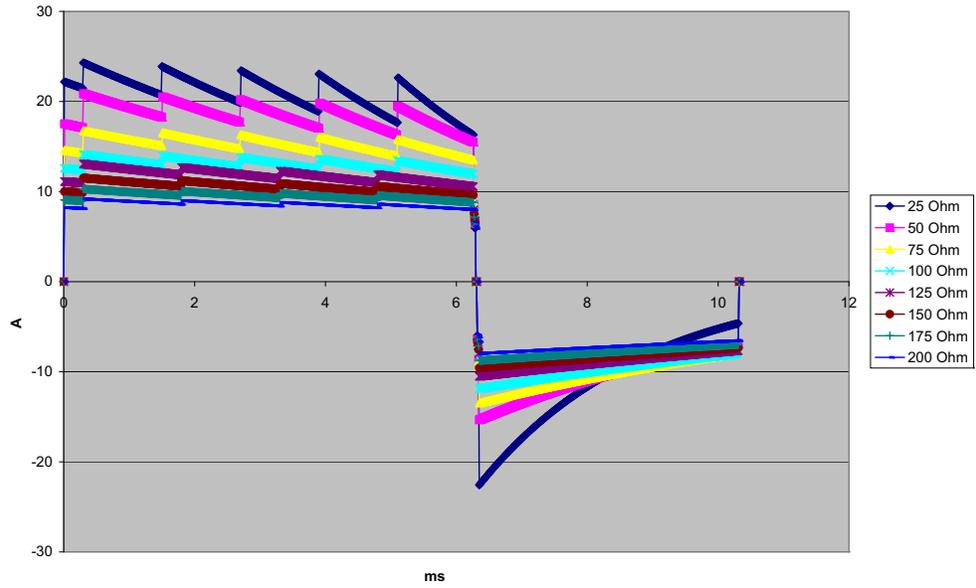


Figure A-3. Rectilinear Biphasic Waveform at 120 Joules

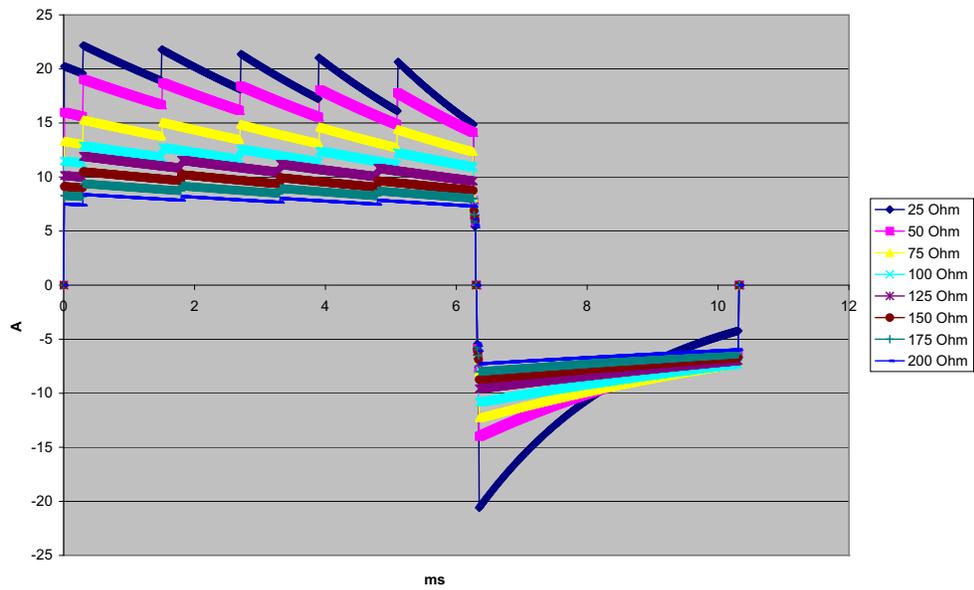


Figure A-4. Rectilinear Biphasic Waveform at 100 Joules

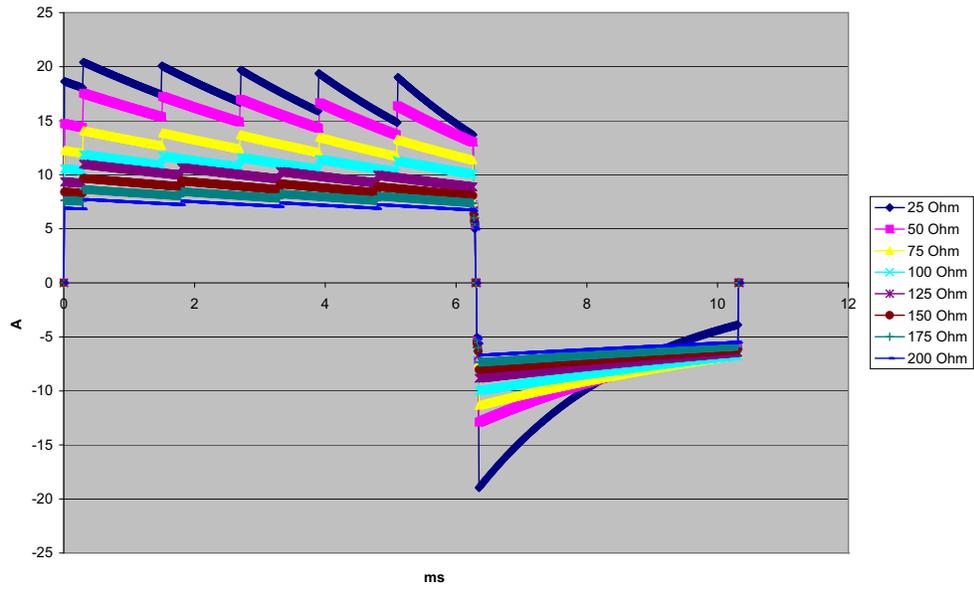


Figure A-5. Rectilinear Biphasic Waveform at 85 Joules

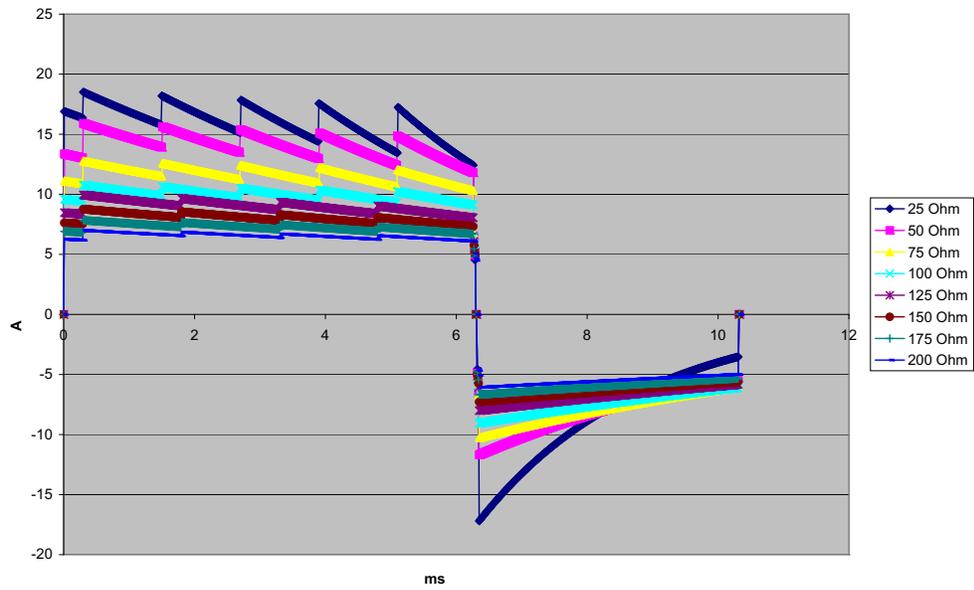


Figure A-6. Rectilinear Biphasic Waveform at 70 Joules

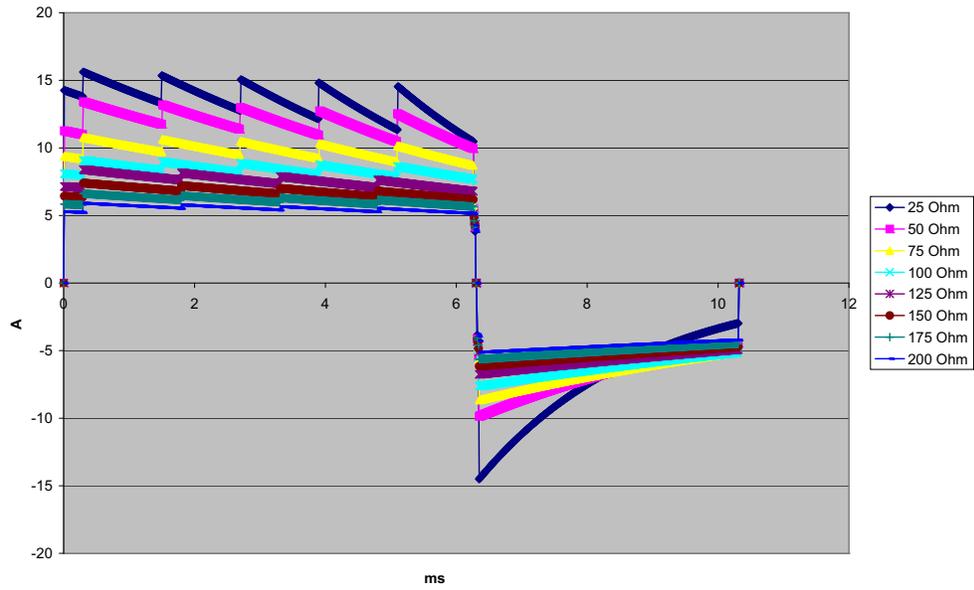


Figure A-7. Rectilinear Biphasic Waveform at 50 Joules

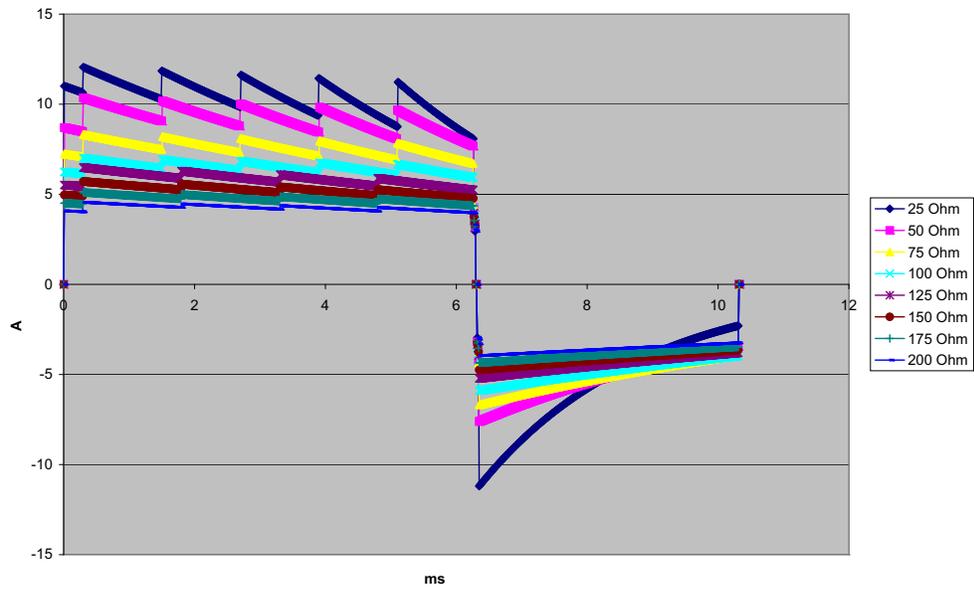


Figure A-8. Rectilinear Biphasic Waveform at 30 Joules

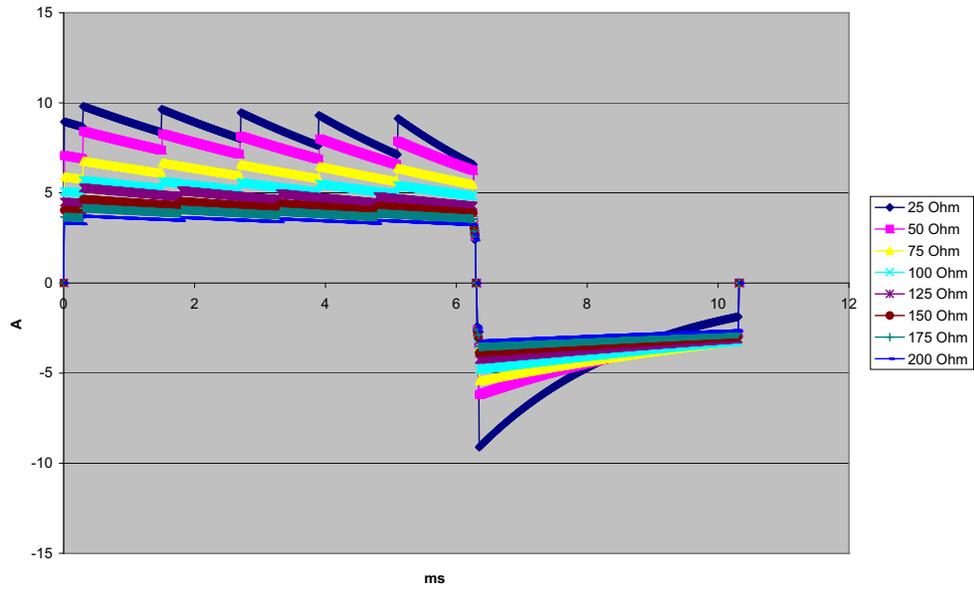


Figure A-9. Rectilinear Biphasic Waveform at 20 Joules

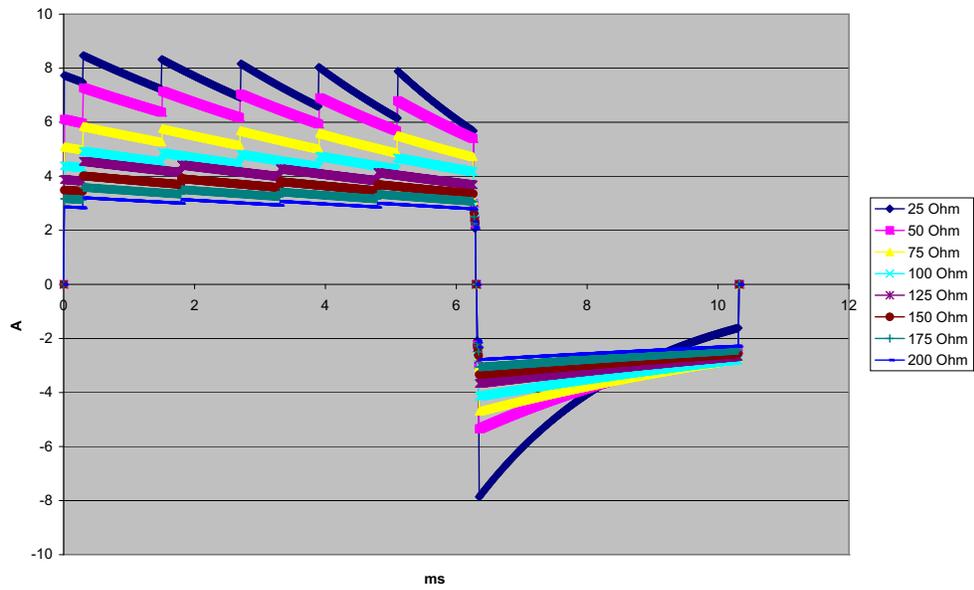


Figure A-10. Rectilinear Biphasic Waveform at 15 Joules

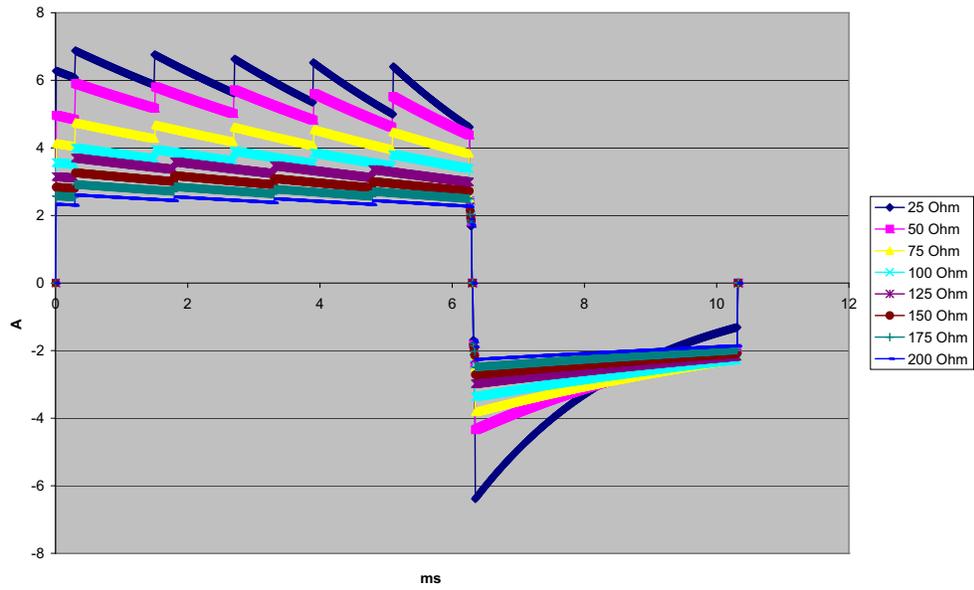


Figure A-11. Rectilinear Biphasic Waveform at 10 Joules

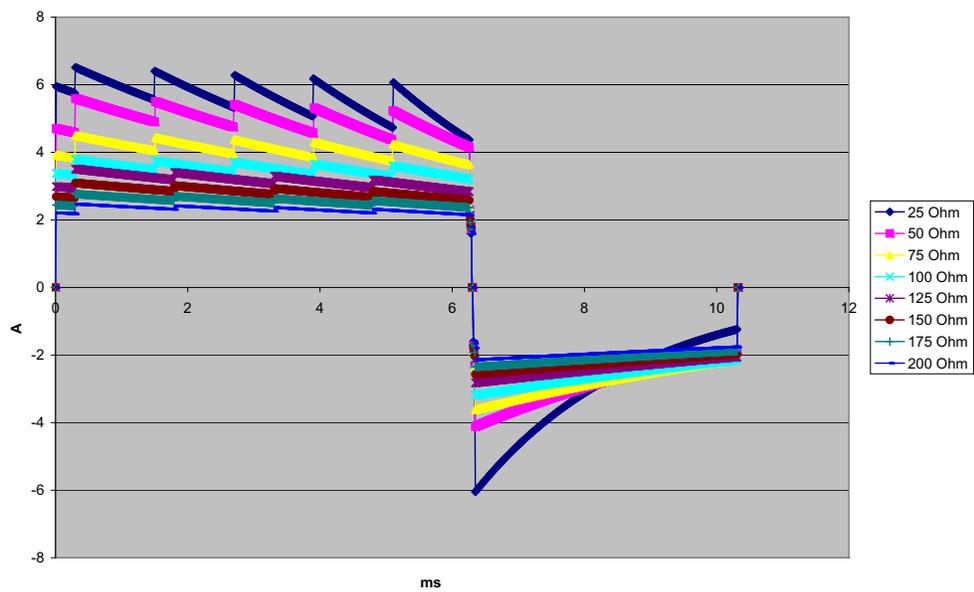


Figure A-12. Rectilinear Biphasic Waveform at 9 Joules

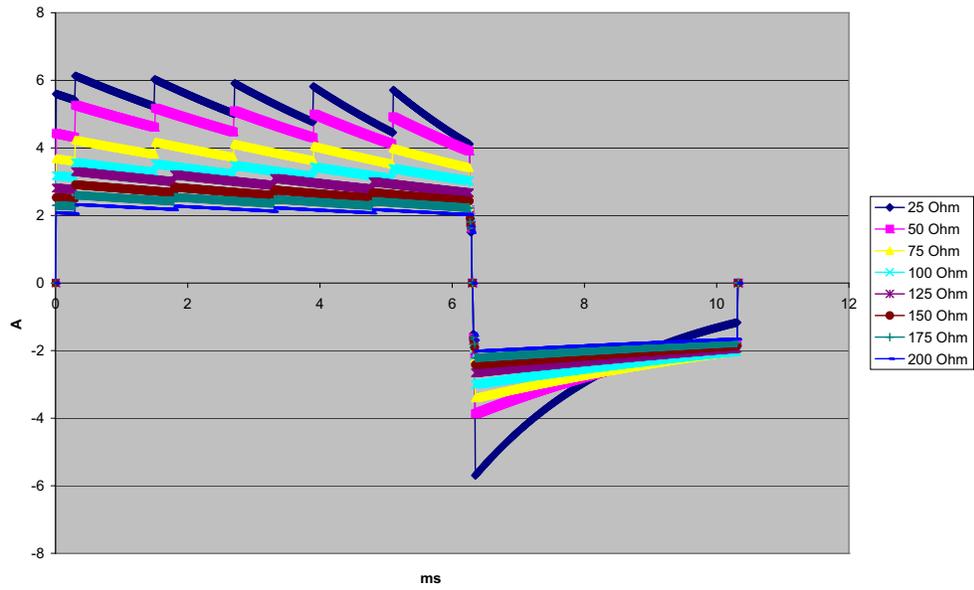


Figure A-13. Rectilinear Biphasic Waveform at 8 Joules

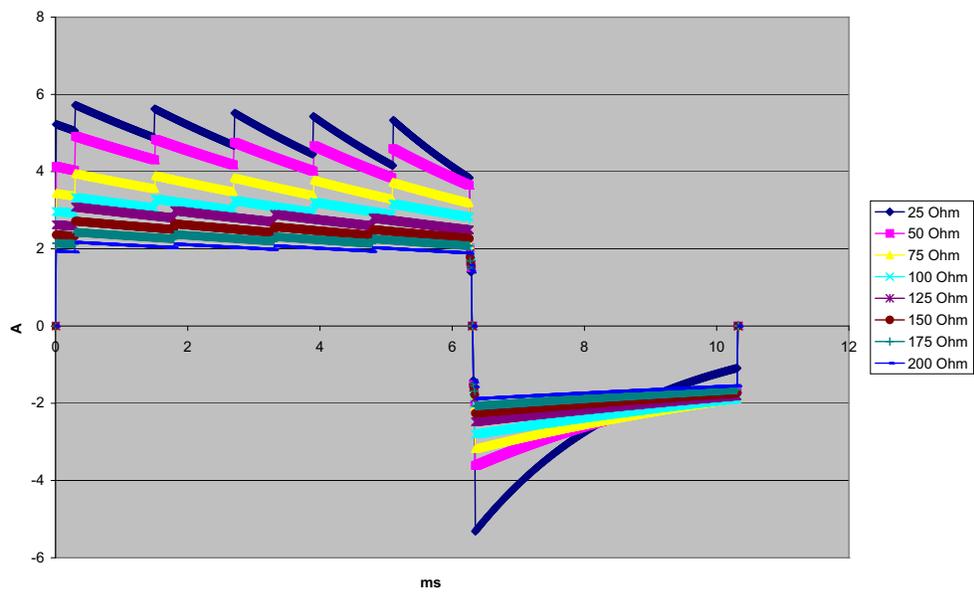


Figure A-14. Rectilinear Biphasic Waveform at 7 Joules

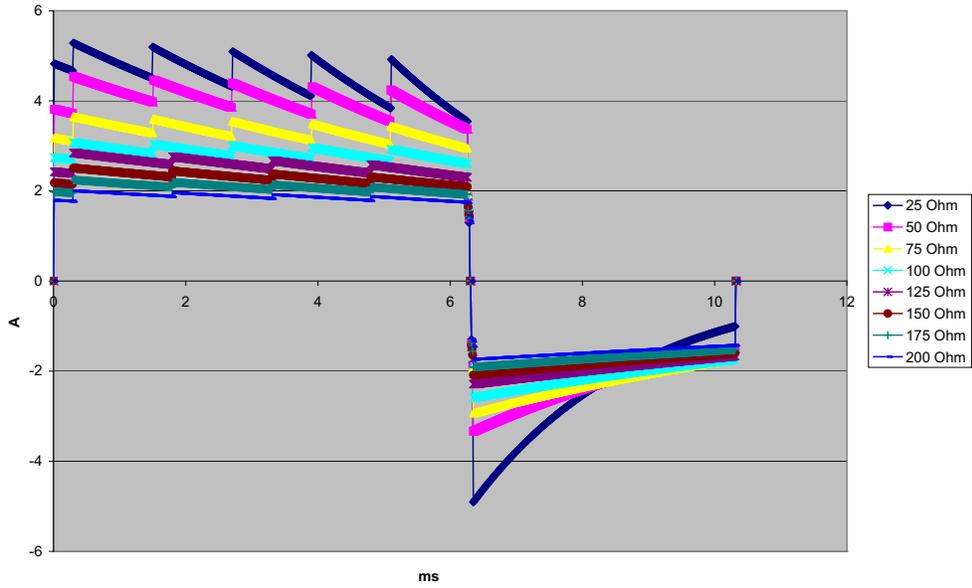


Figure A-15. Rectilinear Biphasic Waveform at 6 Joules

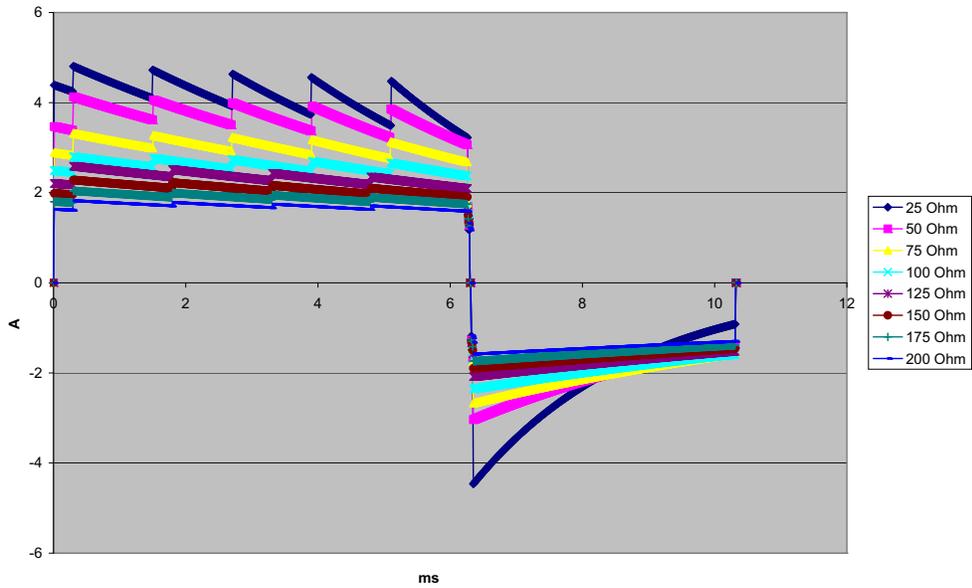


Figure A-16. Rectilinear Biphasic Waveform at 5 Joules

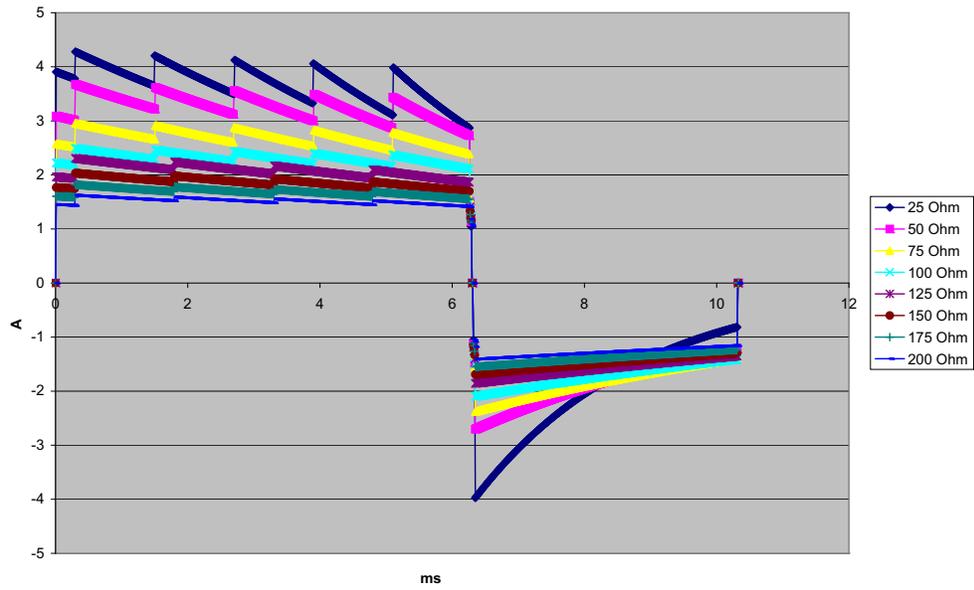


Figure A-17. Rectilinear Biphasic Waveform at 4 Joules

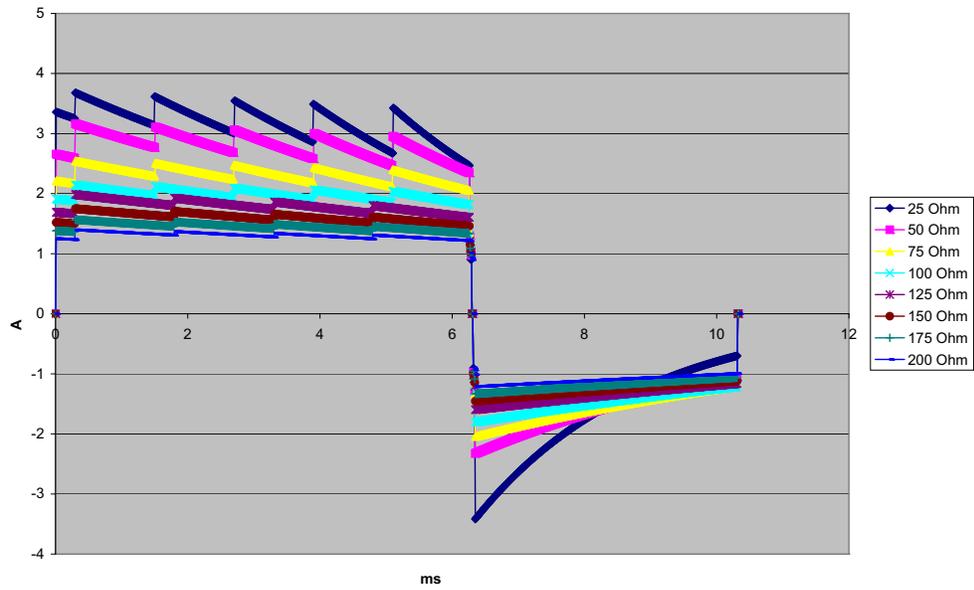


Figure A-18. Rectilinear Biphasic Waveform at 3 Joules

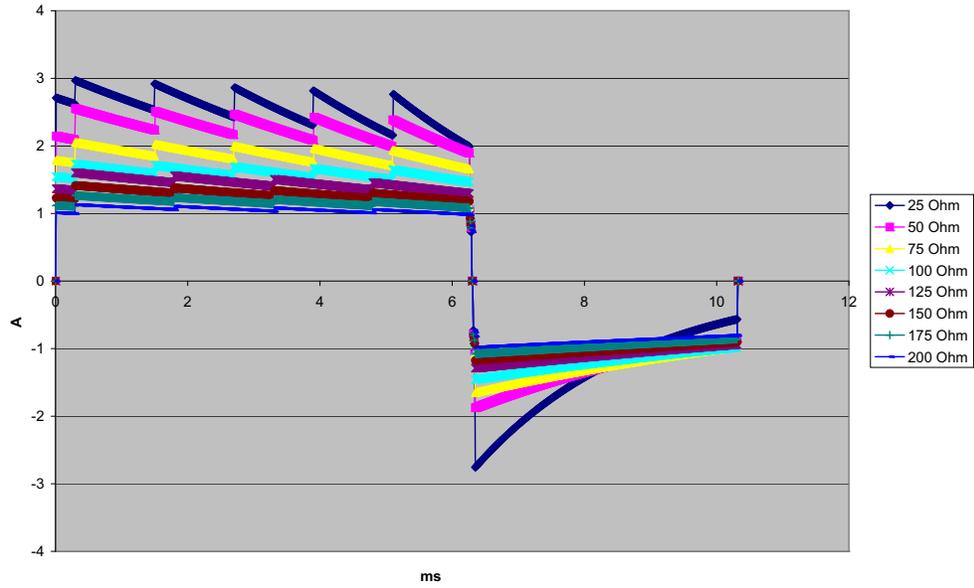


Figure A-19. Rectilinear Biphasic Waveform at 2 Joules

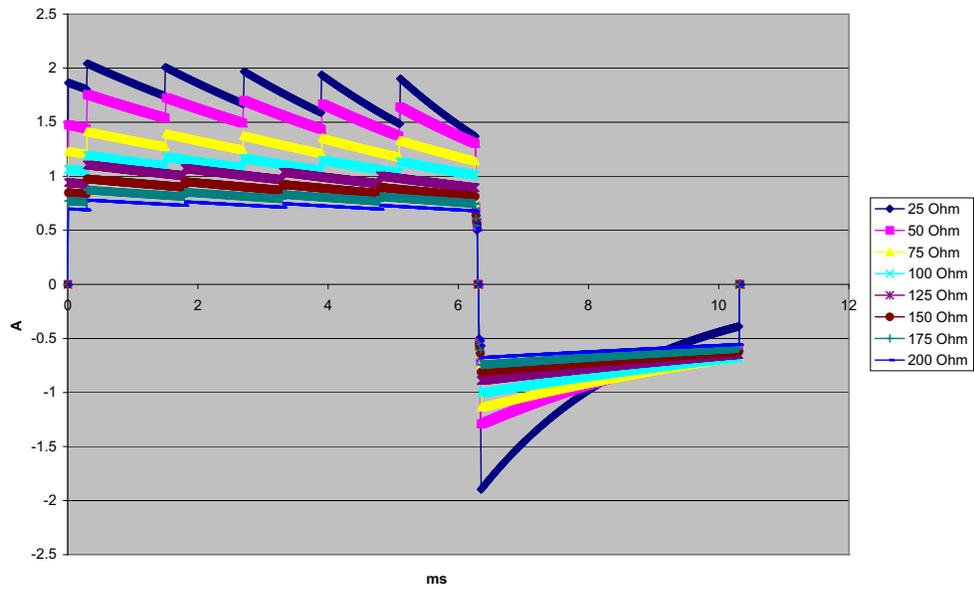


Figure A-20. Rectilinear Biphasic Waveform at 1 Joule

## Monitor/Display

**Input:** 3-lead, 5-lead, or 12-lead patient cable, paddles, or Multifunction electrodes.

**Type:** Color LCD, 640 x 480 pixels, 800 MCD

**Sweep Speed:** 25 mm / sec or 50 mm / sec (User Selectable)

**Lead Selections:** Paddles (Pads), I, II, III, AVR, AVL, AVF, V1-6.

**Frequency Response:**

Pads/Paddles:

0.67 to 20Hz Limited response

3/5/12 Lead Continuous Monitoring (user selectable):

0.67 to 20Hz Limited response

0.67 to 40Hz Monitor response

0.05 to 40Hz EMS Diagnostic response

Acquired 12-lead snapshots (supervisor selectable):

0.25 to 40Hz Filtered Diagnostic response

0.05 to 150Hz Diagnostic response

Per methods a, b, c of EC11 3.2.7.2

(Automatically sets chart recorder response)

**Common Mode Rejection:**

Complies with AAMI EC13-2002 section 4.2.9.10.

**Tall T-Wave Rejection:**

Meets AAMI EC13-2002, section 4.1.2.1c for 0.9 mV

T-wave (0.8 mV with diagnostic response) and 1mV QRS.

**Diagnostic Signals Applied to Patient Connections:**

Leads off / active noise suppression sensing circuit is < 0.1 microamps DC. The impedance /respiration detector signal frequency is  $72 \pm 7$  kHz at 77 microamps RMS pseudo-sinewave into 100 ohms.

**Heart Rate Range:** 30 to 300 BPM.

**Displayed Heart Rate:** Average of last 5 beat-to-beat intervals.

**Heart Rate Alarms:** User-selectable.

**Size:** 0.125, 0.25, 0.5, 1, 2, 4 cm/mv and auto-ranging.

**Heart Rate Meter Response Time:**

Responds to a 40 BPM step increase in heart rate within 4.5 seconds per AAMI EC-13-2002, section 4.1.2.1.f. Responds to a 40 BPM step decrease within 3.9 seconds per AAMI EC-13-2002, section 4.1.2.1.f. Response times include a 1.0-second display update interval.

**Heart Rate Response to Irregular Rhythm:** (AAMI EC13-2002, section 4.1.2.1.e.)

**Ventricular Bigeminy:** 80 BPM (expected)

**Slow Alternating Ventricular Bigeminy:** 60 BPM (expected)

**Rapid Alternating Ventricular Bigeminy:** 120 BPM (expected)

**Bidirectional Systole:** 45 BPM (expected)

**Tachycardia Response Time:**

Response time to tachycardia alarm is on average 3.4 seconds per AAMI EC-13-2002, section 4.1.2.1.g, and IEC 60601-2-27, clause 6.8.2.bb.7. Response times include a 1.0 second display update interval.

**Pacemaker Pulse Rejection:**

(In accordance with AAMI EC13:2002, section 4.1.4 and IEC 60601-2-27:2005, subclause 50.103.13)

- Pulses without overshoot: Rejects all pulses with amplitude of +2 mV to +700 mV and duration of 0.1 ms to 2 ms, with no tail.
- Pulses with overshoot: Rejects all pulses with amplitude of +2 mV to +700 mV and duration of 0.1 ms to 2 ms, with overshoot up to 100 ms.
- A-V sequential pulses: A-V sequential pacemaker pulses may not be rejected.
- Fast ECG signals: Approximately 50% of ECG input pulses with a slew rate of 3 V/s RTI may trigger the pacemaker pulse detector.

**Electrosurgery Protection:** The Propaq MD is suitable for use in the presence of electrosurgery as specified in IEC 60601-2-27. Burn hazard protection via a 1K current limiting resistor contained in each ECG leadwire.

## Impedance Pneumography

**Displayed Data:** Numeric breath rate, Impedance waveform

**Breath rate range:** Adult, Ped: 2 to 150 breaths / minute

Neonates: 3 to 150 breaths / minute

**Breath rate accuracy:** 2% or +/- 2 breaths / minute, whichever is greater

**Displayed Breath Rate:** Average of last 10 breath-to-breath rates.

**Leads:** Lead I (RA – LA), Lead II (RA – LL)

**Sweep Speed:** 3.13, 6.25, 12.5 mm/sec

**Alarm settings:** High and low breath rate alarm

## Alarms

**Heart Rate Alarms:**

**Audible:** 5 pulse, 900 Hz tone, with a PW of 125 msec, a PRI of 250 msec, and a repetition interval of 15 seconds.

**Visual:** Heart Rate Alarm causes the heart rate to be displayed in red, with a white background.

The red device status LED will flash a rate of 1.7 Hz.

**Lead Fault Alarm:**

**Audible:** 3 pulse, 500 Hz, triplet tone with a PW of 200 msec, a PRI of 200 msec. The lead fault tone repeats at a repetition interval of 30 seconds.

**Visual:** Lead Fault condition causes a LEAD FAULT message to be displayed on the trace along with a dashed line the width of the trace.

**Physiological Alarms (NIBP, SpO2, Resp, CO2, IP & Temp):**

**Audible:** Same as Heart Rate Alarm

**Visual:** Physiological alarms cause the alarming parameter to be displayed in Red with a white background. The red device status LED will flash at a rate of 1.7 Hz.

**Silence Duration:** 90 seconds.

**Invalid Operation Alert Tone:**

A short, low-pitched tone is audible when a selected control button is unavailable for use or an invalid entry is detected. Tone frequency is 160 Hz. Duration is 250 msec.

**Delays (hold off time):**

**Heart rate/pulse rate:**

- if source is ECG, 5 seconds
- if source is SpO2, 10 seconds
- if source is IBP, 5 seconds
- if source is NIBP, no hold off

**SpO2 Saturation:** 10 seconds

**EtCO2:** 7 seconds

**FiCO2:** 5 seconds

**IBP (Systolic, Diastolic, Mean):** 2 seconds

**Temperature:** 2 seconds

## Recorder

**Type:** High-resolution thermal array.

**Annotation:** Time, date, ECG lead, ECG gain, heart rate, defibrillation and pacing parameters and treatment summary events.

**Paper Width:** 50 mm.

**Paper Speed:** 25 mm/sec, 50 mm/sec

**Delay:** 6 seconds.

**Frequency Response:** Automatically set to monitor's frequency response.

**Treatment Summary:**

10 switches to record key ACLS events (IV, INTUB, EPI, LIDO, ATROP, etc.).  
Automatically logs into memory the type of event, time and ECG sample.

**Patient Data Log:** A combination of 24 hours of trends at 1-minute intervals, 1000 time-stamped events, and 32 snapshots, including monitor, defibrillator, pacer, and treatment snapshots. The actual number of trends, events, or snapshots stored could be more or less depending on the use profile.

**Record Modes:** Manual and automatic (User-configurable).

## Battery

**Type:** Rechargeable Lithium-Ion, 11.1Vdc, 6.6 Ah, 73Wh

**Capacity:**

With a new, fully charged battery operating at room temperature:

- At least 6 hours of continuous monitoring of ECG, SpO<sub>2</sub>, CO<sub>2</sub>, three Invasive Pressure channels, and 2 channels of Temperature, with NIBP measurements every 15 minutes and 10 200 J shocks (display set to 30%).
- At least 100 discharges at maximum shock energy (200 joules).
- At least 3.5 hours pacing, with ECG, SPO<sub>2</sub>, CO<sub>2</sub>, three Invasive Pressures, temperature, NIBP every 15 minutes and pacing at 180 ppm, and 140 mA.
- At least 10 discharges at maximum shock setting (200 joules) after a Low Battery indication.

**Note:** Proper battery care is required to maintain maximum available capacity.

**Battery Indicators:**

5 Battery capacity LED indicators, Fault indicator, Recalibration indicator

**Recharge Rate:** 100% in 4 hours, when initiated at Low Battery indication.

## General

**Weight:**

9.9 lbs.without battery and paper

11.6 lbs.with battery and paper

**Dimensions:**

Without Handle: 8.9" x 8.75" x 7.9"

With Handle: 8.9" x 10.4" x 7.9"

**Operating:**

**Temperature:** 0 to 50° C

60° C for 6 hours after storage at room temperature

**Humidity:** 15 to 95% RH (non-condensing)

**Vibration:**

- MIL-STD 810G, Method 514.6, Profile for combined UH-1, UH-60, and CH-47 Rotary Wing Aircraft.
- EN 1789 for ambulance.

**Shock:** MIL-STD 810G, Method 516.6, Tested at 75 g

**Drop:** MIL-STD 810G, Method 516-6, Tested at 1 meter with 26 drops

IEC 60601-1, Tested at 2 meters

**Altitude:** 170 M to 4572 M (-557 feet to 15,000 feet)

**Transport and Storage:**

**Temperature:** -30 to 70°C

**Note:** The Propaq MD device may not perform to specifications when stored at the upper or lower extreme limits of storage temperature and immediately put into use.

**Humidity:** 15 to 95% RH (non-condensing)

**Atmospheric pressure:** 572 mbar to 1034 mbar

**Shock/vibration:** ISTA 1A

**Safety Classification:** Class 1 and internal power per IEC/EN 60601-1

**Enclosure Protection:**

**Solid Foreign Object:** IEC 60529, IP5X

**Water:** IEC 60529, IPX5

**Auxiliary Operating Power:**

Propaq MD Auxiliary Power Adapter 8300-0004

Input: 100-240V  $\sphericalcap$  50-60 Hz, 2A

100-115V  $\sphericalcap$  400 Hz, 2A

Output: 14.5V  $\text{---}$  4.15A

80W (peak)

IP Rating: IP23

## Pacer

**Type:** External transcutaneous pacing

**Pacer Rate:** 30 to 180 BPM  $\pm$  1.5%.

**Output Current:** 0 to 140 mA  $\pm$  5% or 5 mA (whichever is greater)

**Modes:** Demand and Fixed

**Status Indicators:**

ECG lead fault, pace marker on monitor and chart, start/stop indicator on display.

**Pulse Type:** Rectilinear, constant current

**Pulse Width:** 40 ms  $\pm$  2 ms

CO<sub>2</sub>

**Range:** 0 to 150 mmHg

**Accuracy CO<sub>2</sub>:**

| CO <sub>2</sub> Partial Pressure* | Accuracy **  |
|-----------------------------------|--|
| 0-38 mmHg                         | ± 2 mmHg   |
| 39-99 mmHg                        | ± (5% of reading + 0.08% for every 1 mmHg above 38 mmHg) |
| 100-150 mmHg                      | ± (5% of reading + 0.08% for every 1 mmHg above 38 mmHg) |

\* At sea level.

\*\* Accuracy applies for breath rates of up to 80 bpm. For breath rates above 80 bpm, accuracy is 4 mmHg or ±12% of reading whichever is greater, for EtCO<sub>2</sub> values exceeding 18 mmHg. This is tested according to and is compliant with ISO 21647. To achieve the specified accuracies for breath rates above 60 breaths/minute, the Microstream FilterLine H Set for Infant/Neonatal must be used. Above 40 C, ± 1mmHg or ± 2.5% (whichever is greater) has to be added to the tolerance of the accuracy specs.

**Drift of Measurement Accuracy:** Over any 24-hour period, the accuracy claims listed above are maintained.

The accuracy specification is maintained to within 4% for the following gas mixtures (all values are in Vol.%).

| CO <sub>2</sub> | N <sub>2</sub> | O <sub>2</sub> | N <sub>2</sub> O | H <sub>2</sub> O | Anesthetic Agents     |
|-----------------|----------------|----------------|------------------|------------------|-----------------------|
| 1.0 to 13       | 0 to 97.5      | 0 to 100       | 0 to 80          | Dry to saturated | According to EN 21647 |

**Respiration Range:** 0 to 149 breaths per minute

**Respiration Rate Accuracy:**

0 to 70 bpm: ±1 bpm

71 to 120 bpm: ±2 bpm

121 to 149 bpm: ±3 bpm

**Flow rate:** 50 ml/min -7.5 + 15 ml/min, flow measured by volume.

**Total System Response Time:** 2.9 seconds typical, 3.9 seconds maximum.

## Pulse Oximeter

**Measurement Range:** 0 to 100%

**Accuracy:** 70 to 100% +/- 3%, 0 to 70% unspecified

**Note:** Since pulse oximeter equipment measurements are statistically distributed, only two-thirds of pulse oximeter equipment measurements can be expected to fall within +/- 3% of the value measured by a CO-oximeter.

**Saturation Update Period:** Updated once per second

**SpO2 Saturation Alarm Hold Off:** 10 seconds

**Pulse Rate Range:** 20 to 250 BPM  $\pm$  3% or 3 BPM whichever is greater

**Pulse Rate Update Period:** Updated once per second

**Pulse Rate and Saturation Display:**

**Normal Response:** Average of pulse rate and saturation signal over 6 to 7 seconds.

**Wavelengths:** Approximately 660nm and 900nm

**Displayed Data:**

Numeric blood oxygen saturation value

Numeric pulse rate

Pulse Amplitude bar indicator

Plethysmographic waveform

**Probe Compatibility:** DS-100A, D-25, R-15, D-25L, D-20, EC-4, EC-8, RS-10

**Power Output:** < 15 mW

## Non-Invasive Blood Pressure

**Technique:** Non-invasive oscillometric method

**Operating Modes:** Automatic and manual

**Automatic Intervals:** 1, 2, 3, 5, 10, 15, 30 and 60 minute intervals.

**Turbocuf:** Maximum measurements allowable in a 5 minute period

**Pressure Measurement Range:**

Systolic: 20 to 260 mmHg

Diastolic: 10 to 220 mmHg

Mean: 13 to 230 mmHg

**Static Pressure Accuracy:** +/- 3 mmHg

**Pulse Rate Range:**

Adult: 30 to 200 +/- 5 BPM

Pediatric:30 to 200 +/- 5 BPM

Neonatal:35 to 220 +/- 5 BPM

**Default Cuff Inflation Pressure:**

Adult: 160 mmHg

Pediatric:120 mmHg

Neonatal:90 mmHg

**Maximum Cuff Inflation Pressure:**

Adult: 270 mmHg

Pediatric:170 mmHg

Neonatal:130 mmHg

**Single Fault Backup Overpressure Limit:**

Adult: 308 mmHg

Pediatric:205 mmHg

Neonatal:154 mmHg

**Typical Determination time without Artifact:**

**Measurements on the deflation:** 30 to 45 seconds

**Measurements on the inflation (SureBP)\*:** 15 to 30 seconds

\* using dual lumen cuffs

**Maximum Determination Time - Measurement on the Inflation**

Adult: 150 seconds

Pediatric:120 seconds

Neonatal:80 seconds

**Blood Pressure Validation:**

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method for adults and pediatric patients and equivalent to an intra-arterial measurement for neonatal patients, within the limits prescribed by the American National Standards Institute (ANSI-AAMI SP10). To receive a copy of the report containing the AAMI SP10 results, contact the ZOLL Technical Service Department.

## Invasive Pressures

**Number of Channels:** 3

**Pressure range:** -30 to 300 mmHg

**Pressure Accuracy:** +/- 2 mmHg or 2% of reading, whichever is greater, plus transducer error.

**Pulse Rate Range:** 25 to 250 BPM

**Pulse Rate Accuracy:** +/- 3 BPM, or +/- 3% of value whichever is greater

**Pulse Rate Display:** Average of last 4 beat-to-beat intervals.

**Zero Adjust:** + / - 200 mmHg

**Transducer:**

Sensitivity: 5uV/V/mmHg

Offset: +/- 125 mmHg including transducer offset

Excitation Impedance Range: 150 to 10,000 ohms

Excitation Voltage: 4.75 +/- 0.25 VDC

Connector: 6-pin circular MS3100 series

|             |         |         |         |         |        |
|-------------|---------|---------|---------|---------|--------|
| Connect to: | A       | B       | C       | D       | E      |
| Signal Type | Sig (-) | Exc (+) | Sig (+) | Exc (-) | shield |

## Temperature

**Number of Channels:** 2

**Measurement Range:** 0° to 50° C

**Accuracy:**

± 0.1° C from 10° C to 50° C

± 0.2° C from 0° C to 10° C

**Resolution:** 0.1° C

**Scale:** Fahrenheit or Celsius.

**Temperature Display Signal:** 20Hz, no averaging.

**Probe:** YSI 400 and 700 series

**Display:** T1, T2, ΔT

**Minimum Measurement Time:** See the probe's Instructions for Use to obtain minimum measurement times for accurate readings. The Propaq MD does not add any clinically significant time to obtain accurate readings.

## Clinical Trial Results for the Biphasic Waveform

The efficacy of the ZOLL Rectilinear Biphasic waveform has been clinically verified during a study of defibrillation of Ventricular Fibrillation (VF) and Ventricular Tachycardia (VT). A feasibility study was performed initially for defibrillation of VF/VT (n=20) on two separate groups of patients to ensure waveform safety and energy selection. Subsequently, a separate, multicenter, randomized clinical trial was performed to verify the waveform’s efficacy. A description of this study is provided below. The study was performed using ZOLL defibrillation systems consisting of ZOLL defibrillators, the ZOLL Rectilinear Biphasic waveform, and ZOLL defibrillation electrodes.

### Randomized Multicenter Clinical Trial for Defibrillation of Ventricular Fibrillation (VF) and Ventricular Tachycardia (VT)

**Overview:** The defibrillation efficacy of the ZOLL Rectilinear Biphasic waveform was compared to a monophasic damped sine waveform in a prospective, randomized, multicenter study of patients undergoing ventricular defibrillation for VF/VT during electrophysiological studies, ICD implants, and test. A total of 194 patients were enrolled in the study. Ten patients who did not satisfy all protocol criteria were excluded from the analysis, leaving a study population of 184.

**Objectives:** The primary goal of this study was to compare the first shock efficacy of the 120 J rectilinear biphasic waveform with a 200 J monophasic waveform. The secondary goal was to compare all shock (three consecutive 120, 150, and 170 joules) efficacy of the rectilinear biphasic waveform with that of a monophasic waveform (three consecutive 200, 300, and 360 joules). A significance level of  $p=0.05$  or less was considered statistically significant using Fischer’s Exact test. Also, differences between the two waveforms were considered statistically significant when the customary 95% or AHA-recommended 90%<sup>1</sup> confidence interval between the two waveforms was greater than 0%.

**Results:** The study population of 184 patients had a mean age of  $63\pm 14$  years. Of these, 143 patients were male. 98 patients were in the biphasic group (ventricular fibrillation/flutter, n=80; ventricular tachycardia, n=18) and 86 patients were in the monophasic group (ventricular fibrillation/flutter, n=76; ventricular tachycardia, n=10). There were no adverse events or injuries related to the study.

The first shock, first induction efficacy of biphasic shocks at 120 J was 99% versus 93% for monophasic shocks at 200 J ( $p=0.0517$ , 95% confidence interval of the difference of  $-2.7\%$  to  $16.5\%$  and 90% confidence interval of the difference of  $-1.01\%$  to  $15.3\%$ ).

|                         | Monophasic            | Biphasic |
|-------------------------|-----------------------|----------|
| First shock efficacy    | 93%                   | 99%      |
| p-value                 | 0.0517                |          |
| 95% confidence interval | $-2.7\%$ to $16.5\%$  |          |
| 90% confidence interval | $-1.01\%$ to $15.3\%$ |          |

1. Kerber RE, et al., “Automated External Defibrillators for Public Access Defibrillation: Recommendations for Specifying and Reporting Arrhythmia Analysis Algorithm Performance, Incorporating New Waveforms, and Enhancing Safety,” *Circ J Am Heart Assoc.* 1997;95:1677-1682.

“... the task force suggests that to demonstrate superiority of an alternative waveform over standard waveforms, the upper boundary of the 90% confidence interval of the difference between standard and alternative waveforms must be  $<0\%$  (ie, alternative is greater than standard).”

Successful defibrillation with rectilinear biphasic shocks was achieved with 58% less delivered current than with monophasic shocks ( $14\pm 1$  amperes versus  $33\pm 7$  amperes,  $p=0.0001$ ).

The difference in efficacy between the rectilinear biphasic and the monophasic shocks was greater in patients with high transthoracic impedance (greater than 90 ohms). The first shock, first induction efficacy of biphasic shocks was 100% versus 63% for monophasic shocks for patients with high impedance ( $p=0.02$ , 95% confidence interval of the difference of  $-0.0217\%$  to  $0.759\%$  and 90% confidence interval of the difference of  $0.037\%$  to  $0.706\%$ ).

|  | Monophasic              | Biphasic |
|--|-------------------------|----------|
| First shock efficacy (high impedance patients) | 63%                     | 100%     |
| p-value  | 0.02                    |          |
| 95% confidence interval                        | $-0.021\%$ to $0.759\%$ |          |
| 90% confidence interval                        | $0.037\%$ to $0.706\%$  |          |

A single patient required a second biphasic shock at 150 joules to achieve 100% efficacy versus six patients for whom monophasic shocks of up to 360 joules were required for 100% total defibrillation efficacy.

**Conclusion:** The data demonstrate the equivalent efficacy of low energy rectilinear biphasic shocks compared to standard high energy monophasic shocks for transthoracic defibrillation for all patients at the 95% confidence level. The data also demonstrate the superior efficacy of low energy rectilinear biphasic shocks compared to standard high energy monophasic shocks in patients with high transthoracic impedance at the 90% confidence level. There were no unsafe outcomes or adverse events due to the use of rectilinear biphasic waveform.

## Randomized Multi-Center Clinical trial for Cardioversion of Atrial Fibrillation (AF)

**Overview:** The defibrillation efficacy of ZOLL's Rectilinear Biphasic Waveform was compared to a monophasic damped sine waveform in a prospective randomized multi-center study of patients undergoing cardioversion of their atrial fibrillation. A total of 173 patients entered the study. Seven (7) patients who did not satisfy all protocol criteria were excluded from the analysis. ZOLL disposable gel electrodes with surface areas of  $78\text{ cm}^2$  (anterior) and  $113\text{ cm}^2$  (posterior) were used exclusively for the study.

**Objective:** The primary goal of the study was to compare the total efficacy of four consecutive rectilinear biphasic shocks (70J, 120J, 150J, 170J) with four consecutive monophasic shocks (100J, 200J, 300J, 360J). The significance of the multiple shocks efficacy was tested statistically via two procedures, the Mantel-Haenszel statistic and the log-rank test, significance level of  $p=0.05$  or less was considered statistically significant. The data are completely analogous to the comparison of two "survival" curves using a life-table approach where shock number plays the role of time.

The secondary goal was to compare the first shock success of rectilinear biphasic and monophasic waveforms. A significance level of  $p=0.05$  or less was considered statistically significant using Fisher Exact tests. Also, differences between the two waveforms were considered statistically significant when the 95% confidence interval between the two waveforms was greater than 0%.

**Results:** The study population of 165 patients had a mean age of  $66\pm 12$  years with 116 male patients.

The total efficacy of consecutive rectilinear biphasic shocks was significantly greater than that of monophasic shocks. The following table displays the Kaplan-Meier (product-limit) "survival" curves for each of the two waveforms. As all patients begin in the failure mode, the estimated life-table probabilities refer to the chance of still being in failure after the  $k^{\text{th}}$  shock ( $k=1,2,3,4$ ):

**Table A-3. Kaplan-Meier Estimate for the Probability of Shock Failure**

| Shock # | Biphasic | Monophasic |
|---------|----------|------------|
| 0       | 1.000    | 1.000      |
| 1       | 0.318    | 0.792      |
| 2       | 0.147    | 0.558      |
| 3       | 0.091    | 0.324      |
| 4       | 0.057    | 0.208      |

As can be seen from the table, the Biphasic experience is superior over the entire course of shocks delivered. The one degree of freedom chi-square statistic for the Mantel-Haenszel test is 30.39 ( $p < 0.0001$ ). Similarly, the log-rank test, also a one degree of freedom chi-square statistic, is 30.38 ( $p < 0.0001$ ). The residual number of patients not successfully treated after four shocks is 5.7% for biphasic compared to 20.8% for monophasic.

There was a significant difference between the first shock efficacy of biphasic shocks at 70J of 68% and that of monophasic shocks at 100J of 21% ( $p = 0.0001$ , 95% confidence interval of the difference of 34.1% to 60.7%).

Successful cardioversion with rectilinear biphasic shocks was achieved with 48% less delivered current than with monophasic shocks ( $11 \pm 1$  vs.  $21 \pm 4$  A,  $p < 0.0001$ ).


One half of the patients who failed cardioversion after four consecutive escalating monophasic shocks were subsequently successfully cardioverted using a biphasic shock at 170J. No patient was successfully cardioverted using a 360J monophasic shock after the patient had failed cardioversion with biphasic shocks.

**Conclusion:** The data demonstrate the superior efficacy of low energy rectilinear biphasic shocks compared to high energy monophasic shocks for transthoracic cardioversion of atrial fibrillation. There were no unsafe outcomes or adverse events due to the use of Rectilinear Biphasic Waveform.

## Synchronized Cardioversion of Atrial Fibrillation

Cardioversion of atrial fibrillation (AF) and overall clinical effectiveness is enhanced by proper pad placement. Clinical studies (refer to above) of the M Series Biphasic Defibrillator Waveform demonstrated that high conversion rates are achieved when defibrillation pads are placed as shown in the following diagram.

Recommended Anterior/Posterior Placement

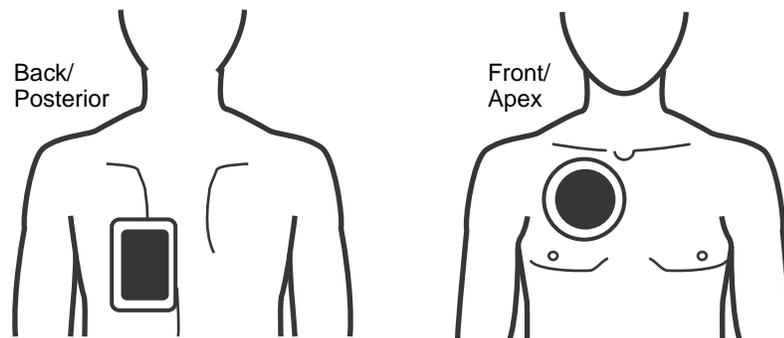

Place the front (apex) pad on the third intercostal space, mid clavicular line on the right anterior chest. The back/posterior pad should be placed in the standard posterior position on the patient's left as shown.

# Electromagnetic Compatibility Guidance and Manufacturer's Declaration

### **In-Flight Use (RTCA/DO-160):**

The Propaq MD unit complies with RTCA/DO-160, Environmental Conditions and Test Procedures for Airborne Equipment, using the methods in Section 21, Category M for Radiated and Conducted Radio Frequency Energy.

### **General Use Environments (IEC 60601-1-2):**

The Propaq MD unit is intended for use in the electromagnetic environment specified below. The customer or the user of the Propaq MD unit should assure that it is used in such an environment.

| <b>Emissions Test</b>   | <b>Compliance</b> | <b>Electromagnetic Environment – Guidance</b>   |
|---|-------------------|---|
| <b>RF Emissions CISPR 11</b>  | Group 1           | The Propaq MD unit uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.   |
| <b>RF Emissions CEM 11</b>  | Class B           |   |
| <b>Harmonic Emission IEC 6100-3-2</b>   | Class A           | The Propaq MD unit is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| <b>Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3</b>  | Complies          |   |
| Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to EMC information provided in this document. |                   |   |

**Guidance and Manufacturer’s Declaration – Electromagnetic (IEC 60601-1-2 Table 202)**

The Propaq MD unit is intended for use in the electromagnetic environment specified below. The customer or the user of the Propaq MD should assure that it is used in such an environment.

| Immunity Test   | IEC 60601 Test Level  | Compliance Level  | Electromagnetic Environment – Guidance   |
|---|---|---|--|
| Electrostatic Discharge (ESD) IEC 61000-4-2   | ± 6 kV contact<br>± 8 kV air  | ± 6 kV contact<br>± 8 kV air  | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.  |
| Electrical Fast Transient / Burst IEC 61000-4-4   | ± 2 kV for power supply lines<br>± 1 kV for input/output lines  | ± 2 kV for power supply lines<br>± 1 kV for input/output lines  | Mains power quality should be that of a typical commercial or hospital environment.  |
| Surge IEC 61000-4-5   | ± 1 kV differential mode<br>± 2 kV common mode  | ± 1 kV differential mode<br>± 2 kV common mode  | Mains power quality should be that of a typical commercial or hospital environment.  |
| Voltage Dips, Short Interruptions and Voltage Variations on Power Supply Input Lines IEC 61000-4-11 | <5% $U_T$ (>95% dip in $U_T$ ) for 0.5 cycle<br>40% $U_T$ (60% dip in $U_T$ ) for 5 cycles<br>70% $U_T$ (30% dip in $U_T$ ) for 25 cycles<br><5% $U_T$ (>95% dip in $U_T$ ) for 5 sec | <5% $U_T$ (>95% dip in $U_T$ ) for 0.5 cycle<br>40% $U_T$ (60% dip in $U_T$ ) for 5 cycles<br>70% $U_T$ (30% dip in $U_T$ ) for 25 cycles<br><5% $U_T$ (>95% dip in $U_T$ ) for 5 sec | Mains power quality should be that of a typical commercial or hospital environment. If the user of the Propaq MD unit requires continued operation during power mains interruptions, it is recommended that the Propaq MD unit be powered from an uninterruptible power supply or a battery. |
| Power Frequency (50/60 Hz) Magnetic Field IEC 61000-4-8   | 3 A/m   | 3 A/m   | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.  |
| <b>Note:</b> $U_T$ is the AC mains voltage prior to application of the test level.                  |   |   |  |

**Guidance and Manufacturer’s Declaration – Electromagnetic Immunity (IEC 60601-1-2 Table 203)**

The Propaq MD unit is intended for use in the electromagnetic environment specified below. The customer or the user of the Propaq MD should assure that it is used in such an environment.

| Immunity Test  | IEC 60601 Test Level  | Compliance Level    | Electromagnetic Environment – Guidance IEC 60601-1-2  |
|--|---|---------------------|---|
|  |   |                     | Portable and mobile RF communications equipment should be used no closer to any part of the Propaq MD unit, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. |
|  |   |                     | Recommended Separation Distance   |
| Conducted RF<br>IEC 61000-4-6  | 3 Vrms<br>150 kHz to 80 MHz<br>outside ISM bands <sup>a</sup> | 3 Vrms              | $d = 1.17 \sqrt{P}$   |
|  | 10 Vrms<br>150 kHz to 80 MHz in<br>ISM bands <sup>a</sup>     | 10Vrms              | $d = 1.2 \sqrt{P}$  |
| Radiated RF<br>IEC 61000-4-3   | 10 V/m<br>80 MHz to 2.5 GHz                                   | 10 V/m              | $d = 1.20 \sqrt{P}$<br>80 MHz to 800 MHz<br><br>$d = 2.30 \sqrt{P}$<br>800 MHz to 2.5 GHz   |
|  | 3 V/m<br>80 MHz to 2.5 GHz<br>(IEC 60601-2-34)                | 3 V/m<br>(IBP only) | $d = 4 \sqrt{P}$<br>80 MHz to 800 MHz<br><br>$d = 7.67 \sqrt{P}$<br>800 MHz to 2.5 GHz  |
|  |   |                     |   |
| <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in meters (m).<sup>b</sup><br/>                     Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey<sup>c</sup>, should be less than the compliance level in each frequency range<sup>d</sup>.<br/>                     Interference may occur in the vicinity of equipment marked with the following symbol:</p> |   |                     |   |
|   |   |                     |   |

**Note 1:** At 80 MHz and 800 MHz, the higher frequency range applies.

**Note 2:** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

- a. The ISM (industrial, scientific, and medical) bands between 150 kHz and 80 MHz are 6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz.
- b. The compliance levels in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2.5 GHz are intended to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas. For this reason, an additional factor of 10/3 is used in calculating the recommended separation distance for transmitters in these frequency ranges.
- c. Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ZOLL Propaq MD unit is used exceeds the applicable RF compliance level above, the Propaq MD unit should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Propaq MD unit.
- d. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than  $[V_1]$  V/m.

### Recommended Separation Distances Between Portable and Mobile RF Communications Equipment and the Propaq MD (IEC 60601-1-2 Table 205)

The Propaq MD unit is intended for use in an environment in which radiated RF disturbances are controlled. The customer or the user of the Propaq MD unit can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Propaq MD unit as recommended below, according to the maximum output power of the communications equipment.

| Separation Distance According to Frequency of Transmitter (meters) |                                     |                                |                   |                    |
|--|-------------------------------------|--------------------------------|-------------------|--------------------|
| Rated maximum output power of transmitter                          | 150 kHz to 80 MHz outside ISM bands | 150 kHz to 80 MHz in ISM bands | 80 MHz to 800 MHz | 800 MHz to 2.5 GHz |
| W  | $d = [3.5/3]P$                      | $d = [12/3]P$                  | $d = [12/10]P$    | $d = [23/10]P$     |
| 0.01   | 0.12                                | 0.40                           | 0.12              | 0.23               |
| 0.1  | 0.37                                | 1.26                           | 0.38              | 0.73               |
| 1  | 1.17                                | 4.00                           | 1.20              | 2.3                |
| 10   | 3.69                                | 12.65                          | 3.79              | 7.27               |
| 100  | 11.70                               | 40.00                          | 12.00             | 23.00              |

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**Note 1:** At 80 Hz and 800 MHz, the separation distance for the higher frequency range applies.

**Note 2:** The ISM (industrial, scientific and medical) bands between 150 kHz and 80 MHz are 6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz.

**Note 3:** An additional factor of 10/3 is used in calculating the recommended separation distance for transmitters in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2.5 GHz to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas.

**Note 4:** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

## ECG Analysis Algorithm Accuracy

Sensitivity and specificity are expressions of ECG analysis algorithm performance when compared to ECG interpretation by a clinician or expert. Sensitivity refers to the algorithm’s ability to correctly identify shockable rhythms (as a percentage of the total number of shockable rhythms). Specificity refers to the algorithm’s ability to correctly identify non-shockable rhythms (as a percentage of the total number of non-shockable rhythms). The data in the following table summarizes the accuracy of the ECG analysis algorithm as tested against ZOLL’s ECG Rhythm Database.

The algorithm sequence takes approximately 9 seconds and proceeds as follows:

- Divides the ECG rhythm into three-second segments.
- Filters and measures noise, artifact, and baseline wander.
- Measures baseline content (‘waviness’ at the correct frequencies — frequency domain analysis) of signal.
- Measures QRS rate, width, and variability.
- Measures amplitude and temporal regularity (‘auto-correlation’) of peaks and troughs.
- Determines if multiple 3 second segments are shockable then displays *SHOCK ADVISED* message.

### Clinical Performance Results

The performance of the incorporated analysis algorithm in a single analysis sequence satisfies the applicable requirements specified in ANSI/AAMI DF80 (section 6.8.3) and the recommendations by Kerber et al. (Circulation. 1997;95(6):1677).

**Table A-4. Clinical Performance Results**

| Rhythms   | Sample Size | Performance Goals  | Observed Performance | 90% One-sided Lower Confidence Limit |
|---|-------------|--------------------|----------------------|--------------------------------------|
| <b>Shockable</b>                                |             | <b>Sensitivity</b> |                      |                                      |
| Coarse VF                                       | 536         | >90%               | >99%                 | >99%                                 |
| Rapid VT  | 80          | >75%               | >99%                 | >96%                                 |
| <b>Non-shockable</b>                            |             | <b>Specificity</b> |                      |                                      |
| NSR   | 2210        | >99%               | >99%                 | >99%                                 |
| AF, SB, SVT, Heart block, idioventricular, PVCs | 819         | >95%               | >99%                 | >99%                                 |
| Asystole  | 115         | >95%               | >99%                 | >97%                                 |
| <b>Intermediate</b>                             |             | <b>Sensitivity</b> |                      |                                      |
| Fine VF   | 69          | Report only        | >89%                 | >81%                                 |
| Other VT  | 28          | Report only        | >96%                 | >84%                                 |

### References:

Young KD, Lewis RJ: “What is confidence? Part 2: Detailed definition and determination of confidence intervals”. *Annals of Emergency Medicine*, September 1997; 30; 311-218

William H. Beyer, Ph.D.: “CRC Standard Mathematical Tables 28<sup>th</sup> Edition,” CRC Press, Inc, Boca Raton, FL., 1981, Percentage Points, F-Distribution Table, pg 573.



# Appendix B

## Accessories

The following accessories are compatible for use with the Propaq MD. To order any of these items, contact your local ZOLL representative.

|   |
|---|
| <b>ECG Accessories</b>  |
| ECG, 3-Lead Cable, AAMI   |
| ECG, 3-Lead Cable, IEC  |
| ECG, 5-Lead Cable, AAMI   |
| ECG, 5-Lead Cable, IEC  |
| ECG, 12-Lead "Breakaway" Patient Cable Complete (Trunk Cable, 4-lead wire set with detachable 6 "V" precordial lead wire set), AAMI |
| ECG, 12-Lead "Breakaway" Patient Cable Complete (Trunk Cable, 4-lead wire set with detachable 6 "V" precordial lead wire set), IEC  |
| ECG, 4-Lead "Breakaway" Trunk Cable and 4 lead wire set only, AAMI  |
| ECG, 4-Lead "Breakaway" Trunk Cable and 4 lead wire set only, IEC   |
| ECG, Detachable 6 precordial lead wire set for "Breakaway" 12-Lead Patient Cable, AAMI  |
| ECG, Detachable 6 precordial lead wire set for "Breakaway" 12-Lead Patient Cable, IEC   |
|   |
| <b>CO2 Accessories (Oridion Filterlines)</b>  |
| Smart CapnoLine Plus, Non-intubated filterline with O2 Delivery, Adult, box of 25   |
| Smart CapnoLine Plus, Non-intubated filterline with O2 Delivery, Pediatric, box of 25   |

ACCESSORIES

|  |
|--|
| FilterLine H Set, Adult/Pediatric, box of 25   |
| FilterLine H Set, Infant/Neonate, box of 25  |
| FilterLine Set, Adult/Pediatric, box of 25   |
| VitaLine H set Adult/Pediatric , box of 25   |
|  |
| <b>NIBP Accessories</b>  |
| <b>Hoses</b>   |
| Hose, Infant/Neonate, 8', w/ female luer cuff connector, single lumen                      |
| Hose, Adult/Pediatric, 10', w/ "twist lock" cuff connector, dual lumen                     |
| <b>Reusable Cuffs (Welch Allyn Blood Pressure Flexiport Cuffs)</b>                         |
| Neonate #1, 3.3 - 5.6 cm single tube w/ male luer connector, box of 10                     |
| Neonate #2, 4.2 - 7.1 cm single tube w/ male luer connector, box of 10                     |
| Neonate #3, 5.4 - 9.1 cm single tube w/ male luer connector, box of 10                     |
| Neonate #4, 6.9 - 11.7 cm single tube w/ male luer connector, box of 10                    |
| Neonate #5, 8.9 - 15.0 cm single tube w/ male luer connector, box of 10                    |
| Neonatal Cuff Kit, one each of sizes #1 - #5, single tube w/ male luer connector, bag of 5 |
| Welch Allyn REUSE-07-2MQ Cuff, Infant, 2-Tube, Twist lock connector                        |
| Welch Allyn REUSE-08-2MQ Cuff, Small Child, 2-Tube, Twist lock connector                   |
| Welch Allyn REUSE-09-2MQ Cuff, Child, 2-Tube, Twist lock connector                         |
| Welch Allyn REUSE-10-2MQ Cuff, Small Adult, 2-Tube, Twist lock connector                   |
| Welch Allyn REUSE-11-2MQ Cuff, Adult, 2-Tube, Twist lock connector                         |
| Welch Allyn REUSE-11L-2MQ Cuff, Adult Long, 2-Tube, Twist lock connector                   |
| Welch Allyn REUSE-12-2MQ Cuff, Lg Adult, 2-Tube, Twist lock connector                      |
| Welch Allyn REUSE-12L-2MQ Cuff, Lg Adult Long, 2-Tube, Twist lock connector                |
| Welch Allyn REUSE-13-2MQ Cuff, Thigh, 2-Tube, Twist lock connector                         |
|  |
| <b>SpO2 Accessories</b>  |
| Nellcor MAX-PAC adhesive SpO2 sensor assortment sample pack                                |
| Nellcor - DS-100A Durasensor (Adult), reusable   |
| Nellcor - 8' Differential Extension Cable, Pulse Oximeter Sensor                           |
|  |
| <b>Temperature Accessories</b>   |
| YSI Reusable Adult, Skin Probe   |
| YSI Reusable Pediatric, Skin Probe   |

|   |
|---|
| YSI Reusable Adult, Esophageal/Rectal   |
| YSI Reusable Pediatric, Esophageal/Rectal   |
| Sensor Adapter Cable for Disposable Probes  |
| YSI Disposable Esophageal/Rectal Probe  |
| YSI Disposable Skin Probe   |
|   |
| <b>Defibrillation Accessories</b>   |
| Propaq MD Multifunction Therapy Cable - allows use of disposable multifunction electrodes and ZOLL M Series CCT External and Internal paddles (sold separately) |
| M Series External Paddles set with controls and built-in pediatric electrodes   |
| ZOLL Sterilizable Internal Handle (black), with Switch, 1.0" spoon, 10' cable   |
| ZOLL Sterilizable Internal Handle (black), with Switch, 1.6" spoon, 10' cable   |
| ZOLL Sterilizable Internal Handle (black), with Switch, 2.0" spoon, 10' cable   |
| ZOLL Sterilizable Internal Handle (black), with Switch, 2.7" spoon, 10' cable   |
| ZOLL Sterilizable Internal Handle (black), with Switch, 3.0" spoon, 10' cable   |
| ZOLL Sterilizable Internal Handle (black), without Switch, 1.0" spoon, 10' cable  |
| ZOLL Sterilizable Internal Handle (black), without Switch, 1.6" spoon, 10' cable  |
| ZOLL Sterilizable Internal Handle (black), without Switch, 2.0" spoon, 10' cable  |
| ZOLL Sterilizable Internal Handle (black), without Switch, 2.7" spoon, 10' cable  |
| ZOLL Sterilizable Internal Handle (black), without Switch, 3.0" spoon, 10' cable  |
| ZOLL Defibrillation Gel   |
|   |
| <b>IBP Accessories</b>  |
| Transducer Interface cable – Abbott   |
| Transducer Interface cable - Edwards  |
| Transducers: 5 $\mu$ V/V/mm Hg, IEC 60601-2-34 and AAMI BP-22 compliant   |
|   |
| <b>Power Accessories</b>  |
| ZOLL Propaq MD Lithium Ion Rechargeable Battery   |
| SurePower Charging Station  |
| SurePower Battery Charger Propaq MD Battery Adapter   |
| Propaq MD Power Adapter   |
| Replacement Power Cord – U.S.   |
|   |

## ACCESSORIES

---

|                                |
|--------------------------------|
| <b>Other Accessories</b>       |
| Propaq MD Molded Carrying Case |
| ECG 50mm Chart Recorder Paper  |