

ZOLL AED 3™
Accessories and Training Products

ZOLL®



ACCESSORIES FOR EVERY RESCUE


DISPOSABLE RESCUE ACCESSORIES

Long-lasting Accessories for Every Rescue

CPR Uni-padz™

Truly universal pads. Use the Child Button to switch easily and quickly from an adult to a pediatric rescue. No pads to change. No key required. Five-year installed life. Provides full CPR feedback.

Part # 8900-000260.

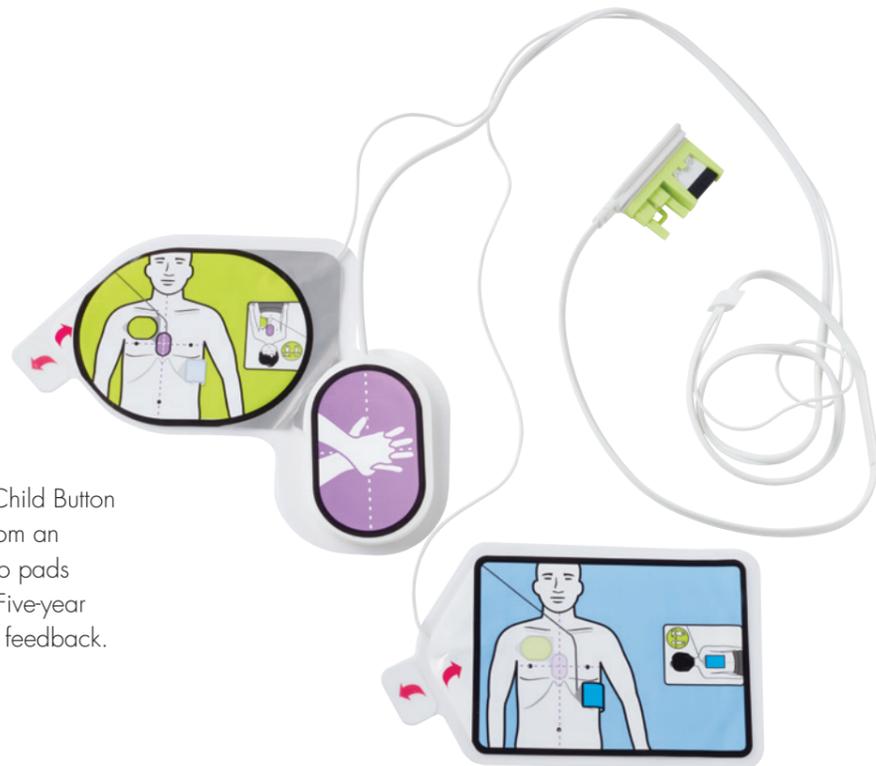

Every set of CPR Uni-padz comes with a Rescue Accessory Kit attached. Includes scissors for cutting clothing to expose the victim's chest, non-latex gloves and breathing mask for rescuer protection, paper towel to assure dry contact with electrodes, razor for removing hair, and a wipe for clean-up.

Battery

In stand-by mode, the ZOLL AED 3 disposable lithium manganese dioxide battery has an installed life of five years. Unless the ZOLL AED 3 is used in a rescue, replace disposable accessories only once every five years.

Part # 8000-000696



TRAINING

Training Simulations for Every Rescuer

Battery-powered Electronic Simulator

Generate either shockable or non-shockable heart rhythms for various rescue scenarios. Includes 9-volt battery.

Part # 8000-000925



Simulation Pads

Connect to ZOLL AED 3 and simulator for simulation with the unit. Includes one set of two reusable training gels.

Part # 8900-000264



Training Gels

Eight sets of two reusable training gels to be used with simulation pads.

Part # 8900-000265



Training Manikin

Simulates 60-kg victim. Removable head for compact storage and hands-only training.

Part # 8000-0835-01





Standard Surface-mounted Wall Cabinet
Part# 8000-001256

WALL CABINETS

Secure and Accessible

Keep your ZOLL AED 3 secure and easily accessible for medical emergencies with a variety of battery-operated, alarmed wall cabinets.* Each holds a ZOLL AED 3 plus a backup set of electrodes and a backup battery should you need them. Choose the surface-mounted model for

easy installation or the semirecessed or fully recessed models for less intrusive storage. All three models include an ILCOR (International Liaison Committee on Resuscitation) AED label set and installation instructions.



Semirecessed Wall Cabinet
Part# 8000-001257



Fully Recessed Wall Cabinet
Part# 8000-001258

*9-volt battery included

AED WALL SIGNS

ILCOR* Format

Flat AED Wall Sign

ILCOR Flush Wall Sign (AED)
Part# 8000-001260



ILCOR Flush Wall Sign (DAE)
Part# 8000-001262



ILCOR Flush Wall Sign (DEA)
Part# 8000-001264



Three-dimensional AED Wall Sign

ILCOR 3D Wall Sign (AED)
Part# 8000-001261



ILCOR 3D Wall Sign (DAE)
Part# 8000-001263



ILCOR 3D Wall Sign (DEA)
Part# 8000-001265



Strobe for Standard
Surface-mounted Wall Cabinet
Part# 8000-001259



Strobe for Semi/Fully Recessed
Wall Cabinet
Part# 8000-001267

*International Liaison Committee on Resuscitation

WALL MOUNTS

A Simple Universal Wall Mount

The Universal Wall Mount holds the ZOLL AED 3 alone or in its carry case and is labeled with the green ILCOR AED symbol. It includes mounting hardware, consumable tracking card, wall-mount template, and installation instructions.



Universal Wall Mount with ZOLL AED 3
alone (without carry bag)
Part# 8000-001255



Universal Wall Mount with ZOLL AED 3,
carry case, and backup CPR Uni-padz
(in carry case)
Part# 8000-001266

CARRY CASES

Secure and Accessible

Choose from among three different carry case models to contain and carry the ZOLL AED 3: the Standard Carry Case (included with ZOLL AED 3 BLS), the slim Hard-shell Carry Case, or the large Hard-shell Carry Case. The hard-shell cases are waterproof, with an IP68 rating for dust and water ingress.


Standard Carry Case

In addition to the ZOLL AED 3 with pre-connected CPR Uni-padz, the Standard Carry Case can also hold a backup set of CPR Uni-padz.

Part# 8000-001250

Replacement Shoulder Strap For ZOLL AED 3 Carry Case

Part# 8000-001252

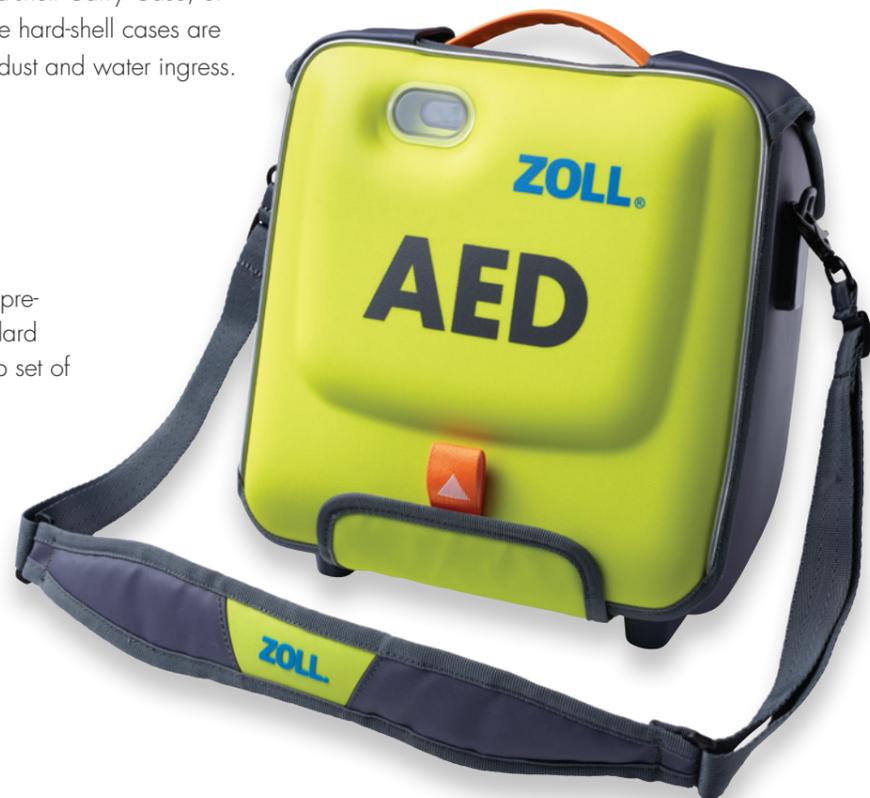

Large Hard-shell Carry Case

Protect and carry everything needed for a vigorous rescue with the large Hard-shell Carry Case, including a backup set of CPR Uni-padz and a backup battery.

Part# 8000-001254



Slim Hard-shell Carry Case

For lightweight maximum protection, choose the slim Hard-shell Carry Case in situations where a vigorous rescue is likely.

Part# 8000-001253



Battery Case

Store the ZOLL AED 3 battery in this convenient pouch for easy identification and access when needed. Stores easily in carry bag.

Part# 8000-001251



ZOLL AED 3 SERVICES

Program Management Onboard with ZOLL's PlusTrac

Seamlessly maintain and track the ZOLL AED 3 with Program Management Onboard™. Cloud-connect it to PlusTrac™, ZOLL's online AED program management system, in a few minutes when you install your ZOLL AED 3.

Program Management Onboard will regularly report, upon completion of every automatic self-check, the status of your ZOLL AED 3. If the self-check detects a fault, PlusTrac will email you the details immediately.

You can also log in to your PlusTrac account at any time to review all the AEDs for which you are responsible, grouped by sites you manage, and showing training status of your volunteer responders.

PlusTrac makes AED program management seamless and easy, and Program Management Onboard allows your ZOLL AED 3 to maintain itself.

With every ZOLL AED 3 purchased, the first year of Program Management Onboard coverage is totally free of charge. You can purchase additional years of Program Management Onboard coverage in increments of one or five years. When you extend PlusTrac coverage through your authorized ZOLL distributor, you will receive an official PlusTrac Certificate with a code that can be used to extend coverage for one or five more years.



PlusTrac Certificate for ONE year of coverage
Part# 8000-1110-01

PlusTrac Certificate for FIVE years of coverage
Part# 8000-1111-01



READILY AVAILABLE ACCESSORIES AND SERVICES

Need to know more about ZOLL AED 3 accessories and services?

Contact your local authorized ZOLL distributor or visit ZOLL at zoll.com/aed3.

To learn more about AED program management with PlusTrac, visit us at plustrac2.com.

Authorized ZOLL distributors can help you with your complete AED program, including:

- Program management and maintenance
- Compliance with local regulations and registries
- Training for your volunteer responders
- Replenishment of accessories
- Adding more AEDs to your program.

ZOLL MEDICAL CORPORATION

An Asahi Kasei Group Company | 269 Mill Road | Chelmsford, MA 01824 | 978-421-9655 | www.zoll.com

Copyright © 2016 ZOLL Medical Corporation. All rights reserved. ZOLL AED 3, PlusTrac, Uni-padz, Program Management Onboard, and ZOLL are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are the property of their respective owners.

The ZOLL AED 3 and Program Management Onboard are not available for sale in the United States. The product has not received regulatory clearance/approval by the US Food and Drug Administration.

Printed in U.S.A.
MCN PP 1606 0183-05

For subsidiary addresses and fax numbers, as well as other global locations, please go to www.zoll.com/contacts.

ZOLL®