

Instructions for Downloading and Installing ZOLL Administrative Software (ZAS) for the AED Plus® on your Computer


Once the zip file containing the software has been downloaded, extract all files and navigate into the subfolders “AEDPlusSoftwareUpgradeSetup\AED Plus 2010 Guidelines Software Update”. Click the application:

“**AEDPlusSoftwareUpgradeSetup**” this will open the executable file that installs the software on your PC.

1. Select “Run” or “Yes” depending on you operating system.



2. Follow the Installer prompts.

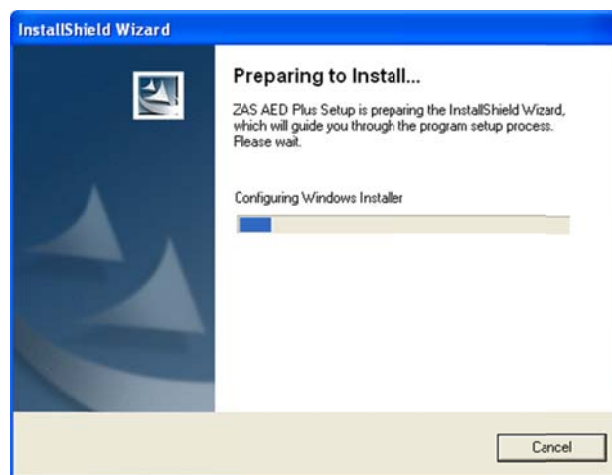


3. If the InstallShield Wizard detects a previous version of ZAS for the AED Plus already installed on your computer, it will prompt you to first remove the old version. Select “Next” to start this process.


Note: If you do *not* have a previous version of ZAS installed on your computer, the InstallShield Wizard will automatically load the program on your computer (See steps 6-12 below).



After you select “Next”, you will need to confirm the removal by selecting “OK”. The InstallShield Wizard will delete your older version of ZAS, preserving all saved configuration files.

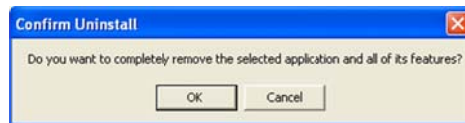


4. When the installer is finished removing the previous version of ZAS, select “Finish.”

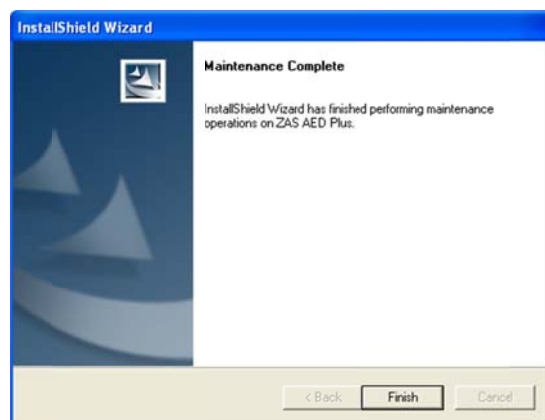

- When the InstallShield Wizard finishes, you have successfully removed your previous version of ZAS. To install ZAS v6.24, select **“AEDPlusSoftwareUpgradeSetup”** a second time to re-open the executable file to install the software.
- Select “Run” or “Yes” depending on you operating system.



- Follow the Installer prompts.

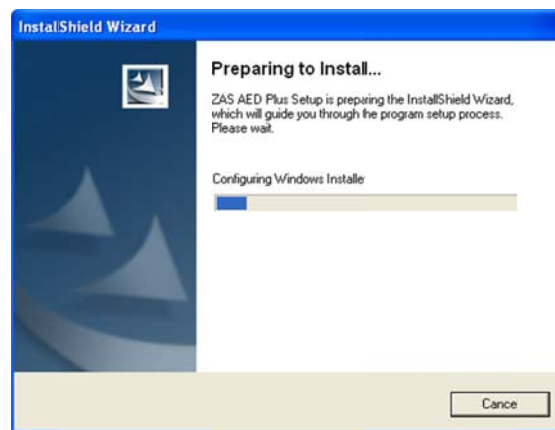

- Please read and accept the License Agreement by selecting “Yes.”




9. Complete the form by filling in your user name and company name. Then select “Next.”



10. When the installation process is complete, select “Finish.”



11. The InstallWizard will install *both* ZAS and the AED Plus Upgrade Wizard onto your computer. When the installation process is complete, the program will close, and a new shortcut icon called “AED Plus Upgrade Wizard” will appear on your desktop. This wizard program allows properly upgrades the internal software on your ZOLL AED Plus via an infrared port.

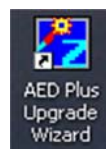

12. When you are ready to upgrade the software on your AED Plus unit(s), be sure your infrared port is functioning, and then open the AED Plus Upgrade Wizard icon on your desktop. For further instructions about how to use the AED Plus Upgrade Wizard to upgrade an AED Plus, refer to the “AED Plus Software Upgrade Instructions” document in the folder (AEDPlusSoftwareUpgradeSetup) you downloaded from the web earlier.

If you experience difficulties installing ZAS v6.24, or if you have further questions, please contact ZOLL Technical Service at 800-348-9011.

* * * * *