

# ZOLL AED 3® Training Power Adapter



## Operate the clinical ZOLL AED 3 device in a training scenario without compromising battery level and performance

The ZOLL AED 3® Training Power Adapter ("TPA") is a mains (wired) power source (AC) which converts the ZOLL AED 3 clinical device to function as a trainer and provides full functionality except delivery of defibrillation therapy.

### Saves on battery power

ZOLL AED 3 Training Power Adapter adapts your ZOLL AED 3 to use AC power, which will preserve the power from the battery on your device. You could even consider using TPA for the setup of clinical settings or Wi-Fi configuration. Furthermore, the TPA has no serviceable parts. When training is complete, simply replace the TPA with the gray ZOLL AED 3 battery pack for clinical use.

### Connect and Practice

With the ZOLL AED 3 Training Power Adapter, you can now practice defibrillation and CPR with unparalleled support, thanks to Real CPR Help® and your clinical ZOLL AED 3 or ZOLL AED BLS 3 device.

Install the blue training power adapter, connect it to the circular connector and mains power, turn on the device and you will immediately be able to run a complete training scenario with real-time feedback from Real CPR Help using the CPR Uni-padz® training kit and simulator.



### Specifications

#### Part Number

8000-002008 - North America  
8000-002009 - Europe  
8000-002010 - United Kingdom  
8000-002011 - Australia  
8000-002012 - China  
8000-002013 - South Korea  
8000-002014 - Japan

#### Accessories

8000-000284  
CPR Uni-padz Training Kit

#### Compatibility

ZOLL AED 3  
ZOLL AED 3 BLS  
(Requires both devices to have software Rel. 6.03.006 installed)

#### Dimensions

Weight: 317.5 grams  
Size: (H x W x D)  
27.75 mm x 133 mm x 88 mm