
Training CPR-D-padz®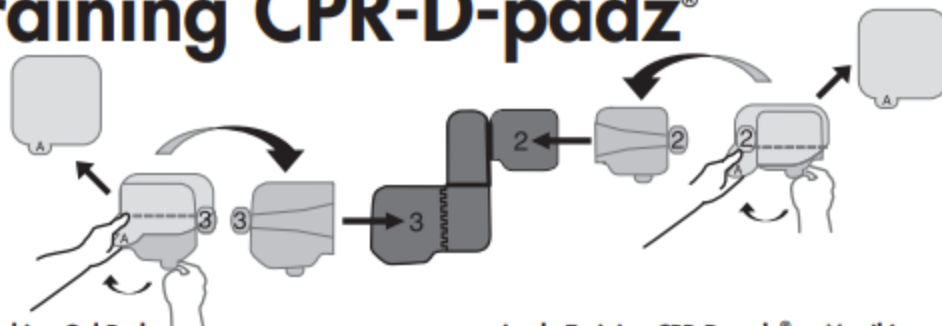

Attaching Gel Pads:

1. Place Training CPR-D-padz® gel side up.
2. Remove "A" from new gel and apply at 2 as shown. Do not remove gel cover 2.
3. Remove "A" from new gel and apply at 3 as shown. Do not remove gel cover 3.

Apply Training CPR-D-padz® to Manikin:

1. Lay electrode on manikin, align CPR device with nipple line of manikin.
2. Remove gel covers 2 and 3, press firmly. Save gel covers for re-use.
3. When finished, remove Training CPR-D-padz® from manikin and re-apply gel covers. Place in poly bag for storage.

OPERATING TEMPERATURE: 0°C to 50°C (32°F to 122°F)

SHORT TERM STORAGE TEMPERATURE: -30°C to 65°C (-22°F to 149°F)

LONG TERM STORAGE TEMPERATURE: 0°C to 35°C (32°F to 95°F)

IMPORTANT:

**Not for clinical use.
For training purposes only.**

ZOLL®

ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA 01824-4105 USA
800-348-9011 Made in USA

ZOLL International Holding B.V.
Newtonweg 18 6662 PV ELST
The Netherlands Tel. +31 (0) 481 366410

NOT MADE WITH
NATURAL RUBBER LATEX.

R2001-39 Rev.J

TRAINING CPR-D-padz®

Applying new gel adhesive electrodes:

1. Position training CPR-D-padz with the gel side facing up.
2. Remove "A" from the new gel and attach to 2, as shown. Do not remove gel cover 2.
3. Remove "A" from the new gel and attach to 3, as shown. Do not remove gel cover 3.

OPERATING TEMPERATURE: 0°C to 50°C (32°F to 122°F)
SHORT-TERM STORAGE TEMPERATURE: -30°C to 65°C (-22°F to 149°F)
LONG-TERM STORAGE TEMPERATURE: 0°C to 35°C (32°F to 95°F)

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Not for clinical use.

Only for training purposes.

Applying training CPR-D-padz® to the anatomical model:

1. Apply electrode to the anatomical model, align the CPR device with the nipple line of the model.
2. Remove gel covers 2 and 3. Press firmly into place. Save gel covers for reuse.
3. When finished, remove training CPR-D-padz® from the model and reattach the gel covers. Place in poly bag for storage.

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