Pedi-padz® II

ENGLISH

Intended Purpose

To transfer energy for defibrillation, cardioversion, non-invasive pacing therapy to the heart and provide ECG monitoring.

Indications for use

For use on pediatric patients with ZOLL® AED Plus®, AED Pro®, AED 3®, AED 3® BLS, R Series®, X Series®, Advanced and Propaq® MD. The pedi-padz II electrode are indicated for use on a patient less than 8 years of age or weighing less than 55lbs(25kg). For patients over 25 kg, adult electrodes are recommended. Do not delay therapy to determine actual weight of patient during emergency.

Important: This symbol indicates that the AED Plus® unit is equipped for treating adults and pediatric patients. An AED Plus unit without this symbol is not equipped to treat pediatric patients and *will not work* with the Pedi-padz® II pediatric electrodes. To upgrade an AED Plus unit for use with Pedi-padz II pediatric electrodes, contact the ZOLL Customer Service Department for information on the ZOLL AED Plus Pediatric Upgrade Kit.

Instructions for Use

1. Wipe skin dry.
2. Apply electrode by attaching one edge securely to the patient and "rolling" smoothly from that edge to the other. Be careful not to trap any pockets of air between the gel and the skin.
3. Connect electrodes.
4. Connect ECG leads for demand pacing.

WARNINGS:

- Transcutaneous pacing may cause burns to the skin. Periodically check the electrode site to ensure that the electrodes are well adhered to the skin.
- During transcutaneous pacing, do not exceed the maximum pacing setting of 1 hour of pacing (75 mA/150 ppm). Doing so can increase the possibility of skin burns.
- Replace electrodes after 24 hours of skin contact or 1 hour of pacing to maximize patient benefit.
- Do not open until ready to use.
- Do not use if gel is dry.
- If repositioning of the electrodes is needed, consider replacement with a new electrode.
- Device disposal shall follow hospital protocol.
- Use only with ZOLL Pacemaker/Defibrillator products.
- Do not use electrodes in the presence of oxygen-environment or other flammable agents. Doing so could cause explosion.
- If any serious incident has occurred in relation to the device, the incident should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
- Do not fold.

OPERATING TEMPERATURE: 0 °C to 50 °C (32 °F to 122 °F)
SHORT TERM STORAGE TEMPERATURE: -30 °C to 65 °C (-22 °F to 149 °F)
LONG TERM STORAGE TEMPERATURE: 0 °C to 35 °C (32 °F to 95 °F)

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