

CPR-D-padz® DEMO PAD

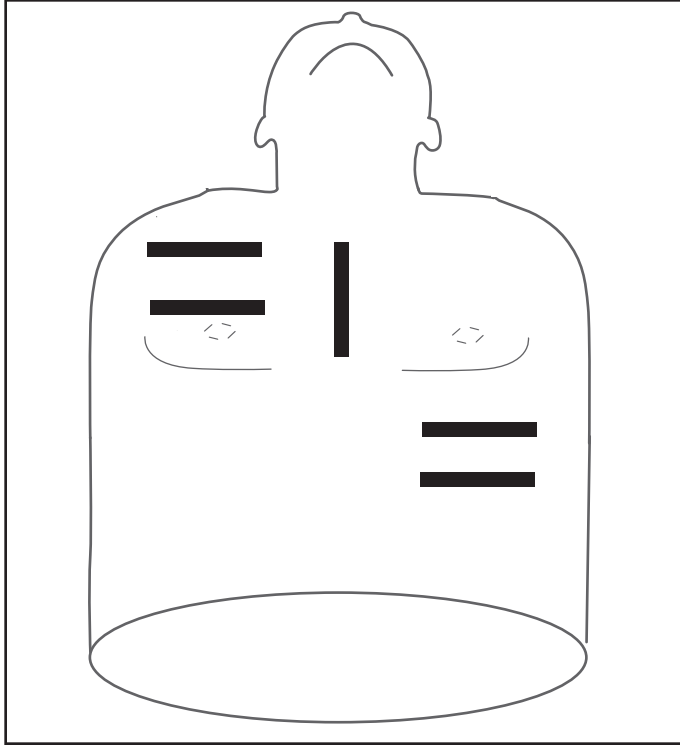


Figure 1 Velcro Placement

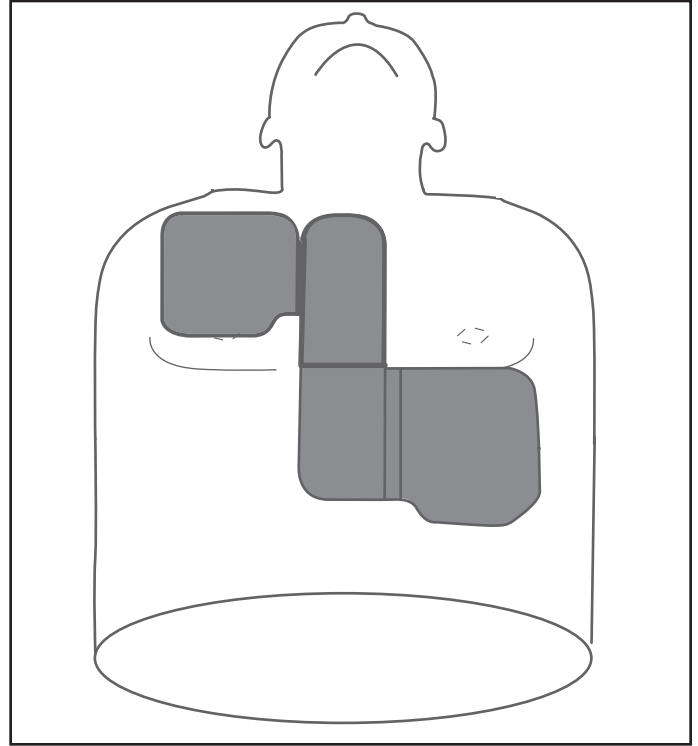


Figure 2 CPR-D Demo Pad Placement

1. Apply velcro loop strips to manikin as shown in Figure 1.
2. Place CPR-D Demo Pad on manikin as shown in Figure 2.
3. Demonstrate gel cover removal as shown in Figure 3 by holding the CPR-D pad with the right hand and pulling the tab on the sternum pad with the left. Press sternum pad to secure to manikin. Holding the CPR-D pad with the left hand, pull the apex tab with the right hand to remove the gel cover. Press the apex pad to secure.
4. Attach green connector to AED plus and multi -function connector to Simulator.

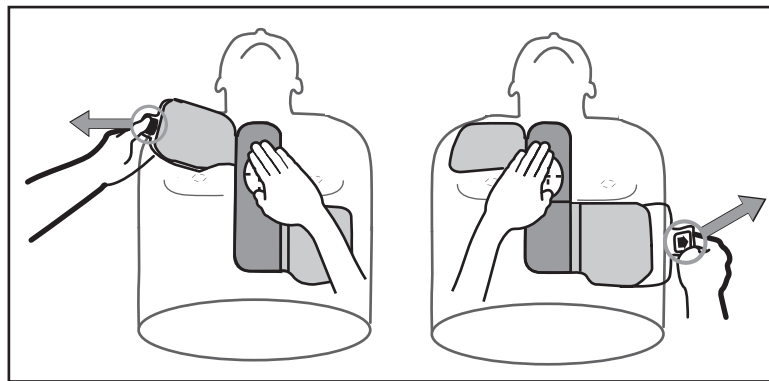



Figure 3 Gel cover removal

ZOLL

ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA 01824-4105 USA
800-348-9011 Made in USA

EC REP ZOLL International Holding B.V.
Newtonweg 18 6662 PV ELST
The Netherlands Tel. +31 (0) 481 366410

OPERATING TEMPERATURE: 0°C to 50°C (32°F to 122°F)
SHORT TERM STORAGE TEMPERATURE: -30°C to 65°C (-22°F to 149°F)
LONG TERM STORAGE TEMPERATURE: 0°C to 35°C (32°F to 95°F)

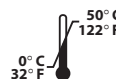

NOT MADE WITH
NATURAL RUBBER LATEX.

R1345-106 Rev. E