INSTRUCTIONS FOR USE

Defibrillation Pads

Use Cardiac Science® defibrillation pads with the Powerheart® G5 AED for emergency treatment of sudden cardiac arrest. The AED automatically identifies electrode pads connected to it, and adjusts the maximum shock energy.

- For patients over 8 years of age or heavier than 55lbs (25kg)
- Intended for short-term use (<8hrs)
- Use before the expiration date listed on the package.
- The AED automatically reads the expiration date of the electrode pads, and notifies the user of expired pads.

WARNING! Do not reuse pads.
Reuse of pads may result in skin burns, improper AED performance or patient-to-patient contamination.

DO NOT open until ready to use.
DO NOT use damaged pads.
Failure to remove blue plastic completely could impact therapy delivery. Before use, check cable and connectors for damage.
DO NOT pull on the lead wires to separate the pads from the blue plastic.
Not intended for use in pacing.

When placing pads:
Place the pad at least an inch from any implanted device.
Ensure patient's skin is clean and dry before use.

Manufactured for:
Cardiac Science Corporation
500 Burdick Parkway
Deerfield, WI 53531 USA
www.cardiacscience.com

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English

Patents pending.
For a complete list of U.S. and foreign patents,
see www.cardiacscience.com.

2797

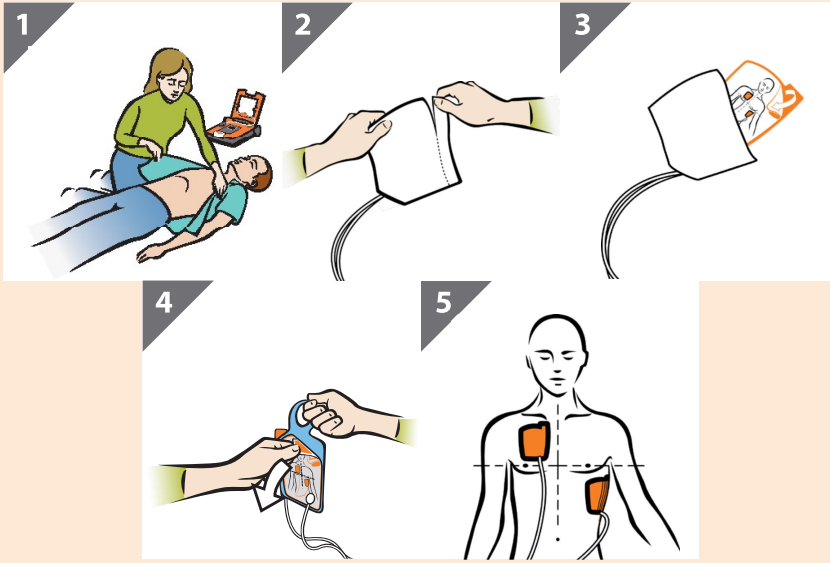
Not made with
natural rubber latex

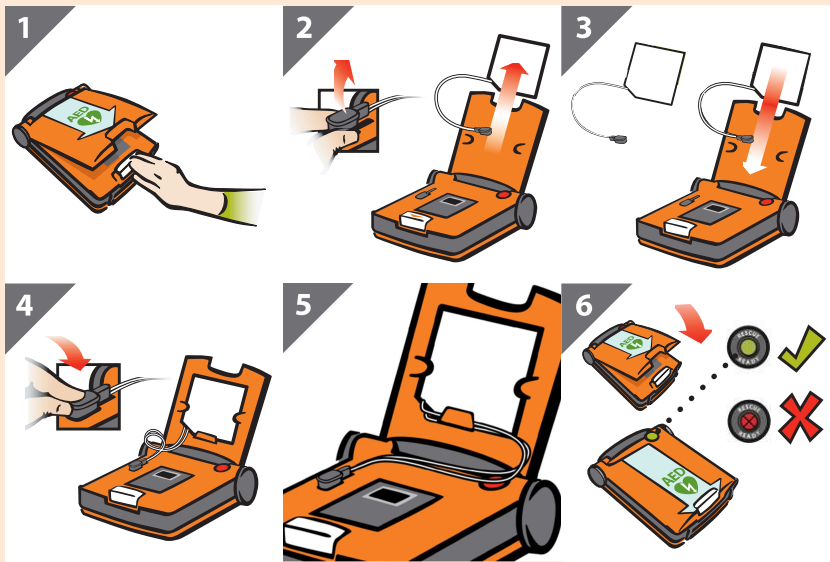
Cardiac Science, Powerheart, the Shielded Heart logo and Rescue Ready are trademarks of Cardiac Science Corporation.
Copyright © 2019 Cardiac Science Corporation.
All Rights Reserved.

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Cardiac Science, Powerheart, le logo Shielded Heart et Rescue Ready sont des marques commerciales de Cardiac Science Corporation. Copyright © 2019 Cardiac Science Corporation. Tous droits réservés.
Cardiac Science, Powerheart, det afskærmede hjerte-logo (Shielded Heart) og Rescue Ready er varemærker tilhørende Cardiac Science Corporation. Copyright © 2019 Cardiac Science Corporation. Alle rettigheder forbeholdes.
Cardiac Science, Powerheart, Shielded Heart logo a Rescue Ready jsou obchodní značky společnosti Cardiac Science Corporation. Copyright © 2019 Cardiac Science Corporation. Všechna práva vyhrazena.
Copyright ©2019 Cardiac Science Corporation.
MDSS GmbH
Telif Hakkı © 2019 Cardiac Science Corporation.