# INSTRUCTIONS FOR USE

## Defibrillation Pads with CPR Device

Use Cardiac Science® defibrillation pads with the Powerheart® G5 AED for emergency treatment of sudden cardiac arrest. The AED automatically identifies electrode pads connected to it, and adjusts the maximum shock energy.

- For patients over 8 years of age or heavier than 55lbs (25kg)
- Intended for short-term use (<8hrs)
- Use before the expiration date listed on the package.
- The AED automatically reads the expiration date of the electrode pads, and notifies the user of expired pads.

**WARNING!** Do not reuse pads.  
Reuse of pads may result in skin burns, improper AED performance or patient-to-patient contamination.

**DO NOT** open until ready to use.  
**DO NOT** use damaged pads.  
Failure to remove blue plastic completely could impact therapy delivery.  
Before use, check cable and connectors for damage.  
**DO NOT** pull on the lead wires to separate the pads from the blue plastic.  
Not intended for use in pacing.

**When placing pads:**  
Place the pad at least an inch from any implanted device.  
Ensure patient's skin is clean and dry before use.

**Manufactured for:**  
Cardiac Science Corporation  
500 Burdick Parkway  
Deerfield, WI 53531 USA  
[www.cardiacscience.com](http://www.cardiacscience.com)

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### English

Patents pending.  
For a complete list of U.S. and foreign patents,  
see [www.cardiacscience.com](http://www.cardiacscience.com).

Not made with  
natural rubber latex

2797

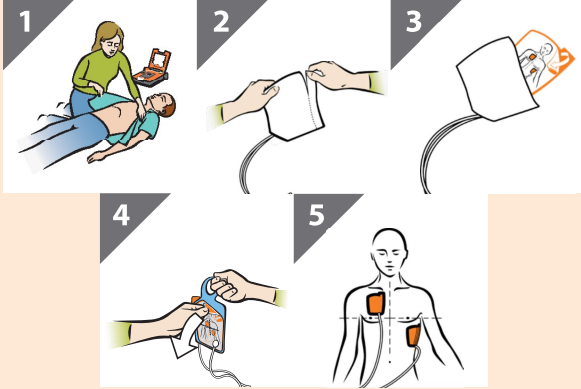
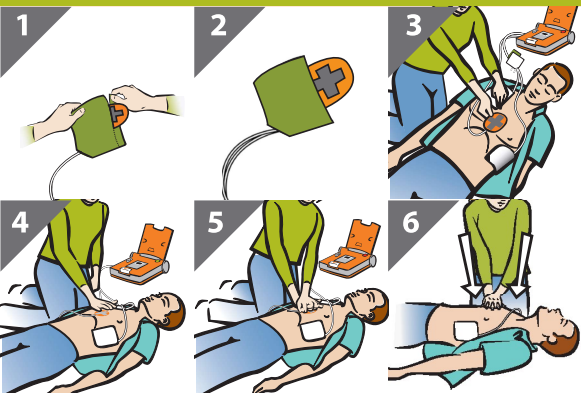
Cardiac Science, Powerheart, the Shielded Heart logo and Rescue Ready are trademarks of Cardiac Science Corporation.  
Copyright © 2019 Cardiac Science Corporation.  
All Rights Reserved.

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Rescue Ready - i Shielded Heart, הלוגו Powerheart, Cardiac Science הם סימני מסחר של Cardiac Science Corporation. Copyright © 2019 Cardiac Science Corporation. כל הזכויות שמורות.
MDSS GmbH
Cardiac Science Corporation şirketinin ticari markalarıdır.
Telif Hakkı © 2019 Cardiac Science Corporation.