

# EEN REDDINGSACTIE UITVOEREN

1



Reageert de patiënt?

2



Bel het noodnummer!

3



Haal AED.

4




Plaats AED op de grond.  
Open deksel.

5



Verwijder kleding.  
Ontbloot borst.

6

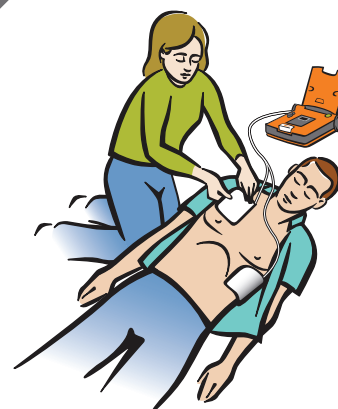

Bevestig pads.

7




Raak de patiënt niet aan!

8




Als opgedragen,  
druk op rode knipperende knop.

9

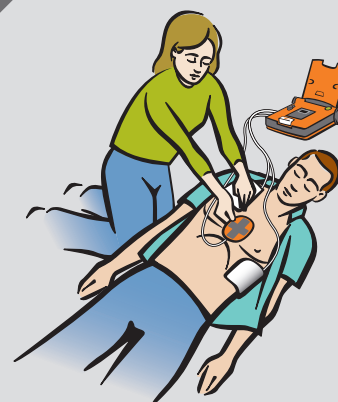

Indien aanwezig,  
plaats reanimatieapparaat.

10

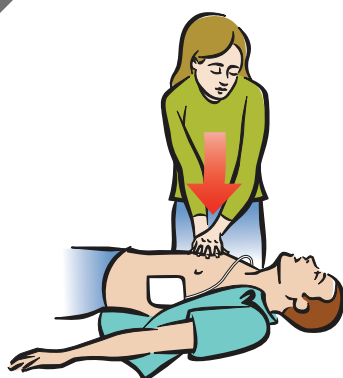

Volg instructies van AED.  
Geef compressies.

11



Als opgedragen,  
geef beademingen.

## Semi-automatische AED



Cardiac Science en het Shielded Heart-logo zijn handelsmerken van Cardiac Science Corporation.  
Copyright © 2019, Alle rechten voorbehouden.  
500 Burdick Parkway, Deerfield, Wisconsin 53531 VS  
[www.cardiacscience.com](http://www.cardiacscience.com)



MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Duitsland



70-01972-86 A



# PERFORMING A RESCUE

1



Is patient responsive?

2



Call emergency services!

3



Get AED.

4




Place AED on ground.  
Open lid.

5



Remove clothing.  
Expose chest.

6

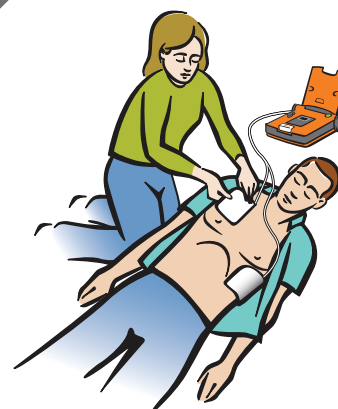

Attach pads.

7




Do not touch the patient.

8




If instructed,  
press red flashing button.

9

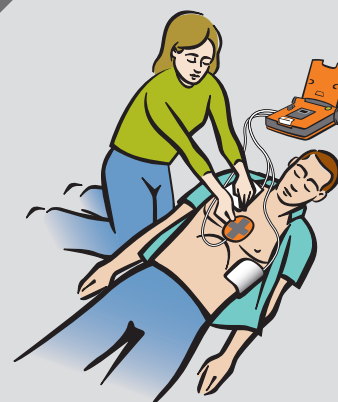

If present, place CPR device.

10

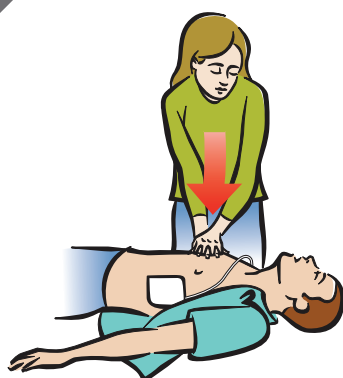

Follow AED prompts.  
Give compressions.

11



If instructed, give breaths.

## Semi-automatic AED



Cardiac Science and the Shielded Heart logo are trademarks of Cardiac Science Corporation.  
Copyright © 2019 All Rights Reserved.  
500 Burdick Parkway, Deerfield, Wisconsin 53531 USA  
[www.cardiacscience.com](http://www.cardiacscience.com)



MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany



70-01972-86 A

