

EEN REDDINGSACTIE UITVOEREN

1



Reageert de patiënt?

2




Bel het noodnummer!

3



Haal AED.

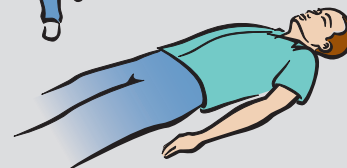

4




Plaats AED op de grond.
Open deksel.

5



Verwijder kleding.
Ontbloot borst.

6

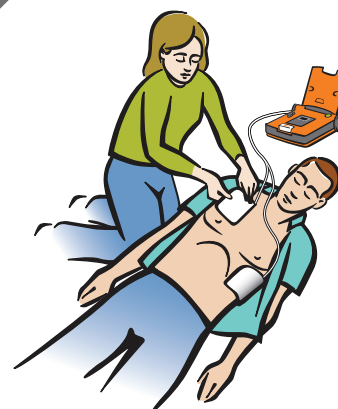

Bevestig pads.

7




Raak de patiënt niet aan!

8




Zo nodig dient AED
automatisch shock toe.

9

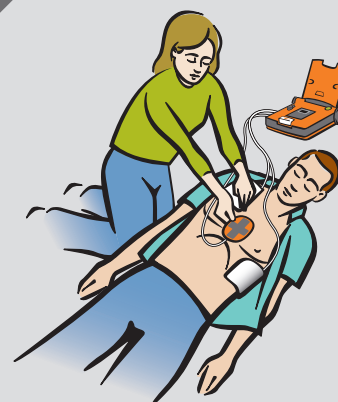

Indien aanwezig,
plaats reanimatieapparaat.

10

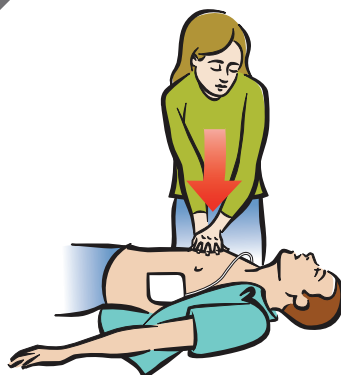

Volg instructies van AED.
Geef compressies.

11



Als opgedragen,
geef beademingen.

Automatische AED



Cardiac Science en het Shielded Heart-logo zijn handelsmerken
van Cardiac Science Corporation.
Copyright © 2019, Alle rechten voorbehouden.
500 Burdick Parkway, Deerfield, Wisconsin 53531 VS
www.cardiacscience.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Duitsland



70-02001-86 A



PERFORMING A RESCUE

1



Is patient responsive?

2




Call emergency services!

3



Get AED.

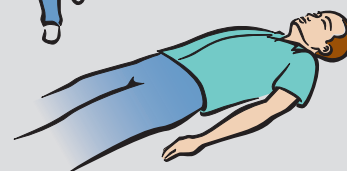

4




Place AED on ground.
Open lid.

5



Remove clothing.
Expose chest.

6

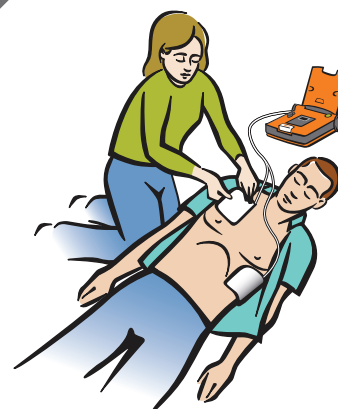

Attach pads.

7




Do not touch the patient.

8




If required, AED will deliver
shock automatically.

9

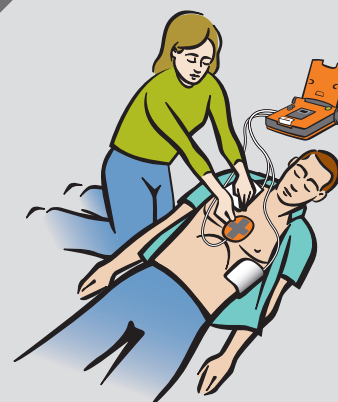

If present, place CPR device.

10

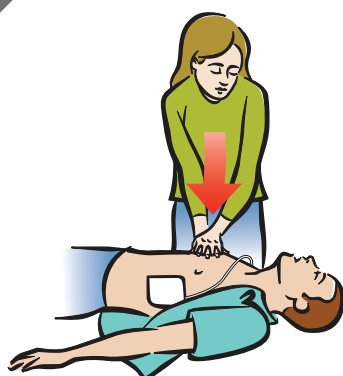

Follow AED prompts.
Give compressions.

11



If instructed, give breaths.

Automatic AED



Cardiac Science and the Shielded Heart logo are trademarks
of Cardiac Science Corporation.
Copyright © 2019 All Rights Reserved.
500 Burdick Parkway, Deerfield, Wisconsin 53531 USA
www.cardiacscience.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany



70-02001-86 A

