

AEDtrax™ Setup Guide for the ZOLL AED 3®



The issue date for the AEDtrax™ Setup Guide for the ZOLL AED 3® (REF 9650-004352-01 Rev. A) is **June, 2026**.

Copyright © 2026 ZOLL Medical Corporation. All rights reserved. AEDtrax, ZOLL AED 3, and ZOLL are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are the property of their respective owners.



ZOLL Medical Corporation

269 Mill Road
Chelmsford, MA USA
01824-4105



ZOLL International Holding B.V.

Einsteinweg 8A
6662 PW Elst
Netherlands



Table of Contents

Connecting to the ZOLL AED 3	1
Software Configuration	4
ZOLL AED 3 Software Configuration and Setup	4
Configure Self-Test Time	6
Confirm USB Transfer	7
Connecting an RFID Card	8
Connecting the RFID Card	8
RFID Card Placement	8

Connecting to the ZOLL AED 3

AEDtrax Shipping Box contents

1. AEDtrax (with batteries)
2. AEDtrax USB Cable
3. AEDtrax RFID Card
4. Torx T8 Tool
5. QR Code for the IFU

Note: To access the AEDtrax User's Guide, you can scan the QR code or go to www.zoll.com/ifu



1. Place ZOLL AED 3® device on its front and remove the CPR Uni-padz from the storage compartment.



2. Remove the USB cover. Using the provided Torx T8 screwdriver, remove the screw of the USB cover on the back of the ZOLL AED 3 device, making sure to keep the screw and washer.



3. Plug the supplied USB cable into the back of the AED and reattach the previously removed screw and washer.



4. Clean the right edge of the AED housing (viewed from the back) with a degreasing agent.



5. Remove the protective film from the cable holder.




6. Attach USB cable to the edge of the AED housing.



7. Turn On AEDtrax

- ✈ Transport Mode
- 🚫 AEDtrax ON - GPS-Off Mode
- 🔌 AEDtrax ON - GPS-On Mode

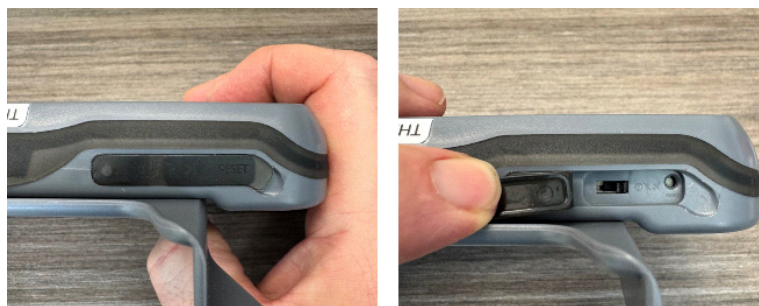


8. Extend the slide plate on AEDtrax.



-
9. Place AEDtrax on top of ZOLL AED 3 Device storage compartment and slip AEDtrax slide plate into place.



10. Insert USB-C cable into AEDtrax.

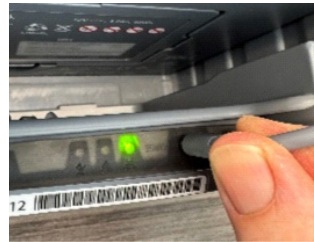

11. Place electrodes in ZOLL AED 3 storage compartment.



12. Maintain ZOLL AED 3 device right side up during storage and use.




Software Configuration

ZOLL AED 3 Software Configuration and Setup

1. Power the ZOLL AED 3 device on to non-rescue mode by pressing and holding the **ON/OFF** button for seven seconds or until you see the ZOLL logo.



2. Select the gears icon (configuration).

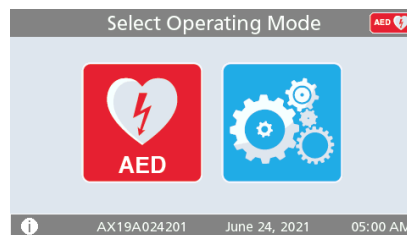

3. Select the lock icon (advanced configuration).



4. Enter access code.
The default is: 123456


Select **OK** once entered.





5. Select the gears icon (basic settings).




6. Select **Advanced**.

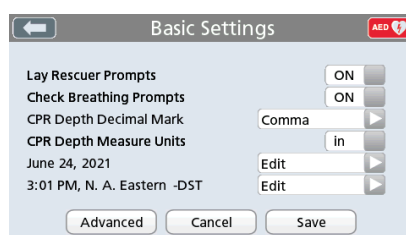


7. Turn USB Self Test Push Enabled to **ON**.

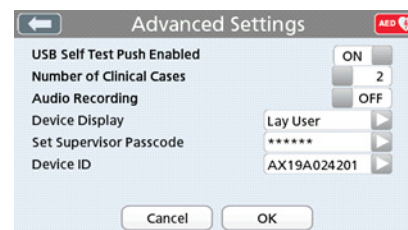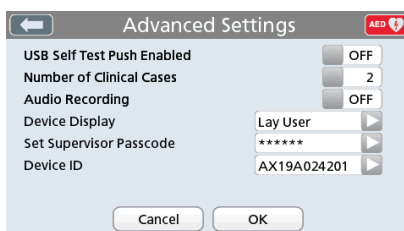

8. Select **OK**.

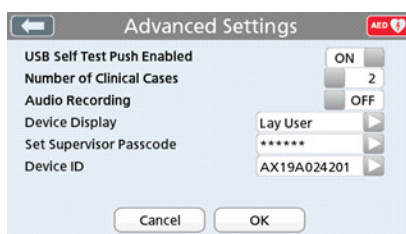

9. Select **Save**.

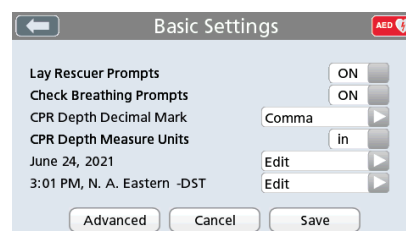

Configure Self-Test Time

To configure the day and time your ZOLL AED 3 completes a self test and transmits to PlusTrac, follow these steps.

1. Select the Wi-Fi icon (Wi-Fi settings).



2. Select **Device History Settings**.



3. Select **Self Test Settings**.



4. Configure:
 - a. Auto Self-Test Report: OFF
 - b. Self-Test Interval: 7
 - c. Custom Settings: ON
 - d. Scheduled Time: choose preferred time.
 - e. Selected Day: choose preferred day.
 - f. Select **OK**.



5. Select **Save** again to save and exit the window.




Confirm USB Transfer

1. Select Files icon (Export Files).



2. Select **Device History**.
3. Select **Save (USB)**.
4. Once the "Complete" message appears, exit and power off your device.

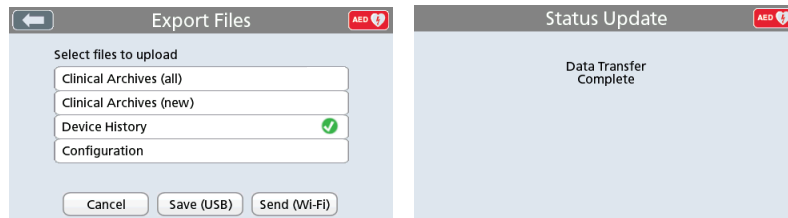

Note: Once you complete the configuration of your first ZOLL AED 3 device, you can copy your device configuration and import that to other ZOLL AED 3 devices. This will import the custom configuration into that device.

Connecting an RFID Card

The RFID card is only used with GPS enabled AEDtrax devices. The RFID card is used to:

- Alert PlusTrac when an AED is placed in a specific location.
- Alert PlusTrac when an AED is removed.
- Activate the GPS deployment tracking feature when an AED is removed.

Connecting the RFID Card

Place the RFID card in close proximity to AEDtrax. Once the card establishes connection, you will hear a beep.



RFID Card Placement

Place the RFID card in the cabinet (or on a firm surface) so that the card can connect with the AEDtrax upper left corner when the ZOLL AED 3 is placed into the cabinet.

Note: If the surface is metallic, use a distance holder of minimum 1 inch (2.5 cm) between the metal surface and the card.