

Is patient responsive?



Place AED on ground. Open lid.



Do not touch the patient.



Follow AED prompts. Give compressions.



Call emergency services!



Remove clothing. Expose chest.

8

11



If required, AED will deliver shock automatically.



If instructed, give breaths.



Get AED.

Attach pads.



If present, place CPR device.

Automatic AED

Cardiac Science and the Shielded Heart logo are trademarks of Cardiac Science Corporation. Copyright © 2021 All Rights Reserved. 500 Burdick Parkway, Deerfield, Wisconsin 53531 USA www.cardiacscience.com

