

ZOLL AED 3™ BLS

Technical Specifications



Enhanced Real CPR Help

The ZOLL AED 3™ BLS provides enhanced Real CPR Help®, which measures the actual depth and rate of each compression and displays it numerically on the CPR Dashboard™ feature. The CPR Dashboard also shows elapsed time, CPR cycle countdown, shocks delivered, and ECG. The Real CPR Help integrated, real-time CPR feedback tells and shows rescuers when they are administering high-quality CPR.

Integrated Pediatric Rescue

Pediatric rescue is made easier with ZOLL's unique CPR Uni-padz™ electrodes and a child-mode setting. Universal pad design provides rescuers with a single, confident solution to treat both adult and pediatric victims of SCA by using the same set of electrode pads and simply activating child mode. The CPR Dashboard displays numerical CPR depth, rate, and cycle time for additional guidance to the rescuer.

Low Total Cost of Ownership

Longer-lasting, 5-year battery and universal electrode pads reduce AED device maintenance requirements and cost. Less maintenance and one set of pads for adults and children can improve readiness while reducing the total cost of ownership.

Wi-Fi and USB Connectivity

The 2015 AHA Guidelines state that a "continuous quality improvement approach to CPR can dramatically improve CPR quality and optimize outcomes...CPR data collection, implementation of best practices, and continuous feedback on performance have been shown to be effective."¹ Using RescueNet® CaseReview, detailed rescue performance data can be exported quickly and easily via USB or transferred directly over Wi-Fi. Data on CPR rate, depth, release velocity, and compression fraction can easily be evaluated and used to improve future responder performance. ZOLL's connectivity also enables fast and easy distribution of event and ECG data to medical personnel.

¹Kleinman ME, et al. *Circulation*. 2015; 132(suppl 2):S423.



The ZOLL CPR Dashboard provides real-time numerical feedback on CPR depth, rate, and cycle time.



Child mode, when activated, invokes the pediatric heart analysis algorithm and reduces energy delivered.



Wi-Fi and USB connectivity enable quick and easy access to event data.



CPR Uni-pads enable treatment for both adult and pediatric victims using the same set of electrode pads.

ZOLL AED 3 BLS Specifications

Defibrillator

Protocol: Semi-automatic

Waveform: ZOLL Rectilinear Biphasic™

Defibrillator Charge Hold Time: 30 seconds

Energy Selection: Factory preprogrammed selection (Adult: 120 J, 150 J, 200 J; Child: 50 J, 70 J, 85 J). User configurable.

Patient Safety: All patient connections are electrically isolated.

Charge Time: Less than 10 seconds with new battery

Pre-shock Pause: 8 seconds with new battery

Electrodes: ZOLL CPR Uni-pad

Self-test: User-configurable automatic self-test every day or every 7 days. Default: Every 7 days. Monthly full-energy test (200 J).

Automatic Self-test Checks: Battery capacity, status, and expiration; electrode connection and expiration; ECG and charge/discharge circuits; microprocessor hardware and software; CPR circuitry and pads sensor; audio circuitry

CPR Metronome Rate: Constant 105 (+/- 2) CPM

Depth Measurement: 1.9 cm to 10.2 cm; 0.75 in to 4 in

Defibrillation Advisory: Evaluates electrode connection and patient ECG to determine if defibrillation is required

Shockable Rhythms: Ventricular fibrillation with average amplitude >100 microvolts and wide complex ventricular tachycardia with rates greater than 150 BPM for adults, 200 BPM for pediatrics. For ECG analysis algorithm sensitivity and specificity, refer to the ZOLL AED 3 Administrator's Guide.

Patient Impedance Measurement Range: 10 ohms to 300 ohms

Defibrillator: Protected ECG circuitry

Display Format: High-resolution LCD with capacitive touch panel

Display Screen Size: 5.39 cm x 9.5 cm; 2.12 in x 3.74 in

Display Sweep Speed: 25 mm/sec

Display Viewing Speed: 3.84 seconds

Data Recording and Storage: User-configurable for 1 or 2 clinical events for total of 120 minutes. Includes ECG, impedance measurements, device prompts, and CPR data. With voice recording enabled, same data with synchronous audio added for total of 60 minutes.

Data Recovery: Controlled by touchscreen, uploaded to USB memory stick, or RescueNet CaseReview, over a Wi-Fi network

Internal Clock Synchronization: Coordinated Universal Time (UTC) synchronization occurs when communicating with the ZOLL Online server.

Device

Size: (H x W x D) 12.7 cm x 23.6 cm x 24.7 cm; 5.0 in x 9.3 in x 9.7 in

Weight: 2.5 kg; 5.5 lbs

Power: Lithium manganese dioxide battery pack

Wireless: 802.11 a/b/g/n

Security Protocols: WPA1, WPA 2, WPA Personal, WPA Enterprise

Port: USB 2.0

Audio Recording: User-configurable on/off (default=off)

Device Classification: Class III and internally powered per EN 60601-1

Design Standards: Meets applicable requirements of EN 60601-1, EN 60601-1-11, IEC 60601-2-4

Environmental

Operating Temperature: 0° to 50°C; 32° to 122°F

Storage Temperature: -30° to 70°C; -22° to 158°F

Humidity: 10% to 95% relative humidity, non-condensing

Vibration: IEC 60068-2-64, Random, Spectrum A.4, Table A.8, Cat. 3b; RTCA/DO-160G, Fixed Wing Aircraft, Section 8.6, Test Cat. H, Aircraft Zone 1 and 2; EN1789, Sweep per EN 60068-2-6 Test Fc

Shock: IEC 60068-2-27; 100G

Altitude: -381 m to 4,573 m; -1,250 to 15,000 ft

Particle and Water Ingress: IP55

Drop Test: 1 meter; 3.28 ft

Battery

Battery Capacity: Typical new battery running at an ambient temperature of +20° C to +25° C (68° F to 77° F) can provide 140 defibrillator discharges at maximum energy (200 joules), or 6 hours of continuous monitoring (with 2-minute CPR periods). **Note:** CPR periods shorter than 2 minutes can decrease the operating time that can be obtained from a new battery.

Type: Disposable, sealed lithium manganese dioxide

Battery Standby Life (once installed): 5 years with weekly self-test. Battery end of life indicated by blank status window (typical remaining shocks: 9).

Battery Shelf Life: Store for up to 2 years at 23°C (77°F) prior to installing in ZOLL AED 3 BLS to maintain battery life detailed above.

Temperature: 0°C to 50°C (32°F to 122°F)

Humidity: 10% to 95% (non-condensing)

Weight: 317.5 grams; 0.7 lbs

Size: (H x W x D) 27.75 mm x 133 mm x 88 mm; 1.0 in x 5.16 in x 3.5 in

Nominal Voltage: 12 volts

CPR Uni-pad

Shelf Life: 5 years

Conductive Gel: Polymer Hydrogel

Conductive Element: Tin

Packaging: Multilayer foil laminate pouch

Impedance Class: Low

Cable Length: 142 (+/- 3.8) cm; 56 (+/- 1.5) in

Design Standards: Meets requirements of IEC 60601-2-4

AED 3 BLS Carry Bag

Size: (H x W x D) 29.2 cm x 27.4 cm x 17.8 cm; 11.5 in x 10.8 in x 7.0 in

Weight: 3.4 kg; 7.5 lbs (ZOLL AED 3 BLS with battery installed and CPR Uni-pad preconnected in carry bag)

Holds: AED 3 BLS with battery inserted and back-up set of CPR Uni-pad

ZOLL Medical Corporation Worldwide Headquarters

269 Mill Road
Chelmsford, MA 01824
978-421-9655
800-804-4356

For subsidiary addresses and fax numbers, as well as other global locations, please go to www.zoll.com/contacts.

Specifications subject to change without notice.

©2017 ZOLL Medical Corporation. All rights reserved. CPR Dashboard, Real CPR Help, RescueNet, Uni-pad, ZOLL, ZOLL AED 3, and ZOLL Rectilinear Biphasic are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are the property of their respective owners.

Printed in U.S.A. MCN PP 1711 0276

ZOLL