

ResQGARD

QUICK REFERENCE GUIDE

ZOLL

The ResQGARD[®] increases circulation in conscious, breathing patients with low blood pressure due to:

- Dehydration
- Heat shock
- Early sepsis
- Orthostatic intolerance
- Blood loss



Do NOT Use if: internal bleeding is suspected or patient has chest trauma or uncontrolled bleeding. Remove if patient develops respiratory distress.

Quick Instructions

- Control bleeding if patient is hemorrhaging.
- Check vital signs and assess need to apply ResQGARD.
- Attach ResQGARD to facemask.
- Place mask over nose and mouth and use headstrap to secure. Ensure tight seal.
- Instruct patient to breathe normally.
- Attach oxygen tubing to oxygen port if needed to deliver up to 15 lpm.
- Reassess vital signs frequently.
- Remove once normal blood pressure levels are re-established.
- Reapply if there is another drop in blood pressure.