
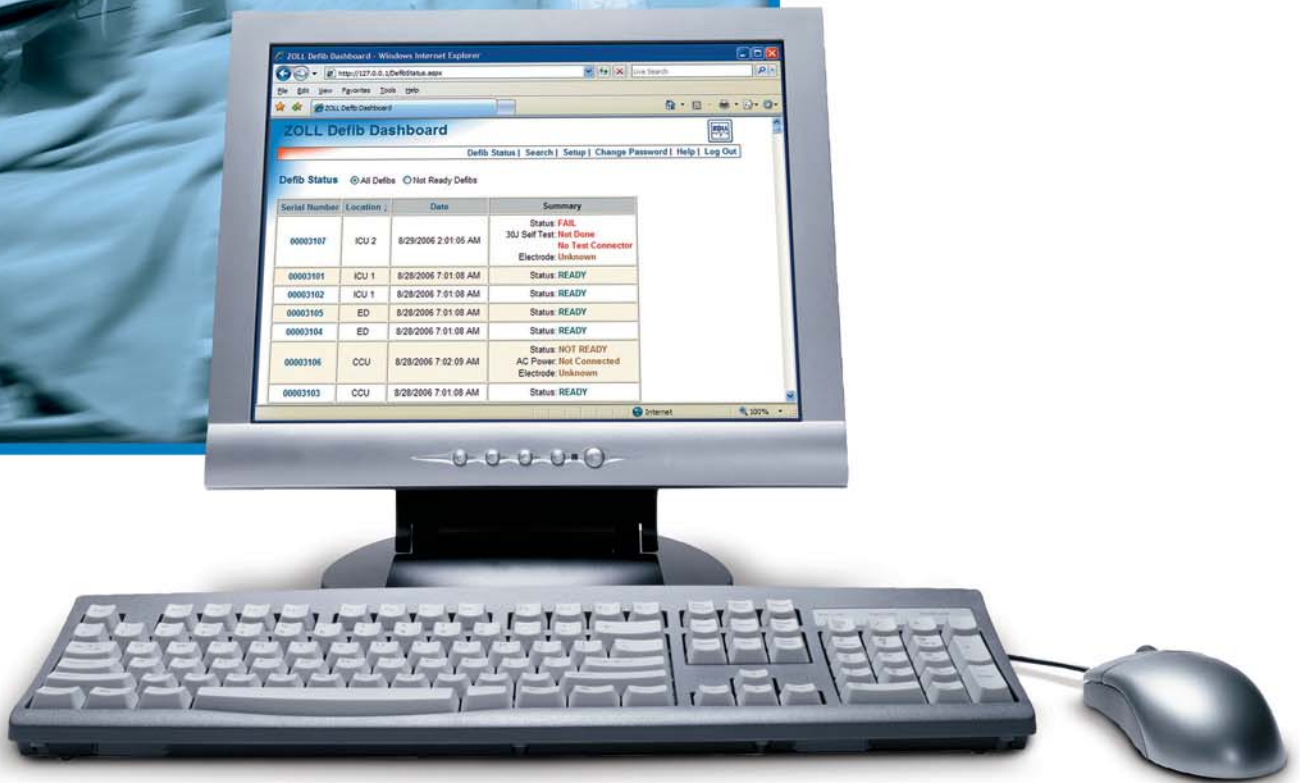


Rseries®

Wireless Networking That Ensures You're Always Code-Ready.®

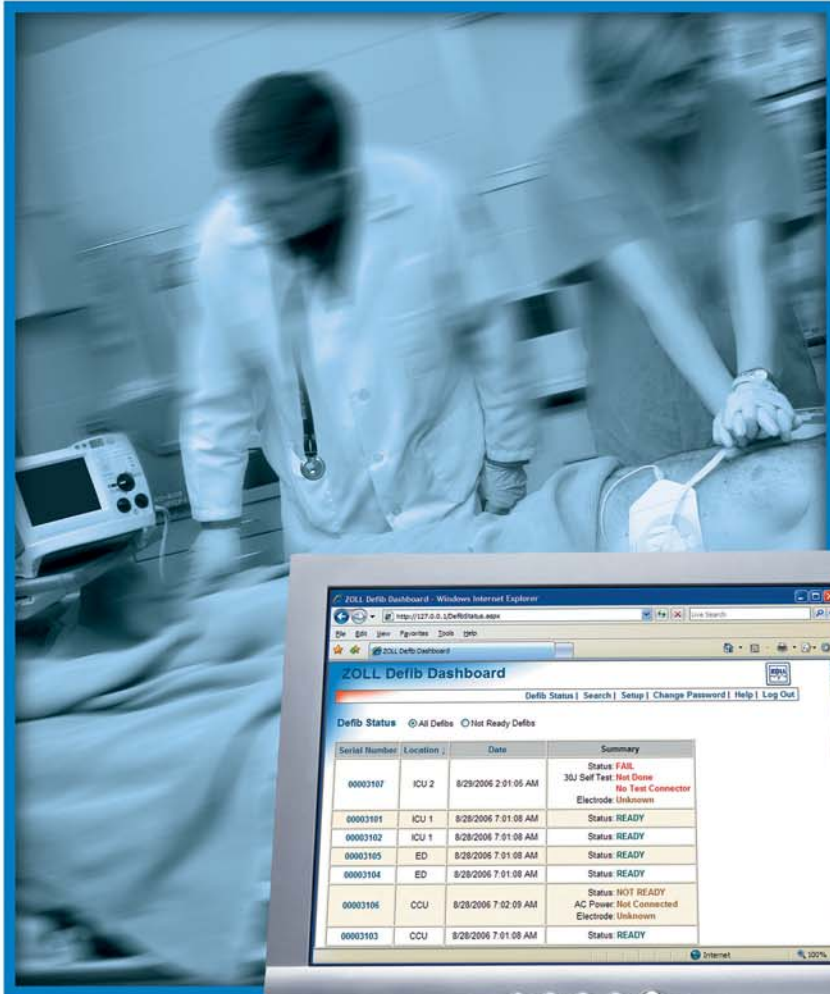


The worst time to find out your defibrillator isn't ready is at the code. Only ZOLL® lets you build a code-readiness network. Based on standard 802.11 WiFi technology, the R Series® automatically sends an alert when its state of readiness is compromised. This early notification maximizes patient safety by letting you intervene before the defibrillator is needed for a code.

ZOLL
Advancing Resuscitation. Today.®


Wireless for Code Readiness

Defibrillators rarely fail. Instead, their readiness gets compromised because someone forgets to plug them in, a cable is missing, or the electrodes are old and dried out. With WiFi, the R Series automatically alerts key personnel when any of the 43 readiness indicators are compromised. The notification identifies contributing factors, and code readiness can be restored with unmatched efficiency as troubleshooting begins from any PC on the network.

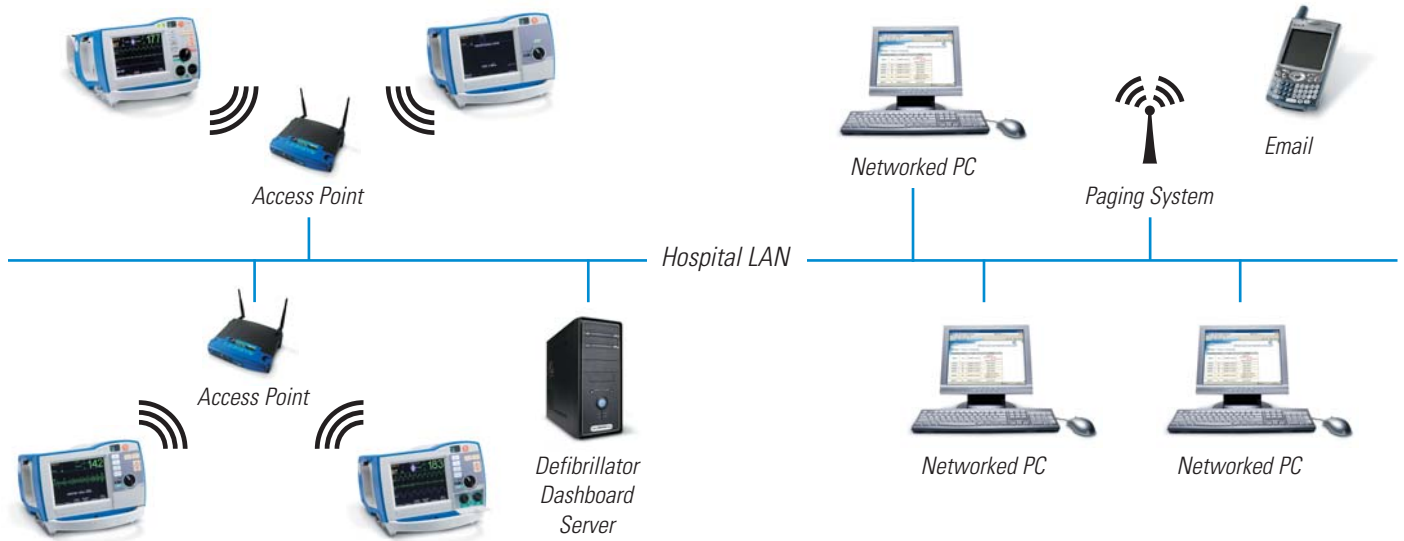

Wireless for Complete Code Documentation

The Defibrillator Summary is an important part of the code record. The R Series WiFi capability simplifies merging this summary with event documentation. At the end of the code, the Defibrillator Summary can be transmitted over a standard wireless network to a server hosting the CodeNet® system, where it is merged and time-synchronized with the Event Report documented on CodeNet Writer.

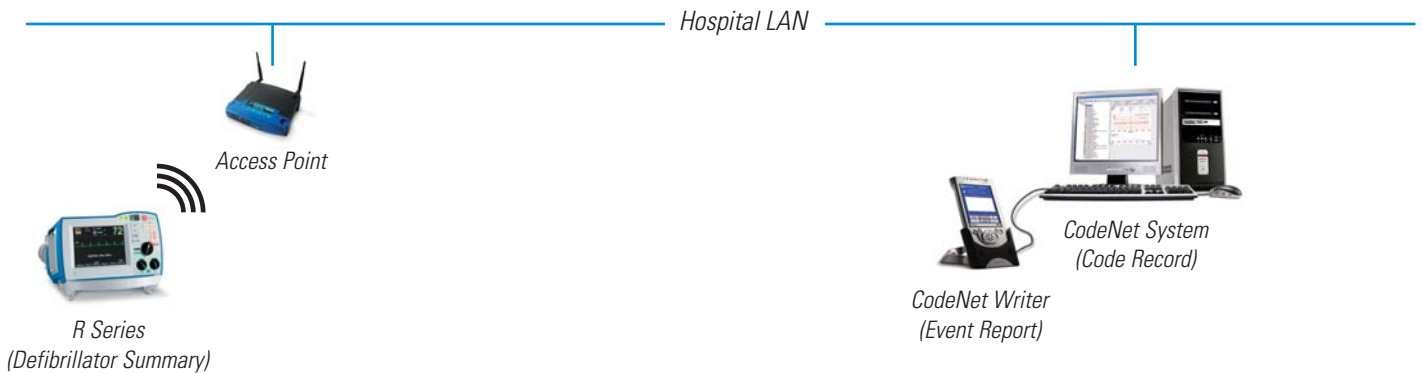

R Series Technical Specifications

Wireless Requirements

Expansion Package	WiFi 802.11b compatibility
WiFi Card	WiFi Compact Flash Card ZOLL P/N: 9214-0203 or Ambicom WL1100C-CF WiFi Card
File Sizes	Max Readiness Test Log: 750K Typical Code Data File: 1.2 MB

Please reference the Defibrillator Dashboard Spec Sheet and CodeNet Technical Note for further information.

ZOLL Medical Corporation
Worldwide Headquarters
 269 Mill Road
 Chelmsford, MA 01824
 978-421-9655
 800-348-9011

For the ZOLL location nearest you,
 visit www.zoll.com/contacts.

© ZOLL Medical Corporation 2008. All rights reserved.
 "Advancing Resuscitation. Today.", CodeNet, Code-Ready, R Series, and ZOLL are registered trademarks of ZOLL Medical Corporation. All other trademarks are the property of their respective owners.
 Specifications subject to change without notice.

Printed in U.S.A. 120810
 9656-0205

ZOLL
 Advancing Resuscitation. Today.®