

Thermogard XP®

CLEANING INSTRUCTIONS



Process for Cleaning the Thermogard XP

- Ensure that the console has been switched off and the power cord has been unplugged.
- Clean the exterior of the console using a soft cloth dampened with a mild detergent and water mixture.
- Never use solvents or abrasive cleaners on the console. Avoid vigorous scrubbing, especially on the front surface of the display.
- For general decontamination and cleaning, we recommend germicidal disposable wipes that include at least 55% isopropyl alcohol or other approved non-corrosive hospital-grade disinfectant.
- ZOLL highly recommends testing the disinfectant on a small area on the outer surface of the console prior to wiping all surfaces of the console. Follow directions provided by the manufacturer of the disinfectant.
- After cleaning, wipe down the outside surfaces with a water-dampened cloth to remove remaining spots or residue.
- Slide out and remove the condensate collection pan from under the front of the console. The pan may be washed in hot soapy water. Never use solvents or abrasive cleaners on the pan. Dry the pan when finished and reinstall it in the console.

Note: Do not operate the console without a condensate collection pan properly installed. Failure to do so may allow water to accumulate on the floor under the console.

- Refer to the Thermogard XP Operation Manual for managing coolant solution, full cleaning and operating instructions.

Products for Cleaning the Thermogard XP

- 70% Isopropyl Alcohol (IPA) Wipes
- Bleach Germicidal Wipes
- Warm Water and Soap